

JAN 17 1997

BioHorizons Implant Systems, Inc.
510(k) Notification
October 24, 1996

K964330

510(k) Summary of Safety and Effectiveness

Proprietary Name

BioHorizons Dental Implant System (name subject to change)

Common Name

Uncoated, titanium plasma spray, and hydroxyapatite screw-form implants, surgical components, and prosthetic components

Classification Name

Endosseous implants, surgical components, and prosthetic attachments

Classification

Class III

Official Contact

R. Steven Boggan, M.S., M.B.A.
President and Chief Operating Officer
BioHorizons Implant Systems, Inc.
2129 Montgomery Highway
Birmingham, AL 35209
(205) 871-1345
FAX (205) 870-0304

Device Description

The BioHorizons Dental Implant System is a comprehensive system containing implants, surgical components, and prosthetic components. The implants are specifically designed to optimize strain distribution to contiguous bone under functional loading in order to promote strain-induced bone growth and interface maintenance over the long term. This improvement in biomechanical performance is achieved by optimizing implant designs specifically for each bone density classification (D1, D2, D3, D4) in the mandible and maxilla.

Four implant designs, corresponding to each bone density classification, are available in four and five mm diameters. Each implant design, manufactured from titanium alloy

BioHorizons Implant Systems, Inc.
510(k) Notification
October 24, 1996

conforming to ASTM F 136, is available in two lengths and may feature a titanium plasma-spray (TPS) or hydroxyapatite (HA) coating. The following table provides a comprehensive summary of implant diameter, length, and coating.

Diameter (mm)	Design	Lengths (mm)	Coating
φ4.00	D1	10, 12	Uncoated
	D2	11, 13	Uncoated
	D3	12, 14	TPS
	D4	13, 15	HA
φ5.00	D1	9, 11	Uncoated
	D2	10, 12	Uncoated
	D3	11, 13	TPS
	D4	12, 14	HA

Product Evaluation

Evaluation of the BioHorizons Dental Implant System consisted mechanical testing of the implant and bioactive coating mechanical tests. These analyses indicate that the BioHorizons Dental Implant System should be safe and effective when used as intended.

Indications

The BioHorizons Dental Implant System may be used in the mandible and maxilla for use as an artificial root structure for single tooth replacement or as abutments for fixed bridgework and denture retention.

Substantial Equivalence Information

The BioHorizons Dental Implant System is substantially equivalent in all features which could affect safety or effectiveness to the BioHorizons Dental Implant System (K960026) and the Steri-Oss[®] Hex-Lock (HL) Threaded Titanium Implants.