

SEP - 2 1997

1C972556

510(k) SUMMARY
As Required By Section 807.92(c)

QuickScreen™ One Step Methamphetamine Test

Immunoassay for the Qualitative Detection of Methamphetamine in Urine

Name of Product: QuickScreen™ One Step Methamphetamine Test

Name Of Packager: Phamatech
9265 Activity Road #112
San Diego, California 92126
USA

Name Of Manufacturer: Phamatech
9520 Padgett Street #108
San Diego, California 92126
USA

Site of Control Testing: Phamatech
9265 Activity Road #112
San Diego, California 92126
USA

Sites of Clinical Testing: Poison Laboratories
7272 Clairemont Mesa Blvd.
San Diego, CA 92111

Quest Diagnostics Incorporated
7470 Mission Valley Road
San Diego, CA 92108

Regulatory Control Number: 011

510 (k) SUMMARY
AS REQUIRED BY SECTION 807.92(C)

Identification: QuickScreen™ One Step Methamphetamine Screening Test (9065)

Description: Immunoassay for the Qualitative Detection of Methamphetamine in Urine

Name Of Manufacturer: Phamatech
9265 Activity Road #112 / 113
San Diego, California 92126, USA

Intended Use: A drug of abuse assay intended for use in clinical toxicology laboratories, physicians' offices, drug-of-abuse clinics and law enforcement agencies is an in-vitro diagnostic test for the qualitative identification of methamphetamine, a central nervous system stimulating drug, in urine. Measurements that are obtained by this device are used in the diagnosis and treatment of methamphetamine use or overdose

Technology: The QuickScreen™ One Step Methamphetamine Test utilizes colloidal gold as the label like other commercially available immunoassays for drug of abuse (methamphetamine) test kits, to qualitatively measure the presence of methamphetamine by visual color sandwich one step immunoassay technology. Examples of such predicate devices include the ABI SureStep (San Diego, CA 92121) and the Syntron Bioresearch Methamphetamine Test (Vista, CA 92083). All of the above devices rely on the basic immunochemical sandwich assay principle of recognition and formation of specific antibody / Methamphetamine / antibody / complexes.

Performance: The product performance characteristics of the QuickScreen™ One Step Methamphetamine Test was evaluated in a clinical sample correlation study and a blind labeled methamphetamine study. The results of these studies demonstrate the Phamatech QuickScreen™ One Step Methamphetamine Test to be substantially equivalent to the reported performance characteristics of other commercially available tests for the qualitative detection of methamphetamine in urine. Correlation studies, using clinical specimens, produced a sensitivity of >95%, specificity of >99% and accuracy >97% when compared to the Syva EMIT II (San Jose, CA 95161). Two clinical laboratory studies were performed, the Phamatech QuickScreen™ exhibited excellent sensitivity (84/86), specificity (39/39), and accuracy (123/125) in the hands of professional laboratory technicians.

Conclusion: For the reasons mentioned above, it may be concluded that the Phamatech QuickScreen™ One Step Methamphetamine Screening Test is substantially equivalent to a variety of qualitative methamphetamine tests currently in commercial distribution.



SEP - 2 1997

Food and Drug Administration
2098 Gaither Road
Rockville MD 20850

Tuan H. Pham
President
Phamatech, Inc.
9265 Activity Road, Suite 112-113
San Diego, California 92126

Re: K972556
QuickScreen™ One Step Methamphetamine Test
Regulatory Class: II
Product Code: LAF
Dated: July 7, 1997
Received: July 9, 1997

Dear Tuan H. Pham:

We have reviewed your Section 510(k) notification of intent to market the device referenced above and we have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (Pre-market Approval), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 895. A substantially equivalent determination assumes compliance with the Current Good Manufacturing Practice requirements, as set forth in the Quality System Regulation (QS) for Medical Devices: General regulation (21 CFR Part 820) and that, through periodic QS inspections, the Food and Drug Administration (FDA) will verify such assumptions. Failure to comply with the GMP regulation may result in regulatory action. In addition, FDA may publish further announcements concerning your device in the Federal Register. Please note: this response to your pre-market notification submission does not affect any obligation you might have under sections 531 through 542 of the Act for devices under the Electronic Product Radiation Control provisions, or other Federal laws or regulations.

Page 2

Under the Clinical Laboratory Improvement Amendments of 1988 (CLIA-88), this device may require a CLIA complexity categorization. To determine if it does, you should contact the Centers for Disease Control and Prevention (CDC) at (770) 488-7655.

This letter will allow you to begin marketing your device as described in your 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801 and additionally 809.10 for in vitro diagnostic devices), please contact the Office of Compliance at (301) 594-4588. Additionally, for questions on the promotion and advertising of your device, please contact the Office of Compliance at (301) 594-4639. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR 807.97). Other general information on your responsibilities under the Act may be obtained from the Division of Small Manufacturers Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its internet address "<http://www.fda.gov/cdrh/dsmamain.html>".

Sincerely yours,

Steven Gutman

Steven I. Gutman, M.D., M.B.A.
Director
Division of Clinical
Laboratory Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

INDICATIONS FOR USE

Applicant: Phamatech

510(k) Number (if known): 972556

Device Name: QuickScreen™ One Step Methamphetamine Test

Indications for Use:

A drug of abuse assay intended for use in clinical toxicology laboratories, physicians' offices, drug-of-abuse clinics and law enforcement agencies is an in-vitro diagnostic test for the qualitative identification of methamphetamine, a central nervous system stimulating drug, in urine. Measurements that are obtained by this device are used in the diagnosis and treatment of methamphetamine use or overdose.

PLEASE DO NOT WRITE BELOW THIS LINE

Concurrence of CDRH Office of Device Evaluation (ODE)



(Division Sign-Off)
Division of Clinical Laboratory Devices
510(k) Number 972556

Division Sign-Off
Division of Clinical Laboratory Devices
510(k) Number:

Prescription Use: _____
Per 21 CFR 801.109

OR Over the Counter _____