

K014206

Special 510(k) Premarket Notification
GE Medical Systems - LOGIQbook Ultrasound System
December 20, 2001

JAN 16 2002

Attachment B:
Summary of Safety and Effectiveness
Prepared in accordance with 21 CFR Part 807.92(c).



GE Medical Systems

General Electric Company
P.O. Box 414, Milwaukee, WI 53201

Section a):

1. Submitter: GE Medical Systems
PO Box 414
Milwaukee, WI 53201

Contact Person: Allen Schuh,
Manager, Safety and Regulatory Engineering
Telephone: 414-647-4385; Fax: 414-647-4090

Date Prepared: December 20, 2001

2. Device Name: GE LOGIQbook Diagnostic Ultrasound System
Ultrasonic Pulsed Doppler Imaging System, 21 CFR 892.1550, 90-IYN
Ultrasonic Pulsed Echo Imaging System, 21 CFR 892.1560, 90-IYO
Diagnostic Ultrasonic Transducer, 21 CFR 892.1570, 90-ITX

3. Marketed Device: GE LOGIQ 100 Diagnostic Ultrasound System, 510(k) Nos: K953752 and K012560 currently in commercial distribution.

4. Device Description: The GE LOGIQbook is a compact and portable diagnostic ultrasound system with integrated keyboard, fold-up LCD type display and interchangeable electronic-array transducers. It has an overall size approximately 33 cm wide, 27 cm deep and 7 cm high in transport configuration and provides digital acquisition, processing and display capability. The user interface includes a computer keyboard, an intuitive layout of specialized controls, color GUI display and Doppler audio.

5. Indications for Use: The device is intended for use by a qualified physician for ultrasound evaluation of Fetal/OB; Abdominal (GYN & Urology); Pediatric; Small Organ (breast, testes, thyroid); Neonatal Cephalic; Cardiac (adult and pediatric); Peripheral Vascular; Musculo-skeletal Conventional; Transrectal; and Transvaginal.

6. Comparison with Predicate Device: The GE LOGIQbook is of a comparable type and substantially equivalent to the currently marketed GE LOGIQ 100. It is a compact and readily portable unit having the same technological characteristics of design, construction, and materials; is comparable in key safety and effectiveness features; and has the same intended uses as the predicate device.

Section b):

1. Non-clinical Tests: The device has been evaluated for acoustic output, biocompatibility, cleaning and disinfection effectiveness as well as thermal, electrical and mechanical safety, and has been found to conform with applicable medical device safety standards.

2. Clinical Tests: None required.

3. Conclusion: Intended uses and other key features are consistent with traditional clinical practice, FDA guidelines, and established methods of patient examination. The design and development process of the manufacturer conforms with 21 CFR 820, ISO 9001 and EN 46001 quality systems. The device conforms to applicable medical device safety standards and compliance is verified through independent evaluation with ongoing factory surveillance. Diagnostic ultrasound has accumulated a long history of safe and effective performance. Therefore, it is the opinion of GE Medical Systems that the GE LOGIQbook Diagnostic Ultrasound is substantially equivalent with respect to safety and effectiveness to devices currently cleared for market.



Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

JAN 16 2002

Mr. Allen Schuh
Manager, GE Ultrasound Safety
and Regulatory Engineering
GE Medical Systems
P.O. Box 414
MILWAUKEE WI 53201

Re: K014206
Trade Name: GE LOGIQbook Diagnostic Ultrasound System
Regulation Number: 21 CFR 892.1550
Regulation Name: Ultrasonic pulsed doppler imaging system
Product Code: 90 IYN
Regulation Number: 21 CFR 892.1560
Regulation Name: Ultrasonic pulsed echo imaging system
Product Code: 90 IYO
Regulatory Class: II
Dated: December 20, 2001
Received: December 21, 2001

Dear Mr. Schuh:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and we have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

This determination of substantial equivalence applies to the following transducers intended for use with the GE LOGIQbook Diagnostic Ultrasound System, as described in your premarket notification:

Transducer Model Number

3C-RS
3MC-RS
CZB-RS

E8C-RS
10LB-RS

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register. Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

This determination of substantial equivalence is granted on the condition that prior to shipping the first device, you submit a postclearance special report. This report should contain complete information, including acoustic output measurements based on production line devices, requested in Appendix G, (enclosed) of the Center's September 30, 1997 "Information for Manufacturers Seeking Marketing Clearance of Diagnostic Ultrasound Systems and Transducers." If the special report is incomplete or contains unacceptable values (e.g., acoustic output greater than approved levels), then the 510(k) clearance may not apply to the production units which as a result may be considered adulterated or misbranded.

The special report should reference the manufacturer's 510(k) number. It should be clearly and prominently marked "ADD-TO-FILE" and should be submitted in duplicate to:

Food and Drug Administration
Center for Devices and Radiological Health
Document Mail Center (HFZ-401)
9200 Corporate Boulevard
Rockville, Maryland 20850

This letter will allow you to begin marketing your device as described in your premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus permits your device to proceed to market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801, please contact the Office of Compliance at (301) 594-4591. Additionally, for questions on the promotion and advertising of your device, please contact the Office of Compliance at (301) 594-4639. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). Other general information on your responsibilities under the Act may be obtained from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or at (301) 443-6597 or at its Internet address "<http://www.fda.gov/cdrh/dsmamain.html>".

Page 3 - Mr. Schuh

If you have any questions regarding the content of this letter, please contact Rodrigo C. Perez at (301) 594-1212.

Sincerely yours,

A handwritten signature in black ink that reads "Nancy C. Brogdon". The signature is written in a cursive style with a large initial 'N'.

Nancy C. Brogdon
Director, Division of Reproductive,
Abdominal and Radiological Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure(s)

Diagnostic Ultrasound Indications for Use Form
GE LOGIQbook with 3MC-RS Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application <i>Anatomy/Region of Interest</i>	Mode of Operation										
	B	M	PW Doppler	CW Doppler	Color Doppler	Color M Doppler	Power Doppler	Combined Modes*	Harmonic Imaging	Coded Pulse	Other
Ophthalmic											
Fetal / Obstetrics	P	P	N		N		N	N	N		
Abdominal ^[1]	P	P	N		N		N	N	N		
Pediatric											
Small Organ (specify)											
Neonatal Cephalic											
Adult Cephalic											
Cardiac ^[3]	P	P	N		N		N	N	N		
Peripheral Vascular											
Musculo-skeletal Conventional											
Musculo-skeletal Superficial											
Other ^[4]											
<i>Exam Type, Means of Access</i>											
Transesophageal											
Transrectal											
Transvaginal											
Transurethral											
Intraoperative (specify)											
Intraoperative Neurological											
Intravascular											
Laparoscopic											

N = new indication; P = previously cleared by FDA; E = added under Appendix E

Notes: [1] Abdominal includes GYN/Pelvic, Renal and Aorta-iliac artery;

[3] Cardiac is Adult and Pediatric.

[*] Combined modes are B/M, B/PWD, B/Color/PWD, B/Power/PWD.

(PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Nancy C Brogdon

(Division Sign-Off)
 Division of **Reproductive, Abdominal,
 and Radiological Devices**
 510(k) Number 1K014206

Diagnostic Ultrasound Indications for Use Form
GE LOGIQbook with CZB-RS Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application <i>Anatomy/Region of Interest</i>	Mode of Operation										
	B	M	PW Doppler	CW Doppler	Color Doppler	Color M Doppler	Power Doppler	Combined Modes*	Harmonic Imaging	Coded Pulse	Other
Ophthalmic											
Fetal / Obstetrics											
Abdominal ^[1]	P	P	N		N		N	N	N		
Pediatric	P	P	N		N		N	N	N		
Small Organ (specify)	P	P	N		N		N	N	N		
Neonatal Cephalic	P	P	N		N		N	N	N		
Adult Cephalic											
Cardiac ^[3]	P	P	N		N		N	N	N		
Peripheral Vascular	P	P	N		N		N	N	N		
Musculo-skeletal Conventional	P	P	N		N		N	N	N		
Musculo-skeletal Superficial											
Other ^[4]											
<i>Exam Type, Means of Access</i>											
Transesophageal											
Transrectal											
Transvaginal											
Transurethral											
Intraoperative (specify)											
Intraoperative Neurological											
Intravascular											
Laparoscopic											

N = new indication; P = previously cleared by FDA; E = added under Appendix E

Notes: [1] Abdominal includes GYN/Pelvic, Renal and Aorta-iliac artery;

[3] Cardiac is Adult and Pediatric.

[*] Combined modes are B/M, B/PWD, B/Color/PWD, B/Power/PWD.

(PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)


 (Division Sign-Off)
 Division of Reproductive, Abdominal,
 and Radiological Devices
 510(k) Number RC4206

Diagnostic Ultrasound Indications for Use Form
GE LOGIQbook with 3C-RS Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application <i>Anatomy/Region of Interest</i>	Mode of Operation										
	B	M	PW Doppler	CW Doppler	Color Doppler	Color M Doppler	Power Doppler	Combined Modes	Harmonic Imaging	Coded Pulse	Other
Ophthalmic											
Fetal / Obstetrics	P	P	N		N		N	N	N		
Abdominal ^[1]	P	P	N		N		N	N	N		
Pediatric	P	P	N		N		N	N	N		
Small Organ (specify)											
Neonatal Cephalic											
Adult Cephalic											
Cardiac ^[3]	P	P	N		N		N	N	N		
Peripheral Vascular											
Musculo-skeletal Conventional											
Musculo-skeletal Superficial											
Other ^[4]	P	P	N		N		N	N	N		
<i>Exam Type, Means of Access</i>											
Transesophageal											
Transrectal											
Transvaginal											
Transurethral											
Intraoperative (specify)											
Intraoperative Neurological											
Intravascular											
Laparoscopic											

N = new indication; P = previously cleared by FDA; E = added under Appendix E

Notes: [1] Abdominal includes GYN;

[3] Cardiac is Adult and Pediatric.

[4] Other use includes Urology;

[*] Combined modes are B/M, B/PWD, B/Color/PWD, B/Power/PWD.

(PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)


 (Division Sign-Off)
 Division of Reproductive, Abdominal,
 and Radiological Devices
 510(k) Number K014206

Diagnostic Ultrasound Indications for Use Form
GE LOGIQbook Ultrasound System

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application <i>Anatomy/Region of Interest</i>	Mode of Operation										
	B	M	PW Doppler	CW Doppler	Color Doppler	Color M Doppler	Power Doppler	Combined Modes	Harmonic Imaging	Coded Pulse	Other
Ophthalmic											
Fetal / Obstetrics	P	P	N		N		N	N	N		
Abdominal ^[1]	P	P	N		N		N	N	N		
Pediatric	P	P	N		N		N	N	N		
Small Organ ^[2]	P	P	N		N		N	N	N		
Neonatal Cephalic	P	P	N		N		N	N	N		
Adult Cephalic											
Cardiac ^[3]	P	P	N		N		N	N	N		
Peripheral Vascular	P	P	N		N		N	N	N		
Musculo-skeletal Conventional	P	P	N		N		N	N	N		
Musculo-skeletal Superficial											
Other ^[4]	P	P	N		N		N	N	N		
<i>Exam Type, Means of Access</i>											
Transesophageal											
Transrectal	P	P	N		N		N	N			
Transvaginal	P	P	N		N		N	N			
Transurethral											
Intraoperative											
Intraoperative Neurological											
Intravascular											
Laparoscopic											

N = new indication; P = previously cleared by FDA; E = added under Appendix E

Notes: [1] Abdominal includes renal, GYN/Pelvic

[2] Small organ includes breast, testes, thyroid.

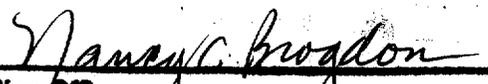
[3] Cardiac is Adult and Pediatric.

[4] Other use includes Urology/Prostate

[*] Combined modes are B/M, B/PWD, B/Color/PWD, B/Power/PWD.

(PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)


 (Division Sign-Off)
 Division of Reproductive, Abdominal,
 and Radiological Devices
 510(k) Number K014206

Prescription User (Per 21 CFR 801.109)

Diagnostic Ultrasound Indications for Use Form
GE LOGIQbook with E8C-RS Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application <i>Anatomy/ Region of Interest</i>	Mode of Operation									
	B	M	PW Doppler	CW Doppler	Color Doppler	Color M Doppler	Power Doppler	Combined Modes*	Harmonic Imaging	Coded Pulse
Ophthalmic										
Fetal / Obstetrics	P	P	N		N		N	N		
Abdominal ^[1]	P	P	N		N		N	N		
Pediatric										
Small Organ (specify)										
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Peripheral Vascular										
Musculo-skeletal Conventional										
Musculo-skeletal Superficial										
Other ^[4]	P	P	N		N		N	N		
<i>Exam Type, Means of Access</i>										
Transesophageal										
Transrectal	P	P	N		N		N	N		
Transvaginal	P	P	N		N		N	N		
Transurethral										
Intraoperative (specify)										
Intraoperative Neurological										
Intravascular										
Laparoscopic										

N = new indication; P = previously cleared by FDA; E = added under Appendix E

Notes: [1] Abdominal includes GYN/Pelvic;

[4] Other use includes Urology/Prostate;

[*] Combined modes are B/M, B/PWD, B/Color/PWD, B/Power/PWD.

(PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)



 (Division Sign-Off)
 Division of Reproductive, Abdominal,
 and Radiological Devices
 510(k) Number K014204

Prescription User (Per 21 CFR 801.109)

Diagnostic Ultrasound Indications for Use Form
GE LOGIQbook with 10LB-RS Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application <i>Anatomy/Region of Interest</i>	Mode of Operation									
	B	M	PW Doppler	CW Doppler	Color Doppler	Color M Doppler	Power Doppler	Combined Modes	Harmonic Imaging	Coded Pulse
Ophthalmic										
Fetal / Obstetrics										
Abdominal	P	P	N		N		N	N		
Pediatric	P	P	N		N		N	N		
Small Organ ^[2]	P	P	N		N		N	N		
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Peripheral Vascular	P	P	N		N		N	N		
Musculo-skeletal Conventional	P	P	N		N		N	N		
Musculo-skeletal Superficial										
Other (specify)										
<i>Exam Type, Means of Access</i>										
Transesophageal										
Transrectal										
Transvaginal										
Transurethral										
Intraoperative										
Intraoperative Neurological										
Intravascular										
Laparoscopic										

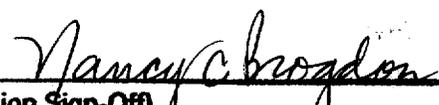
N = new indication; P = previously cleared by FDA; E = added under Appendix E

Notes: [2] Small organ includes breast, testes, thyroid.

[*] Combined modes are B/M, B/PWD, B/Color/PWD, B/Power/PWD.

(PLEASE DO NOT WRITE BELOW THIS LINE - CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)


 (Division Sign-Off)
 Division of Reproductive, Abdominal,
 and Radiological Devices
 510(k) Number K014206

Prescription User (Per 21 CFR 801.109)