

K 983673

510(K) Summary
Cell Robotic's Lasette Laser Skin Perforator

This 510(K) Summary of safety and effectiveness for the Cell Robotic's Lasette Laser Skin Perforator is submitted in accordance with the requirements of the SMDA 1990 and following guidance concerning the organization and content of a 510(K) summary.

Applicant:	Cell Robotics, Inc.
Address:	2715 Broadbent Parkway NE Albuquerque, NM 87107
Contact Person:	Connie White, Manager of Regulatory Affairs
Telephone:	(505) 343-1131 Ext. 108 (505) 344-8112
Preparation Date:	10-3-98
Device Trade Name:	Lasette
Common Name:	Laser skin perforator
Classification:	Class II
Legally Marketed Predicate Device:	Laser Lancet manufactured by TransMedica International, 323 Center Street, Suite 1100, Little Rock, AR 72201.
Description of the Cell Robotic's Lasette Laser Skin Perforator	The Cell Robotics Lasette laser skin perforator is a portable battery operated laser device. The device produces a single pulse of laser light which ablates a small hole in the patient's fingertip comparable to that produced by commonly used stainless steel blood lancets.
Intended use	Ablation of skin for collecting capillary blood samples for subsequent analysis
Indications for use	The Cell Robotic's Lasette Laser Skin Perforator is indicated for use for the perforation of skin to draw capillary blood for screening purposes
Contraindications for Use	The Lasette should not be used to collect samples for use in analyzers that require complex sample transfer procedures.
Nonclinical Performance Data:	None
Clinical Performance Data:	The subject device has been investigated in three human clinical trials under approved Investigational Device Exemptions. Results from the clinical trials indicate that the device is safe and effective.

Conclusion:

The Cell Robotic's Lasette Laser Skin Perforator is substantially equivalent to the predicate device.

Additional Information:

None requested at this time



Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

JAN 15 1999

Mr. Ronald K. Lohrding
President and Chief Executive Officer
Cell Robotics, Inc.
2715 Broadbent Parkway, Northeast
Albuquerque, New Mexico 87107

Re: K983673
Trade Name: Cell Robotics Lasette Laser Skin Perforator
Regulatory Class: II
Product Code: GEX
Dated: October 13, 1998
Received: October 20, 1998

Dear Mr. Lohrding:

We have reviewed your Section 510(k) notification of intent to market the device referenced above and we have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.


If your device is classified (see above) into either class II (Special Controls) or class III (Premarket Approval), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 895. A substantially equivalent determination assumes compliance with the current Good Manufacturing Practice requirement, as set forth in the Quality System Regulation (QS) for Medical Devices: General regulation (21 CFR Part 820) and that, through periodic (QS) inspections, the Food and Drug Administration (FDA) will verify such assumptions. Failure to comply with the GMP regulation may result in regulatory action. In addition, FDA may publish further announcements concerning your device in the Federal Register. Please note: this response to your premarket notification submission does not affect any obligation you might have under sections 531 through 542 of the Act for devices under the Electronic Product Radiation Control provisions, or other Federal laws or regulations.

Page 2 - Mr. Ronald K. Lohrding

This letter will allow you to begin marketing your device as described in your 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801 and additionally 809.10 for in vitro diagnostic devices), please contact the Office of Compliance at (301) 594-4595. Additionally, for questions on the promotion and advertising of your device, please contact the Office of Compliance at (301) 594-4639. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR 807.97). Other general information on your responsibilities under the Act may be obtained from the Division of Small Manufacturers Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its internet address "<http://www.fda.gov/cdrh/dsmamain.html>".

Sincerely yours,

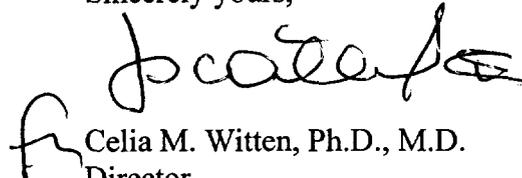A handwritten signature in black ink, appearing to read "Celia M. Witten". The signature is written in a cursive style with a large initial "C" and "W".

Celia M. Witten, Ph.D., M.D.
Director
Division of General and
Restorative Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

INDICATION FOR USE STATEMENT

510(k) Number: New Submission (K983673)

Device Name: Lasette Laser Skin Perforator

Indications for Use:


The Cell Robotic's Lasette Laser Skin Perforator is indicated for use for the perforation of skin to draw capillary blood for screening purposes.

(Please do not write below this line - Continue on another page if needed)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Prescription Use
 (per 21 CFR 801.109)

OR Over-the-Counter Use

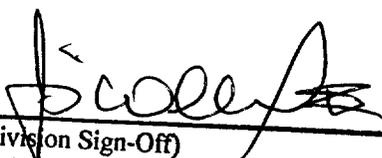

(Division Sign-Off)
Division of General Restorative Devices
510(k) Number K983673