

SEP 1 1999

510(K) SUMMARY OF SAFETY AND EFFECTIVENESS

This summary of safety and effectiveness is provided as part of this Premarket Notification in compliance with 21 CFR, Part 807, Subpart E, Section 807.92.

1) **Submitter's name, address, telephone number, contact person:**

Medison America, Inc.  
6616 Owens Drive  
Pleasanton, CA 94588  
M. T. Kwon  
President  
Telephone: 925.463.1830

Prepared June 11, 1999

2) **Name of the device, including the trade or proprietary name if applicable, the common or usual name, and the classification name, if known:**

Common/Usual Name:

Diagnostic Ultrasound System and Accessories

Proprietary Name:

SonoAce 5500 Diagnostic Ultrasound System and Transducers.

<u>Classification Names:</u>	<u>FR Number</u>	<u>Product Code</u>
Ultrasound Pulsed Echo Imaging System	892.1560	90-IYO
Diagnostic Ultrasound Transducer	892.1570	90-ITX

3) **Identification of the predicate or legally marketed device:**

Medison America, Inc. believes that SA 5500 Ultrasound System is substantially equivalent to the currently marketed SonoAce 6000C/UM 400 systems (K990970) and Voluson 530D (K974813).

4) **Device Description:**

The SA 5500 System is a general purpose, mobile, software controlled, diagnostic ultrasound

system. Its function is to acquire ultrasound data and to display the data as B-mode, M-mode, 3D or in a combination of these modes. M-mode uses the sweep display method which has its images flow from the left to the right on the monitor. The SA 5500 also gives the operator the ability to measure anatomical structures and offers analysis packages that provide information that is used to make a diagnosis by competent health care professionals. The system has real time acoustic output display with two basic indices, a mechanical index and a thermal index, which are both automatically displayed.

Eleven different models of transducers are available and any two array probes along with 1 mechanical sector probe may be connected at the same time. In addition to the initial operational settings for each transducer preprogrammed in the system, user-customized parameter settings for each transducer may be inserted by the operator and stored for recall as needed via the system control panel. Customization includes transmit focusing, filtering, image enhancement processing, dynamic window curve selection. Controls are also provided to select display format (single and various combinations), to activate zoom features, and to utilize the cine loop function. Patient contact materials are the same as those previously cleared in the predicate devices.

The SA 5500 uses digital beamforming technology, and supports a variety of Linear and Convex probes for a wide variety of applications. It is an ultrasound scanner, which provides high resolution, high penetration performance, and various measurement functions. Probes are supported in frequencies from 2.0 MHz to 10.0 MHz.

The system can be used to measure distances and calculate areas, circumferences and volumes, as well as calculate the date of delivery by using BPD (biparietal diameter), OFD (occipito-frontal diameter), HC (head circumference), AC (abdominal circumference), AD (abdominal diameter), FL (femur length), CRL (crown rump length), APTD (anteroposterior trunk diameter), TTD (transverse trunk diameter), GS (gestational sac), LMP (last menstrual period.), Cardiac Analysis and Vascular Analysis.

Biopsy guidelines are provided on screen to assist in the collection of tissue samples, using biopsy guide adapters offered as an optional accessory. The SA 5500 supports the Cine function (capable of storing up to 64 sequential images) and real-time zoom function to the region-of-interest. The system provides the ability to perform remote viewing of images, without compression. Management of patient history is possible by image-filing function. High-resolution images are provided by utilizing a technology called digital dynamic receive focusing.

The SA 5500 has been designed to meet the following electromechanical safety standards:

- < EN 60601-1 (IEC 601-1,) European Norm, Medical Electrical Equipment
- < UL 2601-1, Underwriters Laboratories Standards, Medical Electrical Equipment
- < C22.2 No. 601.1, Canadian Standards Association, Medical Electrical Equipment
- < CEI/IEC 1157:1992, International Electrotechnical Commission, Requirements for the declaration of the acoustic output of medical diagnostic ultrasonic equipment
- < EN 60601-1-2 (IEC 601-1-2,) European Norm, Collateral Standard: Electromagnetic Compatibility
- < Compliant with the European Medical Device Directive Certificate issued by TUV.

**5) Intended Use:**

SonoAce 5500 intended uses as defined FDA guidance documents are:

- < Fetal (includes infertility monitoring of follicle development)
- < Abdominal
- < Intra-operative (abdominal organs and peripheral vessel, neurological)
- < Pediatric
- < Small Organ
- < Neonatal Cephalic
- < Adult Cephalic
- < Cardiac
- < Trans-Rectal
- < Trans-Vaginal
- < Peripheral-Vascular
- < Muscular-Skeletal (conventional, superficial)

Typical examinations performed using the system are:

- < General abdominal and pelvic studies including organ surveys, assessment, and retro-peritoneal cavity studies.
- < Study of small parts including breasts, shoulders, thyroid, and the abdominal wall.
- < Pediatric scans of organs and bony structures.
- < Peripheral vascular applications including carotid arteries, legs, arms, feet, and penile artery.
- < Monitoring procedures for infertility studies (other than in vitro fertilization).
- < First, second and third trimester pregnancy studies.
- < Prostate, prostate biopsy guidance, and rectal wall studies.
- < Neonatal head studies.
- < Cardiac studies in adults and children.
- < Biopsy guidance for tissue or fluid sampling.
- < Conventional podiatry scans.
- < Intraoperative application including soft tissue structures.

**6) Technological Characteristics:**

This device operates identical to the predicate devices in that piezoelectric material in the transducer is used as an ultrasound source to transmit sound waves into the body. Sound waves are reflected back to the transducer and converted to electrical signals that are processed and displayed as a 2D and M-mode, or 3D images. Transducer patient contact materials are biocompatible.

The device's acoustic output limits are:

All Applications:	(Maximum Range)
ISPTA	720 mW/cm <sup>2</sup>
MI	1.9

The limits are the same as predicate Track 3 devices.



SEP 1 1999

Food and Drug Administration  
9200 Corporate Boulevard  
Rockville MD 20850

Medison America, Inc.  
C/o Carole Stamp  
TUV Product Service  
1775 Old Highway 8  
New Brighton, MN 55112-1891

Re: K992761  
SonoAce 5500 Diagnostic Ultrasound System  
Regulatory Class: II (two)  
Product Code: 90-IYO and 90-ITX  
Dated: August 16, 1999  
Received: August 17, 1999

Dear Ms. Stamp:

We have reviewed your Section 510(k) notification of intent to market the device referenced above and we have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

This determination of substantial equivalence applies to the following transducers intended for use with the SonoAce 5500 Diagnostic Ultrasound System, as described in your premarket notification:

Transducer Model Number

HL5-9 7.5MHz/40mm Linear Array  
L5-9/60mm 7.5MHz/60mm Linear Array  
LT5-9 Interoperative 7.5MHz/40mm Linear Array  
C2-4/3OR 3.0MHz/30R/60D Curved Linear Array  
HC2-5 3.5MHz/40R/89D Curved Linear Array  
HC3-6 3.5MHz/60R/60D Curved Linear Array  
C4-7 5.0MHz/40R/60D Curved Linear Array  
C4-9/10R 6.5MHz/10R/140D Curved Linear Array  
EC4-9/10R Endocavity 6.5MHz/10R/140D Curved Linear Array  
ERW7/10AKP Mechanical Sector Probe 7.5 10MHz  
ERW5/7KP Mechanical Sector Probe 5 & 7.5 MHz

If your device is classified (see above) into either class II (Special Controls) or class III (Premarket Approval) it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 895. A substantially equivalent determination assumes compliance with the Good

Manufacturing Practice requirement, as set forth in the Quality System Regulation (QS) for Medical Devices: General (GMP) regulation (21 CFR Part 820) and that, through periodic QS inspections, the FDA will verify such assumptions. Failure to comply with the GMP regulation may result in regulatory action. In addition, the Food and Drug Administration (FDA) may publish further announcements concerning your device in the Federal Register. Please note: this response to your premarket notification does not affect any obligation you may have under sections 531 and 542 of the Act for devices under the Electronic Product Radiation Control provisions, or other Federal laws or regulations.

Please be advised that the determination above is based on the fact that no medical devices have been demonstrated to be safe and effective for in vitro fertilization or percutaneous umbilical blood sampling, nor have any devices been marketed for these uses in interstate commerce prior to May 28, 1976, or reclassified into class I (General Controls) or class II (Special Controls). FDA considers devices specifically intended for in vitro fertilization and percutaneous umbilical blood sampling to be investigational, and subject to the provision of the investigational device exemptions (IDE) regulations, 21 CFR, Part 812. Therefore, your product labeling must be consistent with FDA's position on this use.

This letter will allow you to begin marketing your device as described in your premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus permits your device to proceed to market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801 and additionally 809.10 for in vitro diagnostic devices), please contact the Office of Compliance at (301) 594-4591. Additionally, for questions on the promotion and advertising of your device, please contact the Office of Compliance at (301) 594-4639. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR 807.97). Other general information on your responsibilities under the Act may be obtained from the Division of Small Manufacturers Assistance at its toll-free number (800) 638-2041 or at (301) 443-6597 or at its internet address "<http://www.fda.gov/cdrh/dsmamain.html>" .

If you have any questions regarding the content of this letter, please contact Robert Phillips, Ph.D. at (301) 594-1212.

Sincerely yours,

*for*



CAPT. Daniel G. Schultz, M.D.  
Acting Director, Division of Reproductive,  
Abdominal, Ear, Nose and Throat, and  
Radiological Devices  
Office of Device Evaluation  
Center for Devices and  
Radiological Health

Enclosure

### Diagnostic Ultrasound Indications for Use Form

**510(k) Number:**

**Device Name: SonoAce 5500 Ultrasound System**

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

#### Mode of Operation

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal		N	N						N	Notes 3, 5
Abdominal (See Note 1)		N	N						N	Notes 3, 5
Intra-Operative (See Note 4)		N	N						N	Note 5
Intra-Operative Neurological										
Pediatric		N	N						N	Notes 3, 5
Small Organ (See Note 2)		N	N						N	Notes 3, 5
Neonatal Cephalic		N	N						N	Note 5
Adult Cephalic										
Cardiac		N	N						N	Note 5
Transesophageal										
Trans-Rectal		N	N						N	Notes 3, 5
Trans-Vaginal		N	N						N	Notes 3, 5
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular		N	N						N	Note 5
Laparoscopic										
Muscular-Skeletal Conventional		N	N						N	Notes 3, 5
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

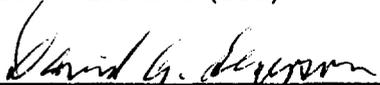
Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

  
 \_\_\_\_\_  
 (Division Sign-Off)  
 Division of Reproductive, Abdominal, ENT,  
 and Radiological Devices  
 510(k) Number     K992761    

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name:** SonoAce 5500 Ultrasound System

**Transducer:** HL5-9 (7.5MHz/40mm, Linear Array Probe)

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal										
Abdominal (See Note 1)										
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric										
Small Organ (See Note 2)		P	P						P	Notes 3, 5
Neonatal Cephalic		P	P						P	Note 5
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal										
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular		P	P						P	Note 5
Laparoscopic										
Muscular-Skeletal Conventional		P	P						P	Notes 3, 5
Muscular-Skeletal Superficial		P	P						P	Notes 3, 5
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

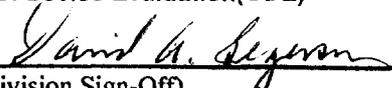
Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

  
 (Division Sign-Off)

Division of Reproductive, Abdominal, ENT, and Radiological Devices

510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name:** SonoAce 5500 Ultrasound System

**Transducer:** L5-9/60mm (7.5MHz/60mm, Linear Array Probe)

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Gombined (B-M)	Other (Specify)
Ophthalmic										
Fetal										
Abdominal (See Note 1)										
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric										
Small Organ		P	P						P	Notes 2, 3, 5
Neonatal Cephalic		P	P						P	Note 5
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal										
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular		P	P						P	Note 5
Laparoscopic										
Muscular-Skeletal Conventional										
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

*David G. Seymour*  
 (Division Sign-Off)

Division of Reproductive, Abdominal, ENT, and Radiological Devices

510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name: SonoAce 5500 Ultrasound System**

**Transducer: LT5-9 (Intraoperative 7.5MHz/40mm, Linear Array Probe)**

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal										
Abdominal (See Note 1)										
Intra-Operative (See Note 4)		P	P						P	Note 5
Intra-Operative Neurological										
Pediatric										
Small Organ (See Note 2)										
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal										
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular		P	P						P	Note 5
Laparoscopic										
Muscular-Skeletal Conventional		P	P						P	Notes 3, 5
Muscular-Skeletal Superficial		P	P						P	Notes 3, 5
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

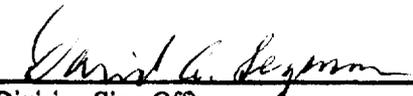
Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)



(Division Sign-Off)  
 Division of Reproductive, Abdominal, ENT,  
 and Radiological Devices

510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name:** SonoAce 5500 Ultrasound System

**Transducer:** C2-4/30R (3.0MHz/30R/60D, Curved Linear Array Probe)

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal		P	P						P	Note 5
Abdominal (See Note 1)		P	P						P	Note 5
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric										
Small Organ (See Note 2)										
Neonatal Cephalic										
Adult Cephalic										
Cardiac		P	P						P	Note 5
Transesophageal										
Trans-Rectal										
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular										
Laparoscopic										
Muscular-Skeletal Conventional										
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

*David C. [Signature]*  
(Division Sign-Off)

Division of Reproductive, Abdominal, ENT, and Radiological Devices

510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name:** SonoAce 5500 Ultrasound System

**Transducer:** HC2-5 (3.5MHz/40R/89D, Curved Linear Array Probe)

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal		P	P						P	Notes 3, 5
Abdominal (See Note 1)		P	P						P	Notes 3, 5
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric		P	P						P	Notes 3, 5
Small Organ (See Note 2)										
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal										
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular										
Laparoscopic										
Muscular-Skeletal Conventional										
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

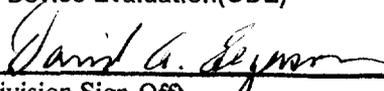
Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

  
 (Division Sign-Off)

Division of Reproductive, Abdominal, ENT, and Radiological Devices

510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name:** SonoAce 5500 Ultrasound System

**Transducer:** HC3-6 (3.5MHz/60R/60D, Curved Linear Array Probe)

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal		N	N						N	Notes 3, 5
Abdominal (See Note 1)		N	N						N	Notes 3, 5
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric		N	N						N	Notes 3, 5
Small Organ (See Note 2)										
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal										
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular										
Laparoscopic										
Muscular-Skeletal Conventional										
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

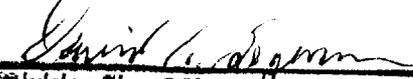
Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

  
 (Division Sign-Off)  
 Division of Reproductive, Abdominal, ENT,  
 and Radiological Devices  
 510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name:** SonoAce 5500 Ultrasound System

**Transducer:** C4-7 (5.0MHz/40R/60D, Curved Linear Array Probe)

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal		P	P						P	Notes 3, 5
Abdominal (See Note 1)		P	P						P	Notes 3, 5
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric										
Small Organ (See Note 2)		P	P						P	Notes 3, 5
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal										
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular										
Laparoscopic										
Muscular-Skeletal Conventional		P	P						P	Notes 3, 5
Muscular-Skeletal Superficial		P	P						P	Notes 3, 5
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

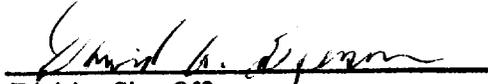
Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

  
(Division Sign-Off)

Division of Reproductive, Abdominal, ENT, and Radiological Devices

510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name:** SonoAce 5500 Ultrasound System

**Transducer:** C4-9/10R (6.5MHz/10R/140D, Curved Linear Array Probe)

**Intended Use:** Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal										
Abdominal (See Note 1)										
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric										
Small Organ (See Note 2)		P	P						P	Note 5
Neonatal Cephalic		P	P						P	Note 5
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal										
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular		P	P						P	Note 5
Laparoscopic										
Muscular-Skeletal Conventional										
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

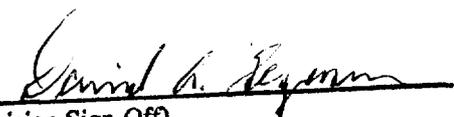
Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

  
 (Division Sign-Off)  
 Division of Reproductive, Abdominal, ENT,  
 and Radiological Devices  
 510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name: SonoAce 5500 Ultrasound System**

**Transducer: EC4-9/10R (Endocavity 6.5MHz/10R/140D, Curved Linear Array Probe)**

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

***Mode of Operation***

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal										
Abdominal (See Note 1)										
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric										
Small Organ (See Note 2)										
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal		P	P						P	Notes 3, 5
Trans-Vaginal		P	P						P	Notes 3, 5
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular										
Laparoscopic										
Muscular-Skeletal Conventional										
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared in K990970, SonoAce 6000C/ Ultramark 400C Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

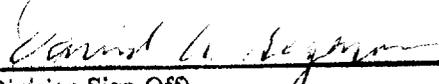
Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

  
 (Division Sign-Off)  
 Division of Reproductive, Abdominal, ENT,  
 and Radiological Devices

510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name:** SonoAce 5500 Ultrasound System

**Transducer:** ERW7/10AKP (Mechanical Sector Probe), 7.5 10 MHz

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal										
Abdominal (See Note 1)										
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric										
Small Organ (See Note 2)										
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal		P	P						P	Notes 3, 5
Trans-Vaginal										
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular										
Laparoscopic										
Muscular-Skeletal Conventional										
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared by "VRW 77 AK" in K940942, , COMBISON 530 Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

*David A. Seymour*  
 (Division Sign-Off)  
 Division of Reproductive, Abdominal, ENT,  
 and Radiological Devices  
 510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)

**Diagnostic Ultrasound Indications for Use Form**

**510(k) Number:**

**Device Name: SonoAce 5500 Ultrasound System**

**Transducer: EW5/7KP (Mechanical Sector Probe), 5 & 7.5 MHz**

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

**Mode of Operation**

Clinical Application	A	B	M	PWD	CWD	Color Doppler*	Power (Amp) Doppler	Color Velocity Imaging	Combined (B-M)	Other (Specify)
Ophthalmic										
Fetal										
Abdominal (See Note 1)										
Intra-Operative (See Note 4)										
Intra-Operative Neurological										
Pediatric										
Small Organ (See Note 2)										
Neonatal Cephalic										
Adult Cephalic										
Cardiac										
Transesophageal										
Trans-Rectal										
Trans-Vaginal		P	P						P	Notes 3, 5
Trans-Urethral										
Intra-Vascular										
Peripheral -Vascular										
Laparoscopic										
Muscular-Skeletal Conventional										
Muscular-Skeletal Superficial										
Others(Specify)										

N = new indication; P = previously cleared by "EW 5/7 K" in K940942, , COMBISON 530 Ultrasound System ; E = added under Appendix E

Other Indications or Modes

Note 1: Abdominal, Solid organs, aneurysms

Note 2: For example: thyroid, breast, testes

Notes 3, 5: Includes imaging for guidance of biopsy

Note 4: Intra-Abdominal Organs

Note 5: 3D Imaging

Concurrence of CDRH, Office of Device Evaluation(ODE)

*David A. Elyman*  
 (Division Sign-Off)  
 Division of Reproductive, Abdominal, ENT,  
 and Radiological Devices

510(k) Number K992761

Prescription Use (Per 21 CFR 801.109)