Premarket Notification Section 510(k) Submission FuStar Steerable Introducers

Exhibit #5 510(k) Summery

Exhibit #5 510(k) Summary Ref No.: A2009-018-075



FEB 1 4 2011

Exhibit #6 510(k) Summary

This 510(k) Summary is being submitted in accordance with the requirements of SMDA 1990 and 21 CFR 807.92.

The 510(k) Number is K100184

Date of Submission: 02 SEPT 2010

Sponsor: Lifetech Scientific (Shenzhen) Co., Ltd

Floor 1-3, Cybio Electronic Building, Langshan 2nd Street, North area of High-tech Park,

Nanshan District, Shenzhen, Guangdong, 518057, P. R. China

Contact Person:

Ms. Xiaoli Shi, International Regulatory / Mr. David Zhang, CTO

TEL:86-13418601356 | FAX:(86-755)86026251

Email:stepbystep0326@hotmail.com

Correspondent: Ms. Diana Hong / Mr. Lee Fu

Shanghai Mid-Link Business Consulting Co., Ltd

Suite 5D, No.19, Lane 999, Zhongshan Road (S-2), Shanghai, 200030, China

TEL: +86-21-22815850 | FAX: +86-21-22815850

Proposed Device FuStar™ Steerable Introducers

Classification: Class !!

Product Code: DQY

Classification Name: catheter, percutaneous Regulation Number: 21 CFR 870.1250

Review Panel: Cardiovascular

Predicate Device: Predicate Device 1:

510(k) Number: K072313

Trade Name: AMPLATZER Torqvue Delivery System

Manufacturer: AGA Medical Corporation

Predicate Device 2: 510(k) Number: K061119

Trade Name: Enpath Medical Steerable Sheath

Manufacturer: Enpath Medical Inc

Premarket Notification Section 510(k) Submission FuStar Steerable Introducers

Exhibit #5 510(k) Summary Ref No.: A2009-018-075



Intended Use:

FuStarTM Steerable Introducer is indicated to provide a pathway through which devices are introduced within the chambers and coronary vasculature of the heart or in the peripheral vasculature.

Device Description:

The FuStarTM Steerable Introducers are designed to perform as a guiding sheath and/or introducer sheath, which provides a pathway through which devices are introduced within the chambers and coronary vasculature of the heart or in the peripheral vasculature. Its low-profile design and atraumatic tip reduce the potential risk for trauma during percutaneous punctures while facilitating passage of a wide variety of catheters and devices.

The proposed introducer can give physicians good control during operation, the curl options are available at 30mm (Small) and 50mm (Large). In each option, there are 2 connector types which are luer lock connector and silicone valve connector, and several specifications combination. Also, the visible knob with scale of controller facilitates the physicians to estimate the deflecting angle of the sheath while intra-operation.

The products are available in different effective length (550mm, 700mm and 900mm) and diameter (French size, 5F, 6F, 7F, 8F, 9F and 10F). The deflectable angle of sheath is from 0 degree to 180 degree.

Testing Conclusion:

Performance testing was conducted to validate and verify that the proposed device met all design specifications including:

Performance Test: Primary Dimension Test; Exterior Surface Condition X-Ray Visible Testing; Bending Test, Cycle (Fatigue) Test: Conical Fitting Test: Hub Test; Connection Strength Test: Corrosion Resistance test, Leakage and Aspiration Test.

Biocompatibility Test: Cytotoxicity, Acute Systemic Toxicity, Intracutaneous Reactivity Test, Hypersensitivity Test, Hemolysis Test, Complement System Test. Thrombosis Test **Sterilization**: Sterilization Validation, Package Integrity Test, Pyrogen Test and Endotoxicity Test.

SE Conclusion:

FuStarTM Steerable Introducers, can be claimed to be Substantially Equivalent (SE) to the predicate device, Predicate Device AMPLATZER Torqvue Delivery System K072313 and Enpath Medical Steerable Sheath, K061119.







Food and Drug Administration 10903 New Hampshire Avenue Document Control Room –WO66-G609 Silver Spring, MD 20993-0002

Lifetech Scientific (Shenzhen) Co., Ltd C/O Ms. Diana Hong Shanghai Mid-Link Business Consulting Co., Ltd Suite 8D, No.19, Lane 999, Zhongshan Road (S-2), Shanghai, 200030, China

FEB 1 4 2011

Re: K100184

Trade/Device Name: FuStarTM Steerable Introducers

Regulation Number: 21 CFR 870.1250 Regulation Name: Percutaneous Catheter

Regulatory Class: Class II Product Code: DQY Dated: January 18, 2011 Received: January 19, 2011

Dear Ms. Hong:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the <u>Federal Register</u>.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21)

Page 2 – Ms. Diana Hong

CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please go to http://www.fda.gov/AboutFDA/CentersOffices/CDRH/CDRHOffices/ucm115809.htm for the Center for Devices and Radiological Health's (CDRH's) Office of Compliance. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm.

Sincerely yours,

Bram D. Zuckerman, M.D.

Director

Division of Cardiovascular Devices Office of Device Evaluation Center for Devices and Radiological Health

Duma R. Vi Anel

Enclosure

Premarket Notification Section 510(k) Submission FuStar Steerable Introducers Section II Indication for Use Statement Ref No.: A2009-018-075



Section II Indication for Use Statement	!	
510(k) Number: CIDSL84 Device Name: FuStar™ Steerable Introduce	èrs	
Indications for Use:		
FuStar™ Steerable Introducer is indica chambers and coronary vasculature of th	ited to provide a pathway through he heart or in the peripheral vascul	n which devices are introduced within the lature.
Prescription UseX (Part 21 CFR 801 Subpart D)	AND/OR	Over-The-Counter Use (21 CFR 801 Subpart C)
(PLEASE DO NOT WRITE	BELOW THIS LINE-CONTINUE ON AN	IOTHER PAGE OF NEEDED)
Concurre	ence of CDRH, Office of Device Evaluat	tion (ODE)
		Page 1 of 1
Division Sign-Of Division of Cardio	r) ovascular Devices	

510(k) Number <u>K10018A</u>