

K103725

800 Turnpike Road Shrewsbury, MA 01545

Phone 508.845.1177 Fax 508.842.8928 www.valeritas.com

FEB 2 3 2011

510(k) Summary

Owner's Name:

Valeritas, Inc.

750 Route 202 South, Suite 100

Bridgewater, NJ 08807

Company Contact:

Scott Huie

Vice President, Regulatory Affairs and Product Compliance

Valeritas, Inc.

800 Boston Turnpike Shrewsbury, MA 01545 (508) 845-1177 X222 shuie@valeritas.com

Date Summary Prepared:

9 February 2011

Trade Name:

Valeritas V-Go™ Insulin Delivery Device

Common Name:

Disposable Insulin Delivery Device

Classification Name:

Pump, Infusion, Insulin/Set, Administration, Intravascular

Classification regulation:

Class II: 21 CFR 880.5725

Panel: 80, LZG External Insulin Infusion Pump

FPA Intravascular Administration Set

Substantial Equivalence:

The Valeritas V-Go Disposable Insulin Delivery Device is substantially equivalent to the Valeritas V-Go Disposable Insulin

Delivery Device K100504, cleared by this Center on December 1,

2010

Device Description:

The Valeritas V-Go Disposable Insulin Delivery Device is a mechanical (no electronics), self-contained, sterile, patient fillable,

single-use disposable insulin infusion device with an integrated

stainless steel subcutaneous needle.

It is designed for the subcutaneous infusion of insulin for the management of diabetes mellitus in persons requiring insulin. After filling the V-Go with insulin using the EZ Fill, the device is secured to the patient's skin over the infusion site with an adhesive-backed foam pad, which is attached to the back of the pump. Once activated, the V-Go delivers a continuous infusion of insulin at a

fixed rate. The device also allows the user to initiate bolus injections to supplement their daily basal insulin requirements. Three device models (20, 30 and 40 Units/day) will be available to address the different basal and bolus requirements of each patient. A window in the top of the pump allows the user to see into the reservoir to check the drug and to monitor the progress of the infusion.

Intended Use:

V-Go 20:

The V-Go Disposable Insulin Delivery Device is indicated for continuous subcutaneous infusion of 20 Units of insulin in one 24-hour time period (0.83U/hr) and on-demand bolus dosing in 2-Unit increments (up to 36 Units per one 24-hour time period) in adult patients requiring insulin

V-Go 30:

The V-Go Disposable Insulin Delivery Device is indicated for continuous subcutaneous infusion of 30 Units of insulin in one 24-hour time period (1.25U/hr) and on-demand bolus dosing in 2-Unit increments (up to 36 Units per one 24-hour time period) in adult patients requiring insulin

V-Go 40:

The V-Go Disposable Insulin Delivery Device is indicated for continuous subcutaneous infusion of 40 Units of insulin in one 24-hour time period (1.67U/hr) and on-demand bolus dosing in 2-Unit increments (up to 36 Units per one 24-hour time period) in adult patients requiring insulin

Technological Characteristics:

The V-Go has two delivery mechanisms, one for basal infusion and one for bolus delivery. Both operate by providing force against a plunger, which is inserted into a cylindrical reservoir filled with high viscosity fluid. This force is directly translated against the rear of the delivery piston at the rear of the insulin reservoir.

The Basal Hydraulic Pump operates by applying a constant force by means of a compressed spring. The spring pushes against a disk inserted internally at the rear of the cylindrical fluid filled basal reservoir, forcing the fluid through a restrictive orifice. This displacement translates into displacement of the medicinal delivery piston, providing a continuous basal infusion. The basal delivery rate can be varied by the substitution of a different orifice into the device during assembly.

The Bolus Hydraulic Pump operates by applying a force by means of a manually depressed actuator (Bolus Button). Manual actuation allows the displacement of a disk inserted internally at the rear of the cylindrical fluid-filled bolus reservoir in pre-set increments that will displace the volume of fluid and the medicinal delivery piston. The manual actuation requires two steps. First, the user must press the Bolus Release Button. Displacement of the Bolus Release allows the Bolus Button to "pop-out" so that it protrudes slightly from the end of the pump. The user then depresses the Bolus Button, which displaces the Bolus Piston and delivers a pre-set bolus increment. This process is repeated until the desired volume has been delivered or the labeled number of actuations are used up. The Bolus Release Button extends from the pump and locks preventing further use as there are no additional bolus actuations remaining.

Substantial Equivalence:

The Valeritas V-Go Insulin Delivery Device is substantially equivalent to the predicate device, the Valeritas V-Go Insulin Delivery Device cleared by this center on 1 December 2010 (K100504). The Valeritas V-Go device has the same intended use and performance characteristics. The primary difference qualifies the use of NovoLog in the device in addition to Humalog.

Non-Clinical Performance Data:

Physical Specifications

Maximum Dimensions: 84 mm x 47 mm x 13 mm

Weight: 20 - 50 grams

Cannula: Stainless Steel' J' shaped needle (J-needle)
Pump Mechanism: Spring and Manual Hydraulic Displacement

Interfaces: Mechanical Buttons

Operational Specifications

Operational Specifications				
Pump Model	V-Go 20	V-Go 30	V-Go 40	
Reservoir volume	560 μL	660 μL	760 μL	
	(56 IU of insulin)	(66 IU of insulin)	(76 IU of insulin)	
Basal Rate	8.3 μL/hr	12.5 μL/hr	16.7 μL/hr	
	(0.6 IU/hr)	(1.2 IU/hr)	(1.8 IU/hr)	
Bolus Increments	20 μL (2 IU)	20 μL (2 IU)	20 μL (2 IU)	
	18 Actuations	18 Actuations	18 Actuations	
Nominal Bolus	360 μL (36 IU)	360 μL (36 IU)	360 μL (36 IU)	
Volume				
Basal Volume	200 μL (20 IU)	300 μL (30 IU)	400 μL (40 IU)	
(24 Hours)				
Accuracy	+/- 10%	+/- 10%	+/- 10%	

Environmental Specifications - Per AAMI ID26, IPX8.

Temperature: +5 to +37°C Humidity: 20 - 90%

Water Ingress Protection: IPX8 (submerged at 1 meter for 24 hours)

Atmospheric Testing: 0-10,000 feet

Biocompatibility – Per ISO 10993-1

Drug and skin contact materials passed all testing per ISO 10993-1. Drug Contact materials were tested and found to be compatible with insulin for the duration of use. Please see the package insert for specific insulin information.

Sterilization and Package Stability

The V-Go and EZ Fill are sterilized using Ethylene Oxide to a Sterility Assurance Level (SAL) of 10⁻⁶ per ANSI/AAMI ST67:2003(R)2008, Sterilization of Health Care Products - Requirements for products labeled "Sterile".

The V-Go and EZ Fill have been tested as pyrogen free using USP <85> Bacterial Endotoxin Testing Gel Clot Method with an acceptable maximum limit of 20EU/device.

Packaging consists of a thermoformed plastic tray with a Tyvek heat sealable lidstock. Packaging validation demonstrated that the packaging can maintain sterility integrity for up to 36 months per AAMI/ISO 11607-1:2006, Packaging for terminally sterilized medical devices - Part 1: Requirements for materials, sterile barrier systems, and packaging systems.

Drug Stability

Humalog has been tested in the V-Go and has been demonstrated to be stable for up to 24 hours storage refrigerated or at room temperature followed by 24 hours wear. NovoLog has been demonstrated to be stable for up to 5 days at refrigerated storage or 3 days at room temperature storage followed by 24 hours wear. The EZ Fill has been demonstrated to be acceptable for filling Humalog and NovoLogfor up to 30 days use.

Clinical Performance Data:

No clinical performance data is required to validate the intended uses and user needs of the system. Design validation is completed by human factors simulated use and clinical evaluation testing.

Conclusion:

Results of performance testing demonstrated substantial equivalence of the V-Go Disposable Insulin Delivery System to the predicate 510(k) device.

DEPARTMENT OF HEALTH & HUMAN SERVICES



Food and Drug Administration 10903 New Hampshire Avenue Document Control Room –WO66-G609 Silver Spring, MD 20993-0002

Mr. Scott Huie Vice President, Regulatory Affairs and Product Compliance Valeritas, Incorporated 800 Boston Turnpike Shrewbury, Massachusetts 01545

FEB 2 3 2011

Re: K103825

Trade/Device Name: V-GoTM 20 Disposable Insulin Delivery Device, V-GoTM 40 Disposable Insulin Delivery Device & V-GoTM 30 Disposable Insulin Delivery

Device

Regulation Number: 21 CFR 880.5725 Regulation Name: Infusion Pump

Regulatory Class: II Product Code: LZG, FPA Dated: December 29, 2010 Received: December 30, 2010

Dear Mr. Huie:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the <u>Federal</u> Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please go to

http://www.fda.gov/AboutFDA/CentersOffices/CDRH/CDRHOffices/ucm115809.htm for the Center for Devices and Radiological Health's (CDRH's) Office of Compliance. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm.

Sincerely yours.

Anthony D. Watson, B.S., M.S., M.B.A.

Director

Division of Anesthesiology, General Hospital, Infection Control and Dental Devices

Office of Device Evaluation

Center for Devices and

Radiological Health

Enclosure

7. Indications for Use Statement

510(k) Number (if known):

Device Name:	V-Go TM 20 Disposable Insulin Delivery Device		
Indications For Use:			
	The V-Go 20 Disposable Insulin Delivery Device is indicated for continuous subcutaneous infusion of 20 Units of insulin in one 24-hour time period (0.83U/hr) and on-demand bolus dosing in 2-Unit increments (up to 36 Units per one 24-hour time period) in adult patients requiring insulin		
		1	
Prescription Use X (Part 21 CFR 801 Subpart D	AND/OR)	Over-The-Counter Use(21 CFR 801 Subpart C)	
(PLEASE DO NOT WRITE NEEDED)	BELOW THIS LIN	E-CONTINUE ON ANOTHER PAGE IF	
Concurre	nce of CDRH, Office	of Device Evaluation (ODE)	
	(I) (I)	Division Sign-Off) Division of Anesthesiology, General Hospital Infection Control, Dental Devices 1 of3	
		1/1 / 202 (
	ļ	510(k) Number: <u>K103825</u>	

Indications for Use Statement

510(k) Number (if known):	
Device Name:	V-Go TM 30 Disposable Insulin Delivery Device
Indications For Use:	
· .	The V-Go 30 Disposable Insulin Delivery Device is indicated for continuous subcutaneous infusion of 30 Units of insulin in one 24-hour time period (1.25U/hr) and on-demand bolus dosing in 2-Unit increments (up to 36 Units per one 24-hour time period) in adult patients requiring insulin
Prescription Use X (Part 21 CFR 801 Subpart D)	AND/OR Over-The-Counter Use (21 CFR 801 Subpart C)
(PLEASE DO NOT WRITE NEEDED)	BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF
Concurrer	nce of CDRH, Office of Device Evaluation (ODE)
Division	of Anesthesiology, General Hospital Control, Dental Devices Page 2 of3
510(k) I	Number: K103825

Indications for Use Statement

510(k) Number (if known):	
Device Name:	V-Go TM 40 Disposable Insulin Delivery Device
Indications For Use:	
	The V-Go 40 Disposable Insulin Delivery Device is indicated for continuous subcutaneous infusion of 40 Units of insulin in one 24-hour time period (1.67U/hr) and on-demand bolus dosing in 2-Unit increments (up to 36 Units per one 24-hour time period) in adult patients requiring insulin
Prescription Use X (Part 21 CFR 801 Subpart D)	AND/OR Over-The-Counter Use (21 CFR 801 Subpart C)
(PLEASE DO NOT WRITE NEEDED)	BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF
Concurren	ce of CDRH, Office of Device Evaluation (ODE)
Infection Contro	sthesiology, General Hospital II, Dental Devices Page 3 of3
510(k) Number	: K103825