

510(k) Summary

Submitted By: MeVis Medical Solutions AG
Universitaetsallee 29
28359 Bremen, Germany

Establishment Name: MeVis Medical Solutions AG

Establishment Registration Number: Applied for and awaiting assignment by FDA

Contact Person: Thomas E. Tynes
President & CEO
MeVis Medical Solutions, Inc.
N27 W24075 Paul Court - Suite 100
Pewaukee, WI 53072
USA
Phone: +1-262-691-9530
Fax: +1-262-691-9531

Date Prepared: 12/14/2011

Trade Name: Visia™ Neuro

Common Name: Medical Image Processing Software

Classification Name: Image Processing System

Classification Regulation Number: 892.2050

Class: II

Panel: Radiology

Product Code: LLZ

Device Description

Visia™ Neuro is a medical image processing software application intended for the visualization of images from various sources such as Magnetic Resonance Imaging systems or from image archives. The system provides viewing, quantification, manipulation, communication, and printing of medical images.

Visia™ Neuro integrates within typical clinical workflow patterns through receiving and transferring medical images over a computer network. The software can be loaded on a standard off-the-shelf personal computer (PC) and can operate as a stand-alone workstation or in a distributed server-client configuration across a computer network.

The software provides functionality for processing and analyzing both anatomical and physiologic/functional imaging datasets. Specifically, the software includes user defined processing modules for image registration, blood oxygen level dependent (BOLD) functional magnetic resonance imaging (fMRI), diffusion imaging, fiber tracking, dynamic imaging, and vessel imaging. Processed images are stored as separate files from the original data such that the original data is preserved.

Images may be displayed based on physician preferences using configurable layouts, or hangings. Visia™ Neuro provides the clinician with a broad set of viewing and analysis tools in both 2D and 3D. The software includes tools to annotate, measure, and output selected image views or user defined reports.

Intended Use

Visia™ Neuro is a medical image processing software application intended for the visualization of images from various sources such as Magnetic Resonance Imaging systems or from image archives. The system provides viewing, quantification, manipulation, communication, and printing of medical images. Visia™ Neuro provides both analysis and viewing capabilities for anatomical and physiologic/functional imaging datasets, including blood oxygen dependent (BOLD) fMRI, diffusion, fiber tracking, dynamic review, and vessel visualization. Data can be visualized in both 2D and 3D views.

BOLD fMRI Review

The BOLD fMRI feature is useful in identifying small susceptibility changes arising from neuronal activity during performance of a specific task.

Diffusion Review

The diffusion review feature is intended for visualization and analysis of the diffusion of water molecules through brain tissue.

Fiber Tracking Review

The fiber tracking feature uses the directional portion of the diffusion vector to track and visualize white matter structures within the brain.

Dynamic Review

Dynamic review feature is intended for visualization and analysis of MRI dynamic studies, showing changes in contrast over time, where such techniques are useful or necessary.

Vessel Visualization

The vessel feature is used to identify and visualize the vascular structures of the brain.

3D Visualization

The 3D visualization feature allows image data to be reconstructed as 3D objects that are visualized and manipulated on a 2D screen.

Predicate Device Information

Product	Predicate Device Name	Predicate 510(k) Submission Reference
Visia™ Neuro	DC Neuro	K081262

Safety and Effectiveness

The Visia™ Neuro labeling contains instructions for use and necessary cautions, warnings and notes to provide for safe and effective use of the device. Risk Management is ensured via MeVis Medical Solution AG's Risk Management procedure, which is used to identify potential hazards. These potential hazards are controlled via software development and verification & validation testing.

Nonclinical Testing and Performance Information

The complete system configuration has been assessed and tested at the manufacturer's facility and has passed all in-house testing criteria. The Validation Test Plan was designed to evaluate all input functions, output functions, and actions performed by the software in each operational mode and followed the process documented in the Validation Test Plan.

Nonclinical testing results are provided in the 510(k). Validation testing indicated that as required by the risk analysis, designated individuals performed all verification and validation activities and that the results demonstrated that the predetermined acceptance criteria were met.

Technological Characteristics

Visia™ Neuro is a software device that does not contact the patient, nor does it control any life sustaining devices. Diagnosis is not performed by the software but by Radiologists, Clinicians and referring Physicians.

A physician, providing ample opportunity for competent human intervention interprets images and information being displayed and printed.

The new device and predicate devices are substantially equivalent in the areas of technical characteristics, general function, application, and intended use. The new device does not raise any new potential safety risks and is equivalent in performance to the existing legally marketed devices.

Conclusion

The 510(k) Pre-Market Notification for Visia™ Neuro contains adequate information, data, and nonclinical test results to enable FDA – CDRH to determine substantial equivalence to the predicate device. MeVis Medical Solutions has determined that its device, Visia™ Neuro, is substantially equivalent to the identified predicate device listed above. A comparison with the legally marketed predicate device indicates that it is substantially equivalent to this device, and that it does not raise any new safety or efficacy concerns. Nonclinical tests demonstrate that the device is safe, effective, and is substantially equivalent to the predicate device.



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Room – WO66-G609
Silver Spring, MD 20993-0002

MeVis Medical Solutions AG
% Mr. Thomas E. Tynes
President and CEO
MeVis Medical Solutions, Inc.
N27 W24075 Paul Court, Suite 100
PEWAUKEE WI 28359

FEB 16 2012

Re: K113701
Trade/Device Name: Visia™ Neuro
Regulation Number: 21 CFR 892.2050
Regulation Name: Picture archiving and communications system
Regulatory Class: II
Product Code: LLZ
Dated: December 14, 2011
Received: December 16, 2011

Dear Mr. Tynes:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into class II (Special Controls), it may be subject to such additional controls. Existing major regulations affecting your device can be found in Title 21, Code of Federal Regulations (CFR), Parts 800 to 895. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Parts 801 and 809); medical device reporting (reporting of

medical device-related adverse events) (21 CFR 803); and good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820). This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Parts 801 and 809), please contact the Office of *In Vitro* Diagnostic Device Evaluation and Safety at (301) 796-5450. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely Yours,

A handwritten signature in black ink that reads "Mary S. Pastel". The signature is fluid and cursive, with the first letters of each name being capitalized and prominent.

Mary S. Pastel, Sc.D.
Director
Division of Radiological Devices
Office of In Vitro Diagnostic Device
Evaluation and Safety
Center for Devices and Radiological Health

Enclosure

Indications for Use Statement

510(k) Number (if known): K113701

Device Name: Visia™ Neuro

Indications for Use:

Visia™ Neuro is a medical image processing software application intended for the visualization of images from various sources such as Magnetic Resonance Imaging systems or from image archives. The system provides viewing, quantification, manipulation, communication, and printing of medical images. Visia™ Neuro provides both analysis and viewing capabilities for anatomical and physiologic/functional imaging datasets, including blood oxygen dependent (BOLD) fMRI, diffusion, fiber tracking, dynamic review, and vessel visualization. Data can be visualized in both 2D and 3D views.

BOLD fMRI Review

The BOLD fMRI feature is useful in identifying small susceptibility changes arising from neuronal activity during performance of a specific task.

Diffusion Review

The diffusion review feature is intended for visualization and analysis of the diffusion of water molecules through brain tissue.

Fiber Tracking Review

The fiber tracking feature uses the directional portion of the diffusion vector to track and visualize white matter structures within the brain.

Dynamic Review

Dynamic review feature is intended for visualization and analysis of MRI dynamic studies, showing changes in contrast over time, where such techniques are useful or necessary.

Vessel Visualization

The vessel feature is used to identify and visualize the vascular structures of the brain.

3D Visualization

The 3D visualization feature allows image data to be reconstructed as 3D objects that are visualized and manipulated on a 2D screen.

Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use _____
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of In Vitro Diagnostic Devices (OIVD)

Mary Speth
Division Sign-Off
Office of In Vitro Diagnostic Device
Evaluation and Safety

510(k) K113701