



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Center - WO66-G609
Silver Spring, MD 20993-0002

October 28, 2015

Pasture Pharma Pte, Ltd
C/O Ms. Sarah Hassan
US MedPharm
38129 Spring Canyon Drive
Murrieta, CA 92563

Re: K141876
Trade/Device Name: Pasture F550S Surgical N95 Respirator, Pasture F550CS Surgical N95 Respirator, Pasture A520S Surgical N95 Respirator, Pasture A520CS Surgical N95 Respirator, Pasture E520S Surgical N95 Respirator, Pasture E520CS Surgical N95 Respirator
Regulation Number: 21 CFR 878.4040
Regulation Name: Surgical Apparel
Regulatory Class: II
Product Code: MSH
Dated: October 15, 2015
Received: October 16, 2015

Dear Ms. Hassan,

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

Tejashri Purohit-Sheth, M.D.

Tejashri Purohit-Sheth, M.D.
Clinical Deputy Director
DAGRID/ODE/CDRH FOR

Erin Keith, M.S.
Director
Division of Anesthesiology,
General Hospital, Respiratory, Infection
Control, and Dental Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)

K141876

Device Name

Pasture F550S Surgical N95 Respirator, Pasture F550CS Surgical N95 Respirator, Pasture A520S Surgical N95 Respirator, Pasture A520CS Surgical N95 Respirator, Pasture E520S Surgical N95 Respirator, Pasture E520CS Surgical N95 Respirator

Indications for Use (Describe)

Pasture F550S, Pasture F550CS, Pasture A520S, Pasture A520CS, Pasture E520S, and Pasture E520CS is a NIOSH certified N95 respirator, and is indicated to be used by the healthcare personnel during procedures to protect both the patient and the healthcare personnel from the transfer of microorganisms, body fluids, and particulate material.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(K) SUMMARY

This 510(K) Summary is being submitted in accordance with requirements of 21 CFR 807.92(c)

The assigned 510(k) number is: K141876

Submitter Name and Address:

Pasture Pharma Pte, Ltd
8 Boon Lay Way #04-01
Trade Hub 21, Singapore 609964
Contact: Lloyd Soong, President & CEO
+65-6515-6561
lloyd@pasturegroup.com

US agent and correspondent:

Mrs. Sarah Hassan
US Med Pharm Supplies, Inc
38129 Spring Canyon Drive
Murrieta, CA 92563
Phone: 951-239-1933
Fax: 951-239-1933
E-mail: sarah@usmedpharm.com

Date Prepared: October 27, 2015

Proprietary Name:

Pasture F550S Surgical N95 Respirator, Pasture F550CS Surgical N95 Respirator, Pasture A520S Surgical N95 Respirator, Pasture A520CS Surgical N95 Respirator, Pasture E520S Surgical N95 Respirator, Pasture E520CS Surgical N95 Respirator

Common name: Surgical N95 NIOSH certified Respirator

Device Classification and Product Code

Classification Name: Surgical Apparel

Regulation: 21 CFR §878.4040

Class: Class II

Classification panel: General and Plastic Surgery

Product Code: MSH

Predicate Device: K070139

WILLSON ONE-FIT HEALTHCARE PARTICLE RESPIRATOR AND SURGICAL MASK, HC-NB095
SURVIVAIR, INC

Intended Use:

Pasture F550S, Pasture F550CS, Pasture A520S, Pasture A520CS, Pasture E520S, and Pasture E520CS is a NIOSH certified N95 respirator, and is indicated to be used by the healthcare personnel during procedures to protect both the patient and the healthcare personnel from the transfer of microorganisms, body fluids, and particulate material.

Device Description:

Pasture F550S and Pasture F550CS respirators are disposable duck bill shaped N95 respirators with NIOSH certification number TC-84A-7504. They contain 5 layers composed of polypropylene, with a nose cushion, a synthetic elastic loop or strap, and a nosepiece which is the combination of zinc wires and embedded polyester inside of layers. F550CS is the same respirator as the F550S, but F550CS contains an adjustable buckle on the headband.

Pasture A520S and Pasture A520CS respirators are cup shaped N95 respirators with NIOSH certification number TC-84A-7454. They contain 4 layers composed of polypropylene with a synthetic elastic loop or a strap. A520CS is the same respirator as the A520S, but A520CS contains an adjustable buckle on the headband.

Pasture E520S and Pasture E520CS respirators are cup shaped N95 respirators with NIOSH certification number TC-84A-7453. They contain 4 layers composed of polypropylene with a

nose cushion, synthetic elastic loop or strap, and an aluminum nosepiece. E520CS is the same respirator as E520S, but E520CS contains an adjustable buckle on the headband.

| Description | Predicate K070139 | Pasture F550S | Pasture F550CS | Pasture A520S | Pasture A520CS | Pasture E520S | Pasture E520CS |
|----------------------------|---|--|--|---|---|---|---|
| NIOSH certification number | N95 TC-84A-4516 | N95 TC-84A-7504 | N95 TC-84A-7504 | N95 TC-84A-7454 | N95 TC-84A-7454 | N95 TC-84A-7453 | N95 TC-84A-7453 |
| Outer layer | Polyester | Polypropylene Spunbond | Polypropylene Spunbond | Polypropylene Spunbond | Polypropylene Spunbond | Polypropylene Spunbond | Polypropylene Spunbond |
| Filter Media | Polypropylene | Polypropylene Meltblown | Polypropylene Meltblown | Polypropylene Meltblown | Polypropylene Meltblown | Polypropylene Meltblown | Polypropylene Meltblown |
| Inner layer | Polypropylene | Polypropylene Spunbond | Polypropylene Spunbond | Polypropylene Spunbond | Polypropylene Spunbond | Polypropylene Spunbond | Polypropylene Spunbond |
| Nose Piece | Not Applicable | combination of zinc wires and embedded polyester | combination of zinc wires and embedded polyester | Not Applicable | Not Applicable | aluminum | aluminum |
| Ear Attachment | synthetic elastic | synthetic elastic | synthetic elastic | synthetic elastic | synthetic elastic | synthetic elastic | synthetic elastic |
| Respirator style | cup shaped | duck bill shaped | duck bill shaped | cup shaped | cup shaped | cup shaped | cup shaped |
| Design features | 3 layers molded-cone with filter web in the middle | 5 layers of non-woven fiber containing filter web in the middle | 5 layers of non-woven fiber containing filter web in the middle | 4 layers molded-cone with filter web in the middle |
| Adjustable buckle | Not Applicable | | Acrylonitrile Butadiene Styrene | | Acrylonitrile Butadiene Styrene | | Acrylonitrile Butadiene Styrene |

| Test | K070139 | F550S F550CS | A520S A520CS | E520S E520CS |
|-------------------------------|---------------------------|---|--|--|
| NIOSH Certification | N95 TC-84A-4516 | N95 TC-84A-7504 | N95 TC-84A-7454 | N95 TC-84A-7453 |
| Fluid Resistance (ASTM F1862) | Not available | Pass 120mmHg | Pass 120mmHg | Pass 160mmHg |
| Flammability (16 CFR 1610) | Not available | Class 1 | Class 1 | Class 1 |
| <i>In vitro cytotoxicity</i> | Not available | Non-cytotoxic under conditions of study | Non-cytotoxic under conditions of study | Non-cytotoxic under conditions of study |
| | Not available | Non-sensitizer under conditions of study | Non-sensitizer under conditions of study | Non-sensitizer under conditions of study |
| | Not available | Non-irritant under conditions of study | Non-irritant under conditions of study | Non-irritant under conditions of study |
| <i>Sensitization</i> | | | | |
| <i>Irritation</i> | | | | |
| Size | | 90±3*200±3*95±3mm | 150±3*130±3*60±3mm | 140±3*125±3*50±3mm |
| Sterility | Non-sterile Single use | Non-sterile Single use | Non-sterile Single use | Non-sterile Single use |
| Indication for Use | Please see below *1 | Pasture F550S, Pasture F550CS, Pasture A520S, Pasture A520CS, Pasture E520S, and Pasture E520CS is a NIOSH certified N95 respirator, and is indicated to be used by the healthcare personnel during procedures to protect both the patient and the healthcare personnel from the transfer of microorganisms, body fluids, and particulate material. | | |

*1 The Willson® ONE-Fit™ HC-NB095 Healthcare Particulate Respirator and Surgical Mask is a NIOSH-approved N95 single use respirator intended for use by healthcare personnel during medical/ surgical procedures to protect both the wearer against the spatter of blood and other potentially infectious materials and reducing the transfer of microorganisms and other airborne particulate matter.

Comparison to Predicated Device:

Pasture F550S, Pasture F550CS, Pasture A520S, Pasture A520CS, Pasture E520S and Pasture E520CS respirators are substantially equivalent to WILLSON ONE-FIT HEALTHCARE PARTICLE RESPIRATOR AND SURGICAL MASK, HC-NB095.

Performance Testing

| <u>Test Performed</u> | <u>Result</u> |
|-----------------------------------|---|
| Fluid Resistance | The Pasture F550S, Pasture F550CS, Pasture A520S, Pasture A520CS, Pasture E520S, and Pasture E520CS met the requirements of ASTM F1862. |
| Flammability | The Pasture F550S, Pasture F550CS, Pasture A520S, Pasture A520CS, Pasture E520S, and Pasture E520CS met the requirements of 16 CFR 1610. |
| Particulate Filtration Efficiency | NIOSH Certification: TC-84A-7504 (F550S and F550CS) NIOSH Certification: TC-84A-7454 (A520S and A520CS) NIOSH Certification: TC-84A-7453 (E520S and E520CS) |
| Bacterial Filtration Efficiency | NIOSH Certification: TC-84A-7504 (F550S and F550CS) NIOSH Certification: TC-84A-7454 (A520S and A520CS) NIOSH Certification: TC-84A-7453 (E520S and E520CS) |
| Differential Pressure | NIOSH Certification: TC-84A-7504 (F550S and F550CS) NIOSH Certification: TC-84A-7454 (A520S and A520CS) NIOSH Certification: TC-84A-7453 (E520S and E520CS) |

