



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Center – WO66-G609  
Silver Spring, MD 20993-0002

January 15, 2015

Boston Scientific Corporation  
Nikki M. Wahlberg  
Regulatory Affairs Specialist  
One Scimed Place  
Maple Grove, MN 55311-1566

Re: K143587

Trade/Device Name: ChoICE, Mailman, Luge, ChoICE PT, ChoICE PT Graphix Guide  
Wires  
ChoICE Magnet, Mailman Magnet, Luge Magnet, ChoICE PT Magnet,  
PT Graphix Magnet Guide Wires

Regulation Number: 21 CFR 870.1330

Regulation Name: Catheter guide wire

Regulatory Class: Class II

Dated: December 17, 2014

Received: December 18, 2014

Dear Ms. Wahlberg:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA).

You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must

comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

<http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

**Melissa A. Torres -S**

For Bram Zuckerman, M.D.  
Director  
Division of Cardiovascular Devices  
Office of Device Evaluation  
Center for Devices and Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)  
K143587

Device Name

ChoICE™, Mailman™, Luge™, ChoICE™ PT, and PT Graphix™ Guide Wires  
ChoICE™ Magnet, Mailman™ Magnet, Luge™ Magnet, ChoICE™ PT Magnet, and PT Graphix™ Magnet Guide Wires

Indications for Use (Describe)

Boston Scientific ChoICE™ PT, PT Graphix™, ChoICE, Luge, and Mailman Guidewires with ICE Hydrophilic Coating are intended to facilitate the placement of balloon dilatation catheters or other therapeutic devices during PTCA or other intravascular interventional procedures. They are not intended for use in the cerebral vasculature.

Boston Scientific ChoICE™ PT Magnet, PT Graphix™ Magnet, ChoICE Magnet, Luge Magnet, and Mailman Magnet Guidewires with ICE Hydrophilic Coating are intended to facilitate the placement of balloon dilatation catheters or other therapeutic devices during PTCA or other intravascular interventional procedures. They are not intended for use in the cerebral vasculature.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

### CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
PRAStaff@fda.hhs.gov

*"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."*

## 510(k) Summary

per 21 CFR §807.92

**Sponsor:** Boston Scientific Corporation  
300 Boston Scientific Way  
Marlborough, MA 01752

**Contact Person:** Nikki M Wahlberg

**Phone Number:** 763-494-2381

**Fax Number:** 763-494-2981

**Prepared:** December 17, 2014

**Trade Name:** ChoICE™, Mailman™, Luge™, ChoICE™ PT, and ChoICE™ PT Graphix Guide Wires  
ChoICE™ Magnet, Mailman™ Magnet, Luge™ Magnet, ChoICE™ PT Magnet, and PT Graphix™ Magnet Guide Wires

**Common Name:** Catheter Guide Wire

**Classification:** II

**Product Code:** DQX per CFR Part 870.1330

**Primary Predicate Device:**

PT<sup>2</sup> Guide Wire and AddWire™ Extension Wire (K030617, 21 May 2003)

**Reference Predicate Device:**

Luge Guide Wires (K973945, 12 January 1998)

Mailman (ChoICE Family Super Support/Super Support II) Guide Wires (K964551, 21 May 1997)

ChoICE Plus, ChoICE PT Plus, ChoICE PT Graphix (K965023, 3 April 1997)

ChoICE Family of Guide Wires (K970244, 28 March 1997)

ChoICE PT Graphix (K962572, 17 December 1996)

**Device Description:**

The Boston Scientific ChoICE™, Mailman™, Luge™, ChoICE™ PT, and ChoICE™ PT Graphix Guide Wires with ICE® Hydrophilic Coating are available with a nominal diameter of 0.014 in (0.37 mm) and in nominal lengths of 182 cm or 300 cm. These guide wires contain a 304 stainless steel core wire. The proximal section of the core wire of all models is coated with polytetrafluoroethylene (PTFE) for lubricity. The distal end of the core wire is formed (flattened) to allow for shaping. All models are available with a shapeable Straight Tip or a preformed “J” Tip to address user preference. Varying tapers along the distal core wire and differing tip materials (spring coil or polymer) provide combinations of rail support and tip flexibility to address user requirements. The 182 cm guide wires are designed with an extension section for exchange of Over-the-Wire systems by using either the MAGNET Exchange Device or the AddWire™ Extension Wire. The 300 cm guide wires allow exchange of therapeutic devices without the use of an extension wire or exchange system.

**Indications for Use / Intended Use:**

Boston Scientific ChoICE™ PT, PT Graphix™, ChoICE, Luge, and Mailman Guidewires with ICE Hydrophilic Coating are intended to facilitate the placement of balloon dilatation catheters or other therapeutic devices during PTCA or other intravascular interventional procedures. They are not intended for use in the cerebral vasculature.

Boston Scientific ChoICE™ PT Magnet, PT Graphix™ Magnet, ChoICE Magnet, Luge Magnet, and Mailman Magnet Guidewires with ICE Hydrophilic Coating are intended to facilitate the placement of balloon dilatation catheters or other therapeutic devices during PTCA or other intravascular interventional procedures. They are not intended for use in the cerebral vasculature.

**Substantial Equivalence:**

The ChoICE™, Mailman™, Luge™, ChoICE™ PT, and ChoICE™ PT Graphix Guide Wires incorporate substantially equivalent designs, device materials, manufacturing processes, fundamental technology, sterilization process, and intended use as those in the predicate devices. The 182 cm guide wires are designed with an extension section for exchange of Over-the-Wire systems by using either the AddWire™ Extension Wire similar to the design of the predicate device PT² Guide Wire (K030617) or the MAGNET Exchange Device similar to the design of the reference predicates.

**Summary of Non-Clinical Testing:**

Design verification testing was performed to verify that the performance of the ChoICE, Mailman, Luge, ChoICE PT, and ChoICE PT Graphix Guide Wires remains substantially equivalent to the predicate device.

Specifically, the following design verification and validation testing was performed:

- ◆ Torqueability
- ◆ Exchange System Coupling Strength
- ◆ Exchange System Connectability

**Biocompatibility Testing:**

The ChoICE, Mailman, Luge, ChoICE PT, and ChoICE PT Graphix Guide Wires were compared to the predicate devices. Based on similarities of the materials used in the subject devices compared with the predicates, the following biocompatibility testing was conducted:

- ◆ Cytotoxicity
- ◆ USP Physiochemical Test
- ◆ Latex

**Summary of Clinical Testing:**

Clinical Evaluation was not required for these devices.

**Conclusion:**

The ChoICE™, Mailman™, Luge™, ChoICE™ PT, and ChoICE™ PT Graphix Guide Wires have the same intended use and technological characteristics such as components, materials, sterilization method, shelf life and operating principle as the predicate devices. Performance data demonstrates that the device functions as intended, and is as safe and effective as the predicate devices.