



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Center – WO66-G609  
Silver Spring, MD 20993-0002

EWOO SOFT Co., Ltd.  
% Ms. Priscilla Chung  
Regulatory Affairs Consultant  
LK Consulting Group USA, Inc.  
2651 E Chapman Avenue, Suite 110  
FULLERTON CA 92831

June 5, 2015

Re: K150761  
Trade/Device Name: Ez3D-i / E3  
Regulation Number: 21 CFR 892.2050  
Regulation Name: Picture archiving and communications system  
Regulatory Class: II  
Product Code: LLZ  
Dated: May 20, 2015  
Received: May 26, 2015

Dear Ms. Chung:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638 2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, “Misbranding by reference to premarket notification” (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

<http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH’s Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

A handwritten signature in black ink that reads "Robert A. Ochs". The signature is written in a cursive style. A faint, semi-transparent "FDA" logo is visible in the background behind the signature.

Robert Ochs, Ph.D.  
Acting Director  
Division of Radiological Health  
Office of In Vitro Diagnostics  
and Radiological Health  
Center for Devices and Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)

K150761

Device Name

Ez3D-i / E3

Indications for Use (Describe)

Ez3D-i is dental imaging software that is intended to provide diagnostic tools for maxillofacial radiographic imaging. These tools are available to view and interpret a series of DICOM compliant dental radiology images and are meant to be used by trained medical professionals such as radiologist and dentist.

Ez3D-i is intended for use as software to load, view and save DICOM images from CT, panorama, cephalometric and intraoral imaging equipment and to provide 3D visualization, 2D analysis, in various MPR (Multi-Planar Reconstruction) functions.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

### CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
[PRASStaff@fda.hhs.gov](mailto:PRASStaff@fda.hhs.gov)

*"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."*

# 510(k) Summary

(K150761)

This summary of 510(k) information is being submitted in accordance with requirements of 21 CFR Part 807.92.

**1. Date:** 05/20/2015

**2. 510k Applicant / Submitter**

EWOOSOFT Co., Ltd.  
S-402, 231 Pangyoyeok-ro, Bundang-Gu  
Seongnam-Si, Gyeonggi-do, 463-440, Republic of Korea  
Tel: +82 31 8015 6171 Fax: +82 31 696 7420  
Contact person: Young Seok Kim  
Email: ys.kim@ewoosoft.com

**3. U.S. Designated Agent**

Priscilla Chung  
LK Consulting Group USA, Inc.  
2651 E. Chapman Ave. Ste 110,  
Fullerton, CA 92831  
Tel: 714.202.5789 Fax: 714.409.3357  
Email: juhee.c@LKconsultingGroup.com

**4. Trade/Proprietary Name:**

Ez3D-i / E3

**5. Common Name:**

Dental Imaging Software

**6. Classification:**

System, image processing, radiological (21CFR 892.2050, Product code LLZ, Class 2, Radiology)

**7. Device Description:**

Ez3D-i is 3D viewing software for prompt and accurate diagnosis dental CT images in DICOM format with a host of useful functions including MPR, 2-dimensional analysis and 3-dimensional image reformation. It provides advanced simulation functions such as Implant Simulation, Drawing Canal, and Implant Environ Bone Density, etc for the benefit of

effective doctor and patient communication and precise treatment planning. Ez3D-i is a useful tool for an easier diagnosis and analysis by processing a 3D image with simple and convenient user interface. Ez3D-i's main functions are;

- Image adaptation through various rendering methods such as Teeth/Bone/Soft tissue/MIP
- Versatile 3D image viewing via MPR Rotating, Curve mode
- "Sculpt" for deleting unnecessary parts to view only the region of interest.
- Implant Simulation for efficient treatment planning and effective patient consultation
- Canal Draw to trace alveolar canal and its geometrical orientation relative to teeth.
- "Bone Density" test to measure bone density around the site of an implant(s)
- Various utilities such as Measurement, Annotation, Gallery, and Report
- 3D Volume function to transform the image into 3D Panorama and the Tab has been optimized for Implant Simulation.

#### **8. Indication for use:**

Ez3D-i is dental imaging software that is intended to provide diagnostic tools for maxillofacial radiographic imaging. These tools are available to view and interpret a series of DICOM compliant dental radiology images and are meant to be used by trained medical professionals such as radiologist and dentist.

Ez3D-i is intended for use as software to load, view and save DICOM images from CT, panorama, cephalometric and intraoral imaging equipment and to provide 3D visualization, 2D analysis, in various MPR (Multi-Planar Reconstruction) functions.

#### **9. Predicate Device:**

- Ez3D-i /E3 (K131616) by EWOO SOFT Co., Ltd.

#### **10. Substantial Equivalence:**

Ez3D-i described in this 510(k) has the same intended use and the same technical characteristics as the unmodified device (K131616, Ez3D-i /E3).

The subject device and the predicate device are substantially equivalent, having the same indications for use and functionalities like operation software, computer platform, picture archiving and communication format, image format, image processing features, windowing, 3D image construction, image edit, measurements and manipulation.

The differences between the subject device and the unmodified device is that the subject device has additional features such as 3D Panorama View, Navigator and Collision Detection for implant simulation. Based on the results of the system level validation tests provided in this submission, we conclude that the subject device is substantially equivalent to the predicate devices and the differences will not raise issues of safety and effectiveness.

## **11. Technological Characteristics:**

Ez3D-i is a software device that does not contact the patient, nor does it control any life sustaining devices. Results produced by the software's diagnostic, treatment planning and simulation tools are dependent on the interpretation of trained and licensed radiologists, clinicians and referring physicians as an adjunctive to standard radiology practices for diagnosis.

## **12. Performance Data:**

Verification, validation and testing activities were conducted to establish the performance, functionality and reliability characteristics of the modified devices. The device passed all of the tests based on pre-determined Pass/Fail criteria.

## **13. Conclusion:**

The new device and the predicate device are substantially equivalent in the areas of technical characteristics, general function, application, and intended use. The new device does not introduce a fundamentally new scientific technology, and the non-clinical tests demonstrated that the device is substantially equivalent to the predicate device.