



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Center – WO66-G609  
Silver Spring, MD 20993-0002

June 17, 2015

NuVasive, Incorporated  
Ms. Cynthia Adams  
Regulatory Affairs Specialist  
7475 Lusk Boulevard  
San Diego, California 92121

Re: K150994

Trade/Device Name: NuVasive<sup>®</sup> CoRoent<sup>®</sup> Thoracolumbar Implants  
Regulation Number: 21 CFR 888.3080  
Regulation Name: Intervertebral body fusion device  
Regulatory Class: Class II  
Product Code: PHM, MAX, MQP  
Dated: May 21, 2015  
Received: May 22, 2015

Dear Ms. Adams:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set

forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

<http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

**Lori A. Wiggins -S**

for

Mark N. Melkerson

Director

Division of Orthopedic Devices

Office of Device Evaluation

Center for Devices and

Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)

K150994

Device Name

NuVasive® CoRoent® Thoracolumbar Implants

Indications for Use (Describe)

The NuVasive® CoRoent Thoracolumbar Implants are devices indicated for the following:

### Intervertebral Body Fusion

The NuVasive CoRoent Thoracolumbar Implants are indicated for intervertebral body fusion of the spine in skeletally mature patients. The implants are designed for use with autogenous bone graft to facilitate fusion and supplemental internal spinal fixation system cleared by the FDA for use in the thoracolumbar spine. The devices are to be used in patients who have had at least six months of non-operative treatment.

### Thoracic Spine:

The CoRoent Thoracolumbar Implants are intended for use for thoracic interbody fusions at one or two contiguous levels in the thoracic spine, from T1-T2 to T11-T12, or at the thoracolumbar junction (T12-L1), following discectomy for the treatment of a symptomatic disc degeneration (DDD), including thoracic disc herniation (myelopathy and/or radiculopathy with or without axial pain).

### Lumbar Spine:

The CoRoent Thoracolumbar Implants are intended for use at either one level or two contiguous levels in the lumbar spine, from L1-L2 to L5-S1, for the treatment of degenerative disc disease (DDD) with up to Grade I spondylolisthesis. DDD is defined as back pain of discogenic origin with degeneration of the disc confirmed by history and radiographic studies.

### Partial Vertebral Body Replacement

The NuVasive CoRoent Thoracolumbar Implants may also be used as a partial vertebral body replacement device indicated for use in the thoracolumbar spine (T1 to L5) to replace a diseased or damaged vertebral body caused by tumor or fracture, to restore height of a collapsed vertebral body, and to achieve decompression of the spinal cord and neural tissues. The implants are intended to be used with supplemental internal spinal fixation systems (e.g., pedicle screw system) that are cleared by the FDA for use in the thoracic and lumbar spine. Allograft or autograft material may be used at the surgeon's discretion.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

**CONTINUE ON A SEPARATE PAGE IF NEEDED.**

---

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
*PRAStaff@fda.hhs.gov*

*“An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number.”*



## 510(k) Summary

In accordance with Title 21 of the Code of Federal Regulations, Part 807, and in particular 21 CFR §807.92, the following summary of information is provided:

### A. Submitted by:

Cynthia Adams  
 Regulatory Affairs Specialist  
 NuVasive, Incorporated  
 7475 Lusk Blvd.  
 San Diego, California 92121  
 Telephone: (858) 909-1800  
 Date Prepared: June 11, 2015

### B. Device Name

Trade or Proprietary Name:	<i>NuVasive® CoRoent® Thoracolumbar Implants</i>
Common or Usual Name:	Intervertebral Body Fusion Device; Spinal Intervertebral Body Fixation Orthosis
Classification Name:	Intervertebral Body Fusion Device; Spinal Intervertebral Body Fixation Orthosis
Device Class:	Class II
Classification:	21 CFR § 888.3080 and § 888.3060
Product Code:	RJ O ."MAX, MQP

### C. Predicate Devices

The subject *NuVasive CoRoent Thoracolumbar Implants* are substantially equivalent to the primary predicate device, *NuVasive CoRoent Thoracolumbar Implants* (K140659) and additional predicate *NuVasive CoRoent System* (K141665).

### D. Device Description

The *CoRoent Thoracolumbar Implants* are manufactured from PEEK-Optima® LT-1 conforming to ASTM F2026 or titanium alloy (Ti-6Al-4V) conforming to ASTM F136. The PEEK device contains radiographic markers manufactured from titanium alloy conforming to ASTM F136 or ASTM F1472. The implants are available in a variety of different shapes and sizes to suit the individual pathology and anatomical conditions of the patient. The hollow aperture allows for packing of autogenous bone graft. Teeth on the superior and inferior surface of the implants provide resistance to expulsion. The device is intended to be used with supplemental internal spinal fixation systems that are cleared by the FDA for use in the thoracolumbar spine. This Special 510(k) is for additional implant offerings for use in the thoracic spine.

**E. Intended Use**

The *NuVasive® CoRoent Thoracolumbar Implants* are devices indicated for the following:

***Intervertebral Body Fusion***

The *NuVasive CoRoent Thoracolumbar Implants* are indicated for intervertebral body fusion of the spine in skeletally mature patients. The implants are designed for use with autogenous bone graft to facilitate fusion and supplemental internal spinal fixation system cleared by the FDA for use in the thoracolumbar spine. The devices are to be used in patients who have had at least six months of non-operative treatment.

***Thoracic Spine:***

The *CoRoent Thoracolumbar Implants* are intended for use for thoracic interbody fusions at one or two contiguous levels in the thoracic spine, from T1-T2 to T11-T12, or at the thoracolumbar junction (T12-L1), following discectomy for the treatment of a symptomatic disc degeneration (DDD), including thoracic disc herniation (myelopathy and/or radiculopathy with or without axial pain).

***Lumbar Spine:***

The *CoRoent Thoracolumbar Implants* are intended for use at either one level or two contiguous levels in the lumbar spine, from L1-L2 to L5-S1, for the treatment of degenerative disc disease (DDD) with up to Grade I spondylolisthesis. DDD is defined as back pain of discogenic origin with degeneration of the disc confirmed by history and radiographic studies.

***Partial Vertebral Body Replacement***

The *NuVasive CoRoent Thoracolumbar Implants* may also be used as a partial vertebral body replacement device indicated for use in the thoracolumbar spine (T1 to L5) to replace a diseased or damaged vertebral body caused by tumor or fracture, to restore height of a collapsed vertebral body, and to achieve decompression of the spinal cord and neural tissues. The implants are intended to be used with supplemental internal spinal fixation systems (e.g., pedicle screw system) that are cleared by the FDA for use in the thoracic and lumbar spine. Allograft or autograft material may be used at the surgeon's discretion.

**F. Technological Characteristics**

As was established in this submission, the subject *NuVasive CoRoent Thoracolumbar Implants* are substantially equivalent to other predicate devices cleared by the FDA for commercial distribution in the United States. There have been no design changes to the implants previously cleared in the predicate 510(k)s. The subject device was shown to be substantially equivalent and have equivalent technological characteristics to its predicate device through comparison in areas including design, labeling/intended use, material composition, and function.

**G. Performance Data**

No performance data was provided to demonstrate that the subject *NuVasive CoRoent Thoracolumbar Implants* are substantially equivalent to the predicate devices.

**H. Conclusions**

Based on the technological characteristics and comparison to predicate devices, the subject *NuVasive CoRoent Thoracolumbar Implants* has been shown to be substantially equivalent to legally marketed predicate devices.