

510(k) SUMMARY

A. Submitter Information:

Submitter: MEDCOMP®
1499 Delp Drive
Harleysville, PA 19438
(215) 256-4201 Telephone
(215) 256-0818 Fax

Contact: Florence A. Caikoski
Regulatory Affairs Associate

Date Prepared: May 20, 2002

B. Trade Name: Medcomp Ultra-Flow™ Catheter
Common Name: Hemodialysis Catheter, Implanted
Classification: 78 MSD
C.F.R. Section: 876.5540

C. Predicate Device: K003207 Medcomp Ultra-Flow™ Catheter
K012562 Medcomp 14.5F Double Lumen
Hemodialysis Catheter

D. Device Description:

The Medcomp Ultra-Flow™ Catheter is a 14.5F polyurethane, double lumen catheter used to remove and return blood through two-segregated lumen passages. Both lumens are "D" shaped, open at the distal tip, with two side holes. The distal venous lumen is tapered and extends beyond the arterial lumen to reduce recirculation. The fixed polyester cuff allows for tissue ingrowth for long term placement.

The lumens are connected to the extensions via a soft pliable hub with suture wing. The arterial and venous extensions are identified by red and blue luer connectors and clamps. Priming volume information is printed on the clamps for ease in identification.

E. Intended Use:

The Medcomp Ultra-Flow™ Catheter is indicated for use in attaining long-term vascular access for hemodialysis and apheresis. It may be inserted percutaneously and is ideally placed in the internal jugular vein of an adult patient. Alternate insertion site is the subclavian vein as required. Catheters greater than 40cm are intended for femoral vein insertion.

The proposed device is a product line extension to the legally marketed device, and is identical in design and materials.

The modifications include:

- 55cm length
- Femoral insertion instructions for use

G. Performance Data:

In-Vitro performance data for the proposed device including recirculation and flow performance demonstrate that this device is substantially equivalent to legally marketed devices intended for hemodialysis and apheresis treatments.

Since the design and materials remain unchanged additional performance testing or supporting documentation is not deemed necessary and is not included in this submission.

Clinical studies were not deemed necessary since in-vitro testing was sufficient to demonstrate safety and effectiveness by way of comparison to legally marketed predicate devices.



Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

Ms. Florence A. Caikoski
Regulatory Affairs Associate
MEDCOMP®
1499 Delp Drive
HARLEYSVILLE PA 19438

AUG 27 2002

Re: K021759

Trade/Device Name: MEDCOMP® 14F X 55CM Ultra-Flow™ Catheter, UFS55 and UFT55
Regulation Number: 21 CFR §876.5540
Regulation Name: Blood access device and accessories
Regulatory Class: III
Product Code: 78 MSD
Dated: May 20, 2002
Received: May 29, 2002

Dear Ms. Caikoski:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (the Act). You may, therefore, market the device, subject to the general controls provisions of Act. However, you are responsible to determine that the medical devices you use as components in the kit have either been determined as substantially equivalent under the premarket notification process (Section 510(k) of the act), or were on the market prior to May 28, 1976, the enactment date of the Medical Device Amendments. *Please note:* If you purchase your device components in bulk (i.e., unfinished) and further process (e.g., sterilize) you must submit a new 510(k) before including these components in your kit. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality

Page 2 – Ms. Florence A. Caikoski

systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

In addition, we have determined that your device kit contains Povidone Iodine Ointment and Swabsticks and Lidocaine HCl 1%, which are subject to regulation as drugs.

Our substantially equivalent determination does not apply to the drug component[s] of your device. We recommend you first contact the Center for Drug Evaluation and Research before marketing your device with the drug component[s]. For information on applicable Agency requirements for marketing this [these] drug[s], we suggest you contact:

Director, Division of Drug Labeling Compliance (HFD-310)
Center for Drug Evaluation and Research
Food and Drug Administration
5600 Fishers Lane
Rockville, Maryland 20857
(301) 594-0101

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801 and additionally Part 809.10 for *in vitro* diagnostic devices), please contact the Office of Compliance at (301) 594-4616. Additionally, for questions on the promotion and advertising of your device, please contact the Office of Compliance at (301) 594-4639. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). Other general information on your responsibilities under the Act, may be obtained from the Division of Small Manufacturers, International and Consumer Assistance at its toll free number (800) 638-2041 or (301) 443-6597, or at its Internet address <http://www.fda.gov/dsma/dsmamain.html>

Sincerely yours,



Nancy C. Brogdon
Director, Division of Reproductive, Abdominal,
and Radiological Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

K021759/A1

INDICATIONS FOR USE

510(k) Number:

Device Name: MEDCOMP ULTRA-FLOW™ CATHETER

Indications for use:

THE MEDCOMP ULTRA-FLOW™ HEMODIALYSIS CATHETER IS INDICATED FOR USE IN ATTAINING LONG TERM VASCULAR ACCESS FOR HEMODIALYSIS AND Apheresis.

IT MAY BE INSERTED PERCUTANEOUSLY AND IS PRIMARILY PLACED IN THE INTERNAL JUGULAR VEIN OF AN ADULT PATIENT.

ALTERNATE INSERTION SITES INCLUDE THE SUBCLAVIAN VEIN AS REQUIRED.

CATHETERS GREATER THAN 40CM ARE INTENDED FOR FEMORAL VEIN INSERTION.

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Prescription Use (Per 21 CFR 801.109)

OR

Over-The-Counter

Nancy C Brogdon
(Division Sign-Off)
Division of Reproductive, Abdominal,
and Radiological Devices
510(k) Number K021759

(Optional Format 1-2-96)