

K022707

Summary of Safety and Effectiveness  
Liquichek™ Urine Toxicology Control Levels S1E and S2E**1.0 Submitter**

Bio-Rad Laboratories  
9500 Jeronimo Road,  
Irvine, California 92618-2017  
Telephone: (949) 598-1200  
Fax: (949) 598-1555

**Contact Person**

Elizabeth Platt  
RA/QA Manager  
Telephone: (949) 598-1285

**Date of Summary Preparation**

August 2, 2002

**2.0 Device Identification**

Product Trade Name: Liquichek™ Urine Toxicology Control Levels S1E and S2E  
Common Name: Drug Mixture Controls  
Classifications: Class I  
Product Code: 91DIF  
Regulation Number: CFR 862.3280

**3.0 Device to Which Substantial Equivalence is Claimed**

Liquichek™ Urine Toxicology Control Levels S1 and S2  
Bio-Rad Laboratories  
Irvine, California

Docket Numbers: K991558

**4.0 Description of Device**

Liquichek™ Urine Toxicology Control Levels S1E and S2E is prepared from human urine with added constituents of animal origin, drugs, drug metabolites, preservatives, and stabilizers. The control is provided in liquid form for convenience.

5.0 **Statement of Intended Use**

Liquichek™ Urine Toxicology Control is intended for use as a quality control urine to monitor the performance of laboratory urine toxicology enzyme immunoassay (EIA) screening procedures.

6.0 **Comparison of the new device with the Predicate Device**

The new Liquichek™ Urine Toxicology Control claims substantial equivalence to the Liquichek™ Urine Toxicology Controls currently in commercial distribution (K991558).

**Table 1.** Similarities and Differences between new and predicate device.

Characteristics	Bio Rad Liquichek™ Urine Toxicology Control, Levels S1E and S2E (New Device)	Bio Rad Liquichek™ Urine Toxicology Control, Levels S1 ans S2 (Predicate Device)
<b>Similarities</b>		
<b>Form</b>	Liquid	Liquid
<b>Matrix</b>	Human urine	Human urine
<b>Storage</b> (Unopened)	2-8° C until expiration date	2-8° C until expiration date
<b>Open Vial Claim</b>	2-8° C for 30 days	2-8° C for 30 days
<b>Differences</b>		
<b>Intended Use</b>	A quality control urine to monitor the performance of laboratory urine toxicology enzyme immunoassay (EIA) screening procedures.	A quality control urine to monitor the performance of laboratory urine toxicology screening procedures.
<b>Levels</b>	Level S1E = Drugs and drug metabolites added during manufacture at concentrations 20-25% <b>below</b> enzyme immunoassay cutoff levels. Level S2E = Drugs and drug metabolites added at concentrations 20-25% <b>above</b> enzyme immunoassay cutoff levels.	Level S1 = Drugs and drug metabolites added at concentrations 20–25% <b>below</b> immunoassay cutoff levels. Level S2 = Drugs and drug metabolites added at concentrations 20-25% <b>above</b> immunoassay cutoff levels.

<b>Characteristics</b>	<b>Bio Rad Liquichek™ Urine Toxicology Control, Levels S1E and S2E (New Device)</b>	<b>Bio Rad Liquichek™ Urine Toxicology Control, Levels S1 and S2 (Predicate Device)</b>
<b>Differences</b>		
<b>Claimed Analytes</b>	Methamphetamine, Secobarbital, Lormetazepam, Tetrahydrocannabinol (THC), Benzoyllecgonine, Ethanol, Lysergic Acid Diethylamide (LSD), Methadone, Methaqualone, Morphine (Free), Phencyclidine, Propoxyphene, Nortriptyline and addition of Creatinine, pH, Specific Gravity.	Amphetamine, Secobarbital, Nordiazepam, Tetrahydrocannabinol (THC), Benzoyllecgonine, Ethanol, Lysergic Acid Diethylamide (LSD), Methadone, Methaqualone, Morphine (Free), Phencyclidine, Propoxyphene, Nortriptyline.

## 2.0 STATEMENT OF SUPPORTING DATA

Stability studies have been performed to determine the open vial stability and shelf life for the Liquichek™ Urine Toxicology Control. Product claims are as follows:

2.1 Open vial: Once the control is opened, all analytes will be stable for 30 days when stored tightly capped at 2-8°C.

2.2 Shelf Life: 2 years when stored unopened at 2-8°C.

Real time studies will be ongoing to support the shelf life of this product.

All supporting data is retained on file at Bio-Rad Laboratories.



DEPARTMENT OF HEALTH & HUMAN SERVICES

AUG 22 2002

Food and Drug Administration  
2098 Gaither Road  
Rockville MD 20850

Ms. Elizabeth Platt  
Regulatory Affairs/Quality Assurance Manager  
Bio-Rad Laboratories, QSD  
9500 Jeronimo Road  
Irvine, CA 92618-2017

Re: k022707  
Trade/Device Name: Liquichek™ Urine Toxicology Control Levels S1E and S2E  
Regulation Number: 21 CFR 862.3280  
Regulation Name: Clinical toxicology control material  
Regulatory Class: Class I  
Product Code: DIF  
Dated: August 2, 2002  
Received: August 14, 2002

Dear Ms. Platt:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Page 2 -

This letter will allow you to begin marketing your device as described in your 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801 and additionally 809.10 for in vitro diagnostic devices), please contact the Office of Compliance at (301) 594-4588. Additionally, for questions on the promotion and advertising of your device, please contact the Office of Compliance at (301) 594-4639. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR 807.97). Other general information on your responsibilities under the Act may be obtained from the Division of Small Manufacturers International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its internet address "<http://www.fda.gov/cdrh/dsma/dsmamain.html>".

Sincerely yours,



Steven I. Gutman, M.D., M.B.A.  
Director  
Division of Clinical Laboratory Devices  
Office of Device Evaluation  
Center for Devices and  
Radiological Health

Enclosure

510 (k) Number (if known): K022707

Device Name: **Liquichek™ Urine Toxicology Control Levels S1E and S2E**

Indications for Use:

Liquichek™ Urine Toxicology Control Levels S1E and S2E is intended for use as a quality control urine to monitor the performance of laboratory urine toxicology enzyme immunoassay (EIA) screening procedures.

(PLEASE DO NOT WRITE BELOW THE LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

---

*Concurrence of CDRH, Office of Device Evaluation (ODE)*

Prescription use  or Over-the Counter use

Carol C Benson for Jean Cooper, DVM  
(Division Sign-Off)  
Division of Clinical Laboratory Devices  
510(k) Number K022707