

AUG 16 2004

### 510(k) Summary for N Latex Cystatin C

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirements of SMDA 1990 and 21 CFR 807.92.

The assigned 510(k) number is: X041878

**1. Manufacturer's Name, Address, Telephone, and Contact Person, Date of Preparation:**

Manufacturer: Dade Behring Marburg GmbH  
Emil-von-Behring Str. 76  
Marburg/Germany

Contact Information: Dade Behring Inc.  
Glasgow Site  
P.O. Box 6101  
Newark, Delaware 19714  
Attn: Kathleen Dray-Lyons  
Tel: 781.826.4551

Preparation date: July 9, 2004

**2. Device Name/ Classification:**

**Device Name:** N Latex Cystatin C  
**Device Description:** Creatinine test system  
**Classification:** Class II  
**Regulation Number:** 21 CFR 862.1225  
**Panel:** Clinical Chemistry  
**Product Code:** NDY

**3. Identification of the Legally Marketed Device:**

N Latex Cystatin C – K003503

**4. Device Description:**

Polystyrene particles coated with specific antibodies to human cystatin C are aggregated when mixed with samples containing human cystatin C. These aggregates scatter a beam of light passed through the sample. The intensity of the scattered light is proportional to the concentration of the relevant protein in the sample. The result is evaluated by comparison with a standard of known concentration.

000043

**5. Device Intended Use:**

N Latex Cystatin C is an *in vitro* diagnostics kit containing reagents for the quantitative determination of cystatin C in human serum and heparinized plasma by means of particle-enhanced immunonephelometry using BN™ Systems. Cystatin C measurements are used in the diagnosis and treatment of renal diseases.

**6. Medical device to which equivalence is claimed and comparison information:**

The modified N Latex Cystatin C assay is substantially equivalent in intended use to the currently marketed N Latex Cystatin C assay (K003503).



DEPARTMENT OF HEALTH & HUMAN SERVICES

Food and Drug Administration  
2098 Gaither Road  
Rockville MD 20850

AUG 16 2004

Ms. Kathleen A. Dray-Lyons  
Regulatory Affairs and Compliance Manager  
Dade Behring, Inc.  
Glasgow Site  
PO Box 6101  
Newark, DE 19714

Re: k041878  
Trade/Device Name: N Latex Cystatin C  
Regulation Number: 21 CFR 862.1225  
Regulation Name: Creatinine test system  
Regulatory Class: Class II  
Product Code: NDY  
Dated: July 9, 2004  
Received: July 15, 2004

Dear Ms. Dray-Lyons:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in Title 21, Code of Federal Regulations (CFR), Parts 800 to 895. In addition, FDA may publish further announcements concerning your device in the Federal Register.


Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Parts 801 and 809); and good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820).

Page 2

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific information about the application of labeling requirements to your device, or questions on the promotion and advertising of your device, please contact the Office of *In Vitro* Diagnostic Device Evaluation and Safety at (301) 594-3084. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its Internet address <http://www.fda.gov/cdrh/dsma/dsmamain.html>.

Sincerely yours,

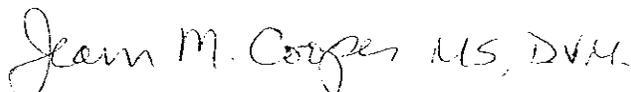

Jean M. Cooper, MS, D.V.M.  
Director  
Division of Chemistry and Toxicology  
Office of *In Vitro* Diagnostic Device  
Evaluation and Safety  
Center for Devices and  
Radiological Health

Enclosure

## Indications Statement

**Device Name:** N Latex Cystatin C

### Indications for Use:

N Latex Cystatin C is an in vitro diagnostics kit containing reagents for the quantitative determination of cystatin C in human serum and heparinized plasma by means of particle-enhanced immunonephelometry using BN™ Systems. Cystatin C measurements are used in the diagnosis and treatment of renal diseases.

Prescription Use   
(Per 21 CFR 801.109)

Over-The-Counter-Use   
(Optional Format 1-2-96)

(PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

*Carol Benson*  
Division Sign-Off

Office of In Vitro Diagnostic  
Device Evaluation and Safety

510(k) K041878

000033