

MAY 17 2005

**510(k) Summary**  
(As required by 21 CFR 807.92)

**A. Submitter Information**

Submitter's Name: St. Jude Medical  
 Address: 14901 DeVeau Place  
 Minnetonka, Minnesota 55345-2126 U.S.A.  
 Telephone Number: (952) 351-1453  
 Contact Person: Mike Burnside  
 Date Submission Prepared: 28-April-2005

**B. Device Information**

Trade Name: Apeel™ CS Plus Catheter Delivery System  
 Common or Usual Name: Percutaneous Catheter Introducer  
 Classification Name: Percutaneous Catheter Introducer  
 (per 21 CFR Part 870.1340)  
 Device Classification: Class II (per 21 CFR Part 870.1340)  
 Panel – Cardiovascular  
 Predicate Device: Apeel CS Catheter Delivery System  
 Reference Device: Spyglass Angiographic Catheter

Device Description: The Apeel™ CS Plus Catheter Delivery System includes a Peelable Introducer Sheath, Dilator, Cannulator, Detachable Hemostasis Valve with Side Port with 3-way Stopcock, Guidewire, Syringe, Needle, and 2 Valve Bypass Tools. The introducer kits are provided sterile, and are intended for single-use only.

Intended Use: The Apeel CS Plus Catheter Delivery System is intended to provide vascular access including the coronary sinus and serve as a conduit for the delivery and support of other devices where minimizing blood loss is essential.

**C. Comparison of Required Technological Characteristics**

All technological characteristics of the Apeel™ CS Plus Catheter Delivery System are substantially equivalent to the predicate and referenced devices including product design, packaging, sterilization, and labeling.

**D. Support of the Substantial Equivalence**

St. Jude Medical considers the Apeel CS Plus Catheter Delivery System to be substantially equivalent to the predicate and referenced devices.



Food and Drug Administration  
9200 Corporate Boulevard  
Rockville MD 20850

MAY 17 2005

Mr. Mike Burnside  
Senior Regulatory Affairs Specialist  
St. Jude Medical  
14900 Minnetonka Ind. Rd.  
Minnetonka, MN 55345

Re: K051096  
Trade/Device Name: Apeel™ CS Plus Catheter Delivery System  
Regulation Number: 21 CFR 870.1340  
Regulation Name: Catheter introducer  
Regulatory Class: II  
Product Code: DYB  
Dated: April 28, 2005  
Received: April 29, 2005

Dear Mr. Burnside:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.


Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set

Page 2 – Mr. Mike Burnside

forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050. This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at (240) 276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,

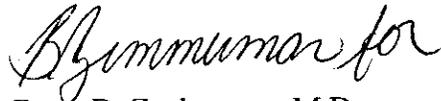

Bram D. Zuckerman, M.D.  
Director  
Division of Cardiovascular Devices  
Office of Device Evaluation  
Center for Devices and  
Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known): K051096

Device Name: Apeel™ CS Plus Catheter Delivery System

Indications for Use:

The Apeel CS Plus Catheter Delivery System is intended to provide vascular access including the coronary sinus and serve as a conduit for the delivery and support of other devices where minimizing blood loss is essential.

Prescription Use                        
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use                       
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE  
OF NEEDED)

---

Concurrence of CDRH, Office of Device Evaluation (ODE)

B. Amman  
(Division Sign-Off)

Division of Cardiovascular Devices

510(k) Number K051096

Page 1 of 1