

K052079 1.62



OCT 21 2005

510(k) SUMMARY

Submitted July 28, 2005

DePuy Orthopaedics, Inc.

PO Box 988
700 Orthopaedic Drive
Warsaw, Indiana 46581-0988
USA

Tel: +1 (574) 267 8143

NAME OF FIRM: DePuy Orthopaedics, Inc.
700 Orthopaedic Drive
Warsaw, IN 46580

510(K) CONTACT: Keli K. Hankee
Clinical Research Associate
DePuy Orthopaedics, Inc.
PO Box 988
700 Orthopaedic Drive
Warsaw, IN 46580

TRADE NAME: DePuy Pinnacle[®] Constrained Acetabular Liners

COMMON NAME: Acetabular Cup Liner

CLASSIFICATION: Class II, per 21 CFR, 888.3310
Hip joint metal/polymer/ metal, constrained, cemented or uncemented prosthesis

DEVICE PRODUCT CODE: 87 KWZ

SUBSTANTIALLY EQUIVALENT DEVICES: **Osteonics** Constrained Acetabular Inserts
DePuy Pinnacle[®] Constrained Acetabular Liners

K052079 2d2

DEVICE DESCRIPTION:

The Pinnacle[®] Constrained Acetabular Liner is part of a modular system designed to replace the natural articular surface of the hip joint in total hip replacement. The liner is manufactured from ultra high molecular weight polyethylene (UHMWPE), which locks into a porous coated, hemispherical outer shell component manufactured from titanium alloy (Ti-6Al-4V). The liner component articulates with a metal femoral head of an appropriate diameter.

The Pinnacle[®] Constrained Acetabular Liner mechanically constrains the femoral head within the ID of the liner by providing greater than 180 degrees femoral head capture combined with a titanium constraining ring which fits over the opening diameter of the liner. The UHMWPE liner is held in the metal shell by means of a titanium locking ring.

The subject Pinnacle[®] Constrained Acetabular Liners are UHMWPE acetabular cup liners that are available in a lateralized neutral or lateralized face-changing orientation. The liners have inner diameters (ID) intended for use with Self-Centering[™] (bipolar) femoral heads within the 40mm-44mm size range. The 62mm-76mm outer diameters (ODs) are geometrically the same as other smaller OD Pinnacle Acetabular Liners cleared in K043058.

INDICATIONS AND INTENDED USE:

Indications:

The Pinnacle[®] Constrained Acetabular Liner is indicated for use as a component of a total hip prosthesis in primary or revision patients at high risk of hip dislocation due to a history of prior dislocation, bone loss, joint or soft tissue laxity, neuromuscular disease or intraoperative instability.

The Pinnacle[®] Constrained Acetabular Liner is indicated for use with the Pinnacle[®] Acetabular Cup in cementless application.

Intended Use:

The subject Pinnacle[®] Constrained Acetabular Liner is intended to be used with the DePuy Pinnacle[®] metal acetabular shells and Self-centering[™] (bipolar) heads to resurface the acetabular socket in cementless total hip arthroplasty.

Basis of Substantial Equivalence:

The DePuy Pinnacle[®] Constrained Acetabular Liner (lateralized neutral and lateralized face-changing), described in this submission is substantially equivalent to the predicate devices based on similarities in design, intended use, material and manufacturing methods. The design is identical to the DePuy predicate and does not raise any new issues of safety or effectiveness.

DePuy believes that the DePuy Pinnacle[®] Constrained Acetabular Liner is substantially equivalent to the previously cleared and approved devices.

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at (240) 276-0210. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its Internet address <http://www.fda.gov/cdrh/dsma/dsmamain.html>.

Sincerely yours,



Mark N. Melkerson
Acting Director
Division of General, Restorative,
and Neurological Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

K052079

Indications for Use

510(k) Number (if known): K052079
Device Name: Pinnacle[®] Constrained Acetabular Liner

Indications for Use:

The Pinnacle[®] Constrained Acetabular Liner is indicated for use as a component of a total hip prosthesis in primary or revision patients at high risk of hip dislocation due to a history of prior dislocation, bone loss, joint or soft tissue laxity, neuromuscular disease or intraoperative instability.

The Pinnacle[®] Constrained Acetabular Liner is indicated for use with the Pinnacle[®] Acetabular Cup in cementless application.


Prescription Use X AND/OR Over-The-Counter Use _____
(Part 21 CFR 801 Subpart D) (21 CFR 807 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Page of

(Posted November 13, 2003)

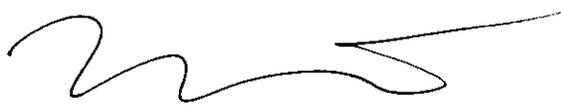

(Division Sign-Off)
Division of General, Restorative,
and Neurological Devices

510(k) Number K052079