

K072279
pg. 1 of 3

Submitter Information

Submitter: Hitachi Medical Systems America, Inc.
1959 Summit Commerce Park
Twinsburg, Ohio 44080-2371
ph: (330) 425-1313
fax: (330) 425-1410

SEP 21 2007

Contact: Douglas J. Thistlethwaite
Date: 08/13/2007

Device Name

Classification Name: System, Nuclear Magnetic Resonance Imaging
Classification Number: 90LNH
Trade/Proprietary Name: OASIS Magnetic Resonance Imaging System
Predicate Device(s): ECHELON Magnetic Resonance Imaging System
(K052172)

Device Intended Use

The MR system is an imaging device and is intended to provide the physician with physiological and clinical information, obtained non-invasively and without the use of ionizing radiation. The MR system produces transverse, coronal, sagittal, oblique, and curved cross-sectional images that display the internal structure of the head, body, or extremities. The images produced by the MR system reflect the spatial distribution of protons (hydrogen nuclei) exhibiting magnetic resonance. The NMR properties that determine the image appearance are proton density, spin-lattice relaxation time (T1), spin-spin relaxation time (T2) and flow. When interpreted by a trained physician, these images provide information that can be useful in diagnosis determination.

Device Description

Function

The OASIS is a Magnetic Resonance Imaging System that utilizes a 1.2 Tesla superconducting open magnet in an open gantry design. The design was based on the ECHELON MRI system. The OASIS has been designed to enhance clinical utility as compared to the ECHELON by taking advantage of open architecture.

Scientific Concepts

Magnetic Resonance Imaging (MRI) is based on the fact that certain atomic nuclei have electromagnetic properties that cause them to act as small spinning bar magnets. The most ubiquitous of these nuclei is hydrogen, which makes it the primary nuclei currently used in magnetic resonance imaging. When placed in a static magnetic field, these nuclei assume a net orientation or alignment with the magnetic field, referred to as a net magnetization vector. The introduction of a short burst of radiofrequency (RF) excitation of a wavelength specific to the magnetic field strength and to the atomic nuclei under consideration can cause a re-orientation of the net magnetization vector. When the RF excitation is removed, the protons relax and return to their original vector. The rate of relaxation is exponential and varies with the character of the proton and its adjacent molecular environment. This re-orientation process is characterized by two exponential relaxation times, called T1 and T2.

A RF emission or echo that can be measured accompanies these relaxation events. The emissions are used to develop a representation of the relaxation events in a three dimensional matrix. Spatial localization is encoded into the echoes by varying the RF excitation, applying appropriate magnetic field gradients in the x, y, and z directions, and changing the direction and strength of these gradients. Images depicting the spatial distribution of the NMR characteristics can be reconstructed by using image processing techniques similar to those used in computed tomography.

Physical and Performance Characteristics

MRI is capable of producing high quality anatomical images without the associated risks of ionizing radiation. The biological properties that contribute to MR image contrast are different from those responsible for x-ray image contrast. In MR imaging, difference in proton density, blood flow, and T1 and T2 relaxation times can all contribute to image contrast. By varying the pulse sequence characteristics, the resulting images can emphasize T1, T2, proton density, or the molecular diffusion of water or other proton containing molecules.

Device Technological Characteristics

The technological characteristics of this device are similar to the primary predicate device. The primary difference is the 1.2 Tesla superconducting open magnet. The control and image processing hardware and the base elements of the system software are identical to the predicate device.

Conclusions

It is the opinion of Hitachi Medical Systems America, Inc. that OASIS MRI system is substantially equivalent to the listed predicate device. The intended use is identical to the listed predicate device.



DEPARTMENT OF HEALTH & HUMAN SERVICES

Food and Drug Administration
9200 Corporate Blvd.
Rockville MD 20850

SEP 21 2007

Mr. Doug Thistlethwaite
Manager, Regulatory Affairs
Hitachi Medical Systems America, Inc.
1959 Summit Commerce Park
TWINSBURG OH 44087

Re: K072279

Trade/Device Name: OASIS, w/V1.0A Operating System Software
Regulation Number: 21 CFR 892.1000
Regulation Name: Magnetic resonance diagnostic device
Regulatory Class: II
Product Code: LNH
Dated: August 13, 2007
Received: August 16, 2007

Dear Mr. Thistlethwaite:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (Premarket Approval), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.



Protecting and Promoting Public Health

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.


This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at one of the following numbers, based on the regulation number at the top of this letter:

21 CFR 876.xxx	(Gastroenterology/Renal/Urology)	240-276-0115
21 CFR 884.xxx	(Obstetrics/Gynecology)	240-276-0115
21 CFR 894.xxx	(Radiology)	240-276-0120
Other		240-276-0100

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,

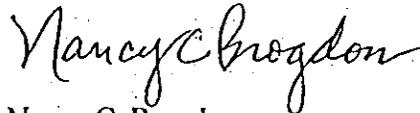

Nancy C. Brogdon
Director, Division of Reproductive,
Abdominal, and Radiological Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

510(k) Number (if known): _____

Device Name: OASIS, w/V1.0A Operating System Software

Indications for Use:

The OASIS MR system is an imaging device, and is intended to provide the physician with physiological and clinical information, obtained non-invasively and without the use of ionizing radiation. The MR system produces transverse, coronal, sagittal, oblique, and curved cross-sectional images that display the internal structure of the head, body, or extremities. The images produced by the MR system reflect the spatial distribution of protons (hydrogen nuclei) exhibiting magnetic resonance. The NMR properties that determine the image appearance are proton density, spin-lattice relaxation time (T1), spin-spin relaxation time (T2), and flow. When interpreted by a trained physician, these images provide information that can be useful in diagnosis determination.

Anatomical Region: Head, Body, Spine, Extremities

Nucleus excited: Proton

Diagnostic uses: T1, T2, proton density weighted imaging
Diffusion weighted imaging
MR Angiography
Image processing


Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-the-Counter Use _____
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

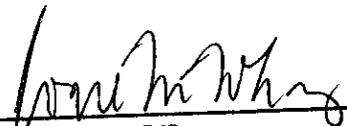

(Division Sign-Off)
Division of Reproductive, Abdominal and
Radiological Devices
510(k) Number K072279