

FEB 12 2008

K072563

510(k) Summary [21 CFR §807.92]

Prepared: August 15, 2007

Device Trade Name: Miltex Rigid Sterilization Container System.

Device Common Name: Rigid Sterilization Container.

Classification Name: Sterilization wrap containers, trays, cassettes, and other accessories.

Class of Device: Class II device, product code KCT

Predicate Device: SteriTite® Rigid Sterilization Container System with MediTray Products- Case Medical, Incorporated- K022978

Official Contact: Audie Margrave, Senior Director QA/RA

Device Description:

The Miltex Rigid Sterilization Container System consists of a family of rigid, re-usable, sealed containers that provides an effective sterilization packaging method for medical devices. Container bottoms and lids, within a given size, are interchangeable. The system is composed of the following components:

- Container bottoms (perforated versions)
- Container baskets,
- Container lids (perforated only)

The container system is designed for sterilant penetration through perforations in the lid and container bottom models that are perforated.

Intended Use:

The Miltex Rigid Sterilization Container System is intended for use in hospitals and health care facilities to contain other medical devices that are to be sterilized and to allow sterilization of the enclosed medical devices, including surfaces and lumens, using gravity steam sterilizers. The recommended gravity steam sterilization cycle parameters are 45 minutes exposure at 121 deg C and a 20 minute dry time. 2 metal lumens 3-mm in diameter and up to 200-mm in length were validated. The container is intended for use with non-porous materials such as stainless steel surgical instruments.

For effective sterilization and drying, the weight of the single container, basket and basket contents should not exceed the following: 16 lbs for the one-half size container, 20 lbs for the three-quarter size container, and 25 lbs for the full size container.

Sterilized devices may be stored and transported within the container. The container is intended to maintain sterility for a maximum of 60 days as long as the integrity of the container has not been compromised.

Containers should not be stacked when used within gravity steam sterilizers.

The device models that are the subject of this pre-market notification on listed on page 2 of this Indication for Use statement.

Part Number	Description	UM
3-5110-10	STERI CONTAINER ½ (11 ¼ in x 11 in) x 4 inches in height PERFORATED BOTTOM, SILVER	EA
3-5110-13	STERI CONTAINER ½ (11 ¼ in x 11 in) x 5 inches in height PERFORATED BOTTOM, SILVER	EA
3-5110-15	STERI CONTAINER ½ (11 ¼ in x 11 in) x 6 inches in height PERFORATED BOTTOM, SILVER	EA
3-5310-10	STERI CONTAINER ¾ (18 ¼ in x 11 in) x 4 inches in height PERFORATED BOTTOM, SILVER	EA
3-5310-13	STERI CONTAINER ¾ (18 ¼ in x 11 in) x 5 inches in height PERFORATED BOTTOM, SILVER	EA
3-5310-15	STERI CONTAINER ¾ (18 ¼ in x 11 in) x 6 inches in height PERFORATED BOTTOM, SILVER	EA
3-5510-10	STERI CONTAINER 1 (23 in x 11 in)x 4 inches in height PERFORATED BOTTOM, SILVER	EA
3-5510-13	STERI CONTAINER 1 (23 in x 11 in)x 5 inches in height PERFORATED BOTTOM, SILVER	EA
3-5510-15	STERI CONTAINER 1 (23 in x 11 in) x 6 inches in height PERFORATED BOTTOM, SILVER	EA
3-6100-00	STERI CONTAINER LID ½ (11 ¼ in x 11 in) PERFORATED, SILVER	EA
3-6300-00	STERI CONTAINER LID ¾ (18 ¼ in x 11 in) PERFORATED, SILVER	EA
3-6500-00	STERI CONTAINER LID 1/1 (23 in x 11 in) PERFORATED, SILVER	EA

Key – ½ indicates one-half size or 11 ¼ inches x 11 inches
¾ indicates three-quarter size or 18 ¼ inches x 11 inches
1 indicates full size or 23 inches x 11 inches

The following container lids can be used for each container base:

Lid Size	Perforated Base to be used with Lid
½ size Lid (11 ¼ in x 11 in) PN 3-6100-00	PN 3-5110-10 (4 in height) PN 3-5110-13 (5 1/3 in height) PN 3-5110-15 (6 in height)
¾ size Lid (18 ¼ in x 11 in) PN 3-6300-00	PN 3-5310-10 (4 in height) PN 3-5310-13 (5 1/3 in height) PN 3-5310-15 (6 in height)
Full size Lid (23 in x 11 in) PN 3-6500-00	PN 3-5510-10 (4 in height) PN 3-5510-13 (5 1/3 in height) PN 3-5510-15 (6 in height)

Technological Characteristics:

A comparison of the technology characteristics of the Miltex Rigid Sterilization Containers to the predicate device.

Properties	Miltex System	SteriTite System
Intended use to contain instruments being sterilized in gravity steam sterilizers	Yes	Yes
Intended to be re-used	Yes	Yes
Closed System	Yes	Yes
Sealed	Yes	Yes
Design		
Incorporates a filter system to permit entry of sterilant agent	Yes	Yes
Incorporates a filter system to prevent microbial migration during transport.	Yes	Yes
Materials		
Container	Aluminum alloy, Stainless Steel, & Silicone	Aluminum alloy, Stainless Steel, & Silicone

Performance Data:

A comparison of the non-clinical performance of the Miltex Rigid Sterilization Containers to the predicate device.

Properties	Miltex System	SteriTite System
Performance Standards		
Conformance to appropriate AAMI standards	Yes, conforms to AAMI ST 77 Draft- <i>Containment Devices for Reusable Medical Device Sterilization</i>	Yes, conforms to AAMI ST-33 – Guidelines for the Selection and Use of Reusable Rigid Sterilization Container Systems for ETO Sterilization and Steam Sterilization in Healthcare Facilities
Validation Testing		
Gravity Steam Efficacy Testing	Yes	Yes
Thermal Profile Study	Sterilant penetration can be achieved in a single layer configuration	Sterilant penetration can be achieved in a stacked configuration
Sterility Maintenance	30-day real time w/ weekly handling events (perforated bottom).	90-day real time w/ weekly handling events. 30-day real time w/ daily handling events.
Load	Up to 16-lbs. (small) Up to 20-lbs. (med.) Up to 25-lbs. (large)	Up to 22-lbs.
Test Organisms/ Inoculated Product		
Inoculated Lumens	Yes-- 3-mm I.D. x 200-mm, metal	Yes
Inoculated Stainless Steel Medical Devices	Yes (hinge area of medical pliers and knurled instruments)	Yes

Conclusion:

The Miltex Rigid Sterilization Containers is substantially equivalent to the SteriTite Container (K022978).



FEB 12 2008

Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

Patricia Kihn D.D.S.
Manager Quality Systems Compliance & Regulatory Affairs
Miltex, Incorporated
589 Davies Drive
York, Pennsylvania 17402

Re: K072563
Trade/Device Name: Miltex Rigid Containers
Regulation Number: 880.6850
Regulation Name: Sterilization Wrap
Regulatory Class: II
Product Code: KCT
Dated: January 25, 2008
Received: January 28, 2008

Dear Dr. Patricia Kihn:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at (240) 276-0115. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,



Chiu Lin, Ph.D.

Director

Division of Anesthesiology, General Hospital,

Infection Control and Dental Devices

Office of Device Evaluation

Center for Devices and

Radiological Health

Enclosure

Indications for Use

510(k) Number (if known): K072563

Device Name: Miltex Rigid Containers

Indications For Use:

The Miltex Rigid Sterilization Container System is intended for use in hospitals and health care facilities to contain other medical devices that are to be sterilized and to allow sterilization of the enclosed medical devices, including surfaces and lumens, using gravity steam sterilizers. The recommended gravity steam sterilization cycle parameters are 45 minutes exposure at 121 deg C and a 20 minute dry time. 2 metal lumens 3-mm in diameter and up to 200-mm in length were validated. The container is intended for use with non-porous materials such as stainless steel surgical instruments.

For effective sterilization and drying, the weight of the single container, basket and basket contents should not exceed the following: 16 lbs for the one-half size container, 20 lbs for the three-quarter size container, and 25 lbs for the full size container.

Sterilized devices may be stored and transported within the container. The container is intended to maintain sterility for a maximum of 60 days as long as the integrity of the container has not been compromised.

Containers should not be stacked when used within gravity steam sterilizers.

Lid Size	Perforated Base to be used with Lid
½ size Lid (11 ¼ in x 11 in) PN 3-6100-00	PN 3-5110-10 (4 in height) PN 3-5110-13 (5 in height) PN 3-5110-15-15 (6 in height)
¾ size Lid (18 ¼ in x 11 in) PN 3-6300-00	PN 3-5310-10 (4 in height) PN 3-5310-13 (5 in height) PN 3-5310-15 (6 in height)
Full size Lid (23 in x 11 in) PN 3-6500-00	PN 3-5510-10 (4 in height) PN 3-5510-13 (5 in height) PN 3-5510-15 (6 in height)


Prescription Use _____
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use X
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

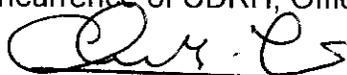

(Division Sign-Off)

Division of Anesthesiology, General Hospital
Infection Control, Dental Devices

510(k) Number: K072563

Page 1 of _____