

K072923

APR 24 2008

510(k) Summary:

Submitter's Name and Address:

ZOLL Medical Corporation
Worldwide Headquarters
269 Mill Road
Chelmsford, MA 01824-4105
(978) 421-9655

Contact Person:

Sean Reynolds
(978) 421-9655, Ext. 9386

Date Summary Prepared:

August 23, 2007

Device:

ZOLL E Series Defibrillator

Classification:

Automated External Defibrillator: Class III (21 CFR 870.5310)

Description:

The ZOLL E Series External Defibrillator is indicated for the defibrillation, Noninvasive Transcutaneous Pacing and multi-parameter monitoring of patient vital signs, including: ECG Monitoring, Pulse Oximetry, end tidal CO₂, 12-Lead ECG monitoring, non-invasive blood pressure measurement, CPR performance and data printing and recording for resting patients in critical care and transport. The ZOLL E Series is intended for use by qualified medical personnel who are trained and authorized to respond to medical emergencies, to facilitate the ability to monitor and assess the physiological characteristics of the indicated patients in a critical care environment. The device is light weight and easy to carry for transport.

The device is capable of providing Basic Life-Saving (BLS) personnel the option of analyzing a patient's ECG signal via the Advisory feature on the device. The Advisory Algorithm will determine if the acquired heart rhythm is shockable or non-shockable and will prompt the end-user to provide therapy in the event of a shock advised determination. The user will be prompted to perform Cardiopulmonary Resuscitation in the event that the Analyze feature determines that CPR compressions are more beneficial to the victim.

Substantial Equivalence:

The features and functions of the enhancement to the ZOLL E Series are substantially equivalent to those of the Philips Medical Systems HeartStart FR2+ AED: 510(k) No. k051632, cleared 10/05/2005.

Indications for Use:**Defibrillator Function**

The E Series products contain a dc defibrillator capable of delivering up to 200 joules of energy. It may be used in synchronized mode to perform synchronized cardioversion by using the R-wave of the patient's ECG as a timing reference. The unit uses paddles or disposable, pre-gelled, MFE Pads for defibrillation.

The E Series products must be prescribed for use by a physician or medical advisor of an emergency response team.

Intended Use — Manual Operation

Use of the E Series products in the manual mode for defibrillation is indicated on victims of cardiac arrest where there is apparent lack of circulation as indicated by these three conditions:

- Unconsciousness
- Absence of breathing, and
- Absence of pulse.

This product should be used only by qualified medical personnel for converting ventricular fibrillation and rapid ventricular tachycardia to sinus rhythm or other cardiac rhythms capable of producing hemodynamically significant heart beats.

In manual mode, the E Series unit may also be used for synchronized cardioversion to terminate atrial fibrillation (AF) or ventricular tachycardias (VT) by using the R-wave of the patient's ECG as a timing reference. A qualified physician must decide when synchronized cardioversion is appropriate.

The advisory function should be used to confirm ventricular fibrillation and wide complex ventricular tachycardia (greater than 150 beats per minute) in patients meeting the three conditions indicating lack of circulation (previously listed).

Intended Use — Semiautomatic Operation (AED)

The E Series AED unit is designed for use by emergency care personnel who have completed training and certification requirements applicable to the use of a defibrillator where the device operator controls delivery of shocks to the patient.

They are specifically designed for use in early defibrillation programs where the delivery of a defibrillator shock during resuscitation involving CPR, transportation, and definitive care are incorporated into a medically-approved patient care protocol.

Use of the device in the Semiautomatic mode for defibrillation is indicated on victims of cardiac arrest where there is apparent lack of circulation.

Specifications for the ECG rhythm analysis function are provided at the end of this section.

Intended Use — CPR Monitoring

The CPR monitoring function provides visual and audio feedback designed to encourage rescuers to perform chest compressions at the AHA/ERC recommended rate of 100 compressions per minute. Voice and visual prompts encourage a compression depth of 1.5 to 2 inches (3.8 to 5.0 cm) for adult patients.

The CPR monitoring function is not intended for use on patients under 8 years of age.

Contraindications for Semiautomatic Operation

Do not use the unit's AED function on patients under 8 years of age.

The rhythm analysis function may not reliably identify ventricular fibrillation in the presence of an implanted pacemaker. Inspection of the electrocardiogram and clinical evidence of cardiopulmonary arrest should be the basis for any treatment of patients with implanted pacemakers.

Do not use the rhythm analysis function during patient movement on a stretcher or in an ambulance or other conveyance. A patient must be motionless during ECG analysis. Do not touch the patient during analysis. Cease all movement of the stretcher or vehicle prior to analyzing the ECG. If you are using the device in an emergency vehicle, bring the vehicle to a halt before activating the analysis function.

Defibrillator Complications

Inappropriate defibrillation or cardioversion of a patient (e.g., with no malignant arrhythmia) may precipitate ventricular fibrillation, asystole, or other dangerous arrhythmias.

Defibrillation without proper application of electrode pads or paddle electrolyte gel may be ineffective and cause burns, particularly when repeated shocks are necessary. Erythema or hyperemia of the skin under the paddles or MFE Pads often occurs; this effect is usually enhanced along the perimeter of the paddle or electrode. This reddening should diminish substantially within 72 hours.

Defibrillator Output Energy

The E Series products may deliver up to 200 joules into a 50 ohm impedance. The energy delivered through the chest wall, however, is determined by the patients transthoracic impedance. An adequate amount of electrolyte gel must be applied to the paddles and a force of 10-12 kilograms (22-26.4 lbs) must be applied to each paddle in order to minimize this impedance. If MFE pads are used, make sure that they are properly applied. (Instructions for proper application are located in "MFE Pad Application/Connection" on page 1-9.

External Pacemaker Function (Pacer version only)

Some E Series products may include an optional transcutaneous demand pacemaker consisting of a pulse generator and ECG sensing circuitry. Non-invasive Transcutaneous Pacing (NTP) is an established and proven technique. This therapy is easily and rapidly applied in both emergency and non-emergency situations when temporary cardiac stimulation is indicated.

Proper operation of the device, together with correct electrode placement, is critical to obtaining optimal results. Every operator must be thoroughly familiar with these operating instructions.

The output current of the pacemaker is continuously variable from 0 to 140 mA. The rate is continuously variable from 30 to 180 pulses per minute (ppm).

The pacing output pulse is delivered to the heart by specially designed ZOLL MFE Pads placed on the back and the precordium.

The characteristics of the output pulse, together with the design and placement of the electrodes, minimize cutaneous nerve stimulation, cardiac stimulation threshold currents, and reduce discomfort due to skeletal muscle contraction.

The unique design of the E Series products allow clear viewing and interpretation of the electrocardiogram (ECG) on the display without offset or distortion during external pacing.

Intended Use — Pacemaker

This product may be used for temporary external cardiac pacing in conscious or unconscious patients as an alternative to endocardial stimulation.

This device must not be connected to internal pacemaker electrodes.

The purposes of pacing include:

Resuscitation from standstill or bradycardia of any etiology

Noninvasive pacing has been used for resuscitation from cardiac standstill, reflex vagal standstill, drug induced standstill (due to procainamide, quinidine, digitalis, b-blockers, verapamil, etc.) and unexpected circulatory arrest (due to anesthesia, surgery, angiography, and other therapeutic or diagnostic procedures). It has also been used for temporary acceleration of bradycardia in Stokes-Adams disease and sick-sinus syndrome. It is safer, more reliable, and more rapidly applied in an emergency than endocardial or other temporary electrodes.

As a standby when standstill or bradycardia might be expected

Noninvasive pacing may be useful as a standby when cardiac arrest or symptomatic bradycardia might be expected due to acute myocardial infarction, drug toxicity, anesthesia or surgery. It is also useful as a temporary treatment in patients awaiting pacemaker implants or the introduction of transvenous therapy. In standby pacing applications, noninvasive pacing may provide an alternative to transvenous therapy that avoids the risks of displacement, infection, hemorrhage, embolization, perforation, phlebitis and mechanical or electrical stimulation of ventricular tachycardia or fibrillation associated with endocardial pacing.

Suppression of tachycardia

Increased heart rates in response to external pacing often suppress ventricular ectopic activity and may prevent tachycardia.

Pacemaker Complications

Ventricular fibrillation does not respond to pacing and requires immediate defibrillation. Therefore, the patient's dysrhythmia must be determined immediately, so that you can employ appropriate therapy. If the patient is in ventricular fibrillation and defibrillation is successful but cardiac standstill (asystole) ensues, you should use the pacemaker.

Ventricular or supraventricular tachycardias may be interrupted with pacing but in an emergency or during circulatory collapse, synchronized cardioversion is faster and more certain. (See "Synchronized Cardioversion" on page 6-1).

Electromechanical dissociation may occur following prolonged cardiac arrest or in other disease states with myocardial depression. Pacing may then produce ECG responses without effective mechanical contractions, and other treatment is required.

Pacing may evoke undesirable repetitive responses, tachycardia, or fibrillation in the presence of generalized hypoxia, myocardial ischemia, cardiac drug toxicity, electrolyte imbalance, or other cardiac diseases.

Pacing by any method tends to inhibit intrinsic rhythmicity. Abrupt cessation of pacing, particularly at rapid rates, can cause ventricular standstill and should be avoided.

Noninvasive Temporary Pacing may cause discomfort of varying intensity, which occasionally can be severe and preclude its continued use in conscious patients.

Similarly, unavoidable skeletal muscle contraction may be troublesome in very sick patients and may limit continuous use to a few hours. Erythema or hyperemia of the skin under the MFE Pads often occurs; this effect is usually enhanced along the perimeter of the electrode.

Comparison of Technological Characteristics

The ZOLL E Series design characteristics have not been changed from those, which were reviewed and cleared by FDA under premarket notification 510(k) K070455. The ZOLL E Series acquires and analyzes ECG signals and provides shock advisory determinations using methods that are very similar to the predicate Philips Medical Systems HeartStart FR2+ Defibrillator.

Performance Testing:

Extensive performance testing ensures that the ZOLL E Series Defibrillator meets all of its functional requirements and performance specifications. Safety testing assures the device complies with applicable sections of recognized industry and safety standards.

Conclusion

Performance and safety testing of the ZOLL E Series Defibrillator demonstrates that its features, functions and incorporated interpretive algorithm are substantially equivalent to that of the indicated commercially distributed predicate devices with regard to performance, safety and effectiveness.



Food and Drug Administration
9200 Corporate Boulevard
Rockville MD 20850

APR 24 2008

Zoll Medical Corporation
Worldwide Headquarters
c/o Ms. Eileen M. Boyle
269 Mill Road
Chelmsford, MA 01824-4105

Re: K072923

Trade Name: Zoll E Series Defibrillator
Regulation Number: 21 CFR 870.5310
Regulation Name: Automated External Defibrillator
Regulatory Class: III (three)
Dated: March 25, 2008
Received: April 7, 2008

Dear Ms. Boyle:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.


If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Center for Devices and Radiological Health's (CDRH's) Office of Compliance at (240) 276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding postmarket surveillance, please contact CDRH's Office of Surveillance and Biometric's (OSB's) Division of Postmarket Surveillance at (240) 276-3474. For questions regarding the reporting of device adverse events (Medical Device Reporting (MDR)), please contact the Division of Surveillance Systems at (240) 276-3464. You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,

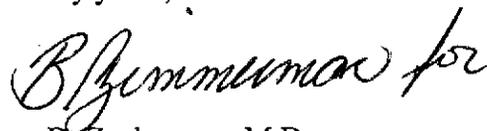

Bram D. Zuckerman, M.D.
Director
Division of Cardiovascular Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

SECTION 4 – INDICATIONS FOR USE

510(k) Number (if known): _____

Device Name: ZOLL E Series

Defibrillator Function

The E Series products contain a DC defibrillator capable of delivering up to 200 joules of energy. It may be used in synchronized mode to perform synchronized cardioversion by using the R-wave of the patient's ECG as a timing reference. The unit uses paddles or disposable, pre-gelled, MFE Pads for defibrillation.

Intended Use — Manual Operation

Use of the E Series products in the manual mode for defibrillation is indicated on victims of cardiac arrest where there is apparent lack of circulation as indicated by:

- Unconsciousness
- Absence of breathing
- Absence of pulse.

This product should be used only by qualified medical personnel for converting ventricular fibrillation and rapid ventricular tachycardia to sinus rhythm or other cardiac rhythms capable of producing hemodynamically significant heart beats.

Intended Use — Semiautomatic Operation (AED)

The E Series products are designed for use by emergency care personnel who have completed training and certification requirements applicable to the use of a defibrillator where the device operator controls delivery of shocks to the patient.

They are specifically designed for use in early defibrillation programs where the delivery of a defibrillator shock during resuscitation involving CPR, transportation, and definitive care are incorporated into a medically-approved patient care protocol.

The E Series products must be prescribed for use by a physician or medical advisor of an emergency response team.

Use of the device in the Semiautomatic mode for defibrillation is indicated on victims of cardiac arrest where there is apparent lack of circulation as indicated by:

- Unconsciousness
- Absence of breathing
- Absence of pulse.

(continued on next page)


Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use _____
(21 CFR 807 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE OF
NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

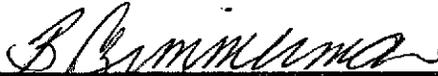

(Division Sign-Off)
Division of Cardiovascular Devices
510(k) Number K072923

Indications for Use
(continued from previous page)

External Pacemaker (Pacer Version Only)

Intended Use — Pacemaker

This product may be used for temporary external cardiac pacing in conscious or unconscious patients as an alternative to endocardial stimulation.

Note: This device must not be connected to internal pacemaker electrodes.

The purposes of pacing include:

Resuscitation from standstill or bradycardia of any etiology:

Noninvasive pacing has been used for resuscitation from cardiac standstill, reflex vagal standstill, drug induced standstill (due to procainamide, quinidine, digitalis, b- blockers, verapamil, etc.) and unexpected circulatory arrest (due to anesthesia, surgery, angiography, and other therapeutic or diagnostic procedures). It has also been used for temporary acceleration of bradycardia in Stokes-Adams disease and sick-sinus syndrome. It is safer, more reliable, and more rapidly applied in an emergency than endocardial or other temporary electrodes.

As a standby when standstill or bradycardia might be expected:

Noninvasive pacing may be useful as a standby when cardiac arrest or symptomatic bradycardia might be expected due to acute myocardial infarction, drug toxicity, anesthesia or surgery. It is also useful as a temporary treatment in patients awaiting pacemaker implants or the introduction of transvenous therapy. In standby pacing applications, noninvasive pacing may provide an alternative to transvenous therapy that avoids the risks of displacement, infection, hemorrhage, embolization, perforation, phlebitis and mechanical or electrical stimulation of ventricular tachycardia or fibrillation associated with endocardial pacing.

Suppression of tachycardia:

Increased heart rates in response to external pacing often suppress ventricular ectopic activity and may prevent tachycardia.

Pediatric Pacing

Pacing can be performed on pediatric patients weighing 33lbs / 15kg or less using special ZOLL pediatric MFE Pads. Prolonged pacing (in excess of 30 minutes), particularly in neonates, could cause burns. Periodic inspection of the underlying skin is recommended.

Monitor

Intended Use — Multi-parameter Monitoring

This product may be used for monitoring various patient vital signs, including: electrocardiogram (ECG), Pulse Oximetry (SpO₂), End Tidal CO₂, 12-Lead ECG, and Non-Invasive Blood Pressure (NIBP).

ECG monitoring is performed by connecting the patient to the unit via the 3 or 5 lead patient cable, MFE Pads, or through the paddles.

SpO₂ monitoring is indicated for detecting arterial oxygen saturation of blood and pulse rate for adult, pediatric and neonatal patients who are well or poorly perfusing, during both no motion and patient motion conditions.

EtCO₂ monitoring is indicated for the continuous measurement of end tidal carbon dioxide (EtCO₂) and respiration rate for adult, pediatric and neonatal patients.

12 Lead ECG analysis is indicated for the diagnosis and treatment of adult and pediatric patients with acute myocardial infarction or other cardiac arrhythmias.

NIBP monitoring is indicated for the measurement of arterial blood pressure for resting adult, pediatric, and neonatal patients.

(continued on next page)

Indications for Use
(continued from previous page)

Intended Use —CPR Monitoring

The CPR monitoring function provides visual and audio feedback designed to encourage rescuers to perform chest compressions at the AHA/ERC recommended rate of 100 compressions per minute. Voice and visual prompts encourage a compression depth of 1.5 to 2 inches (3.8 to 5.0 cm) for adult patients.

The CPR monitoring function is not intended for use on patients under 8 years of age.