



# U.S. Department of Health & Human Services

---

**Food and Drug Administration**

## SAVE REQUEST

**USER:** (kml)  
**FOLDER:** K133997 - 965 pages  
**COMPANY:** LG ELECTRONICS (LGELECB)  
**PRODUCT:** TRANSMITTERS AND RECEIVERS, PHYSIOLOGICAL SIGNAL, RADIOFREQUENCY (DRG)  
**SUMMARY:** Product: LG SMARTHEALTH

**DATE REQUESTED:** Dec 20, 2015

**DATE PRINTED:** Dec 20, 2015

**Note:** Printed





K133997/A001

January 8, 2014

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
Cardiac Diagnostics Devices Branch,  
Division of Cardiovascular Devices  
10903 New Hampshire Avenue  
W066-1244  
Silver Spring, Maryland 20993-0002

FDA/CDRH/DCC  
JAN 10 2014  
RECEIVED

Dear Oscar;

LG Electronics ('LG'), LG SmartHealth, hereby submits this response to the agency's request for information **K133997** email dated January 8, 2014. The purpose of this submission is to address the deficiencies identified during FDA's review of the 510(k) submission.

In regard your comments that the submission contains both, prescription and OTC designation, the OTC check box on the Indications for Use FDA Form was inadvertently selected. This submission is for prescription use only, and is evidenced by the Rx indications in the 510(k) Summary, Executive Summary, User Manual (pg. 4) and Section 12 – Substantial Equivalence table. A revised Indications for Use FDA Form 3881 will be provided as an **Exhibit A**.

As both the predicate and the LG SmartHealth device are indicated for prescription use only, we believe the analysis of the differences between the LG SmartHealth and the predicate is appropriate. It should be noted that the comparison table correctly indicated Rx status for both the predicate and the LG SmartHealth devices and referenced in the 510(k) Summary, Executive Summary, and Section 12 – Substantial Equivalence document.

Per your comment regarding labeling, the User Manual correctly indicated the prescription use statement on page 3 of 39, 026\_Exhibit 13A – UHCTX-MNL 01 v1 0h.pdf. The name and place of business of the manufacturer is provided within the User Manual (Exhibit 13A) in section 1.8, page 4 of 39.

The CD provided with the submission is the official electronic copy of the submission; the eCopy is an exact duplicate of the paper copy.

We appreciate your prompt consideration of this submission. If you have any questions, please contact me at 561.305.5075 or by email at [project.management@emergogroup.com](mailto:project.management@emergogroup.com).

Sincerely,

Diane Sudduth  
Senior Consultant, RA/QA

29

January 8, 2014

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
Cardiac Diagnostics Devices Branch,  
Division of Cardiovascular Devices  
10903 New Hampshire Avenue  
W066-1244  
Silver Spring, Maryland 20993-0002

Dear Oscar;

LG Electronics ('LG'), LG SmartHealth, hereby submits this response to the agency's request for information **K133997** email dated January 8, 2014. The purpose of this submission is to address the deficiencies identified during FDA's review of the 510(k) submission.

In regard your comments that the submission contains both, prescription and OTC designation, the OTC check box on the Indications for Use FDA Form was inadvertently selected. This submission is for prescription use only, and is evidenced by the Rx indications in the 510(k) Summary, Executive Summary, User Manual (pg. 4) and Section 12 – Substantial Equivalence table. A revised Indications for Use FDA Form 3881 will be provided as an **Exhibit A**.

As both the predicate and the LG SmartHealth device are indicated for prescription use only, we believe the analysis of the differences between the LG SmartHealth and the predicate is appropriate. It should be noted that the comparison table correctly indicated Rx status for both the predicate and the LG SmartHealth devices and referenced in the 510(k) Summary, Executive Summary, and Section 12 – Substantial Equivalence document.

Per your comment regarding labeling, the User Manual correctly indicated the prescription use statement on page 3 of 39, 026\_Exhibit 13A – UHCTX-MNL 01 v1 0h.pdf. The name and place of business of the manufacturer is provided within the User Manual (Exhibit 13A) in section 1.8, page 4 of 39.

The CD provided with the submission is the official electronic copy of the submission; the eCopy is an exact duplicate of the paper copy.

We appreciate your prompt consideration of this submission. If you have any questions, please contact me at 561.305.5075 or by email at [project.management@emergogroup.com](mailto:project.management@emergogroup.com).

Sincerely,



Diane Sudduth  
Senior Consultant, RA/QA

## Indications for Use

510(k) Number (if known)

**Device Name**

LG SmartHealth

**Indications for Use (Describe)**

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers(or nurse). Patients can also engage in video conferences with caregivers(or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

**PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON A SEPARATE PAGE IF NEEDED.**

**FOR FDA USE ONLY**

Concurrence of Center for Devices and Radiological Health (CDRH) (Signature)

---

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
*PRAStaff@fda.hhs.gov*

*“An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number.”*

K133997

**Section 3 – 510(k) Cover Letter**

FDA CDRH DMC

DEC 27 2013

Received

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
Document Mail Center – WO66-G609  
10903 New Hampshire Avenue  
Silver Spring, Maryland 20993-0002

Dear Sir or Madam:

This document contains the information for LG Electronics, LG SmartHealth submission for market clearance. In accordance with Section 510(k) of the Federal Food and Drug Cosmetic Act as amended, and in conformance with Title 21 CFR, Part 807, this Pre-market Notification is being submitted at least ninety (90) days prior to the date when LG Electronics proposes to introduce its LG SmartHealth into interstate commerce for commercial distribution. The CD provided with the submission is the official electronic copy of the submission; the eCopy is an exact duplicate of the paper copy. Following contains the regulatory information for the contents of this submission supporting the device's market clearance.

**Administrative Information**

Date of Submission: December 26, 2013

Submission is Completed By: Emergo Group  
816 Congress Avenue  
Suite 1400  
Austin, TX 78701

Application Correspondent: Dr. Diane Sudduth  
Senior Consultant QA, Emergo Group  
Email: [project.management@emergogroup.com](mailto:project.management@emergogroup.com)  
Cellphone number: 561.305.5075  
Home office number: 512.327.9997

Submission Sponsor: LG Electronics  
38 Baumoe-ro, Seocho-gu  
Seoul, 137-724  
Korea  
Chang Hee Lee  
Chief Research Engineer  
Office number: +82-2-526-4370

**Device Identification**

Type of 510(k) Submission: Traditional

Trade or Proprietary Name: LG SmartHealth

Common or Usual Name: Remote Patient Monitoring System

Regulation Classification: Transmitters and Receivers, Physiological Signal, Radiofrequency

Product Code: 870.2910 (Product Code DRG)  
 Class of Device: Class II  
 Panel: Cardiovascular  
 Reason for Submission: New device  
 Prior Related Submissions: No prior submissions for the device

**Design and Use of the Device**

**Table 3A – Principle Factors**

Question	Yes	No
Is the device intended for prescription use (21 CFR 801 Subpart D)?	X	
Is the device intended for over-the-counter use (21 CFR 807 Subpart C)?		X
Does the device contain components derived from a tissue or other biologic source?		X
Is the device provided sterile?		X
Is the device intended for single use?		X
Is the device a reprocessed single use device?		X
If yes, does this device type require reprocessed validation data?		N/A
Does the device contain a drug?		X
Does the device contain a biologic?		X
Does the device use software?	X	
Does the submission include clinical information?		X
Is the device implanted?		X

All information necessary for a substantial equivalence determination is included herein. Should you require any additional data in order to reach a determination of substantial equivalence, please do not hesitate to contact me at 561.305.5075 or by email at [project.management@emergogroup.com](mailto:project.management@emergogroup.com)

Sincerely,



Diane Sudduth  
 Senior Consultant, RA/QA

---

## TABLE OF CONTENTS

Section 1	Medical Device User Fee Cover Sheet (FDA Form 3601)
Section 2	CDRH Premarket Review Cover Sheet (FDA Form 3514)
Section 3	510(k) Cover Letter
Section 4	Indications for Use Statement
Section 5	510(k) Summary
Section 6	Truth and Accuracy Statement
Section 7	Class III Summary and Certification
Section 8	Financial Certification or Disclosure Statement
Section 9	Declaration of Conformity and Summary Reports
Section 10	Executive Summary
Section 11	Device Description
Section 12	Substantial Equivalence Discussion
Section 13	Proposed Labeling
Section 14	Sterilization and Shelf Life – <b>Not Applicable</b>
Section 15	Biocompatibility – <b>Not Applicable</b>
Section 16	Software
Section 17	Electromagnetic Compatibility and Electrical Safety – <b>Not Applicable</b>
Section 18	Performance Testing – Bench – <b>Not Applicable</b>
Section 19	Performance Testing – Animal – <b>Not Applicable</b>
Section 20	Performance Testing – Clinical
Section 21	Exhibits (Other)



**LIST OF SECTIONS**

		<b>Present</b>	<b>Not Applicable</b>
<b>Section 1</b>	<b>Medical Device User Fee Cover Sheet (FDA Form 3601)</b>	X	
<b>Section 2</b>	<b>CDRH Premarket Review Cover Sheet (FDA Form 3514)</b>	X	
<b>Section 3</b>	<b>510(k) Cover Letter</b>	X	
	Administrative Information		
	Device Identification		
	Design and Use of the Device		
	Table 3A – Principle Factors		
<b>Section 4</b>	<b>Indications for Use Statement</b>	X	
<b>Section 5</b>	<b>510(k) Summary</b>	X	
	Submission Sponsor		
	Submission Correspondent		
	Device Identification		
	Predicate Devices		
	Device Description		
	Intended Use		
	Comparison of Technological Characteristics		
	Table 5A – Comparison of Characteristics		
	Non-Clinical Performance Data		
	Clinical Testing		
	Statement of Substantial Equivalence		
<b>Section 6</b>	<b>Truth and Accuracy Statement</b>	X	
<b>Section 7</b>	<b>Class III Summary and Certification</b>		X
<b>Section 8</b>	<b>Financial Certification or Disclosure Statement</b>		X
<b>Section 9</b>	<b>Declaration of Conformity and Summary Reports</b>	X	
	Risk Management Declaration of Conformity		
	Software Design and Development Declaration of Conformity		
	Table 9A – List of Voluntary Standards		
<b>Section 10</b>	<b>Executive Summary</b>	X	
	Summary of Device		
	Indications for Use		
	Device Comparison Summary		
	Table 10A – Device Comparison Table		
	Performance Testing Summary		
<b>Section 11</b>	<b>Device Description</b>	X	
	Description of the Device		
<b>Section 12</b>	<b>Substantial Equivalence Discussion</b>	X	
	Comparison to Legally Marketed Device		
	Table 12A – Device Comparison Chart: Similarities and Differences		
	Performance Testing Summary		
	Table 12B – Performance Summary		
	Comparison Discussion		
<b>Section 13</b>	<b>Proposed Labeling</b>	X	

		Present	Not Applicable
	Labeling		
	Table 13A – Labeling Information		
<b>Section 14</b>	<b>Sterilization and Shelf Life</b>		X
<b>Section 15</b>	<b>Biocompatibility</b>		X
<b>Section 16</b>	<b>Software</b>	X	
	Device Description		
	Level of Concern		
	Table 16A – Major Level of Concern Rationale		
	Table 16B – Moderate Level of Concern Rationale		
	Level of Concern for OTS Software		
	Table 16C – Summary of Software Documentation		
	Software Description		
	Figure 1 – SmartHealth Operating Structure		
	Figure 2 – Televisit Process		
	Figure 3 – Medication Management Process		
	Figure 4 – Remote Patient Monitoring Process		
	Software Development		
	Table 16D – Process Outputs, Tools, Methodology and Technology		
	Table 16E – Standard Reference List of Outputs		
	Table 16F - Infrastructure		
	Figure 5 – Software Development Process Software Requirements Specification		
	Architecture Design Chart		
	Figure 6 – External Interface Specification		
	Figure 7 – Password Encryption Algorithm		
	Figure 8 – LG SmartHealth Architecture Layer		
	Software Design Specification		
	Device Hazard Analysis		
	Traceability Analysis		
	Verification and Validation Documentation		
	Revision Level History		
	Unresolved Anomalies		
	Software Certification		
<b>Section 17</b>	<b>Electromagnetic Compatibility and Electrical Safety</b>		X
<b>Section 18</b>	<b>Performance Testing – Bench</b>		X
<b>Section 19</b>	<b>Performance Testing – Animal</b>		X
<b>Section 20</b>	<b>Performance Testing – Clinical</b>	X	
<b>Section 21</b>	<b>Intentionally left blank</b>		
<b>EXHIBITS</b>			
Exhibit 6A	Signed Truth and Accuracy Statement	X	
Exhibit 9A	Standards Data Report IEC 62304 2006/C11 2008 (Form FDA 3654)	X	
Exhibit 9B	Standards Data Report ISO 14971 2007 (Form FDA 3654)	X	

		Present	Not Applicable
Exhibit 12A	510(k) Summary for Intel Health Guide PHS6000 (K103276)	X	
Exhibit 13A	LG Electronics User Manual	X	
Exhibit 16A-1	Software Requirements Specifications (common)	X	
Exhibit 16A-2	Software Requirements Specifications (Telemedicine)	X	
Exhibit 16A-3	Software Requirements Specifications (Measurement)	X	
Exhibit 16A-4	Software Requirements Specifications (Medication)	X	
Exhibit 16B	Software Architecture Document	X	
Exhibit 16C-1	High Level Design Document (common)	X	
Exhibit 16C-2	High Level Design Document (Telemedicine)	X	
Exhibit 16C-3	High Level Design Document (Measurement)	X	
Exhibit 16C-4	High Level Design Document (Medication)	X	
Exhibit 16D	Requirements Traceability Matrix	X	
Exhibit 16E	Risk Management File	X	
Exhibit 16F	Integration/System Test Plan	X	
Exhibit 16F-1	Integration/System Test Protocol (common)	X	
Exhibit 16F-2	Integration/System Test Protocol (Telemedicine)	X	
Exhibit 16F-3	Integration/System Test Protocol (Measurement)	X	
Exhibit 16F-4	Integration/System Test Protocol (Medication)	X	
Exhibit 16G	Verification Test Report	X	
Exhibit 16H	Validation Test Report	X	
Exhibit 16H	Software Certificate	X	
Exhibit 18A	Bluetooth Communication Test Report	X	
Exhibit 20A	Signed Certificate of Compliance for Clinical Trials (Form FDA 3674)	X	

## **Section 1 - Medical Device User Fee Cover Sheet (FDA Form 3601)**

**Device Name:** LG SmartHealth

The FDA Form 3601 Medical Device User Fee Sheet is contained in this section.

DEPARTMENT OF HEALTH AND HUMAN SERVICES FOOD AND DRUG ADMINISTRATION <b>MEDICAL DEVICE USER FEE COVER SHEET</b>	PAYMENT IDENTIFICATION NUMBER: (b) (4) Write the Payment Identification number on your check.
-----------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------

A completed cover sheet must accompany each original application or supplement subject to fees. If payment is sent by U.S. mail or courier, please include a copy of this completed form with payment. Payment and mailing instructions can be found at: <http://www.fda.gov/oc/mdufma/cover sheet.html>

1. COMPANY NAME AND ADDRESS (include name, street address, city state, country, and post office code)  LG ELECTRONICS INC INCHEON FACTORY 816 Congress Avenue Suite 1400 Austin TX 78701 US  1.1 EMPLOYER IDENTIFICATION NUMBER (EIN)	2. CONTACT NAME Melissa Brightwell 2.1 E-MAIL ADDRESS mbrightwell@emergogroup.com 2.2 TELEPHONE NUMBER (include Area code) 512-222-0256 2.3 FACSIMILE (FAX) NUMBER (Include Area code)
------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

3. TYPE OF PREMARKET APPLICATION (Select one of the following in each column; if you are unsure, please refer to the application descriptions at the following web site: <http://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/GuidanceDocuments/ucm345263.htm>)

Select an application type:

<input checked="" type="checkbox"/> Premarket notification(510(k)); except for third party <input type="checkbox"/> 513(g) Request for Information <input type="checkbox"/> Biologics License Application (BLA) <input type="checkbox"/> Premarket Approval Application (PMA) <input type="checkbox"/> Modular PMA <input type="checkbox"/> Product Development Protocol (PDP) <input type="checkbox"/> Premarket Report (PMR) <input type="checkbox"/> 30-Day Notice	3.1 Select a center <input checked="" type="checkbox"/> CDRH <input type="checkbox"/> CBER 3.2 Select one of the types below <input checked="" type="checkbox"/> Original Application <u>Supplement Types:</u> <input type="checkbox"/> Efficacy (BLA) <input type="checkbox"/> Panel Track (PMA, PMR, PDP) <input type="checkbox"/> Real-Time (PMA, PMR, PDP) <input type="checkbox"/> 180-day (PMA, PMR, PDP)
--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

4. ARE YOU A SMALL BUSINESS? (See the instructions for more information on determining this status)

YES, I meet the small business criteria and have submitted the required qualifying documents to FDA       NO, I am not a small business

4.1 If Yes, please enter your Small Business Decision Number:

5. FDA WILL NOT ACCEPT YOUR SUBMISSION IF YOUR COMPANY HAS NOT PAID AN ESTABLISHMENT REGISTRATION FEE THAT IS DUE TO FDA. HAS YOUR COMPANY PAID ALL ESTABLISHMENT REGISTRATION FEES THAT ARE DUE TO FDA?

YES (All of our establishments have registered and paid the fee, or this is our first device, and we will register and pay the fee within 30 days of FDA's approval/clearance of this device.)

NO (If "NO," FDA will not accept your submission until you have paid all fees due to FDA. This submission will not be processed; see <http://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/HowtoMarketYourDevice/RegistrationandListing/ucm053165.htm> for additional information)

6. IS THIS PREMARKET APPLICATION COVERED BY ANY OF THE FOLLOWING USER FEE EXCEPTIONS? IF SO, CHECK THE APPLICABLE EXCEPTION.

<input type="checkbox"/> This application is the first PMA submitted by a qualified small business, including any affiliates <input type="checkbox"/> This biologics application is submitted under section 351 of the Public Health Service Act for a product licensed for further manufacturing use only	<input type="checkbox"/> The sole purpose of the application is to support conditions of use for a pediatric population <input type="checkbox"/> The application is submitted by a state or federal government entity for a device that is not to be distributed commercially
---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

7. IS THIS A SUPPLEMENT TO A PREMARKET APPLICATION FOR WHICH FEES WERE WAIVED DUE TO SOLE USE IN A PEDIATRIC POPULATION THAT NOW PROPOSES CONDITION OF USE FOR ANY ADULT POPULATION? (If so, the application is subject to the fee that applies for an original premarket approval application (PMA).)

YES       NO

PAPERWORK REDUCTION ACT STATEMENT  
 Public reporting burden for this collection of information is estimated to average 18 minutes per response, including the time for

reviewing instructions, searching existing data sources, gathering and maintaining the data needed, and completing and reviewing the collection of information. Send comments regarding this burden estimate or any other aspect of this collection of information, including suggestions for reducing this burden, to the address below.

Department of Health and Human Services, Food and Drug Administration, Office of Chief Information Officer, 1350 Piccard Drive, 4th Floor Rockville, MD 20850

[Please do NOT return this form to the above address, except as it pertains to comments on the burden estimate.]

8. USER FEE PAYMENT AMOUNT SUBMITTED FOR THIS PREMARKET APPLICATION

(b) (4)

26-Dec-2013

Form FDA 3601 (01/2007)

["Close Window"](#) [Print Cover sheet](#)

Online Payment

Step 3: Confirm Payment

1 | 2 | 3

Thank you.  
Your transaction has been successfully completed.

Pay.gov Tracking Information

Application Name: (b) (4)  
Pay.gov Tracking ID: (b) (4)  
Agency Tracking ID: (b) (4)  
Transaction Date and Time: (b) (4)

Payment Summary

Address Information

Account Holder Name: (b) (4)  
Billing Address: (b) (4)Tr  
Billing Address 2: (b) (4)Trade  
City: (b) (4)T  
State / Province: (b) (4)  
Zip / Postal Code: (b) (4)T  
Country: (b) (4)

Account Information

Card Type: (b) (4)  
Card Number: (b) (4)

Payment Information

Payment Amount: (b) (4)  
Transaction Date and Time: (b) (4)

## **Section 2 – CDRH Premarket Review Submission Cover Sheet (FDA Form 3514)**

**Device Name:** LG SmartHealth

The CDRH Premarket Review Submission Cover Sheet is contained in this section. Standard Data Report Forms (FDA Forms 3654) for applicable standards are referenced in Section 9 Declaration of Conformity and Summary Reports of this submission.



**CDRH PREMARKET REVIEW SUBMISSION COVER SHEET**

Date of Submission 12/26/2013	User Fee Payment ID Number (b)(4)	FDA Submission Document Number (if known)
----------------------------------	--------------------------------------	-------------------------------------------

**SECTION A TYPE OF SUBMISSION**

<b>PMA</b> <input type="checkbox"/> Original Submission <input type="checkbox"/> Premarket Report <input type="checkbox"/> Modular Submission <input type="checkbox"/> Amendment <input type="checkbox"/> Report <input type="checkbox"/> Report Amendment <input type="checkbox"/> Licensing Agreement	<b>PMA &amp; HDE Supplement</b> <input type="checkbox"/> Regular (180 day) <input type="checkbox"/> Special <input type="checkbox"/> Panel Track (PMA Only) <input type="checkbox"/> 30-day Supplement <input type="checkbox"/> 30-day Notice <input type="checkbox"/> 135-day Supplement <input type="checkbox"/> Real-time Review <input type="checkbox"/> Amendment to PMA &HDE Supplement <input type="checkbox"/> Other	<b>PDP</b> <input type="checkbox"/> Original PDP <input type="checkbox"/> Notice of Completion <input type="checkbox"/> Amendment to PDP	<b>510(k)</b> <input checked="" type="checkbox"/> Original Submission: <input checked="" type="checkbox"/> Traditional <input type="checkbox"/> Special <input type="checkbox"/> Abbreviated (Complete section I, Page 5) <input type="checkbox"/> Additional Information <input type="checkbox"/> Third Party	<b>Meeting</b> <input type="checkbox"/> Pre-510(K) Meeting <input type="checkbox"/> Pre-IDE Meeting <input type="checkbox"/> Pre-PMA Meeting <input type="checkbox"/> Pre-PDP Meeting <input type="checkbox"/> Day 100 Meeting <input type="checkbox"/> Agreement Meeting <input type="checkbox"/> Determination Meeting <input type="checkbox"/> Other (specify):
<b>IDE</b> <input type="checkbox"/> Original Submission <input type="checkbox"/> Amendment <input type="checkbox"/> Supplement	<b>Humanitarian Device Exemption (HDE)</b> <input type="checkbox"/> Original Submission <input type="checkbox"/> Amendment <input type="checkbox"/> Supplement <input type="checkbox"/> Report <input type="checkbox"/> Report Amendment	<b>Class II Exemption Petition</b> <input type="checkbox"/> Original Submission <input type="checkbox"/> Additional Information	<b>Evaluation of Automatic Class III Designation (De Novo)</b> <input type="checkbox"/> Original Submission <input type="checkbox"/> Additional Information	<b>Other Submission</b> <input type="checkbox"/> 513(g) <input type="checkbox"/> Other (describe submission):

Have you used or cited Standards in your submission?  Yes  No (If Yes, please complete Section I, Page 5)

**SECTION B SUBMITTER, APPLICANT OR SPONSOR**

Company / Institution Name LG Electronics	Establishment Registration Number (if known)		
Division Name (if applicable) N/A	Phone Number (including area code) ( ) +82-2-526-4370		
Street Address 38 Baumoe-ro, Seocho-gu	FAX Number (including area code) ( )		
City Seoul	State / Province	ZIP/Postal Code	Country Korea
Contact Name Chang Hee Lee			
Contact Title Chief Research Engineer		Contact E-mail Address chp.lee@lge.com	

**SECTION C APPLICATION CORRESPONDENT (e.g., consultant, if different from above)**

Company / Institution Name Emergo Group	Establishment Registration Number (if known)		
Division Name (if applicable)	Phone Number (including area code) ( 512 ) 327-9997		
Street Address 816 Congress Avenue, Suite 1400	FAX Number (including area code) ( 512 ) 327-9998		
City Austin	State / Province TX	ZIP/Postal Code 78701	Country USA
Contact Name Diane Sudduth			
Contact Title Senior Regulatory Consultant		Contact E-mail Address project.management@emergogroup.com	



**SECTION D1**

**REASON FOR APPLICATION - PMA, PDP, OR HDE**

<input type="checkbox"/> Withdrawal <input type="checkbox"/> Additional or Expanded Indications <input type="checkbox"/> Request for Extension <input type="checkbox"/> Post-approval Study Protocol <input type="checkbox"/> Request for Applicant Hold <input type="checkbox"/> Request for Removal of Applicant Hold <input type="checkbox"/> Request to Remove or Add Manufacturing Site	<input type="checkbox"/> Change in design, component, or specification: <input type="checkbox"/> Software / Hardware <input type="checkbox"/> Color Additive <input type="checkbox"/> Material <input type="checkbox"/> Specifications <input type="checkbox"/> Other ( <i>specify below</i> )	<input type="checkbox"/> Location change: <input type="checkbox"/> Manufacturer <input type="checkbox"/> Sterilizer <input type="checkbox"/> Packager
<input type="checkbox"/> Process change: <input type="checkbox"/> Manufacturing <input type="checkbox"/> Sterilization <input type="checkbox"/> Packaging <input type="checkbox"/> Other ( <i>specify below</i> )	<input type="checkbox"/> Labeling change: <input type="checkbox"/> Indications <input type="checkbox"/> Instructions <input type="checkbox"/> Performance <input type="checkbox"/> Shelf Life <input type="checkbox"/> Trade Name <input type="checkbox"/> Other ( <i>specify below</i> )	<input type="checkbox"/> Report Submission: <input type="checkbox"/> Annual or Periodic <input type="checkbox"/> Post-approval Study <input type="checkbox"/> Adverse Reaction <input type="checkbox"/> Device Defect <input type="checkbox"/> Amendment
<input type="checkbox"/> Response to FDA correspondence:		<input type="checkbox"/> Change in Ownership <input type="checkbox"/> Change in Correspondent <input type="checkbox"/> Change of Applicant Address
<input type="checkbox"/> Other Reason ( <i>specify</i> ):		

**SECTION D2**

**REASON FOR APPLICATION - IDE**

<input type="checkbox"/> New Device <input type="checkbox"/> New Indication <input type="checkbox"/> Addition of Institution <input type="checkbox"/> Expansion / Extension of Study <input type="checkbox"/> IRB Certification <input type="checkbox"/> Termination of Study <input type="checkbox"/> Withdrawal of Application <input type="checkbox"/> Unanticipated Adverse Effect <input type="checkbox"/> Notification of Emergency Use <input type="checkbox"/> Compassionate Use Request <input type="checkbox"/> Treatment IDE <input type="checkbox"/> Continued Access	<input type="checkbox"/> Change in: <input type="checkbox"/> Correspondent / Applicant <input type="checkbox"/> Design / Device <input type="checkbox"/> Informed Consent <input type="checkbox"/> Manufacturer <input type="checkbox"/> Manufacturing Process <input type="checkbox"/> Protocol - Feasibility <input type="checkbox"/> Protocol - Other <input type="checkbox"/> Sponsor  <input type="checkbox"/> Report submission: <input type="checkbox"/> Current Investigator <input type="checkbox"/> Annual Progress Report <input type="checkbox"/> Site Waiver Report <input type="checkbox"/> Final	<input type="checkbox"/> Repose to FDA Letter Concerning: <input type="checkbox"/> Conditional Approval <input type="checkbox"/> Deemed Approved <input type="checkbox"/> Deficient Final Report <input type="checkbox"/> Deficient Progress Report <input type="checkbox"/> Deficient Investigator Report <input type="checkbox"/> Disapproval <input type="checkbox"/> Request Extension of Time to Respond to FDA <input type="checkbox"/> Request Meeting <input type="checkbox"/> Request Hearing
<input type="checkbox"/> Other Reason ( <i>specify</i> ):		

**SECTION D3**

**REASON FOR SUBMISSION - 510(k)**

<input checked="" type="checkbox"/> New Device	<input type="checkbox"/> Additional or Expanded Indications	<input type="checkbox"/> Change in Technology
<input type="checkbox"/> Other Reason ( <i>specify</i> ):		

**SECTION E ADDITIONAL INFORMATION ON 510(K) SUBMISSIONS**

Product codes of devices to which substantial equivalence is claimed								Summary of, or statement concerning, safety and effectiveness information <input checked="" type="checkbox"/> 510 (k) summary attached <input type="checkbox"/> 510 (k) statement
1	DRG	2		3		4		
5		6		7		8		

Information on devices to which substantial equivalence is claimed (if known)

	510(k) Number		Trade or Proprietary or Model Name		Manufacturer
1	K103276	1	Intel Health Guide PHS6000	1	INTEL CORPORATION DIGITAL HEALTH GROUP
2		2		2	
3		3		3	
4		4		4	
5		5		5	
6		6		6	

**SECTION F PRODUCT INFORMATION - APPLICATION TO ALL APPLICATIONS**

Common or usual name or classification  
 Remote Patient Monitoring System

	Trade or Proprietary or Model Name for This Device		Model Number
1	LG SmartHealth	1	TBD
2		2	
3		3	
4		4	
5		5	

FDA document numbers of all prior related submissions (regardless of outcome)

1	2	3	4	5	6
N/A					
7	8	9	10	11	12

Data Included in Submission  
 Laboratory Testing     Animal Trials     Human Trials

**SECTION G PRODUCT CLASSIFICATION - APPLICATION TO ALL APPLICATIONS**

Product Code DRG	C.F.R. Section (if applicable) 870.2910	Device Class <input type="checkbox"/> Class I <input checked="" type="checkbox"/> Class II <input type="checkbox"/> Class III <input type="checkbox"/> Unclassified
Classification Panel Cardiovascular		

Indications (from labeling)  
 The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home . Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers(or nurse). Patients can also engage in video conferences with caregivers(or nurse). The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required. The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

Note: Submission of this information does not affect the need to submit a 2891 or 2891a Device Establishment Registration form.

FDA Document Number (if known)

**SECTION H MANUFACTURING / PACKAGING / STERILIZATION SITES RELATING TO A SUBMISSION**

<input checked="" type="checkbox"/> Original <input type="checkbox"/> Add <input type="checkbox"/> Delete		Facility Establishment Identifier (FEI) Number TBD	<input checked="" type="checkbox"/> Manufacturer <input type="checkbox"/> Contract Sterilizer <input type="checkbox"/> Contract Manufacturer <input type="checkbox"/> Repackager / Relabeler	
Company / Institution Name LG Electronics		Establishment Registration Number TBD		
Division Name (if applicable) N/A		Phone Number (including area code) (   ) +82-2-526-4370		
Street Address 38 Baumoe-ro, Seocho-gu		FAX Number (including area code) (   )		
City Seoul		State / Province	ZIP/Postal Code	Country Korea
Contact Name Chang Hee Lee		Contact Title Chief Research Engineer		Contact E-mail Address chl@lge.com

<input type="checkbox"/> Original <input type="checkbox"/> Add <input type="checkbox"/> Delete		Facility Establishment Identifier (FEI) Number	<input type="checkbox"/> Manufacturer <input type="checkbox"/> Contract Sterilizer <input type="checkbox"/> Contract Manufacturer <input type="checkbox"/> Repackager / Relabeler	
Company / Institution Name		Establishment Registration Number		
Division Name (if applicable)		Phone Number (including area code) (   )		
Street Address		FAX Number (including area code) (   )		
City		State / Province	ZIP/Postal Code	Country
Contact Name		Contact Title		Contact E-mail Address

<input type="checkbox"/> Original <input type="checkbox"/> Add <input type="checkbox"/> Delete		Facility Establishment Identifier (FEI) Number	<input type="checkbox"/> Manufacturer <input type="checkbox"/> Contract Sterilizer <input type="checkbox"/> Contract Manufacturer <input type="checkbox"/> Repackager / Relabeler	
Company / Institution Name		Establishment Registration Number		
Division Name (if applicable)		Phone Number (including area code) (   )		
Street Address		FAX Number (including area code) (   )		
City		State / Province	ZIP/Postal Code	Country
Contact Name		Contact Title		Contact E-mail Address

## SECTION I

## UTILIZATION OF STANDARDS

Note: Complete this section if your application or submission cites standards or includes a "Declaration of Conformity to a Recognized Standard" statement.

	Standards No.	Standards Organization	Standards Title	Version	Date
1	IEC 62304	International Electrotechnical Commission (IEC)	Medical device software - software life cycle processes. (Software/Informatics)	2006	8/20/2012
2	ISO 14971	International Standards Organization	Medical devices - Application for risk management to medical devices	2007	3/1/2007
3					
4					
5					
6					
7					

Please include any additional standards to be cited on a separate page.

Public reporting burden for this collection of information is estimated to average 0.5 hour per response, including the time for reviewing instructions, searching existing data sources, gathering and maintaining the data needed, and completing reviewing the collection of information. Send comments regarding this burden estimate or any other aspect of this collection of information, including suggestions for reducing this burden to:

Food and Drug Administration  
CDRH (HFZ-342)  
9200 Corporate Blvd.  
Rockville, MD 20850

*An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB control*



### Section 3 – 510(k) Cover Letter

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
Document Mail Center – WO66-G609  
10903 New Hampshire Avenue  
Silver Spring, Maryland 20993-0002

Dear Sir or Madam:

This document contains the information for LG Electronics, LG SmartHealth submission for market clearance. In accordance with Section 510(k) of the Federal Food and Drug Cosmetic Act as amended, and in conformance with Title 21 CFR, Part 807, this Pre-market Notification is being submitted at least ninety (90) days prior to the date when LG Electronics proposes to introduce its LG SmartHealth into interstate commerce for commercial distribution. The CD provided with the submission is the official electronic copy of the submission; the eCopy is an exact duplicate of the paper copy. Following contains the regulatory information for the contents of this submission supporting the device's market clearance.

#### Administrative Information

Date of Submission: December 26, 2013

Submission is Completed By: Emergo Group  
816 Congress Avenue  
Suite 1400  
Austin, TX 78701

Application Correspondent: Dr. Diane Sudduth  
Senior Consultant QA, Emergo Group  
Email: [project.management@emergogroup.com](mailto:project.management@emergogroup.com)  
Cellphone number: 561.305.5075  
Home office number: 512.327.9997

Submission Sponsor: LG Electronics  
38 Baumoe-ro, Seocho-gu  
Seoul, 137-724  
Korea  
Chang Hee Lee  
Chief Research Engineer  
Office number: +82-2-526-4370

#### Device Identification

Type of 510(k) Submission: Traditional

Trade or Proprietary Name: LG SmartHealth

Common or Usual Name: Remote Patient Monitoring System

Regulation Classification: Transmitters and Receivers, Physiological Signal, Radiofrequency



Product Code: 870.2910 (Product Code DRG)  
 Class of Device: Class II  
 Panel: Cardiovascular  
 Reason for Submission: New device  
 Prior Related Submissions: No prior submissions for the device

**Design and Use of the Device**

**Table 3A – Principle Factors**

Question	Yes	No
Is the device intended for prescription use (21 CFR 801 Subpart D)?	X	
Is the device intended for over-the-counter use (21 CFR 807 Subpart C)?		X
Does the device contain components derived from a tissue or other biologic source?		X
Is the device provided sterile?		X
Is the device intended for single use?		X
Is the device a reprocessed single use device?		X
If yes, does this device type require reprocessed validation data?		N/A
Does the device contain a drug?		X
Does the device contain a biologic?		X
Does the device use software?	X	
Does the submission include clinical information?		X
Is the device implanted?		X

All information necessary for a substantial equivalence determination is included herein. Should you require any additional data in order to reach a determination of substantial equivalence, please do not hesitate to contact me at 561.305.5075 or by email at [project.management@emergogroup.com](mailto:project.management@emergogroup.com)

Sincerely,

Diane Sudduth  
 Senior Consultant, RA/QA



## Indications for Use

510(k) Number (if known)

Device Name

LG SmartHealth

Indications for Use (Describe)

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

**PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON A SEPARATE PAGE IF NEEDED.**

**FOR FDA USE ONLY**

Concurrence of Center for Devices and Radiological Health (CDRH) (Signature)

---

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
*PRASStaff@fda.hhs.gov*

*“An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number.”*

**510(k) Summary**  
**for**  
**LG SmartHealth**

**1. Submission Sponsor**

LG Electronics  
38 Baumoe-ro, Seocho-gu  
Seoul, 137-724  
Korea  
Phone: +82-2-526-4527  
Fax: +82-2-526-4370  
Contact: Chang Hee Lee, Chief Research Engineer

**2. Submission Correspondent**

Emergo Group  
816 Congress Avenue, Suite 1400  
Austin, TX 78701  
Cell Phone: (561) 305-5075  
Office Phone: (512) 327.9997  
Fax: (512) 327.9998  
Contact: Dr. Diane Sudduth, Senior Consultant, QA  
Email: [project.management@emergogroup.com](mailto:project.management@emergogroup.com)

**3. Date Prepared**

December 26, 2013

**4. Device Identification**

Trade/Proprietary Name: LG SmartHealth  
Common/Usual Name: Remote Patient Monitoring System  
Classification Name: Transmitters and Receivers, Physiological Signal,  
Radiofrequency  
Classification Regulation: 870.2910  
Product Code: Product Code DRG  
Device Class: Class II  
Classification Panel: Cardiovascular

**5. Legally Marketed Predicate Device(s)**

Intel Health Guide Express (K103276)

## 6. Device Description

The LG Patient Monitoring System is software planned to operate on the Android operating system (OS) that can be loaded as an application or “app” on a commercially available smartphone. LG intended the software to work on the Android OS, as well as to operate on smartphones as well. The scalability and differences between many smartphones today are not too great to allow this flexibility and option.

The LG Patient Monitoring system connects to commercially available wireless medical devices that are commonly used by patients in a home-care setting. These “sensor devices” are FDA cleared or FDA registered (Exempt) devices that can communicate with the LG Patient Monitoring system software loaded on the smartphone using Bluetooth connectivity. To avoid wireless interference and confusion on the part of the patient, serial readings are performed (i.e. only one Bluetooth sensor used at a time) and MAC address filtering is used for the various medical devices. The LG Patient Monitoring system plans to support the following sensor available medical devices, some initially, and others as the technology becomes available:

- Glucose meters or glucometers
- Blood pressure cuffs
- Weight scales
- Body fat readers
- Activity monitors

On each screen of the LG Patient Monitoring software a help button is present, which when pressed provides step-by-step based help to guide a patient through an interaction.

The medical device takes a reading depending on the frequency previously established and the LG Patient Monitoring software fetches the information for the reading automatically. The LG Patient Monitoring software stores and displays the information on the smartphone and transmits the information to the server. The LG Patient Monitoring system encrypts the information in preparation for transmit using SHA256 bit encryption, complying with HIPAA requirements.

## 7. Indication for Use Statement

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing

its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

## 8. Substantial Equivalence Discussion

The comparison chart below provides evidence to facilitate the substantial equivalence determination between LG SmartHealth to the predicate device (K103276) with respect to intended use, technological characteristics and principles of operation.

**Table 5A – Comparison of Characteristics**

Features and Specifications	Intel® Health Guide Express (K103276)	LG SmartHealth (Subject Device)
<b>Regulatory Information</b>		
<b>Intended Use/ Indications for Use</b>	<p>The Intel® Health Guide Express is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers. Patients can also engage in video conferences with caregivers and answer the caregivers' questions by participating in surveys.</p> <p>The Intel® Health Care Management Suite allows the caregiver to review patient data and initiate video conferencing with patients, or select and send educational and motivational content to patients.</p> <p>The Intel® Health Guide Express is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The Intel® Health Guide Express is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>	<p>The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).</p> <p>The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>
<b>OTC and/or Rx</b>	Rx	Same

User Characteristics		
<b>Intended User</b>	Patient in Home Setting	Same
Device		
<b>Components</b>	<ul style="list-style-type: none"> <li>Web based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>	<ul style="list-style-type: none"> <li>Smart phone app based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>
<b>User Interface Device</b>	Computer Device (i.e. desktop, laptop)	Android Device(i.e. Smart phone, tablet)
<b>Connectivity to Device</b>	Bluetooth or USB wired	Bluetooth
<b>Ability to Update Device Settings</b>	Not available	Automatic deploy
<b>Connectivity to Web-based Services</b>	Login Secured by SSL Encryption	Same
<b>Operating System</b>	Microsoft Windows 7 (32-bit versions)	Android 2.3.3 or higher
<b>Supported Device Types</b>	Medical devices designed for home use: Blood glucose meter Weight scale Blood pressure Pulse Oximeter FEV/PEF	Medical devices designed for home use: Blood glucose meter Weight scale Blood pressure Temperature Activity Monitor
<b>Software Delivery</b>	Web download for the uploader only	Same
Software Features		
<b>Patient Management</b>	Manage patient accounts, filter/sort/search patients, print reports.	The server connected to the mobile device manages patient accounts, filter/sort/search patients
<b>Implementation method of collecting data from sensors</b>	Bluetooth or USB wired	Bluetooth
<b>Communication frequency</b>	Bluetooth: 2.402 to 2.480 GHz	Same
<b>Max data throughput under worst case conditions</b>	Multiply read data can be stored in the medical device and they are resent to the server after recovery of network condition	Same

<b>Use of thresholds/algorithms for determining how thresholds are set and changed</b>	Thresholds set by healthcare professional in server software	Same
<b>Communication method with server</b>	Connected through internet	Same

The LG SmartHealth is a medical device and has similar indications for use statement as the predicate device. The differences in the indications statement between LG SmartHealth and Intel Health Guide Express do not alter the intended use – both devices have essentially the same intended use.

The device also has similar technological characteristics as the predicate device. Both devices have the same data collection software functionality, communication method of patient device with central server, types of sensors which can be interfaced to the patient device, implementation method of collecting data from sensors, sensor software, connectivity, and communication protocol. Since the comparison of the descriptive characteristics of the proposed and predicate devices may not be sufficiently precise to assure equivalence, performance data are provided. The results of the performance testing demonstrate substantial equivalence.

#### **9. Non-Clinical Performance Data**

The device’s software development, verification, and validation have been carried out in accordance with FDA guidelines. The software was tested against the established Software Design Specifications for each of the test plans to assure the device performs as intended. The device Hazard analysis was completed and risk control implemented to mitigate identified hazards. The testing results support that all the software specifications have met the acceptance criteria of each module and interaction of processes. The LG SmartHealth device passed all testing and supports the claims of substantial equivalence and safe operation.

Validation activities included a usability study of the LG SmartHealth under actual use. The study demonstrated:

- Comprehension of the study nurses and participants with the LG SmartHealth,
- Appropriate human factors related to the LG SmartHealth, and
- Ease of use of the LG SmartHealth.

#### **10. Clinical Performance Data**

There was no clinical testing required to support the medical device as the indications for use is equivalent to the predicate device. These types of devices, including the predicate devices, have been on the market for many years with a proven safety and efficacy for the use of the device. The verification and validation testing of the device’s software was found to be acceptable and supports the claims of substantial equivalence.

## **11. Statement of Substantial Equivalence**

The LG SmartHealth has the same or similar intended use as the predicate device and that any technological differences between the LG SmartHealth software and the predicate device do not raise any questions regarding LG SmartHealth ('SmartHealth')'s safety and effectiveness.

The information provided in this submission supports the substantial equivalence to the predicate device and that the system is safe and effective for the users/operators.



## Section 6 – Truthful and Accurate Statement

**Device Name:** LG SmartHealth

As required by 21 CFR 807.87(k), the Truthful and Accurate Statement on company letterhead by a responsible person at LG Electronics is attached in Exhibit 6A.

## Section 7 - Class III Certification and Summary

**Device Name:** LG SmartHealth

**Conclusion:** Not applicable; Class II device

## Section 8 – Financial Certification or Disclosure Statement

**Device Name:** LG SmartHealth

**Conclusion:** Not applicable as clinical trials were not performed with the device.

## Section 9 - Declarations of Conformity and Summary Reports

**Device Name:** LG SmartHealth

The submission is a Traditional 510(k) Submission with no device-specific guidance documents or any declaration of conformity to recognized standards specified by the device regulation. No specific performance standards have been established under Section 514 of the Food, Drug and Cosmetic Act. However, certain voluntary standards and guidance have been used in the design and testing of this device.

### **Risk Management Declaration of Conformity:**

- ISO 14971: 2007, Medical devices -- Application of risk management to medical devices

For the voluntary standards listed above, LG Electronics certifies that the results from the requisite testing performed meet specified acceptance criteria and the LG SmartHealth software conforms to applicable and relevant portions of each respective standard.

### **Software Design and Development Declaration of Conformity:**

The LG SmartHealth software complies with the applicable voluntary standards that include IEC 62304:2006/C11:2008 for Medical Device Software – Software Lifecycle processes and FDA General Principles of Software Validation; Final Guidance for Industry and FDA Staff, dated January 11, 2002. The Software Verification and Validation Summary reports are included in Section 16 – Software section of the submission.

- IEC 62304:2006/C11:2008 – Medical Device Software – Software Lifecycle processes

The Standard Data Report, FDA Form-3654 is attached as Exhibit 9-A and Exhibit 9-B in order to provide more detailed conformance information for the recognized voluntary standards.

**Table 9-A – List of Voluntary Standards**

<b>Standard</b>	<b>FDA Recognition Number</b>	<b>Exhibit</b>
IEC 62304:2006/C11:2008	13-32	Exhibit 9-A
ISO 14971:2007	5-40	Exhibit 9-B

## Section 10 – Executive Summary

**Device Name:** LG SmartHealth

### **Summary of Device:**

The LG Patient Monitoring System is software planned to operate on the Android operating system (OS) that can be loaded as an application or “app” on a commercially available smartphone. LG intended the software to work on the Android OS, as well as to operate on smartphones as well. The scalability and differences between many smartphones today are not too great to allow this flexibility and option.

The LG Patient Monitoring system connects to commercially available wireless medical devices that are commonly used by patients in a home-care setting. These “sensor devices” are FDA cleared or FDA registered (Exempt) devices that can communicate with the LG Patient Monitoring system software loaded on the smartphone using Bluetooth connectivity. To avoid wireless interference and confusion on the part of the patient, serial readings are performed (i.e. only one Bluetooth sensor used at a time) and MAC address filtering is used for the various medical devices. The LG Patient Monitoring system plans to support the following sensor available medical devices, some initially, and others as the technology becomes available:

- Glucose meters or glucometers
- Blood pressure cuffs
- Weight scales
- Body fat readers
- Activity monitors

On each screen of the LG Patient Monitoring software a help button is present, which when pressed provides step-by-step based help to guide a patient through an interaction.

The medical device takes a reading depending on the frequency previously established and the LG Patient Monitoring software fetches the information for the reading automatically. The LG Patient Monitoring software stores and displays the information on the smartphone and transmits the information to the server. The LG Patient Monitoring system encrypts the information in preparation for transmit using SHA256 bit encryption, complying with HIPAA requirements.

### **Indications for Use:**

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

**Device Comparison Summary:**

**Table 10A – Device Comparison Table**

Features and Specifications	Intel® Health Guide Express (K103276)	LG SmartHealth (Subject Device)
<b>Regulatory Information</b>		
<b>Intended Use/ Indications for Use</b>	<p>The Intel® Health Guide Express is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers. Patients can also engage in video conferences with caregivers and answer the caregivers' questions by participating in surveys.</p> <p>The Intel® Health Care Management Suite allows the caregiver to review patient data and initiate video conferencing with patients, or select and send educational and motivational content to patients.</p> <p>The Intel® Health Guide Express is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The Intel® Health Guide Express is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>	<p>The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse).</p> <p>Patients can also engage in video conferences with caregivers (or nurse).</p> <p>The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>
<b>OTC and/or Rx</b>	Rx	Same

<b>User Characteristics</b>		
<b>Intended User</b>	Patient in Home Setting	Same
<b>Device</b>		
<b>Components</b>	<ul style="list-style-type: none"> <li>Web based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>	<ul style="list-style-type: none"> <li>Smart phone app based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>
<b>User Interface Device</b>	Computer Device (i.e. desktop, laptop)	Android Device(i.e. Smart phone, tablet)
<b>Connectivity to Device</b>	Bluetooth or wired USB	Bluetooth
<b>Ability to Update Device Settings</b>	Not available	Automatic deploy
<b>Connectivity to Web-based Services</b>	Login Secured by SSL Encryption	Same
<b>Operating System</b>	Microsoft Windows 7 (32-bit versions)	Android 2.3.3 or higher
<b>Supported Device Types</b>	Medical devices designed for home use: Blood glucose meter Weight scale Blood pressure Pulse Oximeter FEV/PEF	Medical devices designed for home use: Blood glucose meter Weight scale Blood pressure Temperature Activity Monitor
<b>Software Delivery</b>	Web download for the uploader only	Same
<b>Software Features</b>		
<b>Patient Management</b>	Manage patient accounts, filter/sort/search patients, print reports.	The server connected to the mobile device manages patient accounts, filter/sort/search patients
<b>Implementation method of collecting data from sensors</b>	Bluetooth or USB wired	Bluetooth
<b>Communication frequency</b>	Bluetooth: 2.402 to 2.480 GHz	Same

<b>Max data throughput under worst case conditions</b>	Multiply read data can be stored in the medical device and they are resent to the server after recovery of network condition	Same
<b>Use of thresholds/algorithms for determining how thresholds are set and changed</b>	Thresholds set by healthcare professional in server software	Same
<b>Communication method with server</b>	Connected through internet	Same

**Performance Testing Summary:**

The LG Integration/System Test Plan, included in Section 16, Exhibit 16-F of this submission, describes the approach used for verification and validation of the LG SmartHealth Application. Full verification and validation testing of the LG SmartHealth software was performed in accordance with the FDA Guidance Document 'General Principles of Software Validation (2002)'. System Level Integration testing were conducted based on the requirements identified in the LG Software Requirements Documents (Exhibits 16-A through Exhibit 16-G), included in Section 16 of this 510(k) Submission.

A risk based approach to the software integrity level is followed to determine the level of verification that will be performed on each module and includes:

- Ad-hoc Testing
- Formal Testing
- Code Reviews
- System Level Integration Testing

Each module underwent System Level Integration Testing and/or Code Review.

For a summary and details of the verification and validation conducted on the LG SmartHealth, please refer to Section 16 of this 510(k) submission.




## Section 11 – Device Description

**Device Name:** LG SmartHealth

### Description of the Device:

The LG Patient Monitoring System Software (“LG SmartHealth”) is software planned to operate on the Android operating system (OS) that can be loaded as an application or “app” on a commercially available smartphone. LG Electronics intended the software to work on the Android OS, as well as to operate on smartphones as well. The scalability and differences between many smartphones today are not too great to allow this flexibility and option.

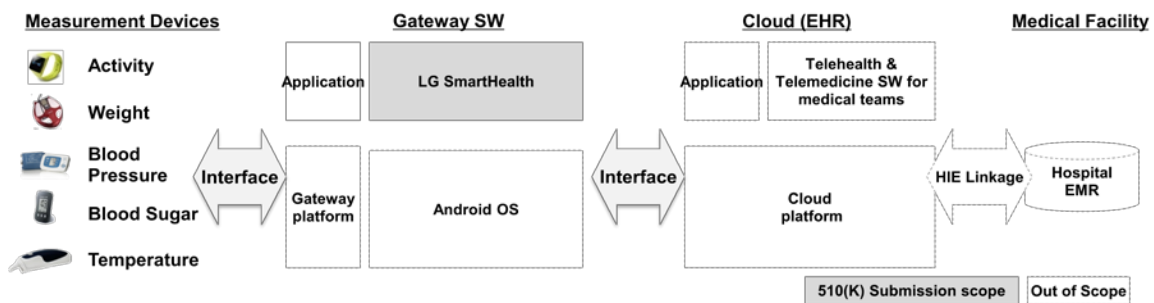

**Figure 1 LG SmartHealth Operation Structure**

LG SmartHealth connects to commercially available wireless medical devices that are commonly used by patients in a home-care setting. These “sensor devices” are FDA cleared or FDA registered (Exempt) devices that can communicate with the LG SmartHealth loaded on the smartphone Bluetooth connectivity. To avoid wireless interference and confusion on the part of the patient, serial readings are performed (i.e. only one Bluetooth sensor used at a time) and MAC address filtering is used for the various medical devices. LG SmartHealth plans to support the following sensor available medical devices, some initially, and then others as the technology become available:

- Glucose meters or glucometers
- Blood pressure cuffs
- Weight scales
- Body fat readers
- Activity monitors

Pressing the measurement guide button in LG SmartHealth provides a step-by-step interactive help guide to the user.

The medical device takes a reading depending on the frequency previously established and the LG Patient Monitoring software fetches the information for the reading automatically. The LG Patient Monitoring software stores and displays the information on the smartphone and transmits the information to the server. The LG Patient Monitoring System encrypts the password using SHA256 algorithm (b)(4)Trade Secret Process-Product Specs [REDACTED] complying with HIPAA requirements.

These medical devices that are sensor, monitoring, and recording devices will all be FDA cleared or FDA Class I, 510(k) Exempt products that are recommended by LG for use with their Patient Monitoring System. A list of the acceptable medical devices that can be used with the LG Patient Monitoring System will be provided in the User’s Manual.

Device Type	Vendor	Model	Connection	k number
Blood pressure monitor	A&D Medical	UA-767PBT	Bluetooth wireless device	K043217
	FORA	P 20 Blood Pressure Monitor IU-RIGHT TD-3 132 Blood Pressure Monitor	Bluetooth wireless device	K092106
	FORA	D30F: Fora Comfort 2 in 1 blood glucose plus blood pressure monitoring system	Bluetooth wireless device	K080014
Glucose monitor	Bayer HealthCare	Contour	Wired device	K121190
	LifeScan, a Johnson & Johnson Company	OneTouch Ultra Mini	USB wired device	K061118
	CareSens	CareSens N LINK Blood Glucose Monitoring System	Bluetooth wireless device	K130696
Weight scale	Biospace	IH-U070B H20	Bluetooth wireless device	K123228
	Tanita Corp. of America	BC590 BT	Bluetooth wireless device	K014009
Temperature Meters	Fora	IR20b Typel/II	Bluetooth wireless device	K090395
Activity Tracker	LG	Life Gram	Bluetooth Wireless device	Exempt

**Naming Convention:**

Please note that LG SmartHealth may also be referred to as “SmartHealth”, “SmartHealth app”, or “LG SmartHealth” in many of the attached documents.

**Indications for Use Statement:**

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

**Mode of Operation and Mechanism of Action:**

(b) (4)



(b) (4)



**Proposed Conditions of Use:**

**Software & Hardware Requirement**

(b)(4) Software



**Labeling:**

The LG SmartHealth is a Smartphone app based software application which will be accessible via a URL address. The user manual will be provided in a downloadable PDF format which can be viewed, saved, or printed. The user manual provides instructions on how to access and operate the software.

## Section 12 – Substantial Equivalence Discussion

**Device Name:** LG SmartHealth

The LG SmartHealth is equivalent to the Intel Health Guide Express, cleared in K103276, on 8 February 2011.

The LG SmartHealth is equivalent to the predicate in terms of data collection software functionality, operating system for the patient device, communication method of patient device with central server, types of sensors, which can be interfaced to the patient device, implementation method of collecting data from sensors, sensor software, connectivity, communication protocol and display method.

A copy of the 510(k) summary for the Intel Health Guide Express is included in Exhibit 12-A as the predicate device for this submission.

### **Comparison to Legally Marketed Device:**

**Table 1** on the following page provides a comparison between the subject system and the predicate device.

**Table 12A – Device Comparison Chart: Similarities and Differences**

Features	LG SmartHealth (Pending)	Intel® Health Guide Express (K103276)	Significant Difference
<b>Regulatory Information</b>			
<b>Intended Use</b>	<p>The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).</p> <p>The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>	<p>The Intel® Health Guide Express is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers. Patients can also engage in video conferences with caregivers and answer the caregivers' questions by participating in surveys.</p> <p>The Intel® Health Care Management Suite allows the caregiver to review patient data and initiate video conferencing with patients, or select and send educational and motivational content to patients.</p> <p>The Intel® Health Guide Express is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The Intel® Health Guide Express is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>	<p>Similar; LG SmartHealth’s Indications Statement is a subset of Intel Health Guide Express in that the predicate extends the indications to allow the caregiver to review patient data and initiate video conferencing, or select and send educational and motivational content to patients.</p>



Features	LG SmartHealth (Pending)	Intel® Health Guide Express (K103276)	Significant Difference
<b>OTC and/or Rx</b>	Rx	Rx	No significant difference
<b>User Characteristics</b>			
<b>Intended User</b>	Patient in Home Setting	Patient in Home Setting	No significant difference
<b>Device</b>			
<b>Components</b>	<ul style="list-style-type: none"> <li>Smartphone app based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>	<ul style="list-style-type: none"> <li>Web based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>	LG SmartHealth is an Android based app
<b>User Interface Device</b>	Android Device(i.e. Smart phone)	Computer Device (i.e. desktop, laptop)	Both of them are off the shelf devices
<b>Connectivity to Device</b>	Bluetooth Interface	Bluetooth Interface or wired USB	LG SmartHealth is a subset of Intel Health Guide Express
<b>Ability to Update Device Settings</b>	Automatic deploy	Automatic deploy	No significant difference
<b>Connectivity to Web-based Services</b>	Login Secured by SSL Encryption	Login Secured by SSL Encryption	No significant difference
<b>Operating System</b>	Android 2.3.3 or higher	Microsoft Windows 7 (32-bit versions)	LG Smartphone operates on different operating system

Features	LG SmartHealth (Pending)	Intel® Health Guide Express (K103276)	Significant Difference
<b>Supported Device Types</b>	Medical devices designed for home use: <ul style="list-style-type: none"> <li>• Blood Glucose Meters</li> <li>• Blood Pressure</li> <li>• Weight</li> <li>• Temperature</li> <li>• Activity tracker(N/A)</li> </ul>	Medical devices designed for home use: <ul style="list-style-type: none"> <li>• Blood Glucose Level</li> <li>• Blood Pressure</li> <li>• Weight</li> <li>• Oxygen Saturation</li> <li>• FEV/PEF</li> </ul>	LG SmartHealth supports a subset of the predicate BGM, blood pressure devices
<b>Software Delivery</b>	Web download for the uploader only	Web download for the uploader only	No significant difference
<b>Software Features</b>			
<b>Patient Management</b>	Manages patient accounts, filter/sort/search patients, print reports	Manage patient accounts, filter/sort/search patients, print reports.	No significant difference

The comparison chart above provides evidence to facilitate the substantial equivalence determination between LG SmartHealth and our chosen predicate, Intel Health Guide Express.

There is a direct correlation between the Indications for Use Statement/Intended Use of LG SmartHealth and Intel Health Guide Express. Both devices are software applications used to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers. They both have the same connectivity to devices, ability to update device settings, connectivity to web-based services, and software delivery.

The difference in the Indications Statement between LG SmartHealth and Intel Health Guide Express is that the Intel® Health Care Management Suite extends the use to allow for the caregiver to review patient data and initiate video conferencing with patients, or select and send educational and motivational content to patients. Both devices add a specific requirement that it should not be used as interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required. Additionally, both LG SmartHealth and Intel Health Guide Express are contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

The technological characteristics between the LG SmartHealth and Intel Health Guide Express are different, as the software operates on different hardware. Although both devices support blood glucose meters, blood pressure monitors and weight scales, the Intel Health Guide Express provides additional functionality in regard to compatibility to oximeters and FEV/PEF sensors. LG SmartHealth does not provide these functionalities. These differences are addressed through labeling and the compatibility with medical devices are evidenced by the results of the Performance testing.

All these facts provide evidence to facilitate the substantial equivalence determination between LG Smartphone and the predicate device, Intel Health Guide Express (K103276).

**Performance Testing Summary:**

Internal verification and validation testing confirms that product specifications are met which are equivalent in design and technological characteristics as the predicate device. The testing results support that the functional testing met for the acceptance of the device. The LG SmartHealth passed all testing and supports the claims of substantial equivalence to the predicate device.

**Table 12B – Performance Summary**

<b>Information or Testing Summarized</b>	<b>Section Located</b>
Proposed Labeling of Device	Section 13
Software	Section 16

**Conclusions:**

In conclusion, LG Electronics believes that we have established substantial equivalence of the subject LG SmartHealth to the predicate. No new issues of safety or effectiveness are introduced by using this device. The basis for our conclusion is reached through successful review of product design specifications and testing results compared to the predicate device. This consideration included review of the requirements for substantial equivalence delineated in the June 30, 1986, Center for Devices and Radiological Health (CDRH) publication entitled *Guidance on the Center for Devices and Radiological Health's Premarket Notification Review Program*, which includes a flow chart entitled 510(k) Substantial Equivalence Decision Making Process (Detailed).

## Section 13 – Proposed Labeling

**Device Name:** LG SmartHealth

**Labeling:**

The LG SmartHealth is a Smartphone app based software application which will be accessible via a URL address. The user manual will be provided in a downloadable PDF format which can be viewed, saved, or printed. The user manual provides instructions on how to access and operate the software.

Copies of the following draft labeling which describes the device in detail and provides user instructions is provided in Exhibit 13A.

Promotional labeling for the LG SmartHealth has not yet been developed. Claims for this device will be consistent with the draft user manual and currently marketed substantially equivalent devices (descriptions and labeling provided later in the “Substantial Equivalence” section of this notification).

## Section 14 – Sterilization and Shelf Life

**Device Name:** LG SmartHealth

**Conclusion:** This is not applicable to software products.

## Section 15 – Biocompatibility

**Device Name:** LG SmartHealth

**Conclusion:** This section is not applicable to software products.

## Section 16 – Software Description

**Device Name:** LG SmartHealth

**Device Description:**

The LG Patient Monitoring System Software (“LG SmartHealth”) is software planned to operate on the Android operating system (OS) that can be loaded as an application or “app” on a commercially available smartphone. LG Electronics intended the software to work on the Android OS, as well as to operate on smartphones as well. The scalability and differences between many smartphones today are not too great to allow this flexibility and option.

LG SmartHealth connects to commercially available wireless medical devices that are commonly used by patients in a home-care setting. These “sensor devices” are FDA cleared or FDA registered (Exempt) devices that can communicate with the LG SmartHealth loaded on the smartphone Bluetooth connectivity. To avoid wireless interference and confusion on the part of the patient, serial readings are performed (i.e. only one Bluetooth sensor used at a time) and MAC address filtering is used for the various medical devices. LG SmartHealth plans to support the following sensor available medical devices, some initially, and then others as the technology become available:

- Glucose meters or glucometers
- Blood pressure cuffs
- Weight scales
- Body fat readers
- Activity monitors

When measurement guide button in LG SmartHealth is present, which when pressed provides step-by-step based help to guide a patient through an interaction.

The medical device takes a reading depending on the frequency previously established and the LG Patient Monitoring software fetches the information for the reading automatically. The LG Patient Monitoring software stores and displays the information on the smartphone and transmits the information to the server. The LG Patient Monitoring System encrypts the password using SHA256 algorithm (b) (4) [REDACTED], complying with HIPAA requirements.

These medical devices that are sensor, monitoring, and recording devices will all be FDA cleared or FDA Class I, 510(k) Exempt products that are recommended by LG for use with their Patient Monitoring System. A list of the acceptable medical devices that can be used with the LG Patient Monitoring System will be provided in the User’s Manual.



**16.1 Level of Concern:**

Based on the Risk Management File ([Exhibit 16-E](#)) and the FDA Guidance document, *Guidance for the Content of Premarket Submissions for Software Contained in Medical Devices*, issued on May 11, 2005, on levels of concern LG has determined that the software level of concern for the LG SmartHealth device is **Moderate**. The rationale for the decision on the software level of concern is shown below in the tables extracted from the Food and Drug Administration guidance document.

**Table 16-A – Major Level of Concern Rationale**

<b>If the answer to any one question below is Yes, the Level of Concern for the Software Device is likely to be Major.</b>		
<b>Question</b>	<b>Answer</b>	<b>Rationale</b>
1. Does the Software Device qualify as Blood Establishment Computer Software?	NO	The device is not a Blood Establishment Computer Software.
2. Is the Software Device intended to be used in combination with a drug or biologic?	NO	The device is not intended to be used in conjunction with a drug or biologic.
3. Is the Software Device an accessory to a medical device that has a Major Level of Concern?	NO	The device is not an accessory to a medical device that has a major level of concern.
4. Prior to mitigation of hazards, could a failure of the Software Device result in death or serious injury, either to a patient or to a user of the device? Examples of this include the following:		
a. Does the Software Device control a life supporting or life sustaining function?	NO	The device does not control a life sustaining or life supporting function.
b. Does the Software Device control the delivery of potentially harmful energy that could result in death or serious injury, such as radiation treatment systems, defibrillators, and ablation generators?	NO	The software does not control the delivery of potentially harmful energy that could result in a death or serious injury.
c. Does the Software Device control the delivery of treatment or therapy such that an error or malfunction could result in death or serious injury?	NO	The device does not control the delivery of treatment or therapy such that an error or malfunction could result in death or serious injury.
d. Does the Software Device provide diagnostic information that directly drives a decision regarding treatment or therapy, such that if misapplied it could result in serious injury or death?	NO	The software device processes (displays) digital information for the assessment of treatment only. This information does not directly mandate or control a decision regarding treatment or therapy as all digital information must be interpreted by a competent medical person together with any additional clinical information required for treatment.
e. Does the Software Device provide vital signs monitoring and alarms for potentially life threatening situations in which medical intervention is necessary?	NO	The device does not provide any vital signs monitoring and/or alarms.

**Table 16-B – Moderate Level of Concern Rationale**

<b>If the answer to any one question below is Yes, the Level of Concern for the Software Device is likely to be Moderate.</b>		
<b>Question</b>	<b>Answer</b>	<b>Rationale</b>
1. Is the Software Device an accessory to a medical device that has a Moderate Level of Concern?	YES	The device is not an accessory to a medical device that has a major level of concern.
2. Prior to mitigation of hazards, could a failure of the Software Device result in Minor Injury, either to a patient or to a user of the device?	NO	A failure of the software device would not result in Minor Injury, either to a patient or to a user of the device if mitigation of hazards were not implemented.
3. Could a malfunction of, or a latent design flaw in, the Software Device lead to an erroneous diagnosis or a delay in delivery of appropriate medical care that would likely lead to Minor Injury?	YES	A malfunction or a latent design flaw in the software device could lead to an erroneous diagnosis that would lead to a minor injury.

(b)(4) Software



(b)(4) Software



**16.2 Software Description:**

(b)(4) Software



(b)(4) Software



(b)(4) Software



**16.3 Software Development:**

(b)(4) Software





(b)(4) Software



(b)(4) Software



(b)(4) Software



**16.4 Software Requirements Specification:**

(b)(4) Software



(b)(4) Software



(b)(4) Software



(b)(4) Software



(b)(4) Software





**16.6 Software Design Specification:**

b(4)Trade Secret Process-Software Specs



**16.7 Device Hazard Analysis:**

(b) (4)



(b)(4) Testing



(b)(4) Testing



(b)(4) Testing



(b)(4) Testing



(b)(4) Testing



(b)(4) Testing





(b)(4) Testing



(b)(4) Testing



(b)(4) Testing



## **Section 17 – Electromagnetic Compatibility and Electrical Safety**

**Device Name:** LG SmartHealth

**Conclusion:** This section is not applicable to software products.

## Section 18 – Performance Testing - Bench

**Device Name:** LG SmartHealth

The evaluation of data transmission for connection and accuracy was completed during verification testing. The requirement for BT pairing connection time, linkage recognition, and measurement data accuracy is according to the Software Requirements Specification (SRS) submitted in Exhibit 16A-1 thru 16A-4 of this submission. A protocol was followed to test all device connection requirements including verifying that the results are transferred to LG SmartHealth on Android Device without distortion.

The result of this protocol was 100% Pass for all three (3) meters, three (3) BP monitors, three (3) weight scales, two (2) temperature monitors, and one (1) activity tracker. See [Exhibit 18A](#), section 5 for results of all device connection and data accuracy requirements.

**Conclusion:** For LG SmartHealth software version 1.2.8, all test cases passed. There were no test failures. LG SmartHealth software version 1.2.8 is considered acceptable for use.

## Section 19 – Performance Testing - Animal

**Device Name:** LG SmartHealth

**Conclusion:** This section is not applicable to software products.

## Section 20 – Performance Testing - Clinical

**Device Name:** LG SmartHealth

Not applicable. Clinical testing is not necessary to demonstrate substantial equivalence of this device as there are no new innovative aspects that have been introduced. These types of software programs and applications have been on the market for several years with no significant incidents of safety or efficacy for the predicate devices. The design, development and testing of the LG SmartHealth has not resulted in the need for any clinical trials.

The Food and Drug Administration Form 3674 Certificate of Compliance for Clinical Trials is attached to the submission as Exhibit 20A.

LG Electronics  
38, Baumoe-ro, Seocho-gu  
SEOUL  
137-724  
Republic Korea

12/11/2013

## Truthful and Accurate Statement

[As required by 21 CFR 807.87(k)]

I certify that, in my capacity as Vice President of LG Electronics, I believe to the best of my knowledge, that all data and information submitted in the premarket notification are truthful and accurate and that no material fact has been omitted.

(b) (6)

Signature

(b) (6)

Typed Name

12/11/2013

Date

\*(Premarket Notification [510(k)] Number)

\*For a new submission, leave the 510(k) number blank



Department of Health and Human Services  
 Food and Drug Administration  
**STANDARDS DATA REPORT FOR 510(k)s**  
 (To be filled in by applicant)

This report and the Summary Report Table are to be completed by the applicant when submitting a 510(k) that references a national or international standard. A separate report is required for each standard referenced in the 510(k).

TYPE OF 510(K) SUBMISSION

Traditional       Special       Abbreviated

STANDARD TITLE <sup>1</sup>

IEC 62304:2006 Medical Device Software - Software Life Cycle

**Please answer the following questions**

Yes    No

Is this standard recognized by FDA <sup>2</sup>? .....    

FDA Recognition number <sup>3</sup> ..... # 13-8

Was a third party laboratory responsible for testing conformity of the device to this standard identified in the 510(k)? .....    

Is a summary report <sup>4</sup> describing the extent of conformance of the standard used included in the 510(k)? .....       
 If no, complete a summary report table.

Does the test data for this device demonstrate conformity to the requirements of this standard as it pertains to this device? .....    

Does this standard include acceptance criteria? .....       
 If no, include the results of testing in the 510(k).

Does this standard include more than one option or selection of tests? .....       
 If yes, report options selected in the summary report table.

Were there any deviations or adaptations made in the use of the standard?.....       
 If yes, were deviations in accordance with the FDA supplemental information sheet (SIS) <sup>5</sup>? .....    

Were deviations or adaptations made beyond what is specified in the FDA SIS?.....       
 If yes, report these deviations or adaptations in the summary report table.

Were there any exclusions from the standard? .....       
 If yes, report these exclusions in the summary report table.

Is there an FDA guidance <sup>6</sup> that is associated with this standard?.....       
 If yes, was the guidance document followed in preparation of this 510k? .....    

Title of guidance: Guidance for the content of Premarket Submissions for Software Contained in Medical Devices

<sup>1</sup> The formatting convention for the title is: [SDO] [numeric identifier] [title of standard] [date of publication]

<sup>2</sup> Authority [21 U.S.C. 360d], <http://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/Standards/default.htm>

<sup>3</sup> <http://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfStandards/search.cfm>

<sup>4</sup> The summary report should include: any adaptations used to adapt to the device under review (for example, alternative test methods); choices made when options or a selection of methods are described; deviations from the standard; requirements not applicable to the device; and the name and

address of the test laboratory or certification body involved in conformance assessment to this standard. The summary report includes information on all standards utilized during the development of the device.

<sup>5</sup> The supplemental information sheet (SIS) is additional information which is necessary before FDA recognizes the standard. Found at <http://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfStandards/search.cfm>

<sup>6</sup> The online search for CDRH Guidance Documents can be found at <http://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/GuidanceDocuments/default.htm>

**EXTENT OF STANDARD CONFORMANCE  
SUMMARY REPORT TABLE**

STANDARD TITLE  
IEC 62304:2006 Medical Device Software - Software Life Cycle

**CONFORMANCE WITH STANDARD SECTIONS\***

SECTION NUMBER ALL	SECTION TITLE ALL	CONFORMANCE? <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No <input type="checkbox"/> N/A
-----------------------	----------------------	------------------------------------------------------------------------------------------------------------------

TYPE OF DEVIATION OR OPTION SELECTED ♦

DESCRIPTION

JUSTIFICATION

SECTION NUMBER	SECTION TITLE	CONFORMANCE? <input type="checkbox"/> Yes <input type="checkbox"/> No <input type="checkbox"/> N/A
----------------	---------------	-------------------------------------------------------------------------------------------------------

TYPE OF DEVIATION OR OPTION SELECTED ♦

DESCRIPTION

JUSTIFICATION

SECTION NUMBER	SECTION TITLE	CONFORMANCE? <input type="checkbox"/> Yes <input type="checkbox"/> No <input type="checkbox"/> N/A
----------------	---------------	-------------------------------------------------------------------------------------------------------

TYPE OF DEVIATION OR OPTION SELECTED ♦

DESCRIPTION

JUSTIFICATION

\* For completeness list all sections of the standard and indicate whether conformance is met. If a section is not applicable (N/A) an explanation is needed under "justification." Some standards include options, so similar to deviations, the option chosen needs to be described and adequately justified as appropriate for the subject device. Explanation of all deviations or description of options selected when following a standard is required under "type of deviation or option selected," "description" and "justification" on the report. More than one page may be necessary.

♦ Types of deviations can include an exclusion of a section in the standard, a deviation brought out by the FDA supplemental information sheet (SIS), a deviation to adapt the standard to the device, or any adaptation of a section.

**Paperwork Reduction Act Statement**

Public reporting burden for this collection of information is estimated to average 1 hour per response, including the time for reviewing instructions, searching existing data sources, gathering and maintaining the data needed, and completing and reviewing the collection of information. Send comments regarding this burden estimate or any other aspect of this collection of information, including suggestions for reducing this burden to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
1350 Piccard Drive, Room 400  
Rockville, MD 20850

*An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB control number.*

Department of Health and Human Services  
 Food and Drug Administration  
**STANDARDS DATA REPORT FOR 510(k)s**  
 (To be filled in by applicant)

This report and the Summary Report Table are to be completed by the applicant when submitting a 510(k) that references a national or international standard. A separate report is required for each standard referenced in the 510(k).

TYPE OF 510(K) SUBMISSION

Traditional       Special       Abbreviated

STANDARD TITLE <sup>1</sup>

ISO 14971:2007 Medical devices - Application of risk management to medical devices

**Please answer the following questions**

Yes      No

Is this standard recognized by FDA <sup>2</sup>? .....      

FDA Recognition number <sup>3</sup> ..... #5-40

Was a third party laboratory responsible for testing conformity of the device to this standard identified in the 510(k)? .....      

Is a summary report <sup>4</sup> describing the extent of conformance of the standard used included in the 510(k)? .....         
 If no, complete a summary report table.

Does the test data for this device demonstrate conformity to the requirements of this standard as it pertains to this device? .....      

Does this standard include acceptance criteria? .....         
 If no, include the results of testing in the 510(k).

Does this standard include more than one option or selection of tests? .....         
 If yes, report options selected in the summary report table.

Were there any deviations or adaptations made in the use of the standard?.....         
 If yes, were deviations in accordance with the FDA supplemental information sheet (SIS) <sup>5</sup>? .....      

Were deviations or adaptations made beyond what is specified in the FDA SIS?.....         
 If yes, report these deviations or adaptations in the summary report table.

Were there any exclusions from the standard? .....         
 If yes, report these exclusions in the summary report table.

Is there an FDA guidance <sup>6</sup> that is associated with this standard?.....         
 If yes, was the guidance document followed in preparation of this 510k? .....      

Title of guidance: \_\_\_\_\_

<sup>1</sup> The formatting convention for the title is: [SDO] [numeric identifier] [title of standard] [date of publication]

<sup>2</sup> Authority [21 U.S.C. 360d], <http://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/Standards/default.htm>

<sup>3</sup> <http://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfStandards/search.cfm>

<sup>4</sup> The summary report should include: any adaptations used to adapt to the device under review (for example, alternative test methods); choices made when options or a selection of methods are described; deviations from the standard; requirements not applicable to the device; and the name and

address of the test laboratory or certification body involved in conformance assessment to this standard. The summary report includes information on all standards utilized during the development of the device.

<sup>5</sup> The supplemental information sheet (SIS) is additional information which is necessary before FDA recognizes the standard. Found at <http://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfStandards/search.cfm>

<sup>6</sup> The online search for CDRH Guidance Documents can be found at <http://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/GuidanceDocuments/default.htm>

**EXTENT OF STANDARD CONFORMANCE  
SUMMARY REPORT TABLE**

STANDARD TITLE  
ISO 14971:2007 Medical devices - Application of risk management to medical devices

**CONFORMANCE WITH STANDARD SECTIONS\***

SECTION NUMBER ALL	SECTION TITLE ALL	CONFORMANCE? <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No <input type="checkbox"/> N/A
-----------------------	----------------------	------------------------------------------------------------------------------------------------------------------

TYPE OF DEVIATION OR OPTION SELECTED ♦

DESCRIPTION

JUSTIFICATION

SECTION NUMBER	SECTION TITLE	CONFORMANCE? <input type="checkbox"/> Yes <input type="checkbox"/> No <input type="checkbox"/> N/A
----------------	---------------	-------------------------------------------------------------------------------------------------------

TYPE OF DEVIATION OR OPTION SELECTED ♦

DESCRIPTION

JUSTIFICATION

SECTION NUMBER	SECTION TITLE	CONFORMANCE? <input type="checkbox"/> Yes <input type="checkbox"/> No <input type="checkbox"/> N/A
----------------	---------------	-------------------------------------------------------------------------------------------------------

TYPE OF DEVIATION OR OPTION SELECTED ♦

DESCRIPTION

JUSTIFICATION

\* For completeness list all sections of the standard and indicate whether conformance is met. If a section is not applicable (N/A) an explanation is needed under "justification." Some standards include options, so similar to deviations, the option chosen needs to be described and adequately justified as appropriate for the subject device. Explanation of all deviations or description of options selected when following a standard is required under "type of deviation or option selected," "description" and "justification" on the report. More than one page may be necessary.

♦ Types of deviations can include an exclusion of a section in the standard, a deviation brought out by the FDA supplemental information sheet (SIS), a deviation to adapt the standard to the device, or any adaptation of a section.

**Paperwork Reduction Act Statement**

Public reporting burden for this collection of information is estimated to average 1 hour per response, including the time for reviewing instructions, searching existing data sources, gathering and maintaining the data needed, and completing and reviewing the collection of information. Send comments regarding this burden estimate or any other aspect of this collection of information, including suggestions for reducing this burden to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
1350 Piccard Drive, Room 400  
Rockville, MD 20850

*An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB control number.*

**510(k) Notification Submission – Abbreviated  
Intel® Health Guide Express**

FEB - 8 2011

**510(k) Summary  
As required by 21 CFR §807.92(c)**

**Submitter**

510(k) Owner: Intel Corporation  
Address: 1900 Prairie City Road, FM7-197, Folsom, CA 95630  
Telephone: 916 847-7794  
Contact Person: Maureen Glynn  
Date Prepared: 01/06/2011

**Device Information**

Trade Name: Intel® Health Guide Express  
Common Name: Remote Patient Monitoring System  
Classification Name: Transmitters and Receivers, Physiological Signal,  
Radiofrequency (21 CFR 870.2910, Product Code DRG)

Substantial Equivalence is claimed to the following device:

1. Intel's Intel® Health Guide PHS6000 (K080798, K083115 and K101178)

**Device Description**

The Intel® Health Guide Express is a communication tool that allows caregivers to remotely access vital sign measurements of patients at home. The Intel® Health Guide Express is a software application running on a Commercial Off The Shelf (COTS) Personal Computer (PC). It collects measurements captured on commercially available wireless or tethered medical devices which are designed for home use and connection to a COTS PC. It displays the collected measurement on the PC, and securely stores the collected information locally on a memory device installed in the PC. The Intel® Health Guide Express also stores the information remotely on a host server, where the caregiver can view the measurement via the host server once synchronization between the host server and Intel® Health Guide Express has been completed. The Intel® Health Guide Express can be used to display educational and motivational content from the caregiver and can facilitate communication between the caregiver and patient via health wellness surveys and optional video conferencing.

The Intel® Health Guide Express is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required. It is contraindicated for patients requiring direct medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

K103276

**510(k) Notification Submission – Abbreviated  
Intel® Health Guide Express**

The Intel® Health Guide PHS Express system consists of the:

(1) Intel® Health Guide Express software application:

The software application captures, stores, displays and transmits information to a secure database on a host server running the Intel® Health Care Management Suite software via a standard telephone line or internet connection. The Intel® Health Guide Express software runs on a Commercial Off The Shelf (COTS) Personal Computer (PC).

(2) Intel® Health Care Management Suite software application:

The software application runs on a host server and allows caregivers to review patient vital signs on the secure website. The Intel® Health Care Management Suite allows for predefining upper and lower limits and, when either limit is exceeded, the system emails and/or pages the caregiver.

### **Indications for Use**

The Intel® Health Guide Express is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers. Patients can also engage in video conferences with caregivers and answer the caregivers' questions by participating in surveys.

The Intel® Health Care Management Suite allows the caregiver to review patient data and initiate video conferencing with patients, or select and send educational and motivational content to patients.

The Intel® Health Guide Express is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The Intel® Health Guide Express is contraindicated for patients requiring direct medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

### **Technological Characteristics**

The Intel® Health Guide Express is substantially equivalent to the predicate device Intel® Health Guide PHS 6000 (K080798, K083115 and K101178) in terms of data collection software functionality, operating system for the patient device, communication method of patient device with central server, types of sensors which can be interfaced to the patient

**510(k) Notification Submission – Abbreviated  
Intel® Health Guide Express**

K103276

device, implementation method of collecting data from sensors, sensor software, connectivity, communication protocol, power source, and display method.

**Table 1: Peripherals Comparison between the predicate device and the Intel® Health Guide Express**

<b>Physiological Parameter</b>	<b>Intel® Health Guide PHS6000 (K080798, K083115 and K101178)</b>	<b>Intel® Health Guide Express</b>
Blood Pressure	A&D UA-767PC (K982481)	
Weight	A&D UC-321PBT (exempt)	
	A&D UC-321PL (exempt)	
Blood Glucose Level	Bayer Diagnostics Ascensia Breeze2 (K062347)	
	Bayer Diagnostics Ascensia Contour Blood Glucose Monitoring System (K062058)	
	LifeScan OneTouch Ultra Family of Blood Glucose Monitoring Systems (K043197)	
	LifeScan OneTouch Ultra 2 of Blood Glucose Monitoring Systems (K053529)	
Oxygen Saturation	Nonin 4100 Pulse Oximeter (K043359)	
	N/A	Onyx® II 9560 (K081285)
FEV/PEF	Microlife PF100 (K031024)	

**510(k) Notification Submission – Abbreviated  
Intel® Health Guide Express**

K103276

Table 2: Hardware Comparison between the COTS PC meeting minimum specifications and the predicate device<sup>1</sup>

<b>Hardware Parameter</b>	<b>Intel® Health Guide PHS6000 (K080798, K083115 and K101178)</b>	<b>COTS PC</b>
Operating System	Microsoft Windows XP embedded	Microsoft Windows 7 (32-bit versions)
CPU Core Frequency	1.5 GHz	Minimum: 1.6 GHz (Single Core)
System Memory Size	512MB	Minimum: 1 Gbyte
Pointing Device	HID compliant pointing device	HID compliant pointing device
Available Storage Capacity	40GB	Minimum: 10 Gbytes
LAN Connection	Externally accessible RJ45 jack for 802.3 10/100 LAN interface.	Externally accessible RJ45 jack for 802.3 10/100 LAN interface.
Ports	4 USB Ports	Minimum: 2 USB Ports
SD Card Slot	N/A	Minimum: 1 memory card SDHC compliant with the SD Card Association 2.00 card specification, Class 4 or faster.
Wireless Peripheral Connection	Bluetooth 2.0 Interface	Bluetooth 2.0 Interface
Display	600 x 800 18 bit color	Minimum: 1024 x 600 24 bit color
Speakers	1 Mono	Minimum: 1 (mono)
Microphone	1 Mono	Minimum: 1 (mono)

<sup>1</sup>Differences are to accommodate the Microsoft Windows 7 operating system



Table 3: Main difference between the COTS PC meeting minimum specifications and the predicate device

<b>Parameter</b>	<b>Intel® Health Guide PHS6000 (K080798, K083115 and K101178)</b>	<b>COTS PC</b>
Safety Standard	ES60601-1:2005 Medical electrical equipment – Part 1: General requirements for basic and essential performance	UL 60950-1:2007 Information Technology Equipment – Safety – Part 1: General Requirements
Patient Leakage Current – From Patient connection to earth <sup>1</sup>	100µA	3.5mA

<sup>1</sup>This is not meant to be an exhaustive list of differences between ES60601-1 and UL 60950-1 but highlights the differences covered in the risk analysis for a COTS PC used with the Intel® Health Guide Express.

### **Safety and Efficacy**

The Intel® Health Guide Express does not rely on an assessment of clinical performance data. The device will conform to FDA’s recognized consensus standards and relies on its conformity to demonstrate the safety and efficacy. The device introduces no new questions concerning the safety or efficacy and is, therefore, substantially equivalent to the predicate device.



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Room -WO66-G609  
Silver Spring, MD 20993-0002

Intel Corporation  
c/o Ms. Maureen Glynn  
1900 Prairie City Rd.  
MS FM7-197  
Folsom, CA 95630

FEB - 8 2011

Re: K103276  
Trade Name: Intel Health Guide Express  
Regulation Number: 21 CFR 870.2910  
Regulation Name: Radiofrequency Physiological Signal Transmitter and Receiver  
Regulatory Class: Class II (two)  
Product Code: DRG  
Dated: January 10, 2011  
Received: January 11, 2011

Dear Ms. Glynn:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

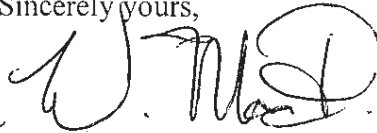
If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Office of Compliance at (240) 276-0120. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (240) 276-3150 or at its Internet address <http://www.fda.gov/cdrh/industry/support/index.html>.

Sincerely yours,

  
Bram D. Zuckerman, M.D.

Director  
Division of Cardiovascular Devices  
Office of Device Evaluation  
Center for Devices and  
Radiological Health

Enclosure

## Indications for Use:

510(k) Number: \_\_\_\_\_

Device Name: Intel® Health Guide Express

### Indications for Use:

The Intel® Health Guide Express is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers. Patients can also engage in video conferences with caregivers and answer the caregivers' questions by participating in surveys.

The Intel® Health Care Management Suite allows the caregiver to review patient data and initiate video conferencing with patients, or select and send educational and motivational content to patients.

The Intel® Health Guide Express is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The Intel® Health Guide Express is contraindicated for patients requiring direct medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

Prescription Use  X   
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use \_\_\_\_\_  
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)



Concurrence of CDRH, Office of Device Evaluation (ODE)

(Division Sign-Off)  
Division of Cardiovascular Devices

Page 1 of 1

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
510(k) Number  K103276

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
# User Manual

UHCTX

b(4)Trade Secret Process-Draft





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














b(4)Trade Secret Process-Draft

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






b(4) Trade Secret Process-Draft

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




# SW Requirements Specifications


LG SmartHealth Development

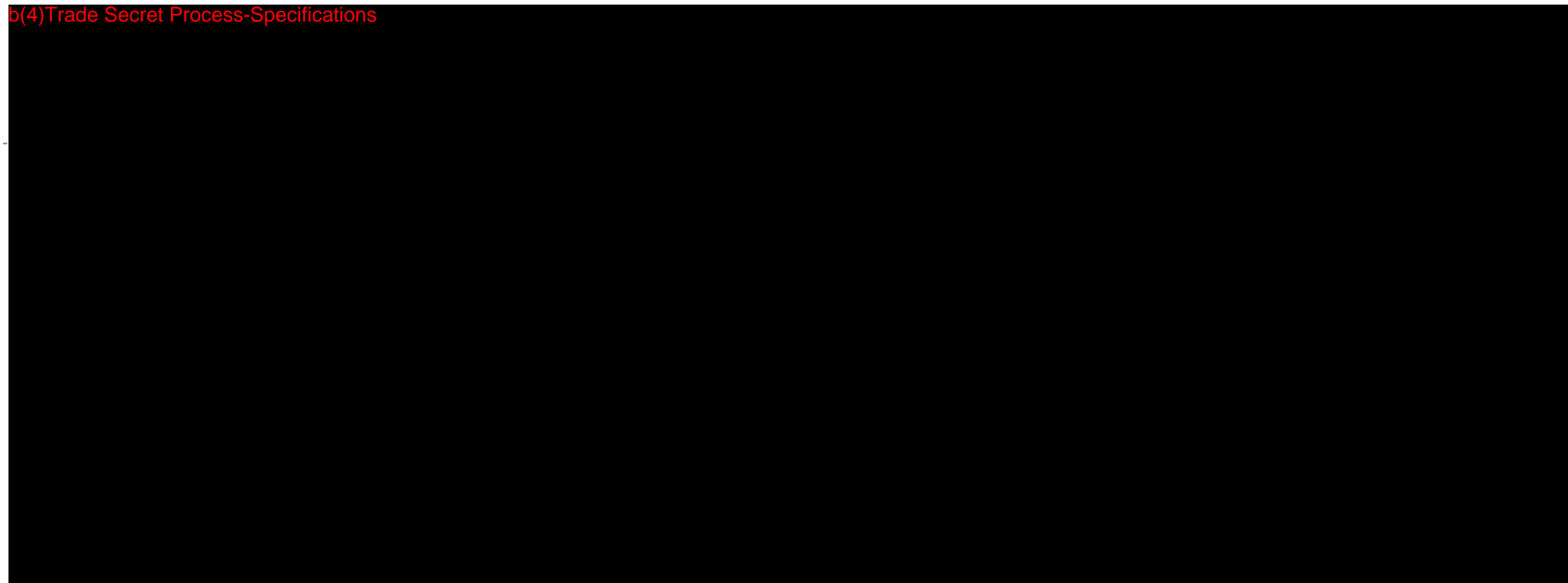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b(4)Trade Secret Process-Specifications

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
















b(4)Trade Secret Process-Specifications

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











# SW Requirements Specifications

LG SmartHealth Development

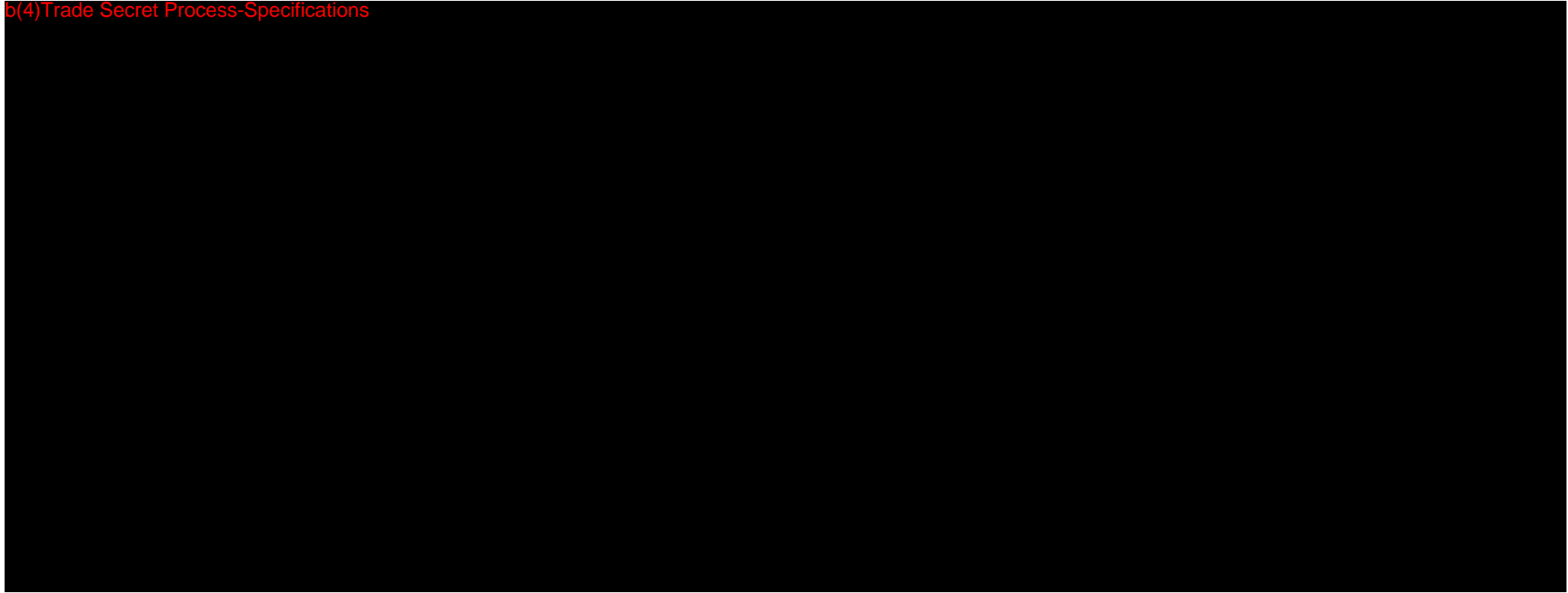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
b(4)Trade Secret Process-Specifications

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
















b(4)Trade Secret Process-Specifications




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











# SW Requirements Specifications


LG SmartHealth Development

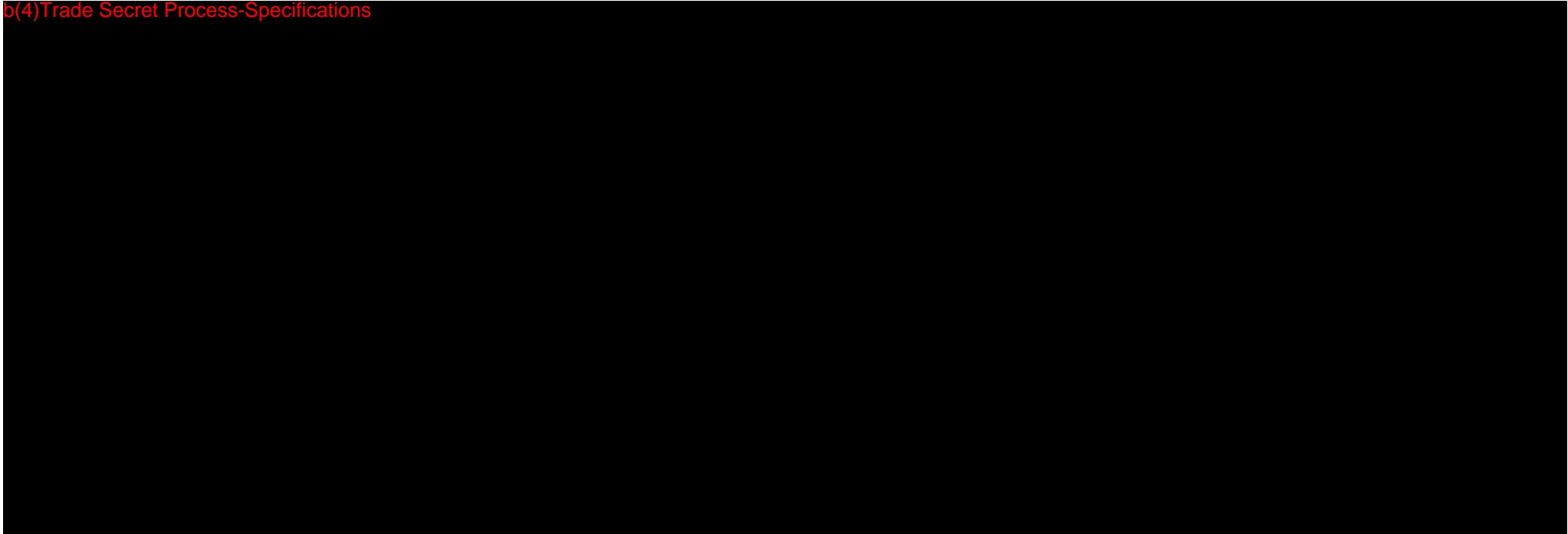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b(4)Trade Secret Process-Specifications












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
b(4) Trade Secret Process-Specifications

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












b(4)Trade Secret Process-Specifications

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


b(4)Trade Secret Process-Specifications

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
b(4) Trade Secret Process-Specifications




b(4)Trade Secret Process-Specifications










b(4)Trade Secret Process-Specifications

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


# SW Requirements Specifications


LG SmartHealth Development

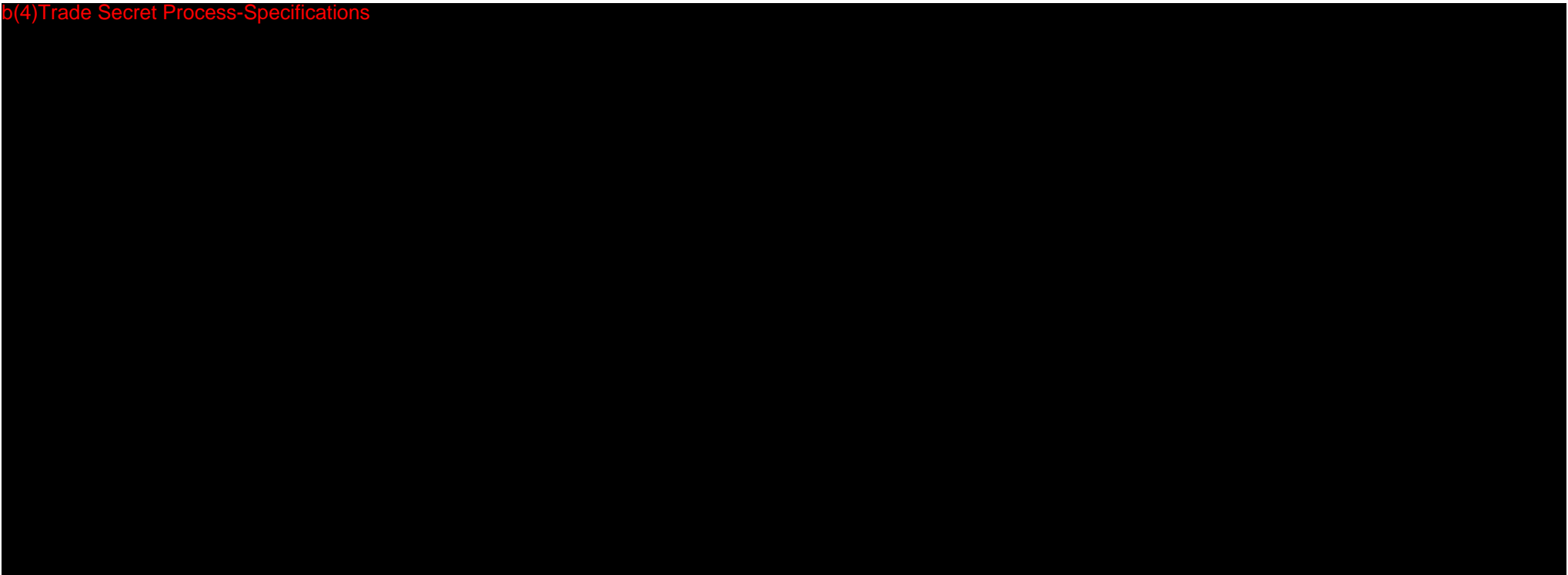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
b(4) Trade Secret Process-Specifications















b(4)Trade Secret Process-Specifications

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b(4) Trade Secret Process-Specifications




b(4)Trade Secret Process-Specifications

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



b(4)Trade Secret Process-Specifications

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
b(4) Trade Secret Process-Specifications

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














# SW Architecture Design

LG SmartHealth Development

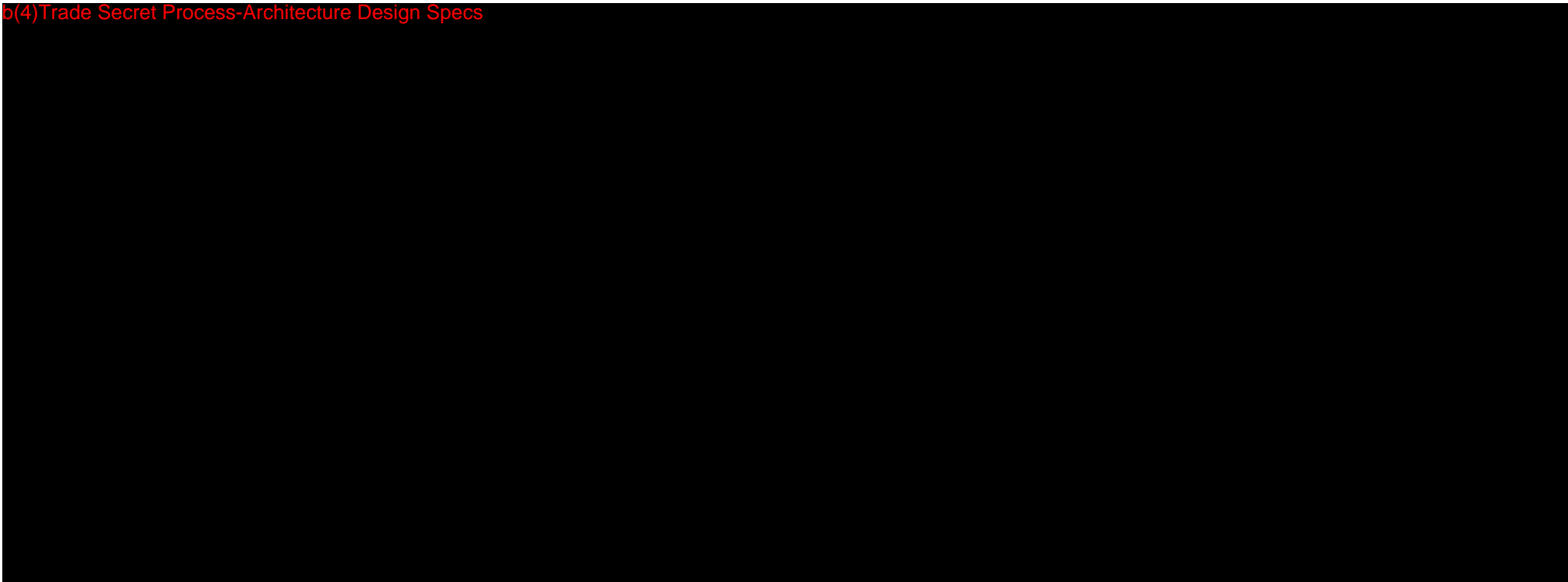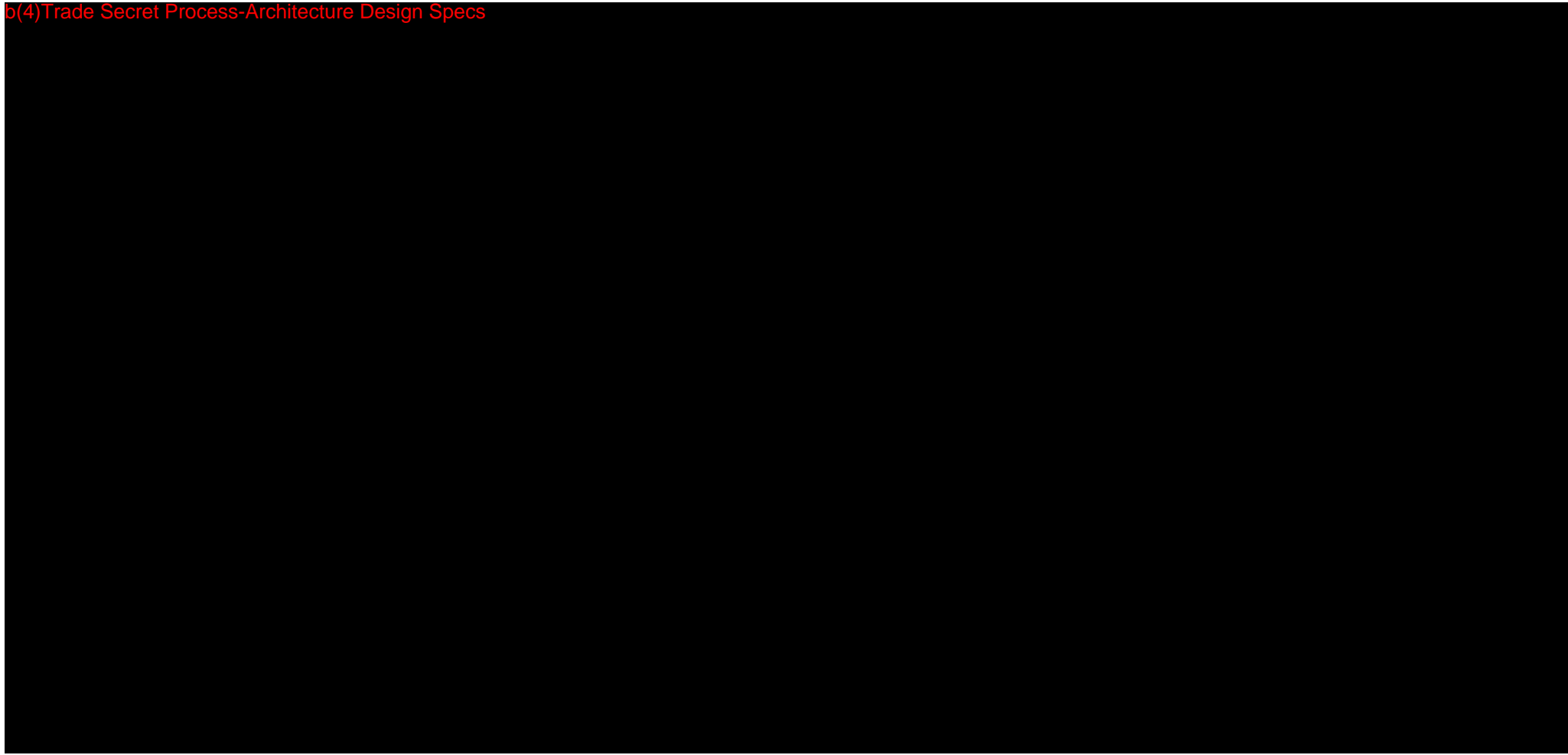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
b(4)Trade Secret Process-Architecture Design Specs

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












# High Level Design

**LG SmartHealth Development**  
[Common Management]

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
b(4)Trade Secret Process-Design Specs



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
b(4)Trade Secret Process-Design Specs



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















b(4)Trade Secret Process-Design Specs










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




b(4) Trade Secret Process-Design Specs



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













# High Level Design

LG SmartHealth Development  
[Telemedicine]

b(4)Trade Secret Process-Design Specs

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b(4)Trade Secret Process-Design Specs





b(4)Trade Secret Process-Design Specs









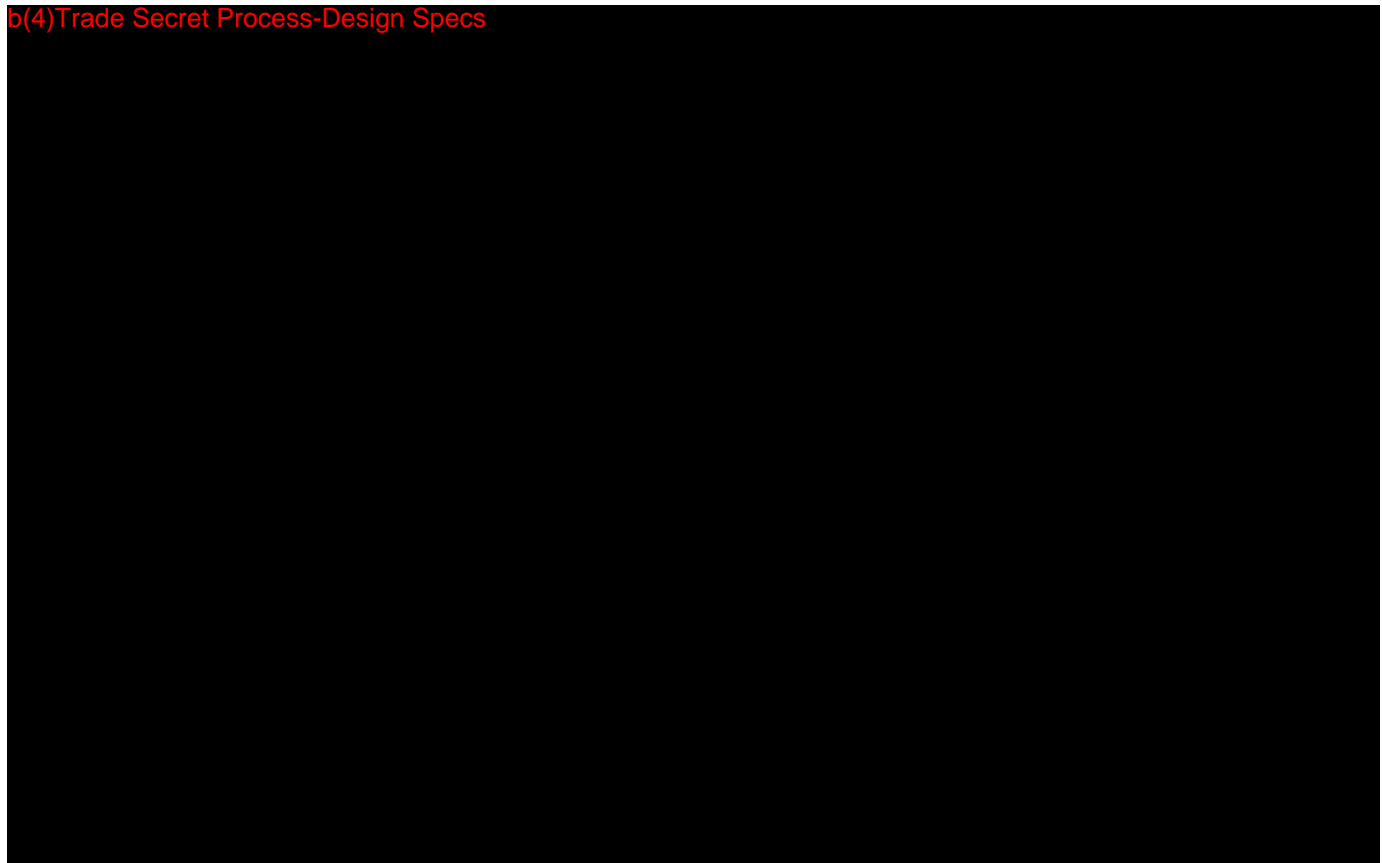


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
b(4)Trade Secret Process-Design Specs















b(4)Trade Secret Process-Design Specs









b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs







b(4)Trade Secret Process-Design Specs




# High Level Design

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b(4)Trade Secret Process-Design Specs

b(4)Trade Secret Process-Design Specs






b(4)Trade Secret Process-Design Specs








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









b(4)Trade Secret Process-Design Specs



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

b(4)Trade Secret Process-Design Specs



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



b(4)Trade Secret Process-Design Specs







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



b(4)Trade Secret Process-Design Specs



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



b(4) Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs






# High Level Design

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
b(4)Trade Secret Process-Design Specs

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
b(4)Trade Secret Process-Design Specs



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














b(4)Trade Secret Process-Design Specs





b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs





b(4)Trade Secret Process-Design Specs




# Requirements Traceability Matrix

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
UHCTX

b(4)Trade Secret Process-Product Specs

b(4)Trade Secret Process-Product Specs







(b)(4) Software



(b)(4) Software



b(4)Trade Secret Process-Product Specs





# Risk Management File

LG SmartHealth Development

b(4)Trade Secret Process-Risk Management Specs

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
b(4) Trade Secret Process-Risk Management Specs



b(4)Trade Secret Process-Risk Management Specs



b(4) Trade Secret Process-Risk Management Specs



b(4)Trade Secret Process-Risk Management Specs



b(4)Trade Secret Process-Risk Management Specs



b(4)Trade Secret Process-Risk Management Specs










# Verification Test Plan

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LG SmartHealth Development

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
b(4)Trade Secret Process-Testing

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



















b(4)Trade Secret Process-Testing

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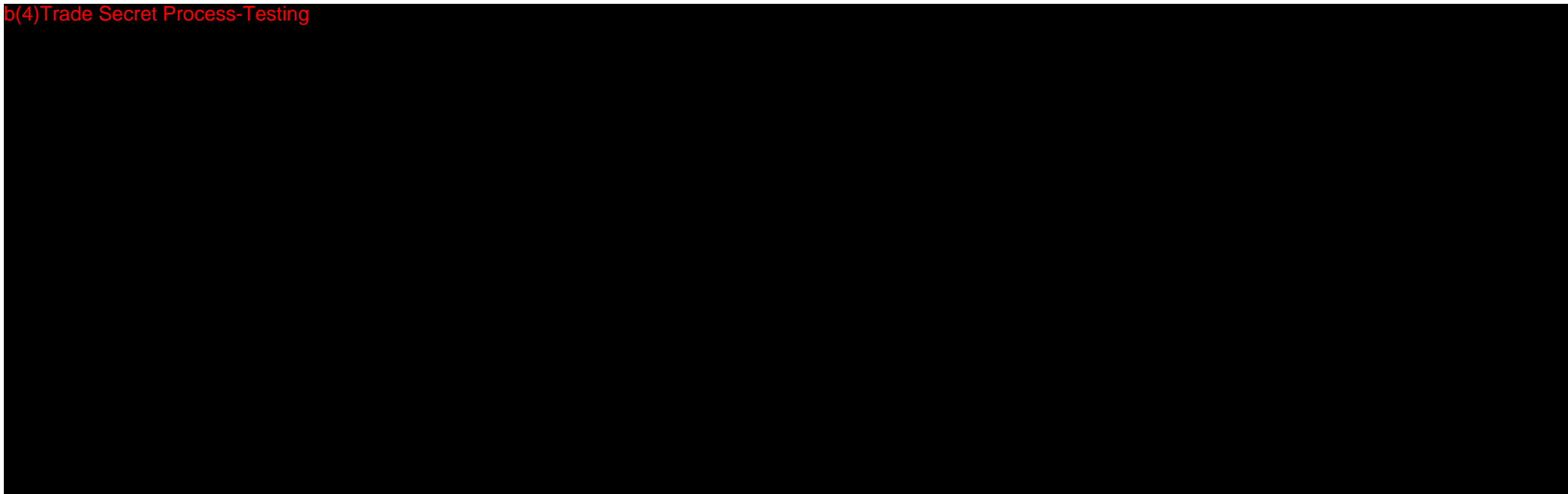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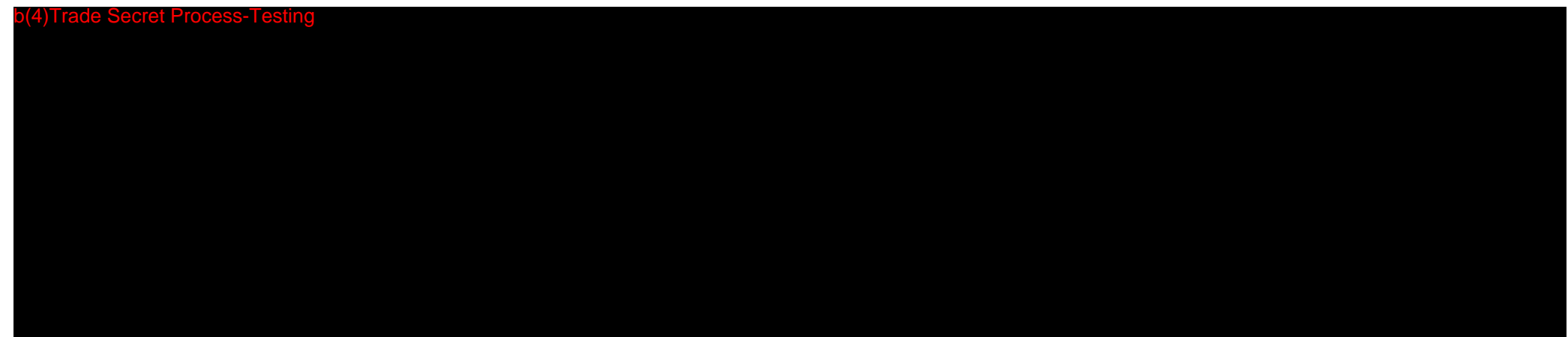




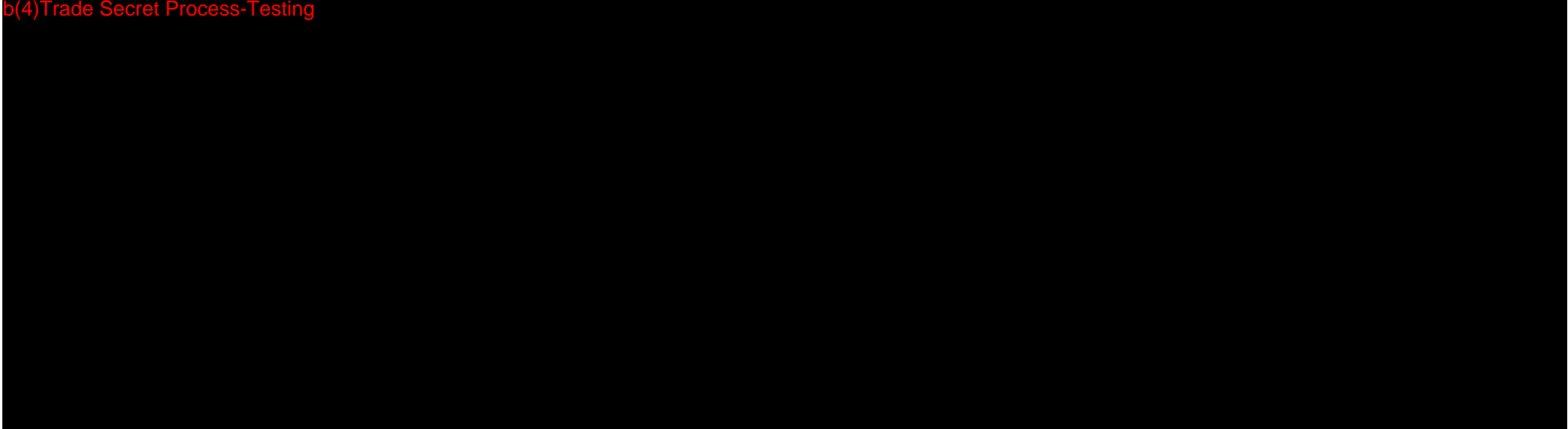




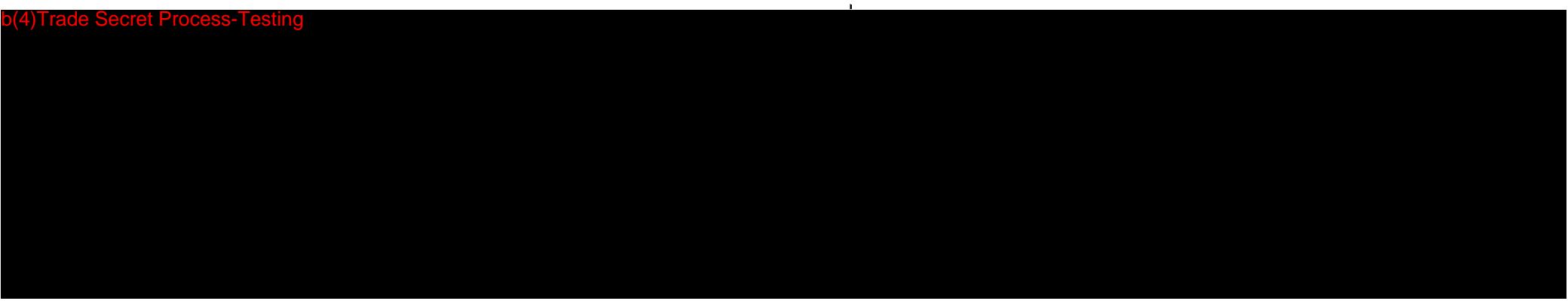














b(4)Trade Secret Process-Testing







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



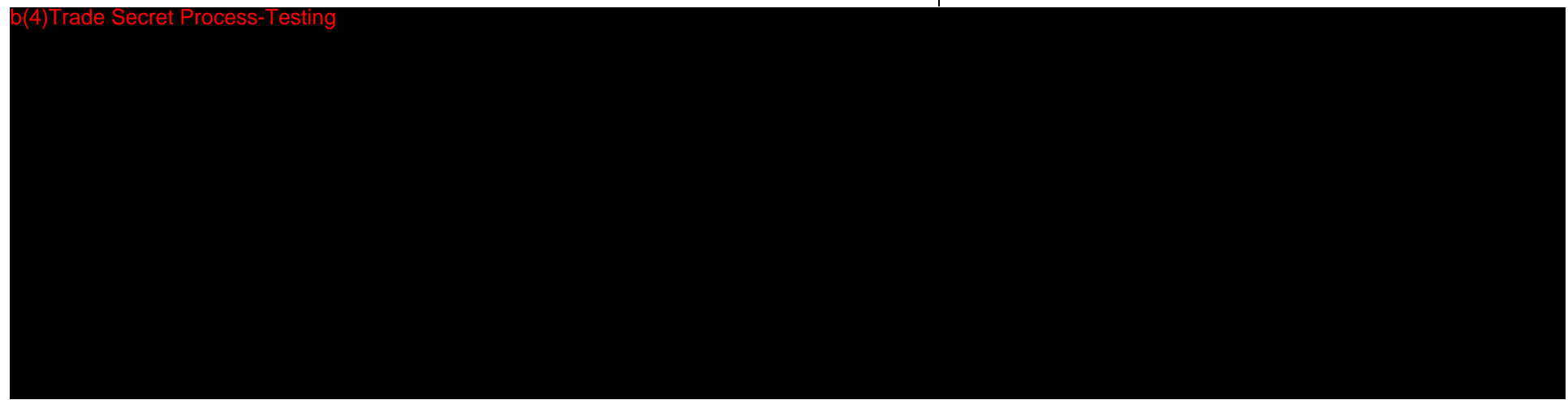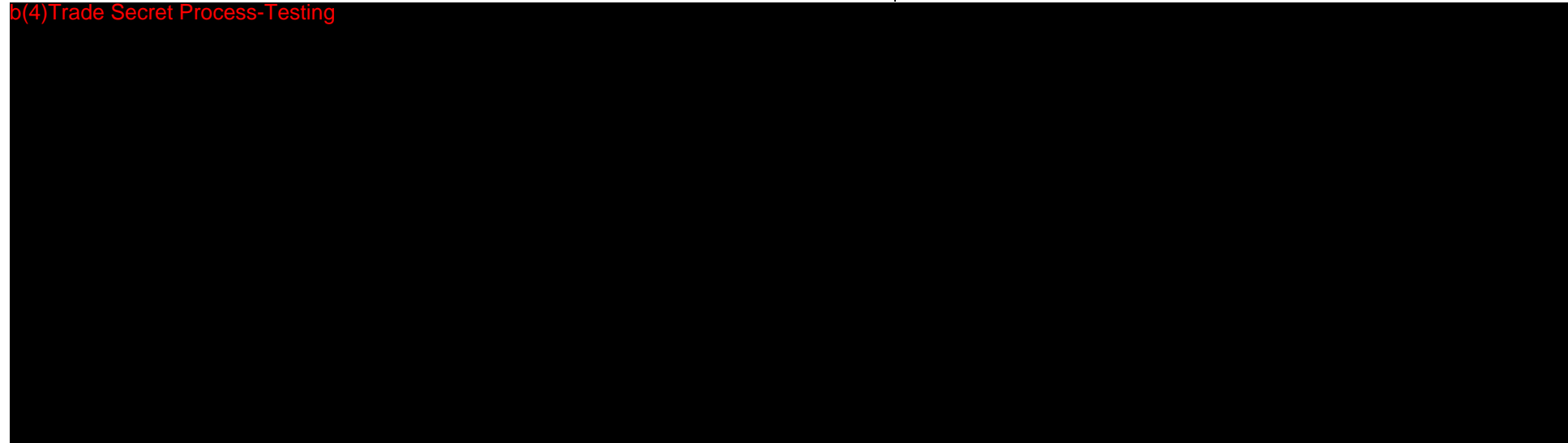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





b(4) Trade Secret Process-Testing



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


b(4)Trade Secret Process-Testing



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
b(4)Trade Secret Process-Testing

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












b(4)Trade Secret Process-Testing

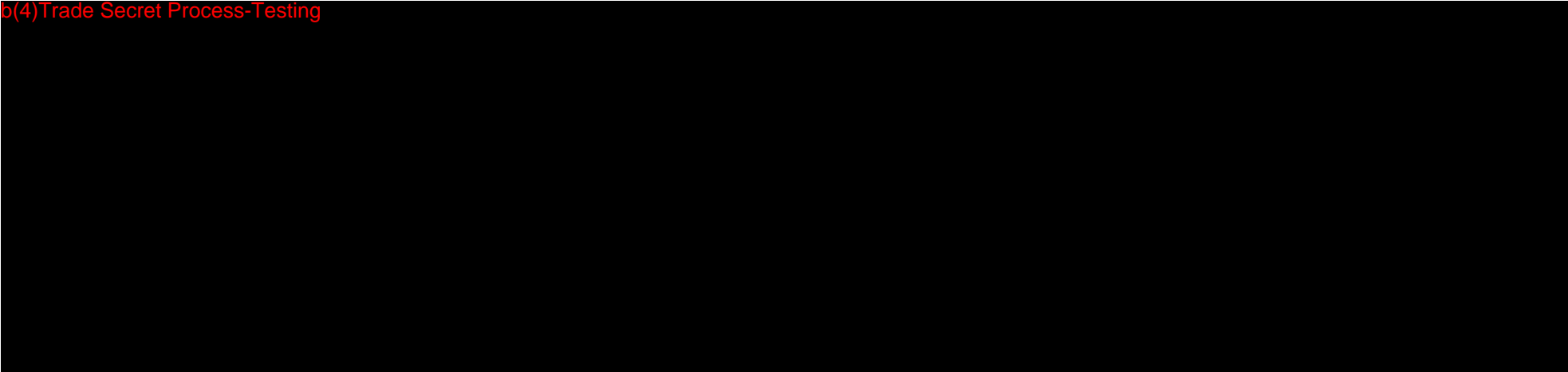


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
b(4)Trade Secret Process-Testing

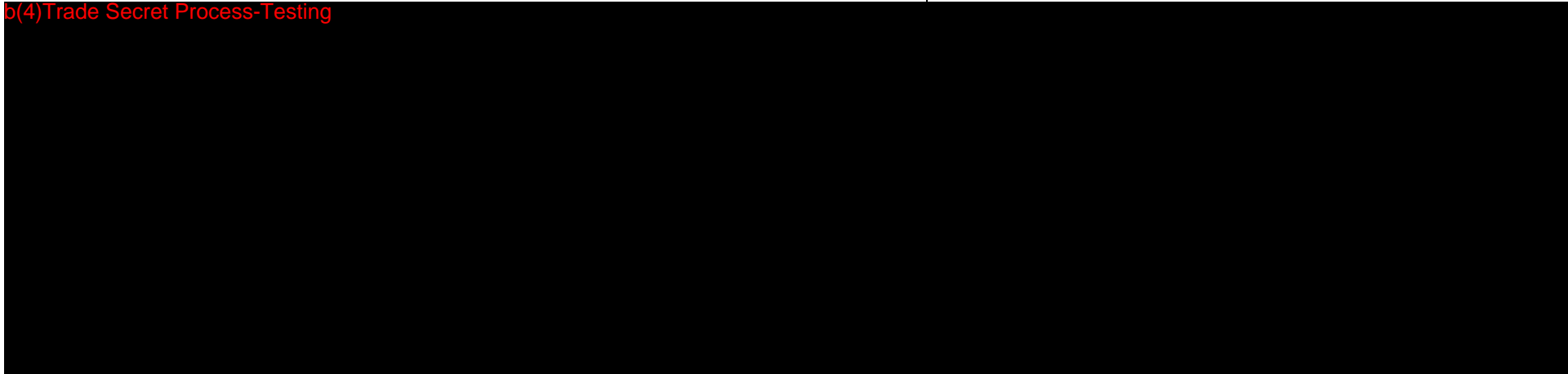


b(4)Trade Secret Process-Testing










b(4)Trade Secret Process-Testing





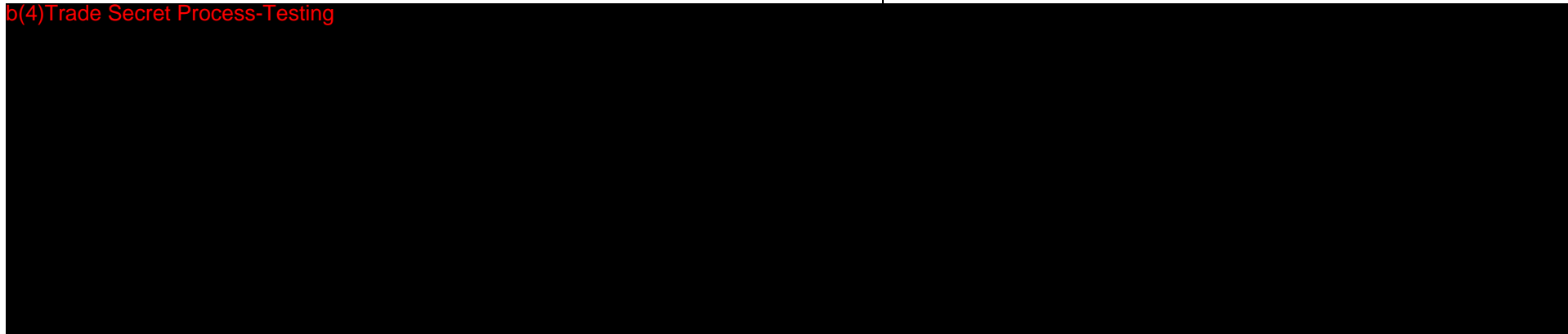b(4)Trade Secret Process-Testing







# Verification Test Report(15<sup>th</sup>)

UHCTX

b(4)Trade Secret Process-Testing

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b(4)Trade Secret Process-Testing

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


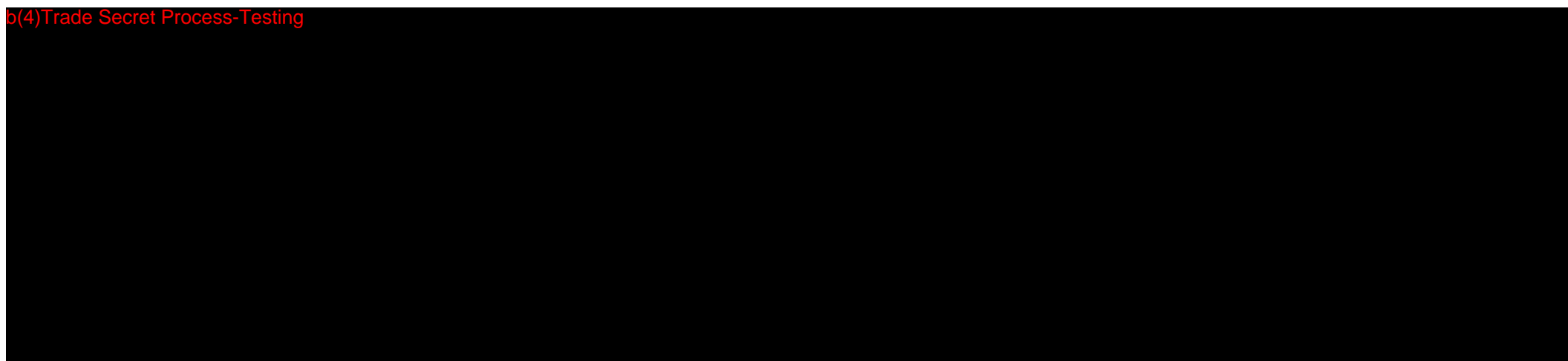













# LG SmartHealth Usability Testing Report

UHCTX

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
b(4)Trade Secret Process-Testing

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












b(4) Trade Secret Process-  
Software Specs

12/11/2013

### Software Certificate

I, (b) (6) certify that the information contained in this 510(k) application about the LG SmartHealth software is correct, and that the procedures described in Section 16 of this application will be used to re-test and re-validate this software when it is revised.

(b) (6)

Signature

(b) (6)

Typed Name

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
12/11/2013

Date

b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing





b(4) Trade Secret Process-Testing



b(4) Trade Secret Process-Testing



b(4) Trade Secret Process-Testing



b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing

b(4)Trade Secret Process-Testing



b(4) Trade Secret Process-Testing



b(4) Trade Secret Process-Testing





b(4)Trade Secret Process-Testing





DEPARTMENT OF HEALTH AND HUMAN SERVICES  
Food and Drug Administration

**Certification of Compliance**

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
Under 42 U.S.C. § 282(j)(5)(B), with Requirements of ClinicalTrials.gov Data Bank (42 U.S.C. § 282(j))

(For submission with an application/submission, including amendments, supplements, and resubmissions, under §§ 505, 515, 520(m), or 510(k) of the Federal Food, Drug, and Cosmetic Act or § 351 of the Public Health Service Act.)

**SPONSOR / APPLICANT / SUBMITTER INFORMATION**

(b) (4)

(b) (4)





EMERGO  GROUP

K133997/S001

January 9, 2014

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
Cardiac Diagnostics Devices Branch,  
Division of Cardiovascular Devices  
10903 New Hampshire Avenue  
W066-1244  
Silver Spring, Maryland 20993-0002

FDA CDRH DMC  
JAN 10 2014  
Received

Dear Oscar;

(b) (4)

Sincerely,

*Diane Sudduth*

Diane Sudduth  
Senior Consultant, RA/QA



January 9, 2014

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
Cardiac Diagnostics Devices Branch,  
Division of Cardiovascular Devices  
10903 New Hampshire Avenue  
W066-1244  
Silver Spring, Maryland 20993-0002

Dear Oscar;

(b) (4)

Sincerely,



Diane Sudduth  
Senior Consultant, RA/QA

## Indications for Use

510(k) Number (if known)

**Device Name**

LG SmartHealth

**Indications for Use (Describe)**

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers(or nurse). Patients can also engage in video conferences with caregivers(or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

**PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON A SEPARATE PAGE IF NEEDED.**

**FOR FDA USE ONLY**

Concurrence of Center for Devices and Radiological Health (CDRH) (Signature)

---

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
*PRAStaff@fda.hhs.gov*

*“An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number.”*



FDA CDRH DMC  
APR 30 2014  
Received

April 29, 2014

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
Cardiac Electrophysiology and Monitoring Branch,  
Division of Cardiovascular Devices  
10903 New Hampshire Avenue  
W066-1244  
Silver Spring, Maryland 20993-0002

RE: K133997 LG SmartHealth Application

Dear Oscar;



Sincerely,

*Diane Sudduth*

Diane Sudduth  
Senior Consultant, RA/QA

25



## TABLE OF CONTENTS

FILE	DESCRIPTION
001	Table of Contents
002	K133997 AI Response
003	510(k) Summary
004	Device Description
005	Software
006	Exhibit 13A User Manual
007	Exhibit 16A-1 Software Requirements Specifications (Common)
008	Exhibit 16A-2 Software Requirements Specifications (Telemedicine)
009	Exhibit 16A-3 Software Requirements Specifications (Measurement)
010	Exhibit 16A-4 Software Requirements Specifications (Medication)
011	Exhibit 16B Software Architectural Design
012	Exhibit 16C-1 High Level Design Document (Common)
013	Exhibit 16C-2 High Level Design Document (Telemedicine)
014	Exhibit 16C-3 High Level Design Document (Measurement)
015	Exhibit 16C-4 High Level Design Document (Medication)
016	Exhibit 16C-5 Server Interface Specification
017	Exhibit 16D Requirements Traceability Matrix
018	Exhibit 16E Risk Management File
019	Exhibit 16F Integration/System Test Plan
020	Exhibit 16F-1 Integration Test Case for Common
021	Exhibit 16F-2 Integration Test Case for Telemedicine
022	Exhibit 16F-3 Integration Test Case for Measurement
023	Exhibit 16F-4 Integration Test Case for Medication
024	Exhibit 16F-5 Bluetooth Performance test case
025	Exhibit 16G Integration/System Test Report
026	Exhibit 16H Usability Testing Report
027	Exhibit 16J-1 Project Management Plan
028	Exhibit 16J-2 Configuration Management Plan
029	Exhibit 16K Basic Documentation for OTS Software

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


April 29, 2014

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
Cardiac Electrophysiology and Monitoring Branch,  
Division of Cardiovascular Devices  
10903 New Hampshire Avenue  
W066-1244  
Silver Spring, Maryland 20993-0002

**RE: K133997 LG SmartHealth Application**

Dear Oscar;




Sincerely,

*Diane Sudduth*

Diane Sudduth  
Senior Consultant, RA/QA



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(b) (4)





(b)(4) Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs

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




(b)(4) Trade Secret Process-Product Specs




b(4) Trade Secret Process-Product Specs






b(4)Trade Secret Process-Product Specs

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






b(4) Trade Secret Process-Product Specs

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



b(4) Trade Secret Process-Product Specs

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




b(4)Trade Secret Process-Product Specs




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b(4)Trade Secret Process-Product Specs





b(4)Trade Secret Process-Product Specs

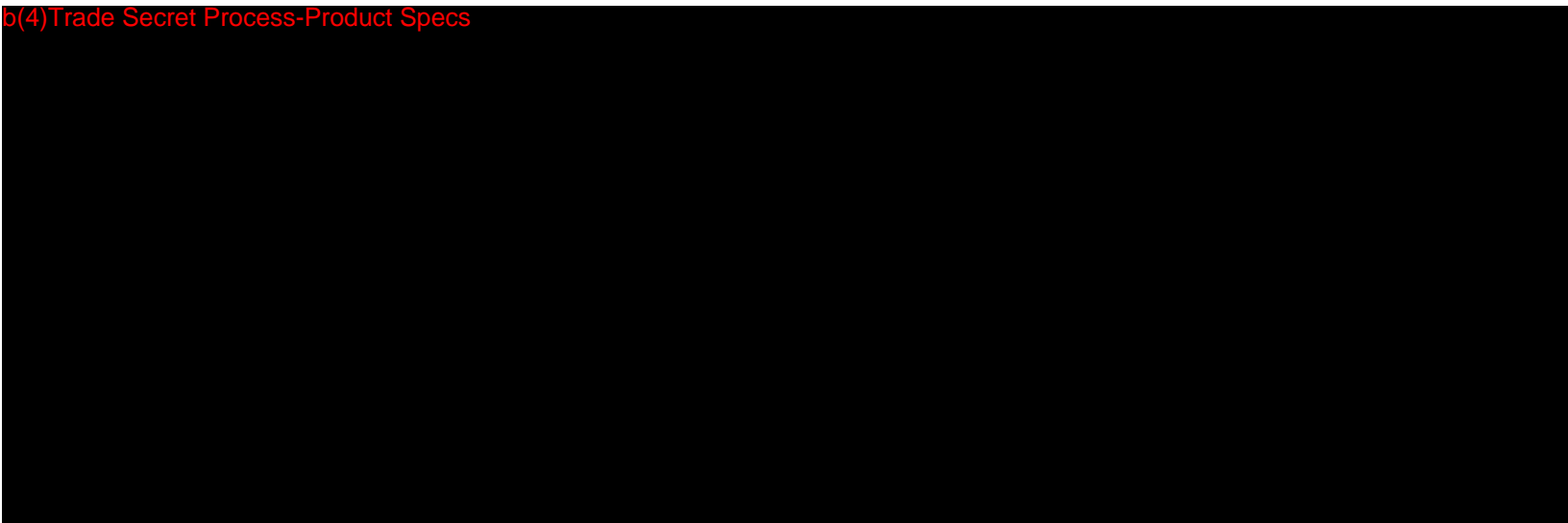

**510(k) Summary**  
**for**  
**LG SmartHealth**

**1. Submission Sponsor**

LG Electronics  
38 Baumoe-ro, Seocho-gu  
Seoul, 137-724  
Korea  
Phone: +82-2-526-4527  
Fax: +82-2-526-4370  
Contact: Chang Hee Lee, Chief Research Engineer

**2. Submission Correspondent**

Emergo Group  
816 Congress Avenue, Suite 1400  
Austin, TX 78701  
Cell Phone: (561) 305-5075  
Office Phone: (512) 327.9997  
Fax: (512) 327.9998  
Contact: Dr. Diane Sudduth, Senior Consultant, QA  
Email: [project.management@emergogroup.com](mailto:project.management@emergogroup.com)

**3. Date Prepared**

December 21, 2013

**4. Device Identification**

Trade/Proprietary Name: LG SmartHealth  
Common/Usual Name: Remote Patient Monitoring System  
Classification Name: Transmitters and Receivers, Physiological Signal,  
Radiofrequency  
Classification Regulation: 870.2910  
Product Code: Product Code DRG  
Device Class: Class II  
Classification Panel: Cardiovascular

**5. Legally Marketed Predicate Device(s)**

Intel Health Guide Express (K103276)

## 6. Device Description

The LG SmartHealth application is software planned to operate on the Android operating system (OS) that can be loaded as an application or “app” on a commercially available smartphone. LG intended the software to work on the Android OS, as well as to operate on smartphones as well. The scalability and differences between many smartphones today are not too great to allow this flexibility and option.

The LG SmartHealth application connects to commercially available wireless medical devices that are commonly used by patients in a home-care setting. These “sensor devices” are FDA cleared or FDA registered (Exempt) devices that can communicate with the LG SmartHealth application software loaded on the smartphone using Bluetooth connectivity. To avoid wireless interference and confusion on the part of the patient, serial readings are performed (i.e. only one Bluetooth sensor used at a time) and MAC address filtering is used for the various medical devices. The LG SmartHealth application plans to support the following sensor available medical devices, some initially, and others as the technology becomes available:

- Glucose meters or glucometers
- Blood pressure cuffs
- Weight scales
- Body fat readers
- Activity monitors

On each screen of the LG SmartHealth application software a help button is present, which when pressed provides step-by-step based help to guide a patient through an interaction.

The medical device takes a reading depending on the frequency previously established and the LG SmartHealth application software fetches the information for the reading automatically. The LG SmartHealth application software stores and displays the information on the smartphone and transmits the information to the server. The LG SmartHealth application encrypts the information in preparation for transmit using SHA256 bit encryption, complying with HIPAA requirements.

## 7. Indication for Use Statement

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing

its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

**8. Substantial Equivalence Discussion**

The comparison chart below provides evidence to facilitate the substantial equivalence determination between LG SmartHealth to the predicate device (K103276) with respect to intended use, technological characteristics and principles of operation.

**Table 5A – Comparison of Characteristics**

Features and Specifications	Intel® Health Guide Express (K103276)	LG SmartHealth (Subject Device)
<b>Regulatory Information</b>		
<b>Intended Use/ Indications for Use</b>	<p>The Intel® Health Guide Express is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers. Patients can also engage in video conferences with caregivers and answer the caregivers' questions by participating in surveys.</p> <p>The Intel® Health Care Management Suite allows the caregiver to review patient data and initiate video conferencing with patients, or select and send educational and motivational content to patients.</p> <p>The Intel® Health Guide Express is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The Intel® Health Guide Express is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>	<p>The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).</p> <p>The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>
<b>OTC and/or Rx</b>	Rx	Same



User Characteristics		
<b>Intended User</b>	Patient in Home Setting	Same
Device		
<b>Components</b>	<ul style="list-style-type: none"> <li>Web based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>	<ul style="list-style-type: none"> <li>Smart phone app based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>
<b>User Interface Device</b>	Computer Device (i.e. desktop, laptop)	Android Device(i.e. Smart phone, tablet)
<b>Connectivity to Device</b>	Bluetooth or USB wired	Bluetooth
<b>Ability to Update Device Settings</b>	Not available	Automatic deploy
<b>Connectivity to Web-based Services</b>	Login Secured by SSL Encryption	Same
<b>Operating System</b>	Microsoft Windows 7 (32-bit versions)	Android 2.3.3 or higher
<b>Supported Device Types</b>	Medical devices designed for home use: Blood glucose meter Weight scale Blood pressure Pulse Oximeter FEV/PEF	Medical devices designed for home use: Blood glucose meter Weight scale Blood pressure Temperature Activity Monitor
<b>Software Delivery</b>	Web download for the uploader only	Same
Software Features		
<b>Patient Management</b>	Manage patient accounts, filter/sort/search patients, print reports.	The server connected to the mobile device manages patient accounts, filter/sort/search patients
<b>Implementation method of collecting data from sensors</b>	Bluetooth or USB wired	Bluetooth
<b>Communication frequency</b>	Bluetooth: 2.402 to 2.480 GHz	Same
<b>Max data throughput under worst case conditions</b>	Multiply read data can be stored in the medical device and they are resent to the server after recovery of network condition	Same

<b>Use of thresholds/algorithms for determining how thresholds are set and changed</b>	Thresholds set by healthcare professional in server software	Same
<b>Communication method with server</b>	Connected through internet	Same

The LG SmartHealth is a medical device and has similar indications for use statement as the predicate device. The differences in the indications statement between LG SmartHealth and Intel Health Guide Express do not alter the intended use – both devices have essentially the same intended use.

The device also has similar technological characteristics as the predicate device. Both devices have the same data collection software functionality, communication method of patient device with central server, types of sensors which can be interfaced to the patient device, implementation method of collecting data from sensors, sensor software, connectivity, and communication protocol. Since the comparison of the descriptive characteristics of the proposed and predicate devices may not be sufficiently precise to assure equivalence, performance data are provided. The results of the performance testing demonstrate substantial equivalence.

#### **9. Non-Clinical Performance Data**

The device’s software development, verification, and validation have been carried out in accordance with FDA guidelines. The software was tested against the established Software Design Specifications for each of the test plans to assure the device performs as intended. The device Hazard analysis was completed and risk control implemented to mitigate identified hazards. The testing results support that all the software specifications have met the acceptance criteria of each module and interaction of processes. The LG SmartHealth device passed all testing and supports the claims of substantial equivalence and safe operation.

Validation activities included a usability study of the LG SmartHealth under actual use. The study demonstrated:

- Comprehension of the study nurses and participants with the LG SmartHealth,
- Appropriate human factors related to the LG SmartHealth, and
- Ease of use of the LG SmartHealth.

#### **10. Clinical Performance Data**

There was no clinical testing required to support the medical device as the indications for use is equivalent to the predicate device. These types of devices, including the predicate devices, have been on the market for many years with a proven safety and efficacy for the use of the device. The verification and validation testing of the device’s software was found to be acceptable and supports the claims of substantial equivalence.

## **11. Statement of Substantial Equivalence**

The LG SmartHealth has the same or similar intended use as the predicate device and that any technological differences between the LG SmartHealth software and the predicate device do not raise any questions regarding LG SmartHealth ('SmartHealth')'s safety and effectiveness.

The information provided in this submission supports the substantial equivalence to the predicate device and that the system is safe and effective for the users/operators.


## Section 11 – Device Description

**Device Name:** LG SmartHealth

**Description of the Device:**

“LG SmartHealth” is an application software (“application” or “app”) intended to operate on the Android operating system (“OS”) that can be loaded on commercially available tablet or smartphone. The scalability and differences between many tablets and smartphones today are not too great to allow this flexibility and option.

LG SmartHealth, an application for use by patients, supports the features of Measurement Management, Medication Management, Telemedicine, and Common Management, in interface with external 3rd party wireless medical devices and healthcare professional server solution.

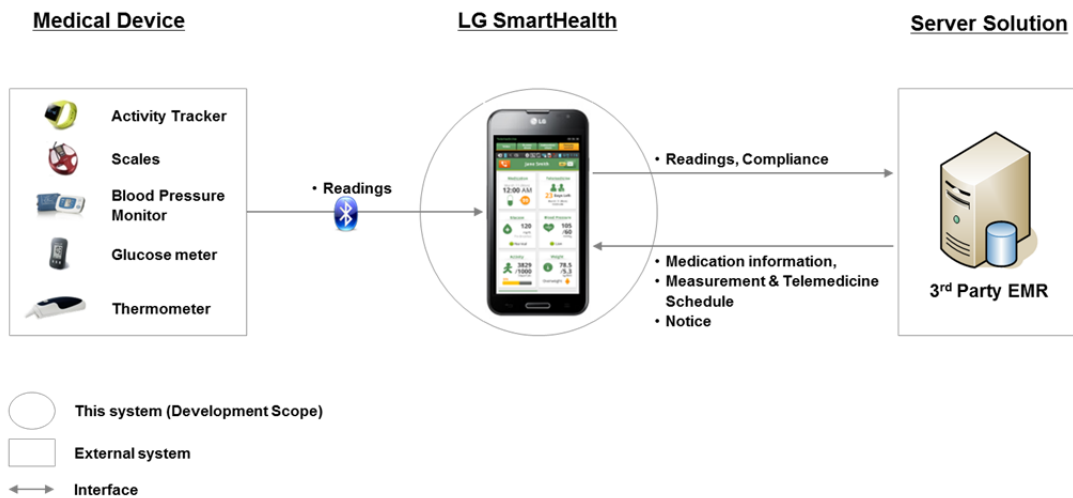

**Figure 1 LG SmartHealth Operation Structure**

LG SmartHealth connects to commercially available wireless medical devices that are commonly used by patients in a home-care setting. These “sensor devices” are FDA cleared or FDA registered (Exempt) devices that can communicate with the LG SmartHealth loaded on the smartphone Bluetooth connectivity. To avoid wireless interference and confusion on the part of the patient, serial readings are performed (i.e. only one Bluetooth sensor used at a time) and MAC address filtering is used for the various medical devices. LG SmartHealth plans to support the following sensor available medical devices, some initially, and then others as the technology become available:

- Glucose meters or glucometers
- Blood pressure cuffs
- Weight scales
- Body fat readers
- Activity monitors
- Thermometer

Pressing the measurement guide button in LG SmartHealth provides step-by-step interactive help guide to the user.

The medical device takes a reading depending on the frequency previously established and LG SmartHealth fetches the information for the reading automatically. LG SmartHealth stores and displays the information on the tablet or smartphone and transmits the information to the 3rd party server and receive the information from the server via external interface.

LG SmartHealth encrypts the password using SHA256 algorithm b(4)Trade Secret Process-Product Specs, complying with HIPAA requirements.

These medical devices that are sensor, monitoring and recording devices will all be FDA cleared or FDA Class I, 510(k) Exempt products that are recommended by LG SmartHealth. A list of the acceptable medical devices that can be used with LG SmartHealth will be provided in the User's Manual.

Device Type	Vendor	Model	Connection	K Number
Blood Pressure Monitor	A&D Medical	UA-767PBT	Bluetooth wireless device	K043217
	FORA	P 20 Blood Pressure Monitor IU-RIGHT TD-3 132 Blood Pressure Monitor	Bluetooth wireless device	K092106
	FORA	D30F: Fora Comfort 2 in 1 blood glucose plus blood pressure monitoring system	Bluetooth wireless device	K080014
Glucose Monitor	Bayer HealthCare	Contour	Wired device	K121190
	LifeScan, a Johnson & Johnson company	OneTouch Ultra Mini	USB wired device	K061118
	CareSens	CareSens N LINK Blood Glucose Monitoring System	Bluetooth wireless device	K130696
Weight Scale	Biospace	H20	Bluetooth wireless device	K123228
	TANIA	BC590 BT	Bluetooth wireless device	K014009
Thermometer	Fora	IR20b Type I /II	Bluetooth wireless device	K090395
Activity Monitors	LG Electronics	LifeGram	Bluetooth wireless device	Exempt

**Naming Convention:**

Please note that LG SmartHealth may also be referred to as "SmartHealth", "SmartHealth app", or "LG SmartHealth" in many of the attached documents.

**Indications for Use Statement:**

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).

Interface with the 3rd party server solution is required in order to provide the content and video conference.

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

**Mode of Operation and Mechanism of Action:**

(b) (4)



b(4) Trade Secret Process-Product Specs





b(4)Trade Secret Process-Product Specs



b(4) Trade Secret Process-Product Specs



## Section 16 – Software Description

**Device Name:** LG SmartHealth

**Device Description:**

“LG SmartHealth” is an application software (“application” or “app”) intended to operate on the Android operating system (“OS”) that can be loaded on commercially available tablet or smartphone. The scalability and differences between many tablets and smartphones today are not too great to allow this flexibility and option.

LG SmartHealth, an application for use by patients, supports the features of Measurement Management, Medication Management, Telemedicine, and Common Management, in interface with external 3rd party wireless medical devices and healthcare professional server solution.

LG SmartHealth connects to commercially available wireless medical devices that are commonly used by patients in a home-care setting. These “sensor devices” are FDA cleared or FDA registered (Exempt) devices that can communicate with the LG SmartHealth loaded on the smartphone Bluetooth connectivity. To avoid wireless interference and confusion on the part of the patient, serial readings are performed (i.e. only one Bluetooth sensor used at a time) and MAC address filtering is used for the various medical devices. LG SmartHealth plans to support the following sensor available medical devices, some initially, and then others as the technology become available:

- Glucose meters or glucometers
- Blood pressure cuffs
- Weight scales
- Body fat readers
- Activity monitors
- Thermometer

Pressing the measurement guide button in LG SmartHealth provides step-by-step interactive help guide to the user.

The medical device takes a reading depending on the frequency previously established and LG SmartHealth fetches the information for the reading automatically. LG SmartHealth stores and displays the information on the tablet or smartphone and transmits the information to the 3rd party server and receive the information from the server via external interface.

LG SmartHealth encrypts the password using SHA256 algorithm (b) (4) [REDACTED], complying with HIPAA requirements.

These medical devices that are sensor, monitoring and recording devices will all be FDA cleared or FDA Class I, 510(k) Exempt products that are recommended by LG SmartHealth. A list of the acceptable medical devices that can be used with LG SmartHealth will be provided in the User’s Manual.

### 16.1 Level of Concern:

Based on the Risk Management File (Exhibit 16-E) and the FDA Guidance document, *Guidance for the Content of Premarket Submissions for Software Contained in Medical Devices*, issued on May 11, 2005, on levels of concern LG has determined that the software level of concern for the LG SmartHealth device is **Moderate**. The rationale for the decision on the software level of concern is shown below in the tables extracted from the Food and Drug Administration guidance document.

**Table 16-A – Major Level of Concern Rationale**

<b>If the answer to any one question below is Yes, the Level of Concern for the Software Device is likely to be Major.</b>		
<b>Question</b>	<b>Answer</b>	<b>Rationale</b>
1. Does the Software Device qualify as Blood Establishment Computer Software?	NO	The device is not a Blood Establishment Computer Software.
2. Is the Software Device intended to be used in combination with a drug or biologic?	NO	The device is not intended to be used in conjunction with a drug or biologic.
3. Is the Software Device an accessory to a medical device that has a Major Level of Concern?	NO	The device is not an accessory to a medical device that has a major level of concern.
4. Prior to mitigation of hazards, could a failure of the Software Device result in death or serious injury, either to a patient or to a user of the device? Examples of this include the following:		
a. Does the Software Device control a life supporting or life sustaining function?	NO	The device does not control a life sustaining or life supporting function.
b. Does the Software Device control the delivery of potentially harmful energy that could result in death or serious injury, such as radiation treatment systems, defibrillators, and ablation generators?	NO	The software does not control the delivery of potentially harmful energy that could result in a death or serious injury.
c. Does the Software Device control the delivery of treatment or therapy such that an error or malfunction could result in death or serious injury?	NO	The device does not control the delivery of treatment or therapy such that an error or malfunction could result in death or serious injury.
d. Does the Software Device provide diagnostic information that directly drives a decision regarding treatment or therapy, such that if misapplied it could result in serious injury or death?	NO	The software device processes (displays) digital information for the assessment of treatment only. This information does not directly mandate or control a decision regarding treatment or therapy as all digital information must be interpreted by a competent medical person together with any additional clinical information required for treatment.
e. Does the Software Device provide vital signs monitoring and alarms for potentially life threatening situations in which medical intervention is necessary?	NO	The device does not provide any vital signs monitoring and/or alarms.

**Table 16-B – Moderate Level of Concern Rationale**

<b>If the answer to any one question below is Yes, the Level of Concern for the Software Device is likely to be Moderate.</b>
-------------------------------------------------------------------------------------------------------------------------------

<b>Question</b>	<b>Answer</b>	<b>Rationale</b>
1. Is the Software Device an accessory to a medical device that has a Moderate Level of Concern?	YES	The device is not an accessory to a medical device that has a major level of concern.
2. Prior to mitigation of hazards, could a failure of the Software Device result in Minor Injury, either to a patient or to a user of the device?	NO	A failure of the software device would not result in Minor Injury, either to a patient or to a user of the device if mitigation of hazards were not implemented.
3. Could a malfunction of, or a latent design flaw in, the Software Device lead to an erroneous diagnosis or a delay in delivery of appropriate medical care that would likely lead to Minor Injury?	YES	A malfunction or a latent design flaw in the software device could lead to an erroneous diagnosis that would lead to a minor injury.

b(4) Trade Secret Process-Software Specs



b

b(4) Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs





b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4) Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4) Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs





b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-  
S ft S

b(4) Trade Secret Process-Software Specs



b(4) Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4) Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs





b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs



b(4)Trade Secret Process-Software Specs

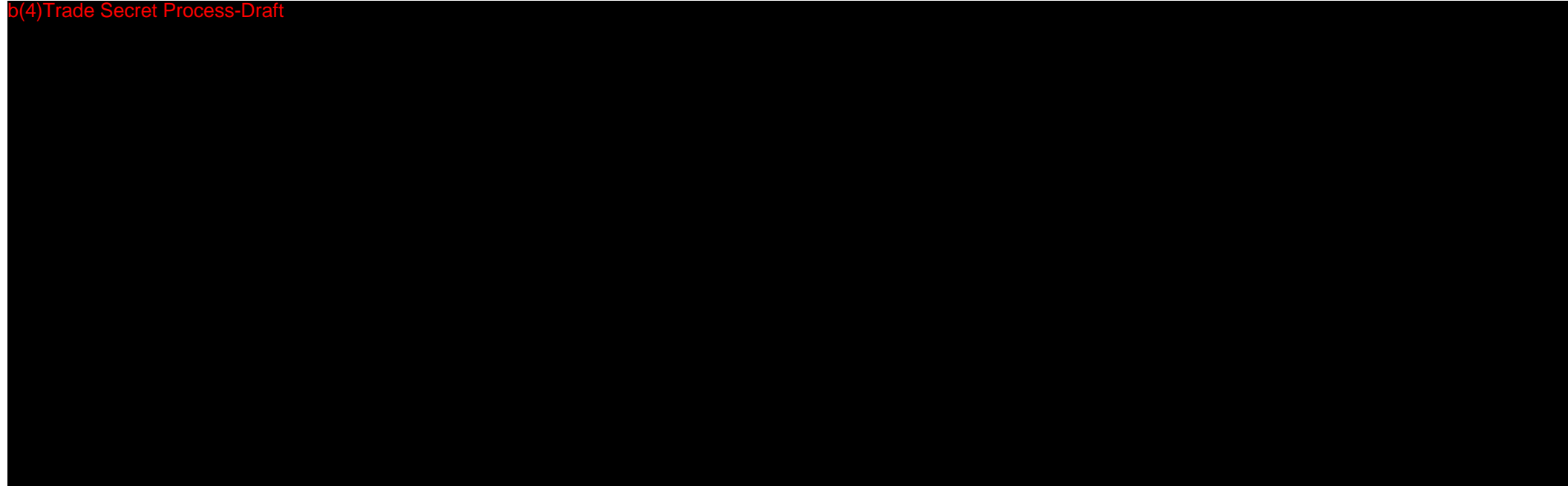


b(4) Trade Secret Process-Software Specs

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
# User Manual

LG SmartHealth Development












b(4)Trade Secret Process-Draft




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












b(4)Trade Secret Process-Draft



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
















# Software Requirements Specifications


LG SmartHealth Development

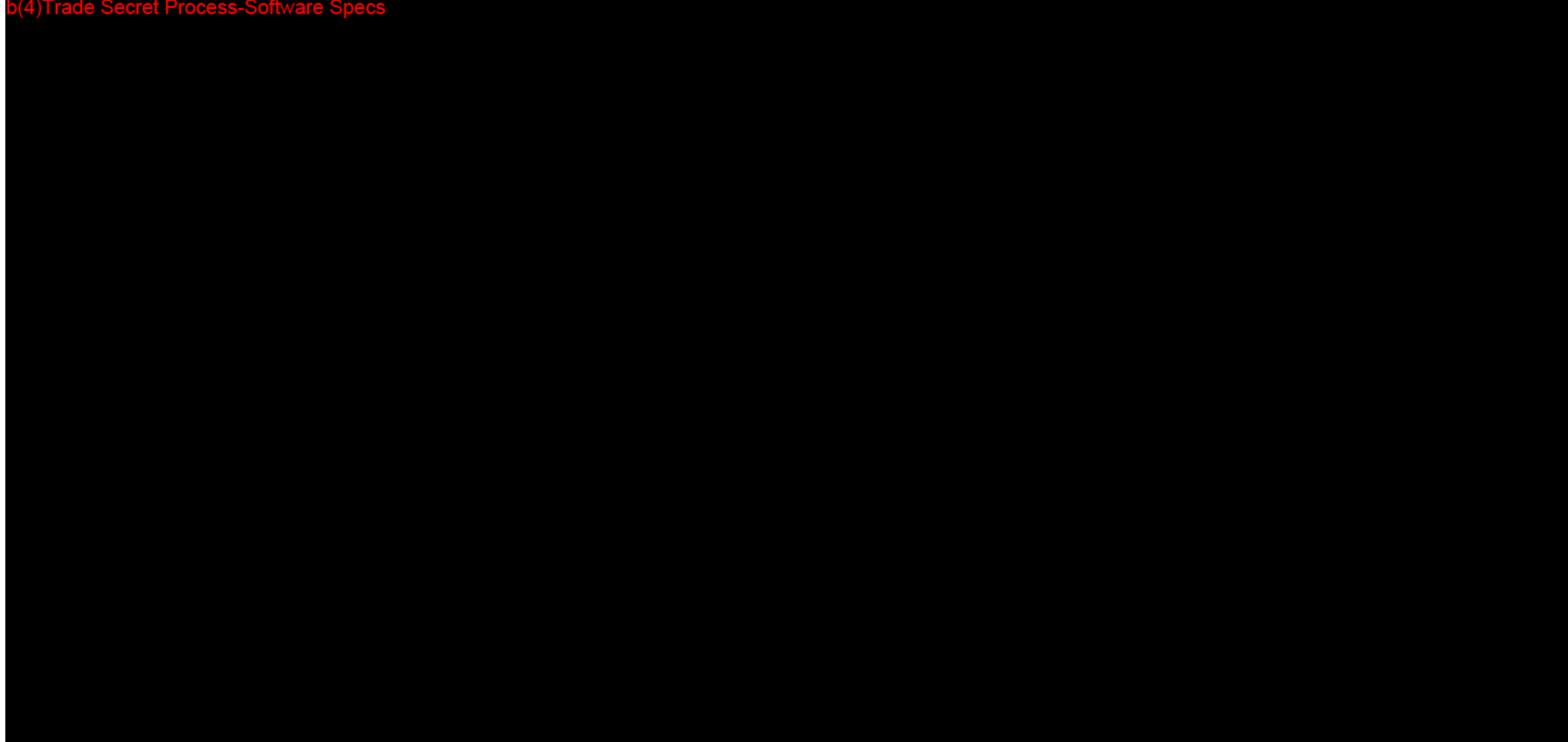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b(4)Trade Secret Process-Software Specs



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

















# Software Requirements Specifications

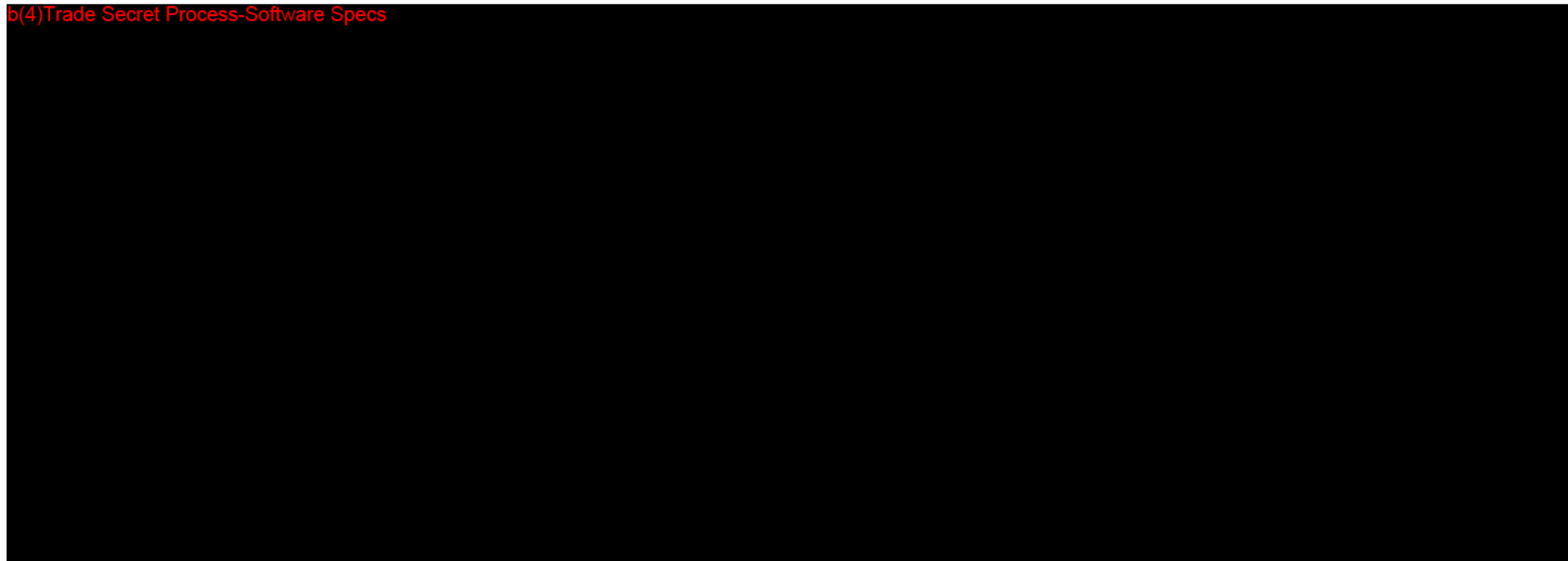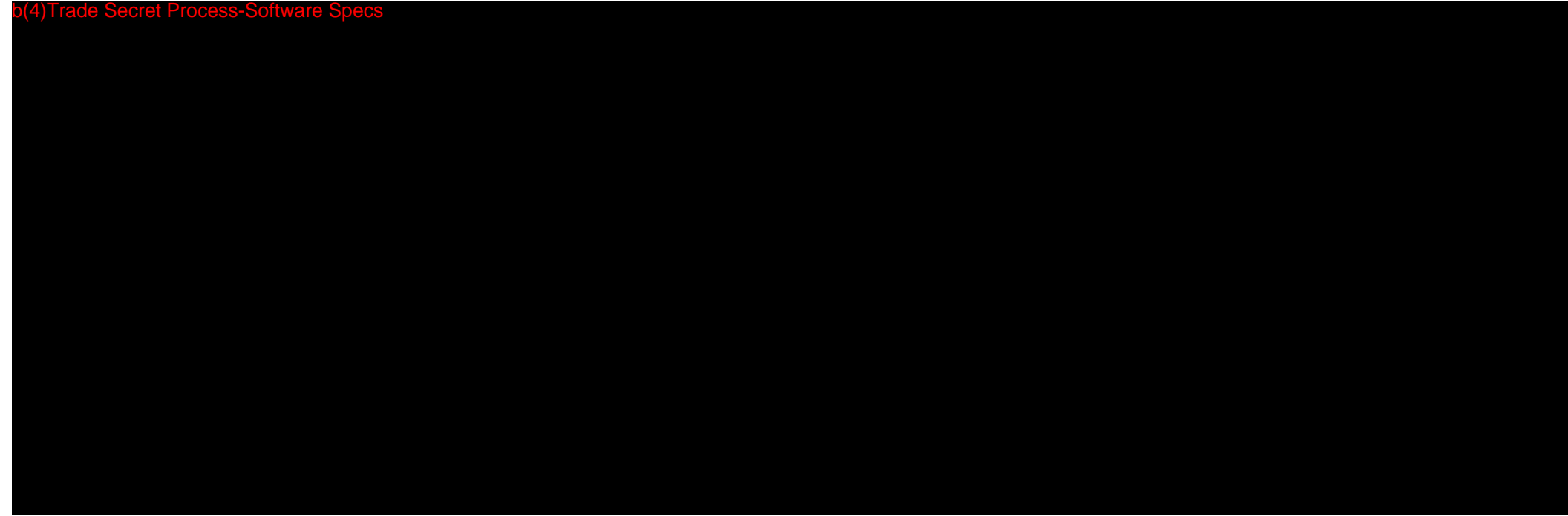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
LG SmartHealth Development





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
















b(4)Trade Secret Process-Software Specs

# Software Requirements Specifications

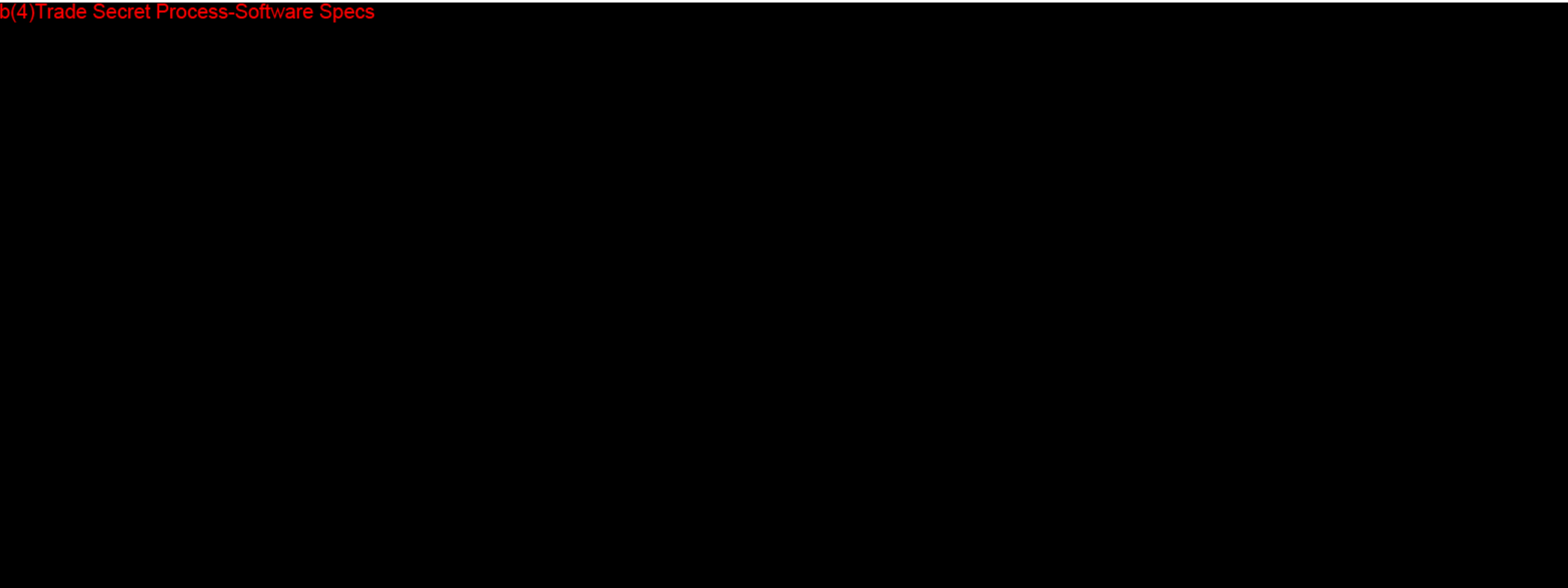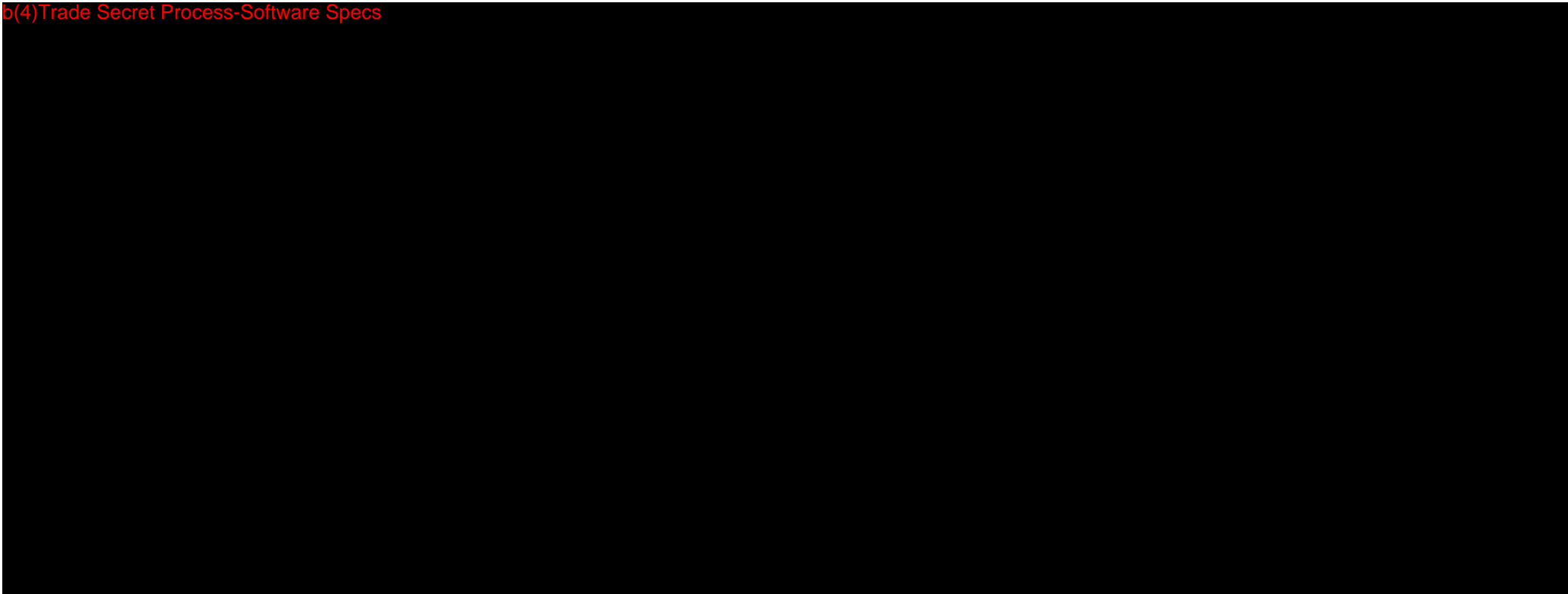
LG SmartHealth Development








b(4)Trade Secret Process-Software Specs

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

















b(4)Trade Secret Process-Software Specs




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b(4)Trade Secret Process-Software Specs










# Software Requirements Specifications

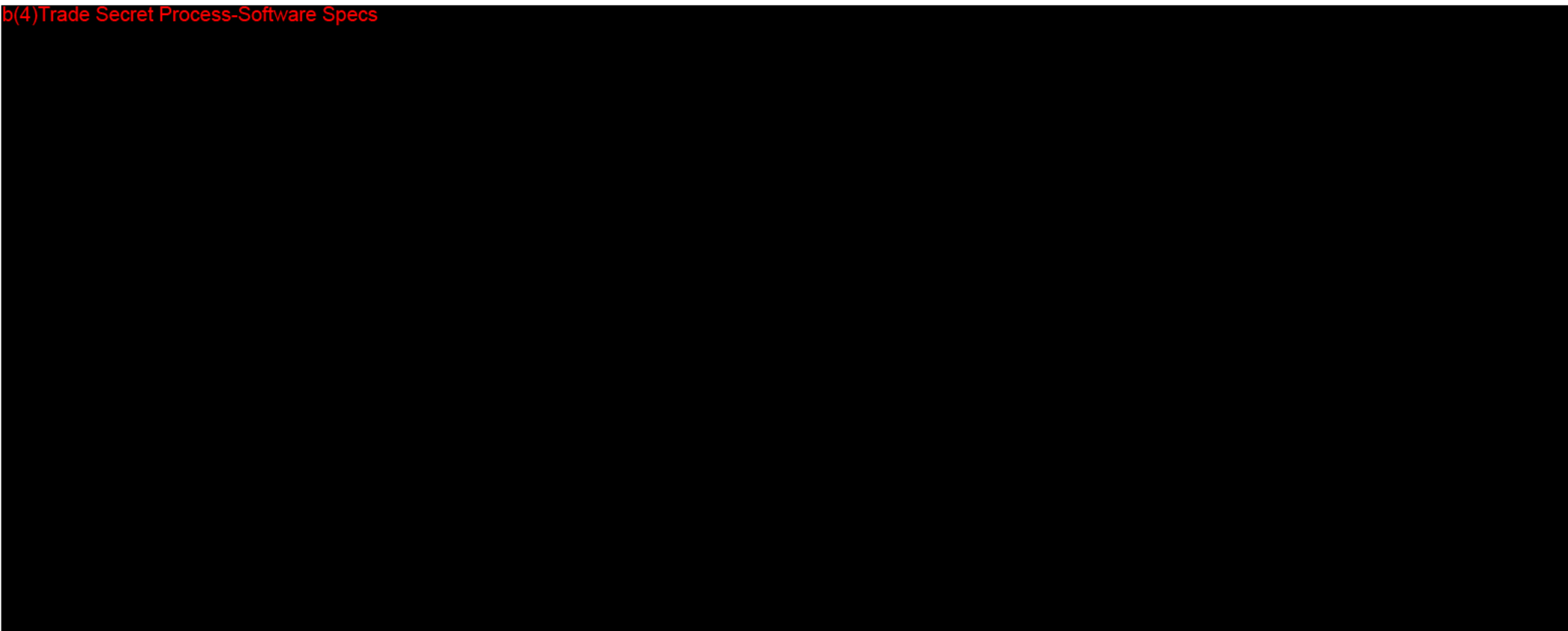
LG SmartHealth Development






b(4)Trade Secret Process-Software Specs



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














b(4)Trade Secret Process-Software Specs

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



# Software Architecture Design


LG SmartHealth Development

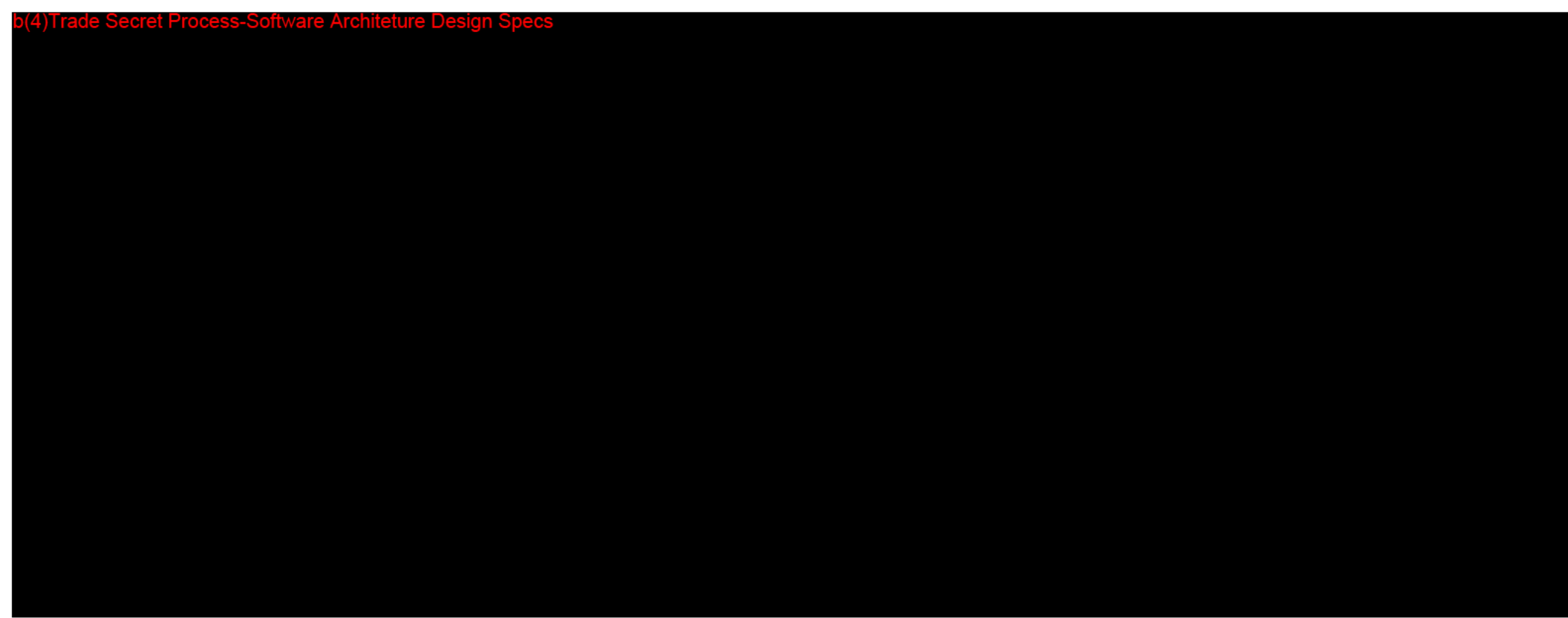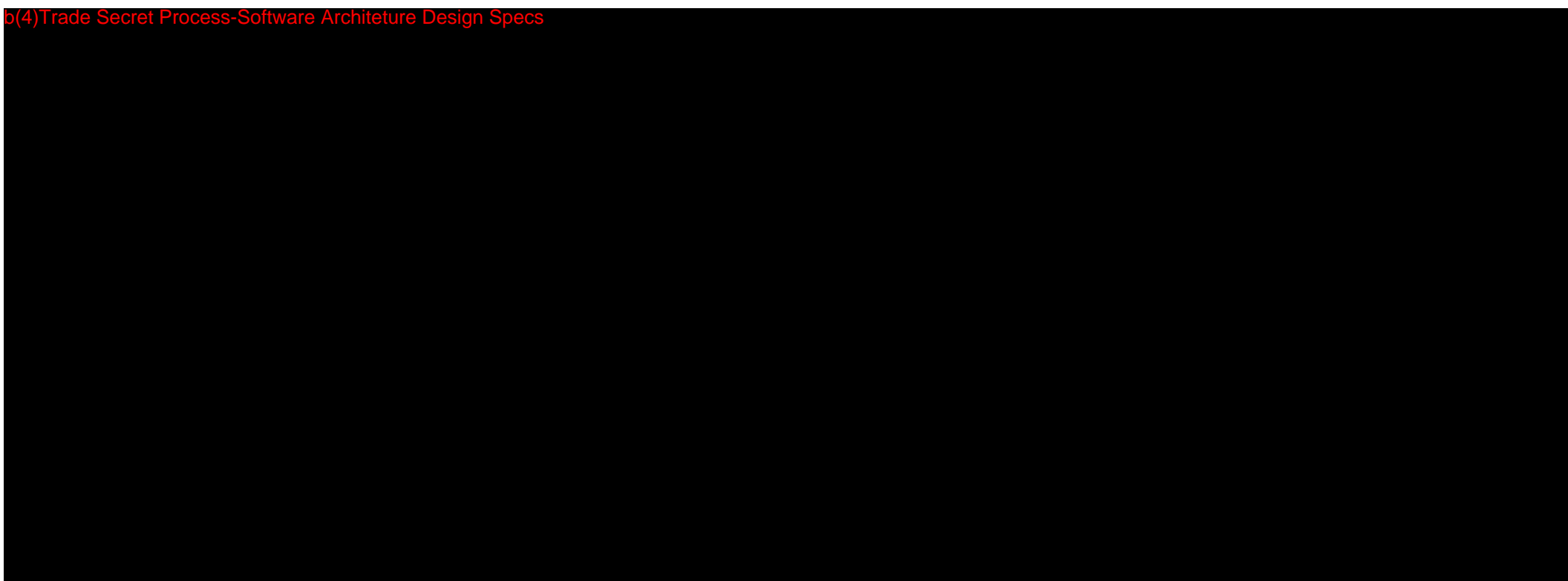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
b(4)Trade Secret Process-Software Architecture Design  
Specs












b(4)Trade Secret Process-Software Architeture Design Specs

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
b(4)Trade Secret Process-Software Architecture Design Specs







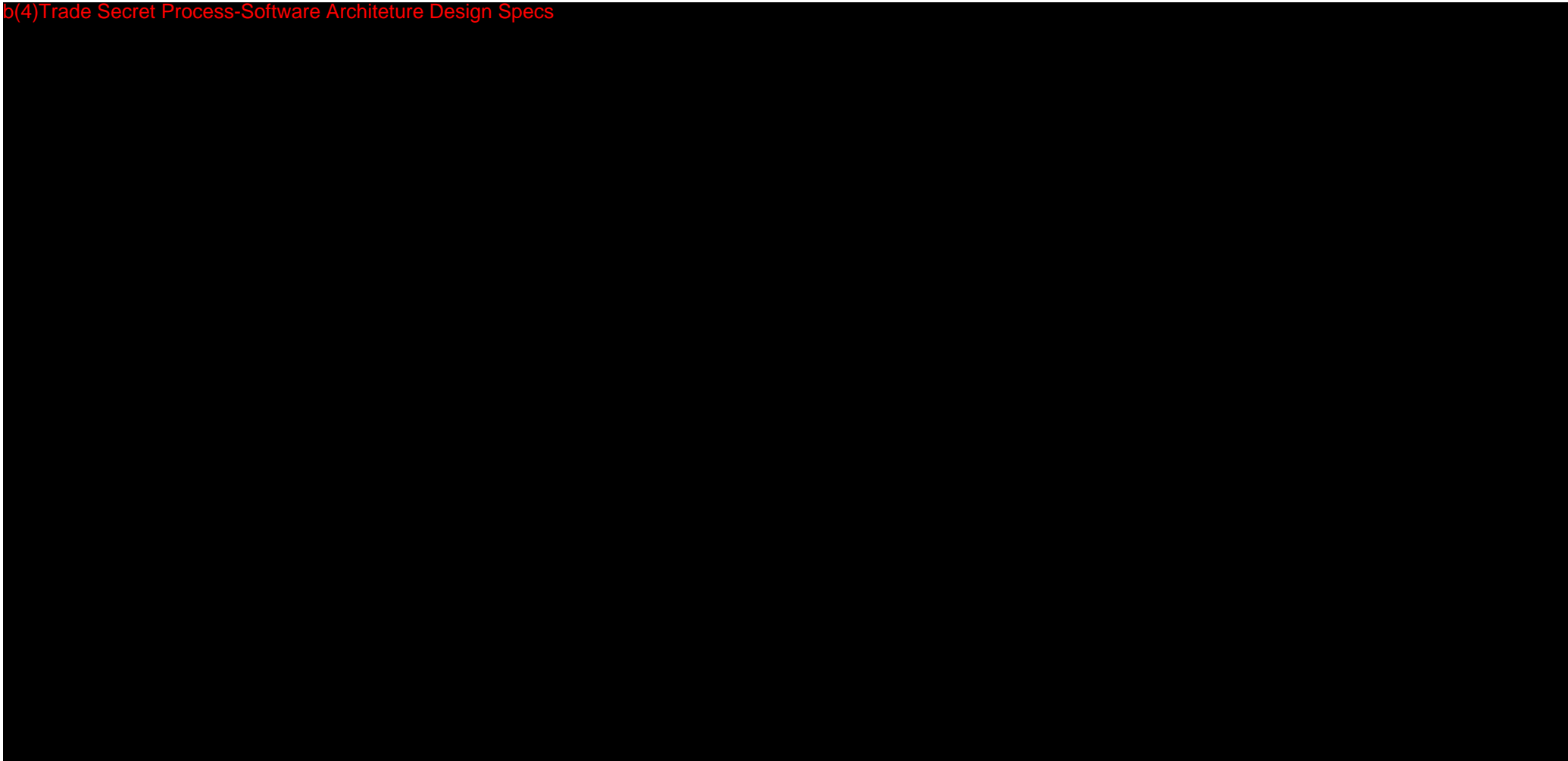b(4)Trade Secret Process-Software Architeture Design Specs



b(4)Trade Secret Process-Software Architeture Design Specs






# High Level Design

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LG SmartHealth Development

b(4)Trade Secret Process-  
Design Specs

b(4)Trade Secret Process-Design Specs

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














b(4)Trade Secret Process-Design Specs



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
b(4)Trade Secret Process-Design Specs

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















b(4)Trade Secret Process-Design Specs



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

b(4)Trade Secret Process-Design Specs

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








b(4)Trade Secret Process-Design Specs



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





b(4)Trade Secret Process-Design Specs







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



b(4)Trade Secret Process-Design Specs



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b(4)Trade Secret Process-Design Specs



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












b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



# High Level Design

LG SmartHealth Development

b(4)Trade Secret  
Process-Design

b(4)Trade Secret Process-Design Specs

b(4) Trade Secret Process-Design Specs











b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs





b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs





b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



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
b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs











b(4)Trade Secret Process-Design Specs



# High Level Design

LG SmartHealth Development

b(4)Trade Secret Process-Design Specs

b(4)Trade Secret Process-Design Specs

b(4)Trade Secret Process-Design Specs

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


b(4)Trade Secret Process-Design Specs

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












b(4)Trade Secret Process-Design Specs



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
b(4)Trade Secret Process-Design Specs





b(4) Trade Secret Process-Design Specs







b(4)Trade Secret Process-Design Specs



b(4) Trade Secret Process-Design Specs





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
b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs




b(4) Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



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



b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs





b(4) Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs







b(4) Trade Secret Process-Design Specs





# High Level Design

LG SmartHealth Development

b(4)Trade Secret Process-Design Specs

b(4)Trade Secret Process-Design Specs



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
b(4)Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs








b(4)Trade Secret Process-Design Specs



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b(4)Trade Secret Process-Design Specs








b(4)Trade Secret Process-Design Specs



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b(4)Trade Secret Process-Design Specs





b(4) Trade Secret Process-Design Specs



b(4)Trade Secret Process-Design Specs



b(4) Trade Secret Process-Design Specs



b(4) Trade Secret Process-Design Specs





b(4)Trade Secret Process-Design Specs





b(4)Trade Secret Process-Design Specs





b(4)Trade Secret Process-Design Specs



# Server Interface Specification

b(4)Trade Secret Process-Product Specs

b(4)Trade Secret Process-Product Specs



b(4) Trade Secret Process-Product Specs





b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs





b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs





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


b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs









b(4)Trade Secret Process-Product Specs



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



b(4) Trade Secret Process-Product Specs



b(4) Trade Secret Process-Product Specs

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


b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs







b(4)Trade Secret Process-Product Specs









# Requirements Traceability Matrix

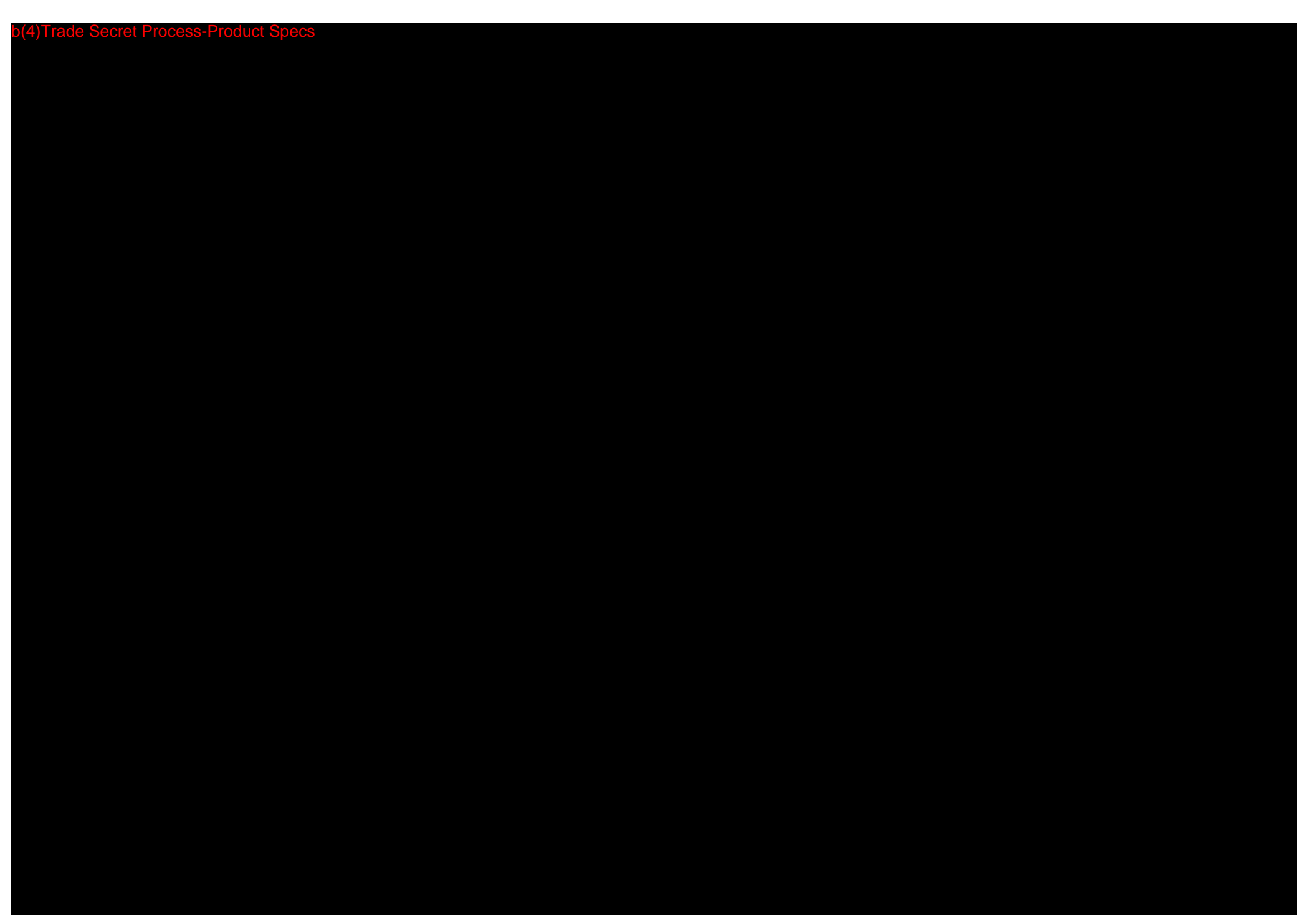
b(4)Trade Secret Process-Product Specs

b(4)Trade Secret Process-Product Specs

b(4) Trade Secret Process-Product Specs







b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs





b(4)Trade Secret Process-Risk  
Management Specs

## Risk Management File

b(4)Trade Secret Process-Risk  
Management Specs

b(4)Trade Secret Process-Risk Management Specs



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
b(4) Trade Secret Process-Risk Management Specs




b(4) Trade Secret Process-Risk Management Specs



b(4)Trade Secret Process-Risk Management Specs



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
b(4)Trade Secret Process-Risk Management Specs




b(4)Trade Secret Process-Risk Management Specs



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b(4)Trade Secret Process-Risk Management Specs



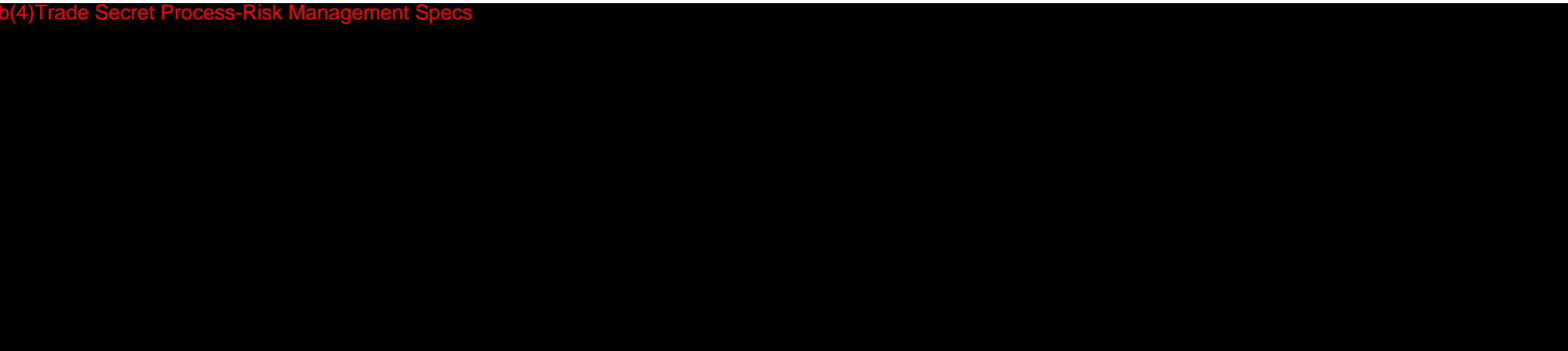b(4)Trade Secret Process-Risk Management Specs



b(4)Trade Secret Process-Risk Management Specs





b(4)Trade Secret Process-Risk Management Specs




b(4)Trade Secret Process-Risk Management Specs



# Integration/System Test Plan

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b(4)Trade Secret Process-Testing

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
b(4)Trade Secret Process-Testing



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














b(4)Trade Secret Process-Testing



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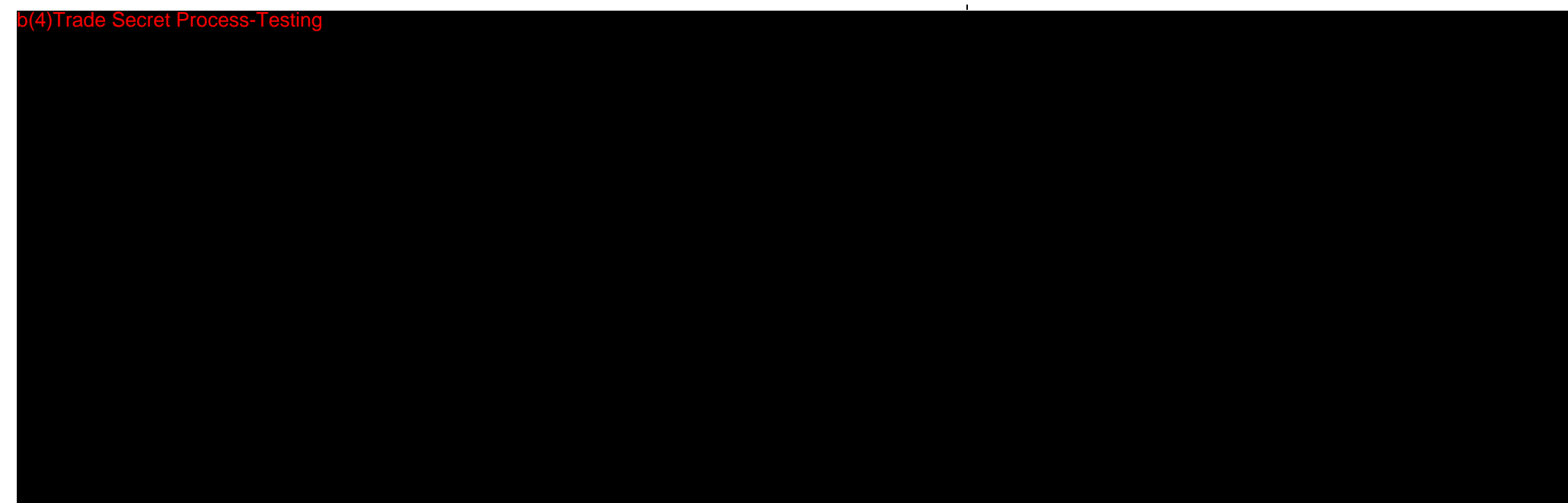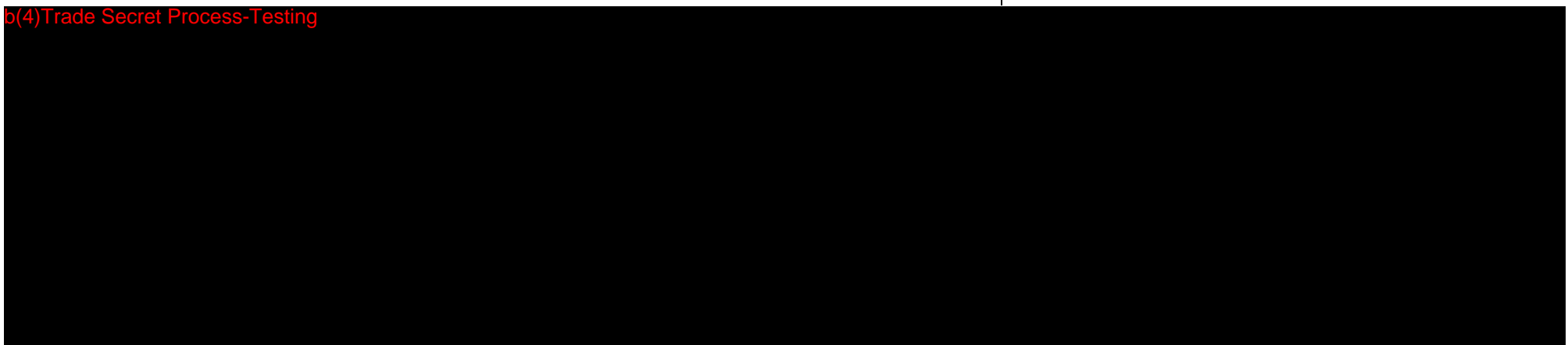














b(4) Trade Secret Process-Testing






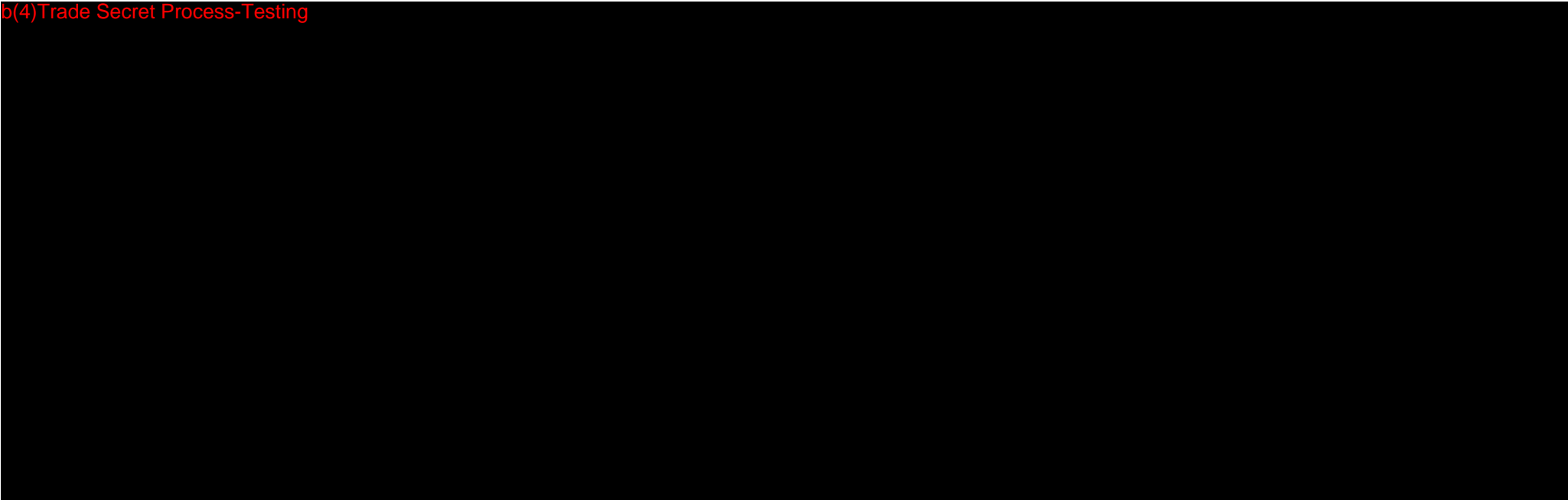





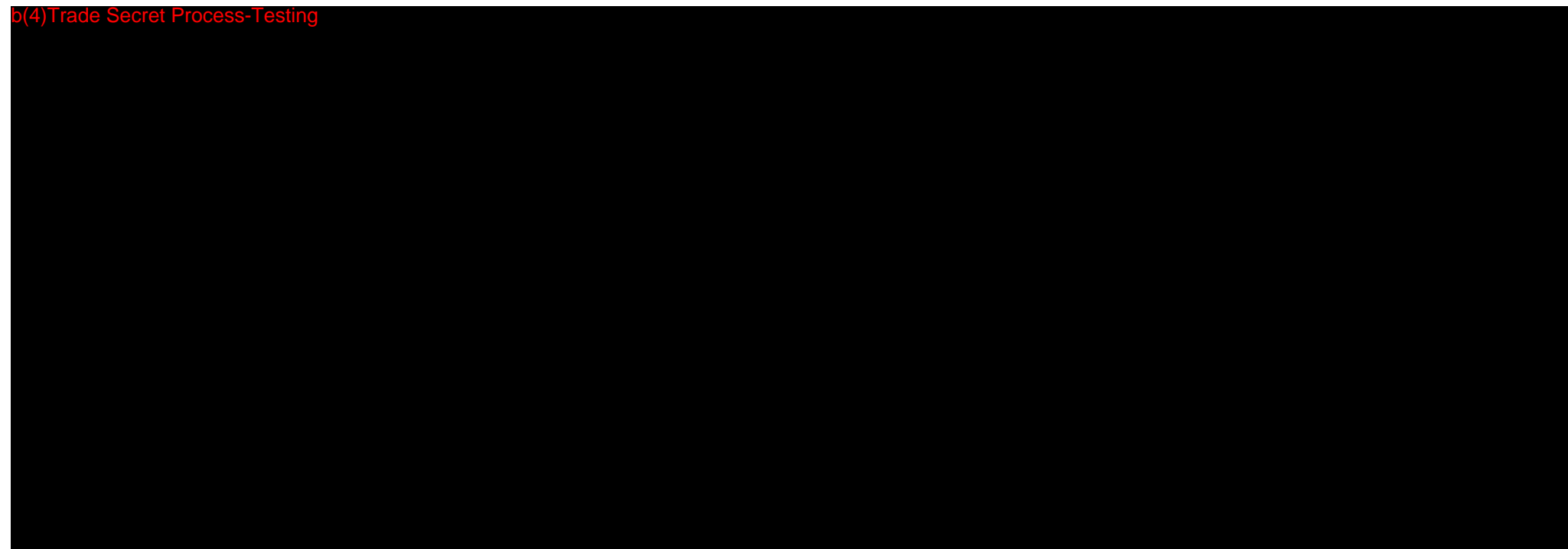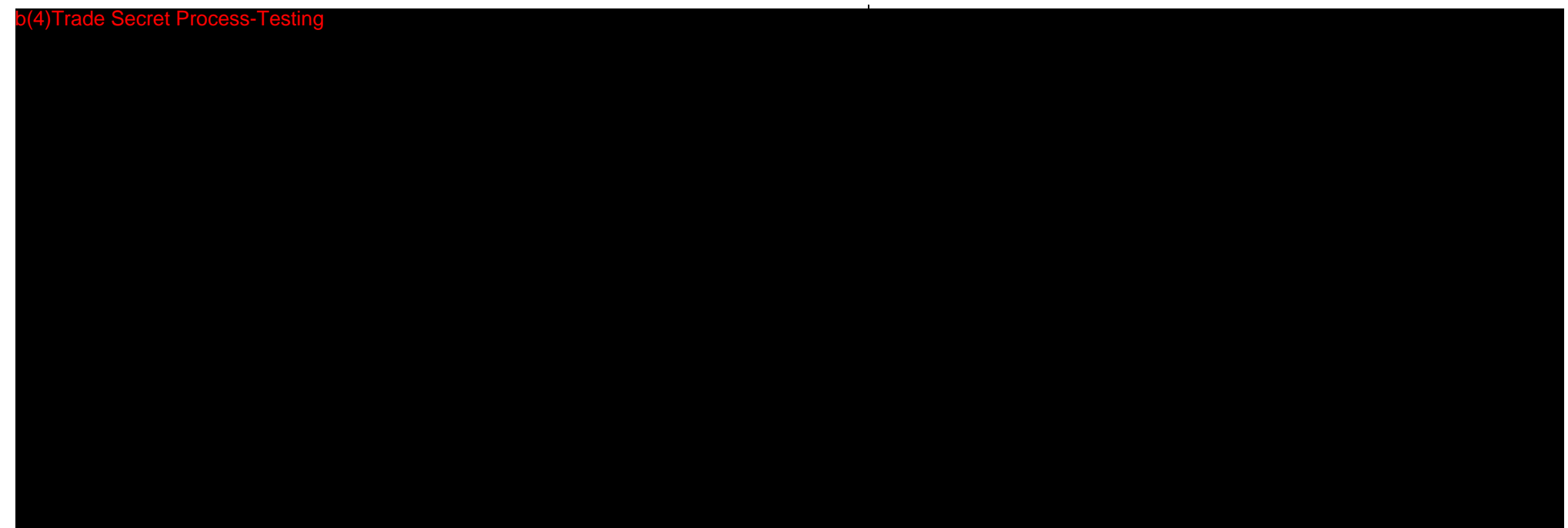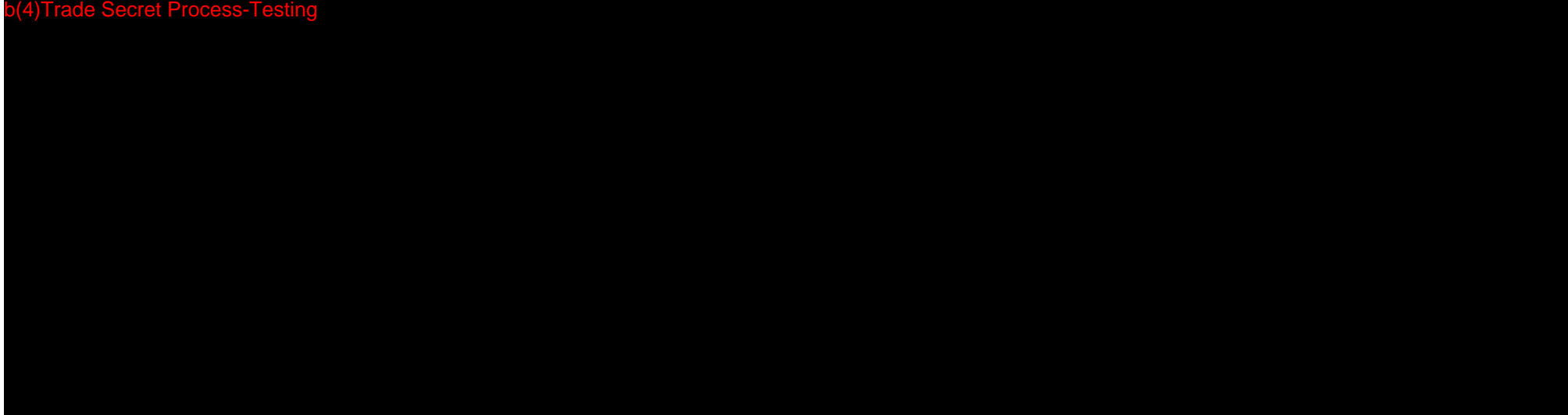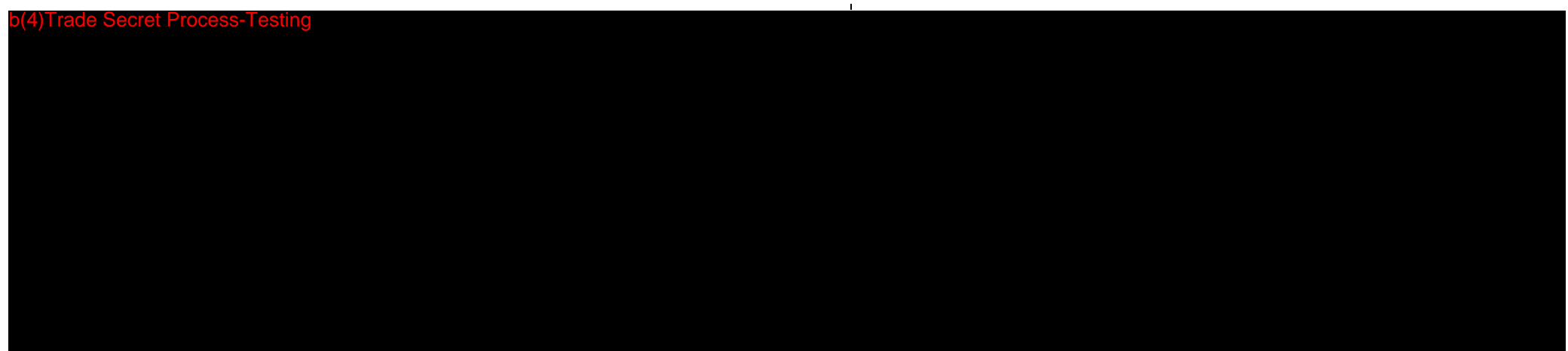











b(4)Trade Secret Process-Testing






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












b(4)Trade Secret Process-Testing



b(4) Trade Secret Process-Testing

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


b(4)Trade Secret Process-Testing

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


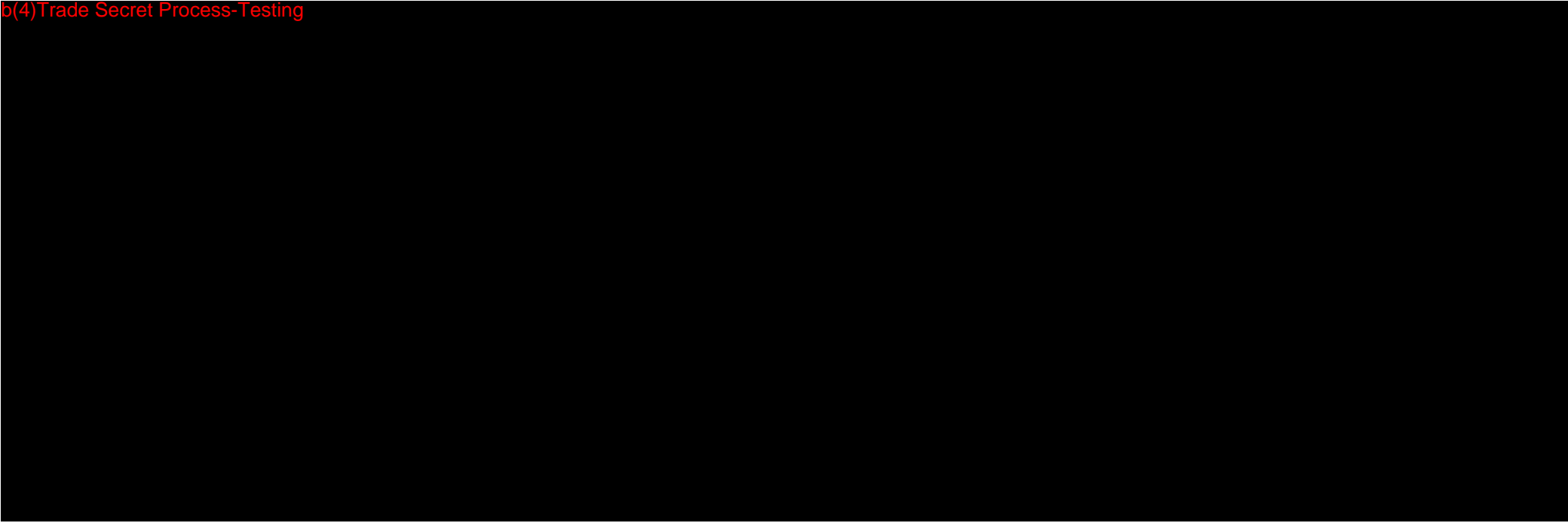
b(4)Trade Secret Process-Testing

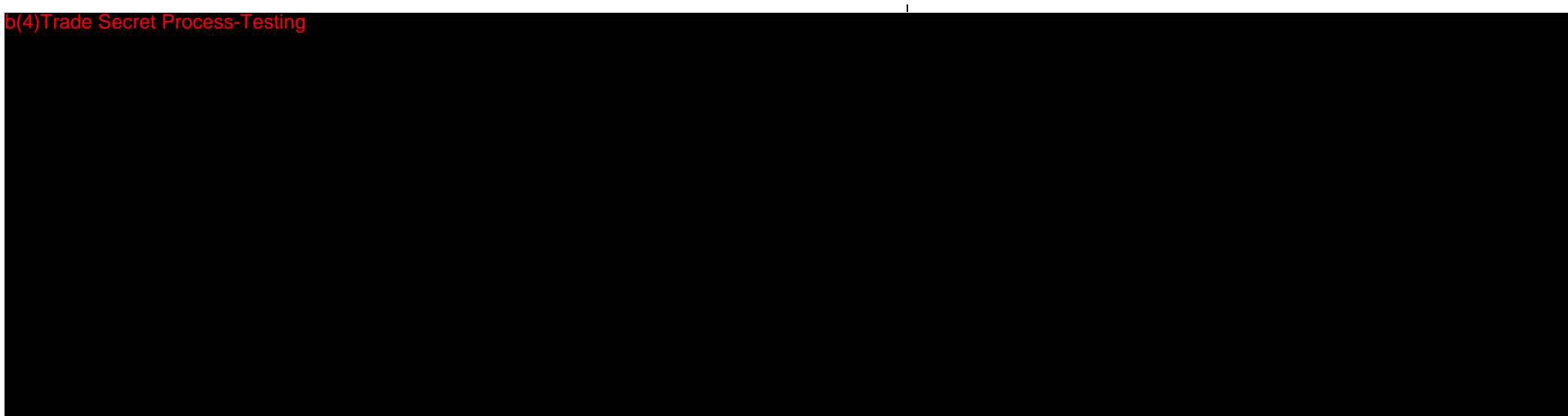


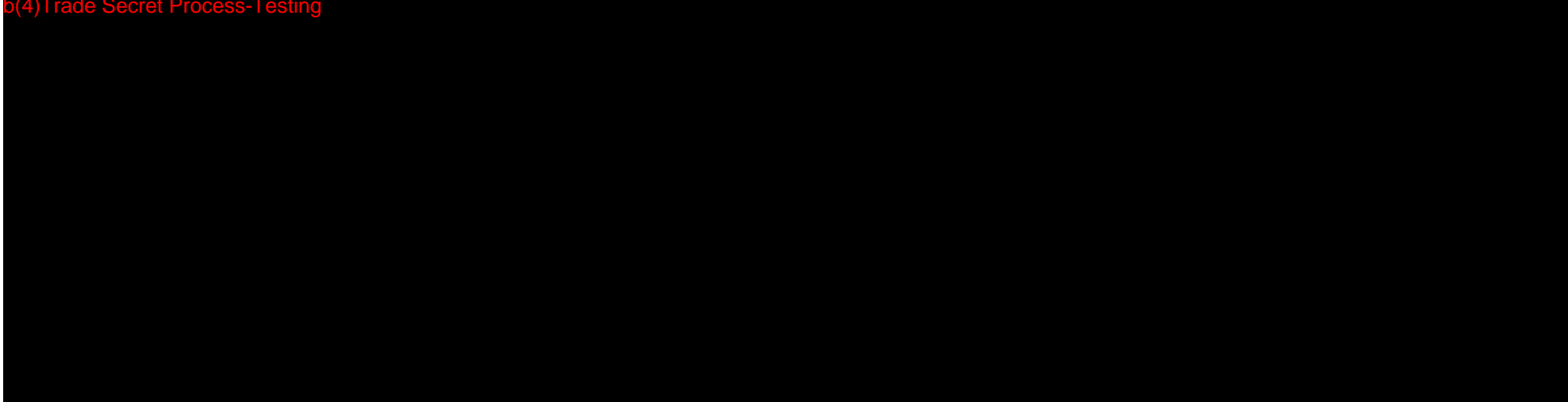














b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing





b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing




b(4)Trade Secret Process-Testing




# Integration/System Test Report(15<sup>th</sup>)

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b(4)Trade  
Secret  
P

b(4)Trade Secret Process-Testing





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














# Usability Testing Report

b(4)Trade Secret Process-Testing

b(4)Trade Secret Process-Testing



b(4)Trade Secret Process-Testing







b(4)Trade Secret Process-Testing





b(4)Trade Secret Process-Testing



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
b(4) Trade Secret Process-Testing

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






b(4)Trade Secret Process-Testing



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
















b(4)Trade Secret Process-Testing






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





b(4)Trade Secret Process-Testing

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



# Project Management Plan

b(4)Trade Secret Process-Product Specs

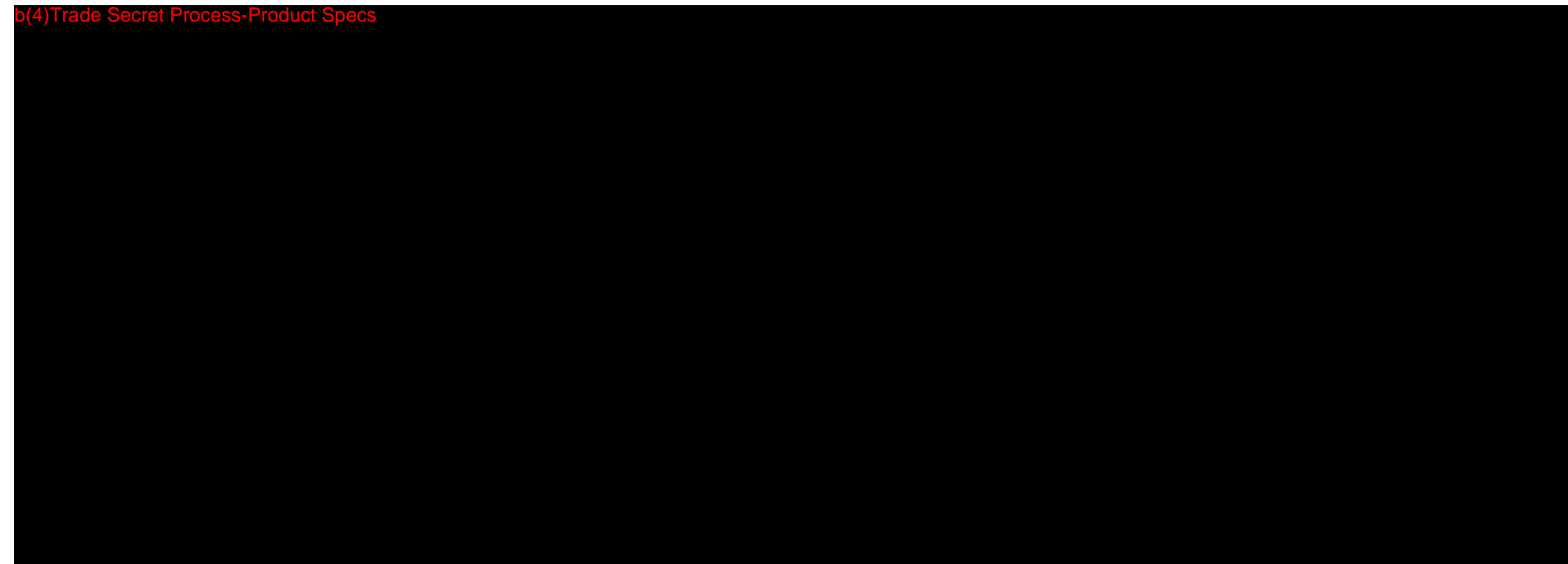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b(4)Trade Secret Process-Product Specs





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












b(4)Trade Secret Process-Product Specs


# Configuration Management Plan

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
b(4)Trade Secret Process-Product Specs

b(4)Trade Secret Process-Product Specs



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
b(4)Trade Secret Process-Product Specs

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




















b(4)Trade Secret Process-Product Specs

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b(4)Trade Secret Process-Product Specs











b(4) Trade Secret Process-Product Specs

b(4)Trade Secret Process-Product Specs



b(4)Trade Secret Process-Product Specs





Emergo Group, Inc.  
816 Congress Avenue, Suite 1400  
Austin, Texas 78701  
UNITED STATES  
Telephone: +1 512 327 9997

K133997/S3

November 19, 2014

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
U.S. Food and Drug Administration  
Building WO66, Room 2558  
10903 New Hampshire Avenue  
Silver Spring, MD 20993  
Attention: Ms. Linda Ricci

FDA CDRH DMC  
NOV 20 2014  
Received

RE: K133997 LG SmartHealth – Supplement S2

Dear Ms Ricci:

(b) (4)



SO 1-CD

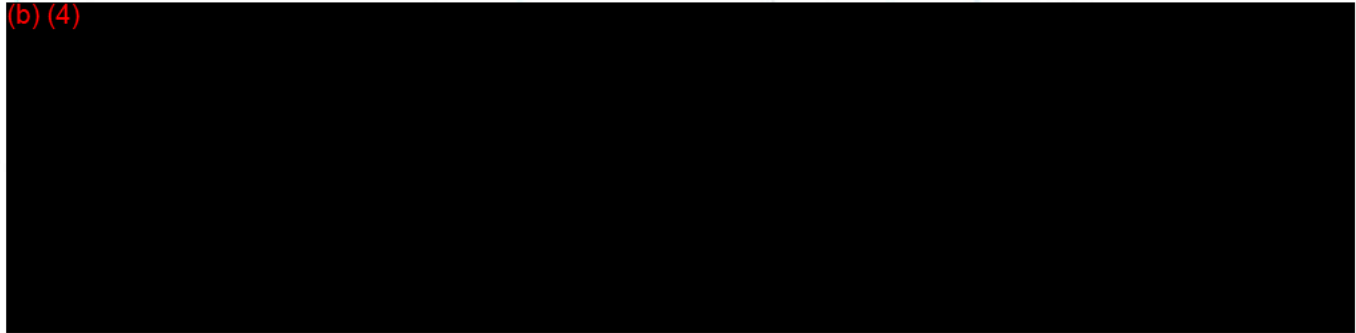
AR

ZA



Emergo Group, Inc.  
816 Congress Avenue, Suite 1400  
Austin, Texas 78701  
UNITED STATES  
Telephone: +1 512 327 9997

(b) (4)



Sincerely,

*Diane Sudduth*

Diane Sudduth  
Sr. Regulatory Consultant  
Emergo

b(4) Trade Secret Process-Product Specs





**Emergo Group, Inc.**

816 Congress Avenue, Suite 1400

Austin, Texas 78701

UNITED STATES

Telephone: +1 512 327 9997

November 19, 2014

U.S. Food and Drug Administration  
Center for Devices and Radiological Health  
U.S. Food and Drug Administration  
Building WO66, Room 2558  
10903 New Hampshire Avenue  
Silver Spring, MD 20993  
Attention: Ms. Linda Ricci

RE: K133997 LG SmartHealth – Supplement S2

Dear Ms Ricci:

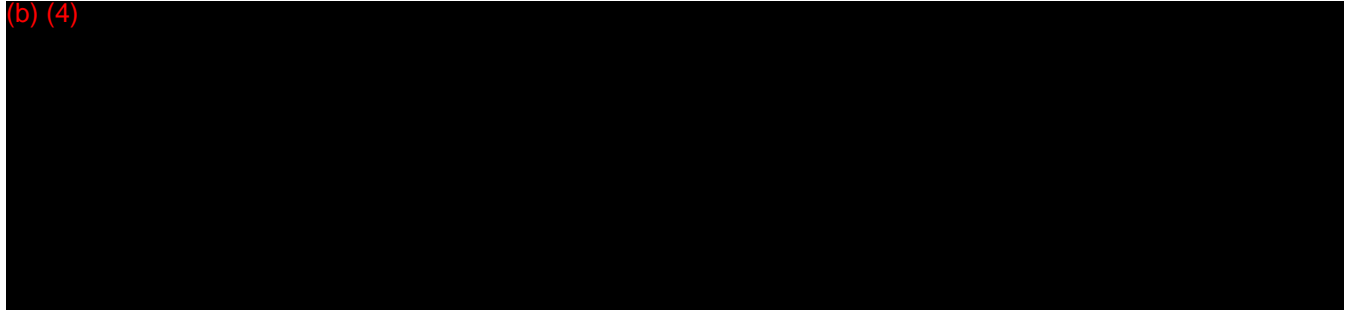
(b) (4)

A large, solid black rectangular redaction box covers the majority of the page's content, starting below the salutation and extending to the bottom of the page.



**Emergo Group, Inc.**  
816 Congress Avenue, Suite 1400  
Austin, Texas 78701  
UNITED STATES  
Telephone: +1 512 327 9997

(b) (4)



Sincerely,

Diane Sudduth  
Sr. Regulatory Consultant  
Emergo

b(4)Trade  
Secret  
Process-  
Product  
Specs

---

(b) (4)

From: Ricci, Linda J [Linda.Ricci@fda.hhs.gov]  
Sent: Friday, August 08, 2014 12:53 PM  
To: Diane Sudduth @ Emergo Group; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 **b(4)Trade Secret Process-Product Specs**

Hello Diane,

(b) (4)

Best regards,  
--Linda

Linda Ricci  
Cardiac Diagnostic Devices Branch Chief  
Division of Cardiovascular Devices  
FDA/CDRH/ODE  
10903 New Hampshire Ave, Bldg 66 Rm 1314  
Silver Spring, MD 20993-0002  
301-796-6325  
linda.ricci@fda.hhs.gov

Excellent customer service is important to us. Please take a moment to provide feedback regarding the customer service you have received: <https://www.research.net/s/cdrhcustomerservice?O=400&D=440&B=442&E=&S=E>

-----Original Message-----

From: Diane Sudduth @ Emergo Group [<mailto:DSudduth@emergogroup.com>]  
Sent: Friday, August 01, 2014 12:03 AM  
To: Ricci, Linda J; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 **b(4)Trade Secret Process-Product Specs**

Dear Linda,

(b) (4)



(b) (4)

Kindest regards,

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

From: Diane Sudduth @ Emergo Group  
Sent: Tuesday, July 22, 2014 11:41 AM  
To: Ricci, Linda J; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 (b) (4)

Linda,

(b) (4)

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

From: Ricci, Linda J [Linda.Ricci@fda.hhs.gov]  
Sent: Tuesday, July 22, 2014 11:15 AM  
To: Diane Sudduth @ Emergo Group; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 (b) (4)

Diane,

(b) (4)

Linda Ricci  
Cardiac Diagnostic Devices Branch Chief  
Division of Cardiovascular Devices  
FDA/CDRH/ODE  
10903 New Hampshire Ave, Bldg 66 Rm 1314  
Silver Spring, MD 20993-0002  
301-796-6325  
linda.ricci@fda.hhs.gov

Excellent customer service is important to us. Please take a moment to provide feedback regarding the customer service you have received: <https://www.research.net/s/cdrhcustomerservice?O=400&D=440&B=442&E=&S=E>

-----Original Message-----

From: Diane Sudduth @ Emergo Group [<mailto:DSudduth@emergogroup.com>]  
Sent: Tuesday, July 22, 2014 11:11 AM  
To: Ricci, Linda J; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group

Subject: RE: K133997 (b) (4)

Linda,

(b) (4)

Regards,

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

From: Ricci, Linda J [Linda.Ricci@fda.hhs.gov]  
Sent: Tuesday, July 22, 2014 10:15 AM  
To: Diane Sudduth @ Emergo Group; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 (b) (4)

Good morning Diane,

(b) (4)

Linda Ricci  
Cardiac Diagnostic Devices Branch Chief  
Division of Cardiovascular Devices  
FDA/CDRH/ODE  
10903 New Hampshire Ave, Bldg 66 Rm 1314  
Silver Spring, MD 20993-0002  
301-796-6325  
linda.ricci@fda.hhs.gov

Excellent customer service is important to us. Please take a moment to provide feedback regarding the customer service you have received: <https://www.research.net/s/cdrhcustomerservice?O=400&D=440&B=442&E=&S=E>

-----Original Message-----

From: Diane Sudduth @ Emergo Group [mailto:DSudduth@emergogroup.com]  
Sent: Tuesday, July 22, 2014 8:56 AM  
To: Ricci, Linda J; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 (b) (4)

Linda,

(b) (4)

Kindest regards,

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

From: Diane Sudduth @ Emergo Group  
Sent: Wednesday, June 25, 2014 3:26 PM  
To: Ricci, Linda J; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 (b) (4)

Linda,

**b(4)Trade Secret Process-Product Specs**

Regards,

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

From: Ricci, Linda J [Linda.Ricci@fda.hhs.gov]  
Sent: Wednesday, June 25, 2014 2:46 PM  
To: Diane Sudduth @ Emergo Group; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 **b(4)Trade Secret Process-Product Specs**

Hi Diane,

**b(4)Trade Secret Process-Product Specs**

Best Regards,

--Linda

Linda Ricci  
Cardiac Diagnostic Devices Branch Chief  
Division of Cardiovascular Devices  
FDA/CDRH/ODE  
10903 New Hampshire Ave, Bldg 66 Rm 1314  
Silver Spring, MD 20993-0002  
301-796-6325  
linda.ricci@fda.hhs.gov

Excellent customer service is important to us. Please take a moment to provide feedback regarding the customer service you have received: <https://www.research.net/s/cdrhcustomerservice?O=400&D=440&B=442&E=&S=E>

-----Original Message-----

From: Diane Sudduth @ Emergo Group [<mailto:DSudduth@emergogroup.com>]  
Sent: Wednesday, June 25, 2014 9:57 AM  
To: Ricci, Linda J; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 **b(4)Trade Secret Process-Product Specs**

Dear Linda,

**b(4)Trade Secret Process-Product Specs**

Kindest regards,

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

From: Ricci, Linda J [Linda.Ricci@fda.hhs.gov]  
Sent: Wednesday, June 25, 2014 9:22 AM  
To: Diane Sudduth @ Emergo Group; Luciana Dall Astra @ Emergo Group; Tovar-Calderon, Oscar  
Subject: RE: K133997 **b(4)Trade Secret Process-Product Specs**

**b(4)Trade**

Linda Ricci  
Cardiac Diagnostic Devices Branch Chief  
Division of Cardiovascular Devices

FDA/CDRH/ODE  
10903 New Hampshire Ave, Bldg 66 Rm 1314  
Silver Spring, MD 20993-0002  
301-796-6325  
linda.ricci@fda.hhs.gov

Excellent customer service is important to us. Please take a moment to provide feedback regarding the customer service you have received: <https://www.research.net/s/cdrhcustomerservice?O=400&D=440&B=442&E=&S=E>

-----Original Message-----

From: Diane Sudduth @ Emergo Group [<mailto:DSudduth@emergogroup.com>]  
Sent: Monday, June 23, 2014 6:00 PM  
To: Ricci, Linda J; Luciana Dall Astra @ Emergo Group; Tovar-Calderon, Oscar  
Subject: RE: K133997 (b) (4)  
Importance: High

Dear Linda,

(b) (4)

[Redacted]

[Redacted]

[Redacted]

[Redacted]

Kindest regards,

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

From: Ricci, Linda J [[Linda.Ricci@fda.hhs.gov](mailto:Linda.Ricci@fda.hhs.gov)]  
Sent: Wednesday, June 18, 2014 4:52 PM  
To: Luciana Dall Astra @ Emergo Group; Diane Sudduth @ Emergo Group; Tovar-Calderon, Oscar  
Subject: Re: K133997 (b) (4)

That will be great.

Thanks!

----- Original Message -----

From: Luciana Dall Astra @ Emergo Group [<mailto:ldallastra@emergogroup.com>]  
Sent: Wednesday, June 18, 2014 04:14 PM  
To: Ricci, Linda J; Diane Sudduth @ Emergo Group <[DSudduth@emergogroup.com](mailto:DSudduth@emergogroup.com)>; Tovar-Calderon, Oscar  
Subject: RE: K133997 (b) (4)

Good afternoon Linda,

(b) (4)

[Redacted]

Thanks!

Best regards,

Luciana Dall'Astra, PMP  
EMERGO GROUP | International Project Manager  
US Office: +1.512.600.7617

This electronic mail message contains information which is PRIVILEGED, PROPRIETARY AND CONFIDENTIAL and is intended only for the use of the Addressee(s) named herein. No part of this document or any attachments may be reproduced or transmitted without permission from Emergo Group, Inc. The services of Emergo Group Inc., its subsidiaries and/or affiliates, are offered only in jurisdictions where they may be lawfully sold. Services provided in non-US jurisdictions may be provided by Emergo Group, Inc.'s subsidiaries and/or affiliates.

-----Original Message-----

From: Ricci, Linda J [<mailto:Linda.Ricci@fda.hhs.gov>]  
Sent: Wednesday, June 18, 2014 11:52 AM  
To: Diane Sudduth @ Emergo Group; Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 (b) (4)

Dear Diane,

(b) (4)

Regards,  
--Linda

Linda Ricci  
Cardiac Diagnostic Devices Branch Chief  
Division of Cardiovascular Devices  
FDA/CDRH/ODE  
10903 New Hampshire Ave, Bldg 66 Rm 1314 Silver Spring, MD 20993-0002  
301-796-6325  
[linda.ricci@fda.hhs.gov](mailto:linda.ricci@fda.hhs.gov)

Excellent customer service is important to us. Please take a moment to provide feedback regarding the customer service you have received: <https://www.research.net/s/cdrhcustomerservice?O=400&D=440&B=442&E=&S=E>

-----Original Message-----

From: Diane Sudduth @ Emergo Group [<mailto:DSudduth@emergogroup.com>]  
Sent: Wednesday, June 18, 2014 12:40 PM  
To: Tovar-Calderon, Oscar; Ricci, Linda J  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 (b) (4)

Dear Linda,

(b) (4)

Kindest regards,

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

From: Diane Sudduth @ Emergo Group  
Sent: Tuesday, June 17, 2014 9:13 AM  
To: Tovar-Calderon, Oscar  
Cc: Luciana Dall Astra @ Emergo Group  
Subject: RE: K133997 (b) (4)

Dear Oscar,

(b) (4)

Regards,

Dr. Diane Sudduth, MS, MPH  
EMERGO GROUP | Senior Regulatory Consultant  
Phone: 561.305.5075  
Online: <http://www.EmergoGroup.com>

---

**510(k) Summary**  
**for**  
**LG SmartHealth**

**1. Submission Sponsor**

LG Electronics  
38 Baumoe-ro, Seocho-gu  
Seoul, 137-724  
Korea  
Phone: +82-2-526-4527  
Fax: +82-2-526-4370  
Contact: Chang Hee Lee, Chief Research Engineer

**2. Submission Correspondent**

Emergo Group  
816 Congress Avenue, Suite 1400  
Austin, TX 78701  
Cell Phone: (561) 305-5075  
Office Phone: (512) 327.9997  
Fax: (512) 327.9998  
Contact: Dr. Diane Sudduth, Senior Consultant, QA  
Email: [project.management@emergogroup.com](mailto:project.management@emergogroup.com)

**3. Date Prepared**

December 21, 2013

**4. Device Identification**

Trade/Proprietary Name: LG SmartHealth  
Common/Usual Name: Remote Patient Monitoring System  
Classification Name: Transmitters and Receivers, Physiological Signal,  
Radiofrequency  
Classification Regulation: 870.2910  
Product Code: Product Code DRG  
Device Class: Class II  
Classification Panel: Cardiovascular

**5. Legally Marketed Predicate Device(s)**

Intel Health Guide Express (K103276)

## 6. Device Description

The LG SmartHealth application is software planned to operate on the Android operating system (OS) that can be loaded as an application or “app” on a commercially available smartphone. LG intended the software to work on the Android OS, as well as to operate on smartphones as well. The scalability and differences between many smartphones today are not too great to allow this flexibility and option.

The LG SmartHealth application connects to commercially available wireless medical devices that are commonly used by patients in a home-care setting. These “sensor devices” are FDA cleared or FDA registered (Exempt) devices that can communicate with the LG SmartHealth application software loaded on the smartphone using Bluetooth connectivity. To avoid wireless interference and confusion on the part of the patient, serial readings are performed (i.e. only one Bluetooth sensor used at a time) and MAC address filtering is used for the various medical devices. The LG SmartHealth application plans to support the following sensor available medical devices, some initially, and others as the technology becomes available:

- Glucose meters or glucometers
- Blood pressure cuffs
- Weight scales
- Body fat readers
- Activity monitors

On each screen of the LG SmartHealth application software a help button is present, which when pressed provides step-by-step based help to guide a patient through an interaction.

The medical device takes a reading depending on the frequency previously established and the LG SmartHealth application software fetches the information for the reading automatically. The LG SmartHealth application software stores and displays the information on the smartphone and transmits the information to the server. The LG SmartHealth application encrypts the information in preparation for transmit using SHA256 bit encryption, complying with HIPAA requirements.

## 7. Indication for Use Statement

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing



its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

## 8. Substantial Equivalence Discussion

The comparison chart below provides evidence to facilitate the substantial equivalence determination between LG SmartHealth to the predicate device (K103276) with respect to intended use, technological characteristics and principles of operation.

**Table 5A – Comparison of Characteristics**

Features and Specifications	Intel® Health Guide Express (K103276)	LG SmartHealth (Subject Device)
<b>Regulatory Information</b>		
<b>Intended Use/ Indications for Use</b>	<p>The Intel® Health Guide Express is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers. Patients can also engage in video conferences with caregivers and answer the caregivers' questions by participating in surveys.</p> <p>The Intel® Health Care Management Suite allows the caregiver to review patient data and initiate video conferencing with patients, or select and send educational and motivational content to patients.</p> <p>The Intel® Health Guide Express is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The Intel® Health Guide Express is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>	<p>The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home.</p> <p>Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers (or nurse). Patients can also engage in video conferences with caregivers (or nurse).</p> <p>The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.</p> <p>The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.</p>
<b>OTC and/or Rx</b>	Rx	Same

User Characteristics		
<b>Intended User</b>	Patient in Home Setting	Same
Device		
<b>Components</b>	<ul style="list-style-type: none"> <li>Web based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>	<ul style="list-style-type: none"> <li>Smart phone app based software application</li> <li>Data Transfer Tool (Integrated)</li> </ul>
<b>User Interface Device</b>	Computer Device (i.e. desktop, laptop)	Android Device(i.e. Smart phone, tablet)
<b>Connectivity to Device</b>	Bluetooth or USB wired	Bluetooth
<b>Ability to Update Device Settings</b>	Not available	Automatic deploy
<b>Connectivity to Web-based Services</b>	Login Secured by SSL Encryption	Same
<b>Operating System</b>	Microsoft Windows 7 (32-bit versions)	Android 2.3.3 or higher
<b>Supported Device Types</b>	Medical devices designed for home use: Blood glucose meter Weight scale Blood pressure Pulse Oximeter FEV/PEF	Medical devices designed for home use: Blood glucose meter Weight scale Blood pressure Temperature Activity Monitor
<b>Software Delivery</b>	Web download for the uploader only	Same
Software Features		
<b>Patient Management</b>	Manage patient accounts, filter/sort/search patients, print reports.	The server connected to the mobile device manages patient accounts, filter/sort/search patients
<b>Implementation method of collecting data from sensors</b>	Bluetooth or USB wired	Bluetooth
<b>Communication frequency</b>	Bluetooth: 2.402 to 2.480 GHz	Same
<b>Max data throughput under worst case conditions</b>	Multiply read data can be stored in the medical device and they are resent to the server after recovery of network condition	Same

<b>Use of thresholds/algorithms for determining how thresholds are set and changed</b>	Thresholds set by healthcare professional in server software	Same
<b>Communication method with server</b>	Connected through internet	Same

The LG SmartHealth is a medical device and has similar indications for use statement as the predicate device. The differences in the indications statement between LG SmartHealth and Intel Health Guide Express do not alter the intended use – both devices have essentially the same intended use.

The device also has similar technological characteristics as the predicate device. Both devices have the same data collection software functionality, communication method of patient device with central server, types of sensors which can be interfaced to the patient device, implementation method of collecting data from sensors, sensor software, connectivity, and communication protocol. Since the comparison of the descriptive characteristics of the proposed and predicate devices may not be sufficiently precise to assure equivalence, performance data are provided. The results of the performance testing demonstrate substantial equivalence.

#### **9. Non-Clinical Performance Data**

The device’s software development, verification, and validation have been carried out in accordance with FDA guidelines. The software was tested against the established Software Design Specifications for each of the test plans to assure the device performs as intended. The device Hazard analysis was completed and risk control implemented to mitigate identified hazards. The testing results support that all the software specifications have met the acceptance criteria of each module and interaction of processes. The LG SmartHealth device passed all testing and supports the claims of substantial equivalence and safe operation.

Validation activities included a usability study of the LG SmartHealth under actual use. The study demonstrated:

- Comprehension of the study nurses and participants with the LG SmartHealth,
- Appropriate human factors related to the LG SmartHealth, and
- Ease of use of the LG SmartHealth.

#### **10. Clinical Performance Data**

There was no clinical testing required to support the medical device as the indications for use is equivalent to the predicate device. These types of devices, including the predicate devices, have been on the market for many years with a proven safety and efficacy for the use of the device. The verification and validation testing of the device’s software was found to be acceptable and supports the claims of substantial equivalence.

## **11. Statement of Substantial Equivalence**

The LG SmartHealth has the same or similar intended use as the predicate device and that any technological differences between the LG SmartHealth software and the predicate device do not raise any questions regarding LG SmartHealth ('SmartHealth')'s safety and effectiveness.

The information provided in this submission supports the substantial equivalence to the predicate device and that the system is safe and effective for the users/operators.

## Indications for Use

510(k) Number (if known)

**Device Name**

LG SmartHealth

**Indications for Use (Describe)**

The LG SmartHealth is intended to collect vital sign measurements from physiological measurement devices intended for use in the home. Patients can review the stored vital sign measurement information and receive educational and motivational content from caregivers(or nurse). Patients can also engage in video conferences with caregivers(or nurse).

The LG SmartHealth is not interpretive, nor is it intended for diagnosis or as a substitute for medical care, and it is not intended to provide real time data. It is made available to patients when time-critical care is not required.

The LG SmartHealth is contraindicated for patients requiring direct-medical supervision or emergency intervention. It is intended for patients who are willing and capable of managing its use. Clinical judgment and experience by a caregiver are required to check and interpret the information delivered.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

**PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON A SEPARATE PAGE IF NEEDED.**

**FOR FDA USE ONLY**

Concurrence of Center for Devices and Radiological Health (CDRH) (Signature)

---

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
*PRAStaff@fda.hhs.gov*

*“An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number.”*