

K102187

[Attachment 2]

SPECIAL 510(K) SUMMARY

DEC 10 2010

This summary of Special 510(k) safety and effectiveness information is being submitted in accordance with the requirements of SMDA 1990 and 21 CFR §807.92.

The assigned 510(k) number is:

1. Submitter's Identification:

Radiant Innovation Inc.,
1F, No.3 Industrial E. 9th Rd., Science-Based
Industrial Park, HsinChu, Taiwan

Contact:

Mr. James Huang
QA Department Manager
Radiant Innovation Inc.
TEL: +886 3 6111666 Ext. 100
FAX: +886 3 5644170
E-mail: jameshuang@radiantek.com.tw

Date Summary Prepared: July/26/2010

2. Name of the Device:

Infrared Ear Thermometer TH709L(E)
Classification Name: Thermometer, Electronic, Clinical
Regulation Number: 21 CFR 880.2910

3. Current Clearance Device:

Radiant Innovation Infrared Ear Thermometer, Models TH60N (510(k)#: K101253), TH01BN (510(k)#: K070976), TH5XY series (510(k)#: K063185), TH00XY series (510(k)#: K051269), THXXN series (510(k)#: K040377), TH1 series (510(k)#: K030324), TH88 series (510(k)#: K020504) and TH8 series (510(k)#: K011059).

4. Device Description:

The Radiant Innovation Inc., Infrared Ear Thermometer, Model TH709L(E) is electronic thermometer using an infrared detector (thermopile detector) to detect body temperature from the auditory canal. Its operation is based on measuring the

natural thermal radiation emanating from the tympanic membrane and the adjacent surfaces of the patient.

To measure ear temperature, the ear thermometer is inserted into a patient's outer ear canal. A start button is pressed to start the measurement through the radiation exchanges. The electrical signal read out from the detector is fed to the circuit for amplification and calculation. The measured temperature then appears on a LCD display. The total operation takes a few seconds.

5. Intended Use:

The device is an electronic clinical thermometer using an infrared sensor to detect body temperature from the auditory canal in the neonatal, pediatric and adult population used in the home setting.

6. Technological Characteristics and Substantial Equivalence:

Radiant Infrared Ear Thermometer TH709L(E) is an ear thermometer with probe cover and some sub-functions such as: temperature unit switchable, recall memory, last reading display, ambient temperature, and backlight.

And, the model number will indicate backlight is with or not.

Both the subject device (TH709L(E)) and predicate device (TH8 series) have the same intended use and fundamental technology. The comparison table of subject device (TH709L(E)) and predicate device (TH8 series) is included in the 510(k) submission. The subject device (TH709L(E)) is substantially equivalence to the technological features as the predicate device (TH8 series).

The basic technological characteristics between subject device vs. predicate device.

| Features | Predicate device(TH8 series) | Subject device (TH709L(E)) |
|-----------------------|---|----------------------------|
| 510(k)# | K011059 | K |
| Accuracy | 35.5~42°C (95.9~107.6°F)±0.2°C (0.4°F), other ±0.3°C (0.5°F). | |
| Temp. Range | 34.0-42.2°C | |
| Ambient Range | 10-40°C | |
| Response Time | 1sec | |
| Read modes | Ear (Oral) | |
| Scale Selection | °C/°F | |
| Display Type | LCD | |
| Probe Cover | With | |
| Activation | Scan Button | |
| Memory | 9 sets | 25 sets |
| SENSOR TYPE | Thermopile | |
| Case | ABS | |
| Weight | 70g | 57.7g |
| Dimension (LxWxH) | 14*3.8*3 cm | 12*3.8*4 cm |
| Battery | 3V Battery, CR2032 * 1 | |
| Backlight Source | EL (Optional) | LED (Optional) |
| 2 Phase Battery Alarm | Yes | |

7. **Discussion of Non-Clinical Tests Performed for Determination of Substantial Equivalence are as follows:**

Compliance to applicable voluntary standards includes ASTM E1965-98 and EN12470-5:2003, as well as EN 60601-1 (IEC 60601-1) and EN 60601-1-2 (IEC 60601-1-2) requirements.

Guidance Documents included the FDA "*Guidance On The Content of Premarket Notification (510(k)) Submissions for Clinical Electronic Thermometers*", "*How to Prepare A Special 510(k)*", "*Deciding When to Submit a 510(k) for a Change to an Existing Device*"..

8. **Conclusions:**

The RII Infrared Ear Thermometer TH709L(E), have the same intended use and similar characteristics as the cleared device TH8 series. Moreover, bench testing contained in this submission supplied demonstrate that the modification of TH709L(E) do not raise any new questions of safety or effectiveness. Thus, the RII Infrared Ear Thermometer, Model TH709L(E) is substantially equivalent to the predicate device.



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Room - WO66-G609
Silver Spring, MD 20993-0002

Mr. James Huang
Quality Assurance Department Manager
Radiant Innovation, Incorporated
1F, Number 3, Industrial E. 9th Rd
Science- Based Industrial Park
Hsinchu China (Taiwan) 300

DEC 10 2010

Re: K102187

Trade/Device Name: RII Infrared Ear Thermometer
Regulation Number: 21 CFR 880.2910
Regulation Name: Clinical Electronic Thermometer
Regulatory Class: II
Product Code: FLL
Dated: November 9, 2010
Received: November 10, 2010

Dear Mr. Huang:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.


Page 2- Mr. Huang

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please go to <http://www.fda.gov/AboutFDA/CentersOffices/CDRH/CDRHOffices/ucm115809.htm> for the Center for Devices and Radiological Health's (CDRH's) Office of Compliance. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

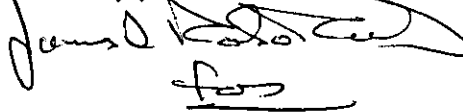

Anthony D. Watson, B.S., M.S., M.B.A.

Director

Division of Anesthesiology, General Hospital,

Infection Control and Dental Devices

Office of Device Evaluation

Center for Devices and

Radiological Health

Enclosure

K102187
EXHIBIT # B

INDICATIONS FOR USE

DEC 10 2010

510(k) Number (if known):

Device Name: Radiant Innovation Inc. Infrared Ear Thermometer TH709L(E)

Indications For Use:

The device is an electronic clinical thermometer using an infrared sensor to detect body temperature from the auditory canal in the neonatal, pediatric and adult population used in the home setting.

Prescription Use _____
(Part 21 CFR 801 Subpart D)

AND/OR
(21 CFR 807 Subpart C)

Over-The-Counter Use

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

 12/10/10

Concurrence of CDRH, Office of Device Evaluation (ODE)
Division of Anesthesiology, General Hospital
Infection Control, Dental Devices

510(k) Number: K102187

Page 1 of 1