

K102930
Pg 1 of 4

510(k) Summary

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirement of SMDA and 21 CFR 807.92.

1.0 submitter's information

NOV 23 2010

Name: Andon Health Co., Ltd.
 Address: No 3, Jinping Street Ya An Road, Nankai District,
 Tianjin, P.R. China
 Phone number: 86-22-6052 6161
 Fax number: 86-22-6052 6162
 Contact: Liu Yi
 Date of Application: 09/27/2010

2.0 Device information

Trade name: Fully Automatic Electronic Blood Pressure Monitor
 Common name: Noninvasive blood pressure measurement system
 Classification name: Noninvasive blood pressure measurement system

3.0 Classification

Production code: DXN- Noninvasive blood pressure measurement system.
 Regulation number: 870.1130
 Classification: II
 Panel: Cardiovascular

4.0 Predicate device information

1	Manufacturer: Andon Health Co., Ltd. Device: KD-556 Fully Automatic Electronic Blood Pressure Monitor 510(k) number: K090963
2	Manufacturer: Andon Health Co., Ltd. Device: KD-738 Fully Automatic Electronic Blood Pressure Monitor 510(k) number: K092045

K102930
P3 2 of 4

5.0 Device description

KD-512 Fully Automatic Electronic Blood Pressure Monitor are for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the upper arm. The cuff circumference is limited to 22cm-48cm.

KD-712 Fully Automatic Electronic Blood Pressure Monitor is for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist. The cuff circumference is limited to 14cm-25cm.

It is designed and manufactured according to ANSI/AAMI SP10--manual, electronic or automated sphygmometers.

The operational principle is based on oscillometric and silicon integrate pressure sensor technology. It can calculate the systolic and diastolic blood pressure, and display the result on the LCD. If any irregular heartbeat is detected, it can also be shown on the LCD. More over, it also calculates the average of the last three measurements.

6.0 Intended use

KD-512 Fully Automatic Electronic Blood Pressure Monitor are for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the upper arm. The cuff circumference is limited to 22cm-48cm.

KD-712 Fully Automatic Electronic Blood Pressure Monitor is for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist. The cuff circumference is limited to 14cm-25cm.

The intended use and the indication for use of KD-512, as described in the

labeling are the same as their predicated device KD-556. While the intended use and the indication for use of KD-712 is the same as its predicated devices KD-738.

7.0 Summary comparing technological characteristics with predicate device

Technological Characteristics	Comparison result
Design principle	Identical
Appearance	Similar
Patients contact Materials	Identical
Performance	Similar
Biocompatibility	Identical
Mechanical safety	Identical
Energy source	Identical
Standards met	Identical
Electrical safety	Identical
EMC	Identical
Function	Similar

K102930
13 4 of 4

8.0 Performance summary

KD-512 and KD-712 Fully Automatic Electronic Blood Pressure Monitor conforms to the following standards:

- IEC 60601-1, Medical Electrical Equipment - Part 1: General Requirements for Safety, 1988; Amendment 1, 1991-11, Amendment 2, 1995.
- EN 60601-1-2, Medical Electrical Equipment - Part 1-2: General Requirements for Safety - Collateral standard: Electromagnetic Compatibility - Requirements and Tests, 2007.
- AAMI SP10:2002, Manual, electronic or automated sphygmomanometers.
- AAMI / ANSI SP10:2002/A1:2003 --, Amendment 1 to ANSI/AAMI SP10:2002 Manual, electronic, or automated sphygmomanometers.
- AAMI / ANSI SP10:2002/A2:2006 --, Amendment 2 to ANSI/AAMI SP10:2002 Manual, electronic, or automated sphygmomanometers.

9.0 Comparison to the predicate device and the conclusion

Our device KD-512 Fully Automatic Electronic Blood Pressure Monitor is substantially equivalent to the Fully Automatic Electronic Blood Pressure Monitor KD-556 whose 510(k) number is K090963. And KD-712 Fully Automatic Electronic Blood Pressure Monitor is substantially equivalent to the Fully Automatic Electronic Blood Pressure Monitor KD-738 whose 510(k) number is K092045.

KD-512 is very similar with its predicate device in the intended use, the design principle, the material, the performance and the applicable standards. Only their appearance, the memory time, the display information and the averaging measurement function are different.

KD-712 is very similar with the predicate device KD-738 in the intended use, the design principle, the material, the performance and the applicable standards. Only their appearance, the memory time and the display information are different.

However, the test in this submission provides demonstration that these small differences do not raise any new questions of safety and effectiveness to the new devices.



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Room -WO66-G609
Silver Spring, MD 20993-0002

Andon Health Co., Ltd.
C/O Mr. Liu Yi, President
No. 3 Jinping Street Ya'an Road
Nankai District, Tianjin 300190
China

NOV - 3 2010

Re: K102930

Trade/Device Name: Fully Automatic Electronic Blood Pressure Monitor, Models: KD-512, KD-712

Regulation Number: 21 CFR 870.1130

Regulation Name: Non-Invasive Blood Pressure Measurement System

Regulatory Class: Class II

Product Code: DXN

Dated: Not Dated

Received: October 4, 2010

Dear Mr. Yi:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.


Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must

comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please go to <http://www.fda.gov/AboutFDA/CentersOffices/CDRH/CDRHOffices/ucm115809.htm> for the Center for Devices and Radiological Health's (CDRH's) Office of Compliance. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

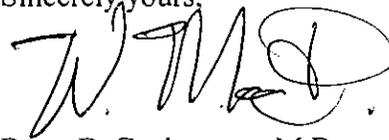

For Bram D. Zuckerman, M.D.

Director
Division of Cardiovascular Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

Statement of Indications for Use

NOV - 3 2010

510(k) Number : K102930

Device name: KD-512 and KD-712 Fully Automatic Electronic Blood Pressure Monitor

Indications for use:

KD-512 Fully Automatic Electronic Blood Pressure Monitor are for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the upper arm. The cuff circumference is limited to 22cm-48cm.

KD-712 Fully Automatic Electronic Blood Pressure Monitor is for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist. The cuff circumference is limited to 14cm-25cm.

Prescription use _____ AND/OR Over-The-Counter Use YES
Part 21 CFR 801 Subpart D) (21 CFR 807 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-COUNTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)



(Division Sign-Off)
Division of Cardiovascular Devices

Page 1 of 1

510(k) Number K102930