

K111172

pg 1 of 2

AUG 29 2011

510(k) Summary

The 510(k) Summary is submitted in accordance with 21 CFR Part 807, Section 807.92

Submitter's Name: The Daavlin Distributing Company

Registration Number: 1526255

Address: 205 West Bement Street
Bryan, Ohio 43506

Telephone: 419.636.6304

Contact: Michele Thiel

Date Prepared: 31 March 2011

Device Trade Name: 4 Series Phototherapy Unit

Device Common Name: 4 Series Partial- Body Phototherapy Unit

Device Classification: Class II

Product Code: FTC

Regulation Number: CFR 878.4630

Regulation Name: Ultraviolet lamp for dermatologic/skin disorders

Predicate Device: Daavlin Distributing Company
1 Series Phototherapy Unit
K100378

Device Description:

The 4 Series Unit is a partial-body phototherapy panel ultraviolet light source controlled by an integrated digital timer or a microprocessor controller. The operator interface consists of two main components: a LCD, and a Membrane with 4 buttons. When the operator sets the treatment time or the desired dose using the operator interface located on the control panel, the treatment begins by illuminating the lamps which emit ultraviolet light. The spectral output at peak wavelengths of 305nm,311nm,350nm,or 365nm depending on lamp. The 4 Series Unit is a therapeutic product designed for individuals who require ultraviolet radiation for diagnosed skin disorders.

Predicate Device Comparison:

The 4 Series Unit is constructed in the same design configuration as the predicate device, utilizing identical energy sources (UV lamps) and materials of identical composition. The 4 Series varies from the predicate device, in that the body is slightly larger and therefore contains more lamps to maintain the same performance. The intended use, general and specific indications for use, spectral output, mode of operation, labeling, accessories, and general operating principals of the 4 Series ultraviolet light-emitting medical device are the same of the predicate device. The treatment area, while still partial body, is similar , but larger on the 4 Series unit than on the predicate device.

Intended Use:

The 4 Series Unit is a therapeutic product designed for individuals who require ultraviolet radiation for diagnosed skin disorders.

Performance Data:

The 4 Series Phototherapy Unit performance data is the same as that of the claimed predicate devices. The Controller, UV lamps and cabinet construction used in the production of the predicate device and the 4 Series Phototherapy Unit are identical, giving equivalent spectral/power outputs.

Conclusion:

On the basis of the information provided in this Summary, the Daavlin Distributing Company believes the 4 Series Phototherapy Unit is substantially equivalent to the legally commercialized predicate devices.



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Room -WO66-G609
Silver Spring, MD 20993-0002

Daavlin Distributing Company
% Ms. Michele Thiel
Management Representative
205 West Bement Street
Bryan, Ohio 43506

AUG 29 2011

Re: K111172
Trade/Device Name: 4 Series Phototherapy Unit
Regulation Number: 21 CFR 878.4630
Regulation Name: Ultraviolet lamp for dermatologic disorders
Regulatory Class: II
Product Code: FTC
Dated: August 22, 2011
Received: August 22, 2011

Dear Ms. Thiel:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21


Page 2 - Ms. Michele Thiel

CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please go to <http://www.fda.gov/AboutFDA/CentersOffices/CDRH/CDRHOffices/ucm115809.htm> for the Center for Devices and Radiological Health's (CDRH's) Office of Compliance. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

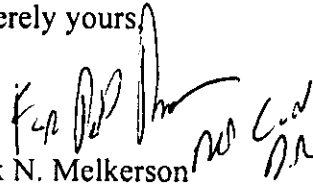

Mark N. Melkerson
Director
Division of Surgical, Orthopedic
and Restorative Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

K111172 pg 1 of 1

Indication for Use

510(k) Number

Device Name 4 Series Phototherapy Unit

Indications for Use

The 4- Series Unit is a therapeutic product designed for individuals who require ultraviolet radiation for diagnosed skin disorders.

Prescription Use X OR Over-the-Counter Use
(per 21 CFR 801.109)

(Please do not write below this line – Continue on another page if needed)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Neil R. DeLorenzo

(Division Sign-Off)
Division of Surgical, Orthopedic,
and Restorative Devices

510(k) Number K111172