

NOV 21 2011

CD HORIZON® Spinal System
510(k) Summary – K
October 26, 2011

- I. Company:** Medtronic Sofamor Danek USA, Inc.
 1800 Pyramid Place
 Memphis, TN 38132
 (901) 396-3133
- Contact:** Chris McKee
 Sr. Regulatory Affairs Specialist
- II. Proprietary Trade Name:** CD HORIZON® Spinal System
- III. Classification Name:** Spinal Interlaminar Fixation Orthosis, Spinal Intervertebral Body Fixation Orthosis, and Pedicle Screw Spinal System
 (21 CFR 888.3050, 888.3060 and 888.3070)
- IV. Classification:** Class III (Pre-amendment)
- V. Product Codes:** KWP, KWQ, MNH, MNI, NKB and OSH

VI. Product Description

The CD HORIZON® Spinal System consists of a variety of shapes and sizes of rods, hooks, screws, CROSSLINK® Plates, staples and connecting components, as well as implant components from other Medtronic spinal systems, which can be rigidly locked into a variety of configurations, with each construct being tailor-made for the individual case.

A subset of CD HORIZON® Spinal System components may be used for posterior pedicle screw fixation in pediatric cases. These constructs may be comprised of a variety of shapes and sizes of rods (ranging in diameter from 3.5mm to 6.35mm), hooks, screws, CROSSLINK® Plates, and connecting components. Similarly to the CD HORIZON® implants used in adult cases, these components can be rigidly locked into a variety of configurations, with each construct being tailor-made for the individual case.

Certain components within the CD HORIZON® Spinal System are specifically excluded for use in pediatric patients. These include PEEK rods, Shape Memory Alloy Staples, SPIRE™ Plates and DYNALOK® bolts. All screws used in pediatric cases are only cleared for use via a posterior approach. All of the components used in pediatric cases are fabricated from medical grade stainless steel, medical grade titanium, titanium alloy, and medical grade cobalt-chromium-molybdenum alloy.

Certain implant components from other Medtronic spinal systems can be used with the CD HORIZON® Spinal System in non-pediatric cases. These components include TSRH® rods, hooks, screws, plates, CROSSLINK® plates, connectors, staples, washers, GDLH® rods, hooks, connectors and CROSSLINK® bar and connectors; LIBERTY® rods and screws; DYNALOK® PLUS and DYNALOK CLASSIC® bolts along with rod/bolt connectors; and Medtronic Multi-Axial rods and screws. Please note that certain

components are specifically designed to connect to specific rod diameters, while other components can connect to multiple rod diameters. Care should be taken so that the correct components are used in the spinal construct.

CD HORIZON® hooks are intended for posterior use only. CD HORIZON® staples and CD HORIZON® ECLIPSE® rods and associated screws are intended for anterior use only. However, for patients of smaller stature and pediatric patients, CD HORIZON® 4.5mm rods and associated components may be used posteriorly.

The CD HORIZON® Spinal System implant components are fabricated from medical grade stainless steel, medical grade titanium, titanium alloy, medical grade cobalt-chromium-molybdenum alloy, or medical grade PEEK OPTIMA-LTI. Certain CD HORIZON® Spinal System components may be coated with hydroxyapatite. No warranties expressed or implied are made. Implied warranties of merchantability and fitness for a particular purpose or use are specifically excluded. See the MDT Catalog for further information about warranties and limitations of liability.

Never use stainless steel and titanium implant components in the same construct.

Medical grade titanium, titanium alloy, and/or medical grade cobalt-chromium-molybdenum alloy may be used together. Never use titanium, titanium alloy, and/or medical grade cobalt-chromium-molybdenum alloy with stainless steel in the same construct.

The CD HORIZON® Spinal System also includes anterior staples made of Shape Memory Alloy (Nitinol – NiTi). Shape Memory Alloy is compatible with titanium, titanium alloy, and cobalt-chromium-molybdenum alloy. Do not use with stainless steel. These staples are not to be used in pediatric patients.

PEEK OPTIMA-LTI implants may be used with stainless steel, titanium, or cobalt-chromium-molybdenum alloy implants. CD HORIZON® PEEK Rods are not to be used with CROSSLINK® Plates or in pediatric patients.

To achieve best results, do not use any of the CD HORIZON® Spinal System implant components with components from any other system or manufacturer unless specifically allowed to do so in this or another Medtronic document. As with all orthopaedic and neurosurgical implants, none of the CD HORIZON® Spinal System components should ever be reused under any circumstances.

The purpose of this 510(k) was to add additional components to the system, namely SOLERA™ 5.5mm and 6.0mm diameter rods and associated multi-axial screws, hydroxyapatite coated multi-axial screws, reduction multi-axial screws, fixed angle screws and setscrews. The components are manufactured out of medical grade titanium, titanium alloy and/or grade cobalt-chromium-molybdenum alloy with some components coated with hydroxyapatite. In addition, previously cleared TSRH® 5.5mm diameter rods were cleared for use with the CD HORIZON® Spinal System.

VII. Indications

The CD HORIZON® Spinal System with or without SEXTANT® instrumentation is intended for posterior, non-cervical fixation as an adjunct to fusion for the following indications: degenerative disc disease (defined as back pain of discogenic origin with degeneration of the disc confirmed by history and radiographic studies); spondylolisthesis; trauma (i.e., fracture or dislocation); spinal stenosis; curvatures (i.e., scoliosis, kyphosis and/or lordosis); tumor; pseudarthrosis; and/or failed previous fusion.

Except for hooks, when used as an anterolateral thoracic/lumbar system, the CD HORIZON® Spinal System may also be used for the same indications as an adjunct to fusion.

With the exception of degenerative disc disease, the CD HORIZON® LEGACY™ 3.5mm rods and the CD HORIZON® Spinal System PEEK rods and associated components may be used for the aforementioned indications in skeletally mature patients as an adjunct to fusion. The 3.5mm rods may be used for the specific pediatric indications noted below.

When used for posterior non-cervical pedicle screw fixation in pediatric patients, the CD HORIZON® Spinal System implants are indicated as an adjunct to fusion to treat adolescent idiopathic scoliosis. Additionally, the CD HORIZON® Spinal System is intended to treat pediatric patients diagnosed with the following conditions: spondylolisthesis/spondylolysis and fracture caused by tumor and/or trauma. These devices are to be used with autograft and/or allograft. Pediatric pedicle screw fixation is limited to a posterior approach.

The CD HORIZON® SPIRE™ Plate is a posterior, single level, non-pedicle supplemental fixation device intended for use in the non-cervical spine (T1-S1) as an adjunct to fusion in skeletally mature patients. It is intended for plate fixation/attachment to spinous processes for the purpose of achieving supplemental fixation in the following conditions: degenerative disc disease (as previously defined); spondylolisthesis; trauma; and/or tumor.

In order to achieve additional levels of fixation, the CD HORIZON® Spinal System rods may be connected to the VERTEX® Reconstruction System with the VERTEX® rod connector. Refer to the VERTEX® Reconstruction System Package Insert for a list of the VERTEX® indications of use.

VIII. Summary of the Technological Characteristics

The subject devices provide additional fixation options for connecting to 5.5mm and 6.0mm diameter rods. The primary change from the predicate devices is that the subject devices can connect to both 5.5mm and 6.0mm rod diameters as opposed to just 4.75mm diameter rods. The materials, thread forms, and sizes are the same as the predicate devices.

IX. Identification of Legally Marketed Devices

The design features, materials and indications for use of the subject devices are substantially equivalent to the predicate CD HORIZON® SOLERA™ screws and setscrews for use with 4.75mm diameter rods (K091974 SE 09/02/09, K102555 SE 11/17/10, K112473 SE 10/21/11), CD HORIZON® HA coated screws (K112555 SE

09/27/11), CD HORIZON® 5.5mm diameter rods (K040962 SE 05/14/04), CD HORIZON® 6.0mm diameter rods (K040072 SE 02/12/04) and CD HORIZON® screws for use with 4.5mm and 5.5mm rods (K042025 SE 08/25/04). The TSRH® 5.5mm diameter rods were previously cleared in K103049 (SE 12/23/10) and K111942 (08/30/11).

The labeling is identical to that cleared in K112473.

X. Discussion of the Non-Clinical Testing

Non-clinical testing in the form of mechanical testing was performed on the worst case subject devices in the form of interconnection testing in accordance with ASTM F1798 and construct testing in accordance with ASTM F1717. The results were compared to the previously listed predicate devices. Test methods included the following:

ASTM F1798

- Axial Grip
- Axial Torsion
- Static Flexion/Extension
- Flexion/Extension Fatigue

ASTM F1717

- Static Axial Compression
- Static Axial Torsion
- Axial Compression Fatigue

For all test methods, the subject devices met the pre-determined acceptance criteria.

Wear debris analysis was performed on the worst case subject devices and the results compared to the worst case predicate devices. The results demonstrated that the subject devices do not introduce a new worst case to the CD HORIZON® Spinal System. A component analysis was also performed on the subject devices to assess the potential for corrosion. No signs of corrosion were detected.

XI. Conclusions

A risk analysis was completed and non-clinical mechanical testing was performed in accordance with ASTM F1798 and ASTM F1717. Wear debris analysis and component analysis was also performed. Based on the test results and additional supporting documentation provided in this pre-market notification, the subject devices demonstrated substantial equivalence to the previously listed predicate devices.



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Center – WO66-G609
Silver Spring, MD 20993-0002

June 27, 2013

Medtronic Sofamor Danek USA, Inc.
% Mr. Chris McKee
Sr. Regulatory Affairs Specialist
1800 Pyramid Place
Memphis, Tennessee 38132

Re: K113174
Trade/Device Name: CD HORIZON Spinal System
Regulation Number: 21 CFR 888.3070
Regulation Name: Pedicle screw spinal system
Regulatory Class: Class III
Product Code: NKB, OSH, MNI, MNH, KWP, KWQ
Dated: October 26, 2011
Received: October 27, 2011

Dear Mr. McKee:

This letter corrects our substantially equivalent letter of November 21, 2011.

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA).

You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21

CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638 2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

Erin F. Keith

For

Mark N. Melkerson
Director
Division of Orthopedic Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

510(k) Number (if known):

Device Name: CD HORIZON® Spinal System

Indications for Use:

The CD HORIZON® Spinal System with or without SEXTANT® instrumentation is intended for posterior, non-cervical fixation as an adjunct to fusion for the following indications: degenerative disc disease (defined as back pain of discogenic origin with degeneration of the disc confirmed by history and radiographic studies); spondylolisthesis; trauma (i.e., fracture or dislocation); spinal stenosis; curvatures (i.e., scoliosis, kyphosis and/or lordosis); tumor; pseudarthrosis; and/or failed previous fusion.

Except for hooks, when used as an anterolateral thoracic/lumbar system, the CD HORIZON® Spinal System may also be used for the same indications as an adjunct to fusion.

With the exception of degenerative disc disease, the CD HORIZON® LEGACY™ 3.5mm rods and the CD HORIZON® Spinal System PEEK rods and associated components may be used for the aforementioned indications in skeletally mature patients as an adjunct to fusion. The 3.5mm rods may be used for the specific pediatric indications noted below.

When used for posterior non-cervical pedicle screw fixation in pediatric patients, the CD HORIZON® Spinal System implants are indicated as an adjunct to fusion to treat adolescent idiopathic scoliosis. Additionally, the CD HORIZON® Spinal System is intended to treat pediatric patients diagnosed with the following conditions: spondylolisthesis/spondylolysis and fracture caused by tumor and/or trauma. These devices are to be used with autograft and/or allograft. Pediatric pedicle screw fixation is limited to a posterior approach.

The CD HORIZON® SPIRE™ Plate is a posterior, single level, non-pedicle supplemental fixation device intended for use in the non-cervical spine (T1-S1) as an adjunct to fusion in skeletally mature patients. It is intended for plate fixation/attachment to spinous processes for the purpose of achieving supplemental fixation in the following conditions: degenerative disc disease (as previously defined); spondylolisthesis; trauma; and/or tumor.

In order to achieve additional levels of fixation, the CD HORIZON® Spinal System rods may be connected to the VERTEX® Reconstruction System with the VERTEX® rod connector. Refer to the VERTEX® Reconstruction System Package Insert for a list of the VERTEX® indications of use.


Prescription Use X
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use _____
(21 CFR Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

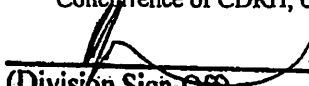

(Division Sign-Off)
Division of Surgical, Orthopedic,
and Restorative Devices

Page 1 of 1

510(k) Number K113174