

## 510(K) Summary

JUL 11 2013

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirements of SMDA 1990 and 21 CFR 807.92.

The assigned 510(k) number is: K123075

### 1. Submitter's Identifications:

Well Life Healthcare Limited  
1F., No. 16, Lane 454, Jungjeng Road, Yunghe City, Taipei County 234, Taiwan, ROC

Contact: Jenny Hsieh  
Telephone: + 886 2 2928 2112

Date of Summary Preparation: May 25, 2013

### 2. Name of the Device:

Trade/Device Name: Buttock Muscle Stimulator, model: WL-2413E.  
Regulation Number: 21 CFR 890.5850  
Regulation Name: Power muscle stimulator.  
Regulatory Class: II  
Product Code: NGX

### 3. Information of the 510(k) Cleared Device (Predicate Device):

SPORT-ELEC models Body Control System "BS"( K092142)

### 4. Device Description:

The Buttock Muscle Stimulator, model WL-2413E, is a two channel battery operated muscle stimulation system specifically designed to exercise the buttocks' muscle. The device comprises mainly the electronic stimulator module which generates the required stimulation signals.

The Buttock Muscle Stimulator, model WL-2413E, comprises a short with silicon pad electrodes which connect the signals from the stimulator to the skin. The electrodes are placed on the inner surface of the accessories. The product is supplied with a 510(K) cleared conductive gel, a user's manual, 3AAA batteries, 2 lead wires and a storage bag.

The required power of the device is derived from 3 batteries located in a compartment protected by a removable battery cover. The electrodes are in the inner surface of the shorts which is worn as shown on the picture of user manual. There is no current passed from side to side. The user cannot access the wiring or connectors within the shorts.

The Buttock Muscle Stimulator, model WL-2413E, is intended for use by healthy persons to apply transcutaneous electrical muscle stimulation(EMS) through skin contact electrode for the purpose of improvement of muscle tone of the buttocks muscles

5. Intended Use:

The Buttock Muscle Stimulator, model WL-2413E, is intended for use by healthy persons to apply transcutaneous electrical muscle stimulation(EMS) through skin contact electrode for the purpose of improvement of muscle tone of the buttocks muscles

6. Discussion of Non-Clinical Tests Performed Determination of Substantial Equivalence are as follows:

Compliance to applicable voluntary standards includes IEC 60601-2-10, as well as IEC 60601-1, and IEC 60601-1-2 requirement.

In addition to the compliance of voluntary standards, the software verification has been carried out according to the FDA software guidance.

Discussion: The Compliance to applicable voluntary standards as above mentioned indicates that the new device in this submission used the same standards as that of predicate device in addition to the software validation standard. For our new device, the software validation was prepared according to FDA guidance, however the predicate device used IEC 60601-1-4 to prepare their software validation report. For this difference we believe that we used more adequate reference tool to prepare software validation report than the predicate device. Therefore, we consider that the compliance of standards included in our submission is adequate for the determination of substantial equivalence.

7. Comparison of Significant device features

Comparison Features	Models to be Compared	
	SPORT-ELEC Body Control System	Buttock Muscle Stimulator
Model	"BS"	WL-2413E
510(K) No.	K092142	K123075
Prescription or OTC	OTC	Same
Indication for use	intended for use by healthy persons to apply transcutaneous electrical muscle stimulation(EMS) through skin contact electrode for the purpose of improvement of muscle tone of the buttocks muscles	Same
FDA product code	NGX	Same
Electrode Used	Self Adhesive Electrode( 5 X 5 cm)	K082065, Silicon Pad Electrode ( 7.0 cm diameter)
Manufacturer	SPORT-ELECT	Well-Life

### 8. Comparison of Unit Characteristics & Output Specification

		Predicate Device	New Device
Mode or Program Name		BS	WL-2413E
Waveform (e.g., pulsed monophasic, biphasic)		Biphasic	Biphasic
Shape (e.g., rectangular, spike, rectified sinusoidal)		Rectangular	Rectangular
Maximum Output Voltage (volts)- (+/- 20%)		31.2V @500Ω	40.8V @500Ω
		66.0V @2kΩ	70.0V @2kΩ
		66.0V @10kΩ	106.0V @10kΩ
Maximum Output Current (mA)- (+/- 20%)		62.4A @500Ω	81.6mA @500Ω
		33.0A @2kΩ	35.0mA @2kΩ
		6.6A @10kΩ	10.6mA @10kΩ
Duration of primary phase (usec)		200 Max	300 Max
Pulse Duration (usec)		540 Max.	720 Max.
Frequency (Hz) or Rate (pps)		70 Max	70 Max
For multiphasic waveforms only:	Symmetrical phases?	Yes	Yes
	Phase Duration (include units), (Stage range, if applicable), (both phases, if asymmetrical)	Not applicable	Not applicable
	510(K) Number	K092142	K123075
Device Name and Model		BS	WL-2413E
Manufacturer		SPORT-ELECT	Well-Life
Power Source(s)		1.5Vx3 (AAA Size)	1.5Vx3 (AAA Size)
- Method of Line current Isolation		Type BF	Type BF
- Patient Leakage Current		---	---
- Normal condition (uA)		Under 0.1	Under 0.1
- single Fault condition (uA)		Under 0.5	Under 0.5
Average DC current through electrodes when device is on but no pulses are being applied (uA)		Not applicable	Not applicable
Number of Output Modes		4	4
Number of Output Channels:	Synchronous or Alternating?	Synchronous	Synchronous
	Method of Channel Isolation	Output Coil	Output Coil
Regulated Current or Regulated Voltage?		Voltage	Voltage
Software/Firmware/Microprocessor control?		Yes	Yes
Automatic Overload Trip?		No	No
Automatic No-Load Trip?		Yes	Yes
Automatic Shut Off?		Yes	Yes
User Override control?		No	No
Indicator Display:	On/Off Status?	Yes	Yes
	Low Battery?	Yes	Yes
	Voltage/Current Level?	Yes	Yes
Timer Range (Minutes)		20-40	20-40
Compliance with Voluntary Standards?		IEC 60601-2-10	IEC 60601-2-10
Compliance with 21 CFR 898?		Yes	Yes
Weight (g) including battery		171.6	80
Dimensions (mm.) [W x H x D]		87x43x69	64x90x20
Housing Materials and construction		ABS	ABS
Pulse per burst		Same for each program	Same for each program
Burst per second		Same for each program	Same for each program
Burst duration		Same for each program	Same for each program
Duty Cycle		Same for each program	Same for each program
Method of achieving zero net charge for net charge/pulse		Biphasic symmetric wave for each pulse	Biphasic symmetric wave for each pulse

9. Significant output characteristics comparison table:

Comparison feature	Model		Remarks
	SPORT-ELECT BS	WL-2413E	
Net charge	0	0	Note I
Max. phase charge	14 uc	24 uc	
Max. current Density	0.0784 mA/cm <sup>2</sup>	0.0997 mA/cm <sup>2</sup>	
Max. Average current (RMSA)	500 Ω	38.43mA	
	2K Ω	14.892mA	
	10 Ω	4.996mA	
Max. Power Density	0.00274 Watts/ cm <sup>2</sup>	0.00399 Watts/ cm <sup>2</sup>	
Burst Mode	Yes	Yes	

Note I: Net Charge is Zero for the symmetrical stimulation biphasic wave

10. Summary for the technology comparison.

Basically the WL-2413E has the similar technological characteristics with the predicate device in the product design, material, energy source type, main program mode and the main output waveform... etc. There exist some difference in the detailed output parameters( mainly in the pulse duration and electrode sizes). Through the detailed calculation comparison of stimulation output energy for each operation mode(in particular the output current density and power density), we found the output level in each operation mode for our WL-2413E and predicate device are very close and within the acceptable range as specified in FDA guidance. So we believe the difference in detailed output parameters does not affect the determination of substantial equivalence.

11. Conclusions

The Buttock Muscle Stimulator, model: WL-2413E have the same intended use and the similar technological characteristics as the cleared device- SPORT-ELEC models Body Control System "BS"( K092142). Moreover, verification and validation tests contained in this submission demonstrate that the difference in the submitted models could maintain the same safety and effectiveness as that of cleared device.

In the other words, those engineering difference do not: (1) affect the intended use or (2) alter the fundamental scientific technology of the device.



July 11, 2013

Well-Life Healthcare Ltd  
C/O Ms. Jenny Hsieh  
1F, No. 16, Lane 454, Jungjeng Road  
Yunghe City, Taipei County, Taiwan  
R.O.C.

Re: K123075

Trade/Device Name: Well-Life Buttock Muscle Stimulator, model WL-2413E  
Regulation Number: 21 CFR 890.5850  
Regulation Name: Powered muscle stimulator  
Regulatory Class: Class II  
Product Code: NGX  
Dated: June 2, 2013  
Received: June 5, 2013

Dear Ms. Hsieh:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the [Federal Register](#).

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21

CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

**Joyce M. Whang -S**

for Victor Krauthamer, Ph.D.  
Acting Director  
Division of Neurological  
and Physical Medicine Devices  
Office of Device Evaluation  
Center for Devices and  
Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known): K123075

Device Name: Well-Life Buttock Muscle Stimulator, model: WL 2413E

### Indications For Use:

The Buttock Muscle Stimulator, model WL-2413E, is intended for use by healthy persons to apply transcutaneous electrical muscle stimulation(EMS) through skin contact electrode for the purpose of improvement of muscle tone of the buttocks muscles.

Prescription Use \_\_\_\_\_  
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use  \_\_\_\_\_  
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

---

Concurrence of CDRH, Office of Device Evaluation (ODE)

<p><b>Joyce M. Whang -S</b></p> <hr/> <p>(Division Sign Off) Division of Neurological and Physical Medicine Devices (DNPMD)</p> <p>510(k) Number _ K123075 _____</p>
--