

510(k) Summary**TYBER MEDICAL Interbody System**

Submitter by:	Tyber Medical LLC 89 Headquarters Plaza North, #1464 Morristown, New Jersey 07960
Contact Person	Jeff Tyber CEO and President Phone: (303) 717-5060 Fax: (866) 889-9914 Email: jtyber@tybermed.com
Date Prepared	July 22, 2013
Common Names	Intervertebral Body Fusion Device
Trade Name	Tyber Medical Interbody System – ACIF, ALIF, PLIF, TLIF, and DLIF
Classification Name and Number	Intervertebral body fusion device (21 CFR 888.3080)
Product Code	ODP and MAX
Predicate Devices	<ol style="list-style-type: none"> 1. X-SPINE; Calix – K112036 2. SPINAL ELEMENTS; Lucent and Lucent Magnum – K110632 3. DEPUY SPINE; Bengal, Concorde, Cougar, Devex, and Leopard – K081917 4. AESCULAP; ASpace, CESpace, Prospace, T-Space – K071983 5. SYNTHES; Opal and Oracle Spacer - K072791 6. SYNTHES; T-Pal - K100089 7. CHOICE SPINE; STEATH PEEK Cervical Fusion Device– K120570
Device Description	The Tyber Medical Interbody System, manufactured from PEEK-Optima [®] , consist of implants available in various foot prints, heights and lordotic configurations with an open architecture to accept packing of autograft materials. The exterior of the device has "teeth" or other generally sharp engagement members on the superior and inferior surfaces to help prevent the device from migrating once it is surgically positioned. The device comes in a PEEK or PEEK with a plasma-sprayed commercially pure titanium coating on the superior and inferior surfaces.
Intended Use/ Indications for use	The Tyber Medical Interbody Systems are indicated for use as intervertebral body fusion devices in skeletally mature patients with degenerative disc disease (defined as discogenic back pain with degeneration of the disc confirmed by patient history and radiographic studies) at one or two contiguous levels of the lumbar spine (L2-S1). Patients should have six months of non-operative treatment prior to surgery. These implants are used to facilitate fusion in the lumbar spine and are placed via either a posterior, transforaminal, lateral or anterior approach using autogenous bone. When used as interbody fusion devices these implants are intended for use with supplemental fixation systems cleared for use in the thoracolumbar spine.

SEP 30 2013

Cervical System Indications

The Tyber Medical Cervical Interbody Systems System is indicated for use as an intervertebral body fusion device in skeletally mature patients with degenerative disc disease (defined as discogenic back pain with degeneration of the disc confirmed by patient history and radiographic studies) at one level of the cervical spine with accompanying radicular symptoms. Patients should have six weeks of non-operative treatment prior to surgery. Cervical implants are used to facilitate fusion in the cervical spine (C2-T1) and are placed via an anterior approach using autogenous bone. When used as an interbody fusion device, supplemental fixation must be used.

**Performance Data
(Non-Clinical and/or
Clinical)**

Non-clinical mechanical testing was performed consisting of Static and Dynamic Compression, Torsion, Compression-Shear per ASTM F2077. Additionally, Subsidence Testing per ASTM F2267 and Expulsion testing was performed. The coating characterization tests include Static Shear per ASTM F 1044, Static Tension per ASTM F 1147, and Abrasion per ASTM F 1978. All data indicates the device is substantial equivalence to the predicate systems

Clinical data and conclusions were not needed for this device.

**Statement of
Technological
Comparison**

The Tyber Medical Interbody System and its predicate devices have the same indications for use; same design; are made of similar materials, same application, and have the same anatomic mechanical properties.

Conclusion

The Tyber Medical Interbody System is substantially equivalent to its predicate devices. This conclusion is based upon indications for use, materials, design, test data and principles of operation.



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Center - WC066-G609
Silver Spring, MD 20993-0002

Tyber Medical LLC
Mr. Jeff Tyber
CEO and President
89 Headquarters Plaza North, #1464
Morristown, New Jersey 07960

September 30, 2013

Re: K130573
Trade/Device Name: Tyber Medical Interbody System
Regulation Number: 21 CFR 888.3080
Regulation Name: Intervertebral body fusion device
Regulatory Class: Class II
Product Code: MAX, ODP
Dated: August 10, 2013
Received: August 12, 2013

Dear Mr. Tyber:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set

Page 2 – Mr. Jeff Tyber

forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

Mark N. Melkerson -S

Mark N. Melkerson
Director
Division of Orthopedic Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

Indications for Use

510(k) Number (if known): K130573

Device Name: Tyber Medical Interbody System

Indications for Use:

The Tyber Medical Interbody Systems are indicated for use as intervertebral body fusion devices in skeletally mature patients with degenerative disc disease (defined as discogenic back pain with degeneration of the disc confirmed by patient history and radiographic studies) at one or two contiguous levels of the lumbar spine (L2-S1). Patients should have six months of non-operative treatment prior to surgery. These implants are used to facilitate fusion in the lumbar spine and are placed via either a posterior, transforaminal, lateral or anterior approach using autogenous bone. When used as interbody fusion devices these implants are intended for use with supplemental fixation systems cleared for use in the thoracolumbar spine.

Cervical System Indications

The Tyber Medical Cervical Interbody Systems System is indicated for use as an intervertebral body fusion device in skeletally mature patients with degenerative disc disease (defined as discogenic back pain with degeneration of the disc confirmed by patient history and radiographic studies) at one level of the cervical spine with accompanying radicular symptoms. Patients should have six weeks of non-operative treatment prior to surgery. Cervical implants are used to facilitate fusion in the cervical spine (C2-T1) and are placed via an anterior approach using autogenous bone. When used as an interbody fusion device, supplemental fixation must be used.

Prescription Use AND/OR Over-the-counter

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Anton E. Dmitriev, PhD
Division of Orthopedic Devices