

JUL 18 2013

**8.0 510(K) SUMMARY**

<b>Submitter's Name and Address</b>	ConforMIS Inc. 28 Crosby Drive Bedford, MA 01730
<b>Establishment Registration Number</b>	3009844603
<b>Date of Summary</b>	July 17, 2013
<b>Contact Person</b>	Amita S. Shah, Sr. Vice President, Regulatory and Quality Affairs
<b>Telephone Number</b>	(781) 345-9164
<b>Fax Number</b>	(781) 345-0104
<b>Name of the Device</b>	ConforMIS iTotal® CR Knee Replacement System (iTotal CR KRS)
<b>Common or Usual Name</b>	Cruciate Retaining Total Knee Replacement System
<b>Classification Name</b>	Knee joint patellofemorotibial polymer/metal/polymer semi-constrained cemented prosthesis

**510(K) Summary Continued****Device Classification**

Product Code:

JWH: Knee joint patellofemorotibial polymer/metal/polymer semi-constrained cemented prosthesis.

OOG: Knee Arthroplasty Implantation System.

Intended to be used to assist in the implantation of a specific knee arthroplasty device or a set of specific knee arthroplasty devices. Indicated to include guiding alignment, making or establishing cuts, selecting, sizing, attaching, positioning or orienting implant components.

OIY: Prosthesis, knee, patellofemorotibial, semi-constrained, cemented polymer + additive/metal/polymer + additive. This generic type of device includes prosthesis that have a femoral component made of alloys, such as cobalt-chromium-molybdenum, and a tibial component(s) and/or a retropatellar resurfacing component made of ultra-high molecular weight polyethylene plus an additive, such as a-tocopherol.

**Indications for Use**

The iTotal CR Knee Replacement System (KRS) is intended for use as a total knee replacement in patients with knee joint pain and disability whose conditions cannot be solely addressed by the use of a prosthetic device that treats only one or two of the three knee compartments, such as a unicompartmental, patellofemoral or bicompartamental prosthesis.

The Indications for Use include:

- Painful joint disease due to osteoarthritis, traumatic arthritis, rheumatoid arthritis or osteonecrosis of the knee.
- Post traumatic loss of joint function.
- Moderate varus, valgus or flexion deformity in which the ligamentous structures can be returned to adequate function and stability.
- Failed osteotomies, hemiarthroplasties, and unicompartmental, patellofemoral or bicompartamental implants.
- Revision procedures provided that anatomic landmarks necessary for alignment and positioning of the implant are identifiable on patient imaging scans.

**This implant is intended for cemented use only.**

**510(K) SUMMARY CONTINUED**

<b>Identification of the Legally Marketed Device (Predicate Device)</b>	<p>ConforMIS iTotal CR Knee Replacement System (ITOTAL CR KRS)</p> <p>Device Class: II</p> <p>Product Code: JWH, OOG, OIY</p> <p>Regulation Number: 21 CFR 888.3560</p> <p>510(k) number: K122870</p>
<b>Device Description</b>	<p>The iTotal CR Knee Replacement System (hereafter referred to as the "iTotal CR KRS") is a patient specific tricompartmental faceted posterior cruciate ligament (PCL) retaining knee replacement system. The iTotal CR KRS is a semi-constrained cemented knee implant which consists of a femoral, tibial and patellar component.</p> <p>Using patient imaging (either CT or MR scans) and a combination of proprietary and off the shelf software a patient-specific implant is designed, that best meets the geometric and anatomic requirements of the specific patient. The femoral component is manufactured from cobalt chromium molybdenum ("CoCrMo") alloy. The tibial component includes a metal tray manufactured from CoCrMo alloy and either one or two polyethylene inserts. The polyethylene inserts may be manufactured from either UHMWPE or a highly cross-linked Vitamin E infused polyethylene (iPoly XE™). The patellar component is also manufactured from either UHMWPE or from a highly cross-linked Vitamin E infused polyethylene (iPoly XE).</p> <p>For user convenience, and similar to the predicate iTotal CR KRS, accessory orthopedic manual surgical instruments designed for use with the modified iTotal CR KRS are provided to assist with implantation. The ancillary instruments are provided sterile and for single-use only. These patient specific instruments are provided to assist in the positioning of total knee replacement components intra-operatively and in guiding the marking of bone before cutting.</p> <p>The function and general design features of the patient specific ancillary instruments remain similar to those described in the predicate 510(k) K122870.</p>

**510(K) SUMMARY CONTINUED**

**Substantial  
Equivalence:**

The product subject of this premarket notification is substantially equivalent to the iTotol Cruciate Retaining Knee Replacement System (K122870 cleared January 14, 2013). The following testing was performed to establish substantial equivalence:

- Patella Fixation Testing
- Steam Sterilization Validation and Product Performance for Metal Implants and iJig Instrumentation post steam sterilization to support instructions provided for sterilization of components in case of a loss of sterility.

## 510(k) Summary continued

## Device Comparison

Attribute	Predicate iTotal CR Knee Replacement System (K122870)	Modified Device iTotal CR Knee Replacement System (This submission)
<b>Components</b>	<ul style="list-style-type: none"> <li>• Femoral Component</li> <li>• Tibial Implant</li> <li>• Metal Backed Tibial Component</li> <li>• Patellar component</li> </ul>	<ul style="list-style-type: none"> <li>• Femoral Component</li> <li>• Tibial Implant</li> <li>• Metal Backed Tibial Component</li> <li>• Patellar component</li> </ul>
<b>Materials</b>	<ul style="list-style-type: none"> <li>• Femoral Implant: CoCrMo</li> <li>• Metal Backed Tibial Components:               <ul style="list-style-type: none"> <li>○ Tibial tray: CoCrMo</li> <li>○ Tibial Inserts: UHMWPE or iPoly XE</li> </ul> </li> <li>• All Polymer Patellar Component: UHMWPE or iPoly XE</li> </ul>	<ul style="list-style-type: none"> <li>• Femoral Implant: CoCrMo</li> <li>• Metal Backed Tibial Components:               <ul style="list-style-type: none"> <li>○ Tibial tray: CoCrMo</li> <li>○ Tibial Inserts: UHMWPE or iPoly XE</li> </ul> </li> <li>• All Polymer Patellar Component: UHMWPE or iPoly XE</li> </ul>
<b>Design</b>	Knee joint patellofemorotibial semi-constrained cemented prosthesis	Knee joint patellofemorotibial semi-constrained cemented prosthesis
<b>Principle of Operation</b>	Cemented Use Fixed Bearing Design	Cemented Use Fixed Bearing Design

Attribute	Predicate iTotal CR Knee Replacement System (K122870)	Modified Device iTotal CR Knee Replacement System (This submission)
<b>Indications for Use</b>	<p>The iTotal CR Knee Replacement System (KRS) is intended for use as a total knee replacement in patients with knee joint pain and disability whose conditions cannot be solely addressed by the use of a prosthetic device that treats only one or two of the three knee compartments, such as a unicondylar, patellofemoral or bicompartamental prosthesis.</p> <p>The Indications for Use include:</p> <ul style="list-style-type: none"> <li>• Painful joint disease due to osteoarthritis, traumatic arthritis, rheumatoid arthritis or osteonecrosis of the knee.</li> <li>• Post traumatic loss of joint function.</li> <li>• Moderate varus, valgus or flexion deformity in which the ligamentous structures can be returned to adequate function and stability.</li> <li>• Failed osteotomies, hemiarthroplasties, and unicondylar, patellofemoral or bicompartamental implants.</li> <li>• Revision procedures provided that anatomic landmarks necessary for alignment and positioning of the implant are identifiable on patient imaging scans.</li> </ul> <p>This implant is intended for cemented use only.</p>	<p>The iTotal CR Knee Replacement System (KRS) is intended for use as a total knee replacement in patients with knee joint pain and disability whose conditions cannot be solely addressed by the use of a prosthetic device that treats only one or two of the three knee compartments, such as a unicondylar, patellofemoral or bicompartamental prosthesis.</p> <p>The Indications for Use include:</p> <ul style="list-style-type: none"> <li>• Painful joint disease due to osteoarthritis, traumatic arthritis, rheumatoid arthritis or osteonecrosis of the knee.</li> <li>• Post traumatic loss of joint function.</li> <li>• Moderate varus, valgus or flexion deformity in which the ligamentous structures can be returned to adequate function and stability.</li> <li>• Failed osteotomies, hemiarthroplasties, and unicondylar, patellofemoral or bicompartamental implants.</li> <li>• Revision procedures provided that anatomic landmarks necessary for alignment and positioning of the implant are identifiable on patient imaging scans.</li> </ul> <p>This implant is intended for cemented use only.</p>
<b>Patient Matched</b>	Yes	Yes

Attribute	Predicate iTotal CR Knee Replacement System (K122870)	Modified Device iTotal CR Knee Replacement System (This submission)
<b>Tibial Implant</b>	<ul style="list-style-type: none"> <li>• Configuration: Metal Backed Tibial Implant</li> <li>• Tibial Insert – UHMWPE or iPoly XE</li> <li>• Single or Dual inserts</li> <li>• Insert sizes: 6-16mm</li> <li>• Profile: patient-specific</li> </ul>	<ul style="list-style-type: none"> <li>• Configuration: Metal Backed Tibial Implant</li> <li>• Tibial Insert – UHMWPE or iPoly XE</li> <li>• Single or Dual inserts</li> <li>• Insert sizes: 6-16mm</li> <li>• Profile: patient-specific</li> </ul>
<b>Posterior Cruciate Ligament (PCL) Sparing</b>	Yes	Yes
<b>Instrumentation</b>	Patient specific Nylon jigs	Patient specific Nylon jigs
<b>Packaging</b>	Device components are individually double-pouched using Tyvek®/film pouches which are sealed and labeled	Device components are individually double-pouched using Tyvek®/film pouches which are sealed and labeled
<b>Sterility Method/ Assurance Level</b>	VHP Gas Plasma $1 \times 10^{-6}$	VHP Gas Plasma $1 \times 10^{-6}$
<b>Instructions for Intra-Operative Sterilization</b>	None	Steam Sterilization (SAL of $1 \times 10^{-6}$ )
<b>Initial Shelf-Life</b>	6 months	6 months
<b>Labeled Non-pyrogenic</b>	No	No
<b>Patella Design and Sizes (diameter)</b>	Symmetrical, offered in sizes: 32mm, 35mm, 38mm, 41mm, 44mm	Symmetrical, offered in sizes, 29mm, 32mm, 35mm, 38mm, 41mm, 44mm

**510(k) Summary continued**

<b>Description and Conclusion of Testing</b>	<b>Nonclinical Testing:</b> The determination of substantial equivalence for this device was based on a detailed device description. The following non-clinical laboratory testing was performed demonstrating that the device is safe and can be considered substantially equivalent to the predicate device for the intended use: <ul style="list-style-type: none"><li>• Patella fixation testing</li><li>• Steam Sterilization Validation and Product Performance post steam sterilization to support instructions provided for sterilization of components in case of a loss of sterility.</li></ul>
<b>Safety and Performance</b>	The determination of substantial equivalence for this device was based on a detailed device description. Non-clinical laboratory testing was performed demonstrating that the device is safe and can be considered substantially equivalent to the predicate device for the proposed intended use. Clinical data is not necessary to demonstrate substantial equivalence.
<b>Conclusion</b>	Based on the testing conducted it is concluded that the modified device is substantially equivalent to the iTotal Cruciate Retaining Knee Replacement System K122870 cleared January 14, 2013.



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Center - WO66-G609  
Silver Spring, MD 20993-0002

July 18, 2013

ConforMIS, Incorporated  
% Ms. Amita Shah  
Senior Vice President, Regulatory and Quality Affairs  
28 Crosby Drive  
Boston, Massachusetts 01730

Re: K131467

Trade/Device Name: iTotal® CR Knee Replacement System (KRS)

Regulation Number: 21 CFR 888.3560

Regulation Name: Knee joint patellofemorotibial polymer/metal/polymer semi-constrained  
cemented prosthesis

Regulatory Class: Class II

Product Code: JWH, OOG, OIY

Dated: May 31, 2013

Received: June 4, 2013

Dear Ms. Shah:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical

device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

**Erin L. Keith**

For

Mark N. Melkerson  
Director  
Division of Orthopedic Devices  
Office of Device Evaluation  
Center for Devices and  
Radiological Health

Enclosure

### Indications for Use

510(k) Number (if known): K131467

Device Name: iTotal CR Knee Replacement System

#### Indications for Use:

The iTotal CR Knee Replacement System (KRS) is intended for use as a total knee replacement in patients with knee joint pain and disability whose conditions cannot be solely addressed by the use of a prosthetic device that treats only one or two of the three knee compartments, such as a unicondylar, patellofemoral or bicompartamental prosthesis.

#### The Indications for Use include:

- Painful joint disease due to osteoarthritis, traumatic arthritis, rheumatoid arthritis or osteonecrosis of the knee.
- Post traumatic loss of joint function.
- Moderate varus, valgus or flexion deformity in which the ligamentous structures can be returned to adequate function and stability.
- Failed osteotomies, hemiarthroplasties, and unicondylar, patellofemoral or bicompartamental implants.
- Revision procedures provided that anatomic landmarks necessary for alignment and positioning of the implant are identifiable on patient imaging scans.

**This implant is intended for cemented use only.**

Prescription Use   X    
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use \_\_\_\_\_  
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE OF  
NEEDED)

---

Concurrence of CDRH, Office of Device Evaluation (ODE)

**Ronald P. Jean -S**

---

(Division Sign-Off)  
Division of Orthopedic Devices  
510(k) Number: K131467