



28, chemin du Petit Bois  
59130 Ecully - FRANCE

TEL: +33 (0)4 72 29 26 26  
FAX: +33 (0)4 72 29 26 29

contact@in2bones.com  
www.in2bones.com

**JUL 30 2014**

**510(k) SUMMARY  
For In2Bones DUAFIT<sup>®</sup> interphalangeal implant**

<b>Sponsor identification</b>	In2Bones SAS 28 chemin du Petit Bois 69130 Ecully - France Phone: +33.4.72.29.26.26 Fax: +33.4.72.29.26.29
<b>Establishment registration number</b>	New company. Will register following FDA clearance (Owner/Operator Number: 10046803)
<b>Date of preparation</b>	July 23, 2014
<b>Contact person</b>	Norman Estrin Estrin Consulting Group LLC 9109 Copenhaver Drive Potomac, MD 20854 Phone: (301) 279-2899 Fax: (301) 294-0126 Email: estrin@yourFDAconsultant.com
<b>Authorized Agent in the United States</b>	Norman Estrin Estrin Consulting Group LLC 9109 Copenhaver Drive Potomac, MD 20854 Phone: (301) 279-2899 Fax: (301) 294-0126 Email: estrin@yourFDAconsultant.com
<b>Proprietary Name</b>	DuaFit <sup>®</sup> interphalangeal implant
<b>Common name</b>	Intramedullary bone fastener
<b>Device classification regulation</b>	21 CFR 888.3040: Smooth or threaded metallic bone fixation fastener Class II
<b>Device Product Code and Panel</b>	HWC: screw, fixation, bone 87 orthopedics
<b>Device Description</b>	The DUAFIT <sup>®</sup> interphalangeal implant is an intramedullary implant, designed to act as a bone fastener for proximal interphalangeal arthrodesis of the lesser rays.  Design is a combination of a proximal taper with barbs and a distal blade for enhanced stabilization.

---

It is made available in multiple lengths and diameters, and in 3 different angles.

The implant is manufactured from PEEK-OPTIMA<sup>®</sup>, polymer from Invibio<sup>®</sup>, and is designed for single use only.

Sizes:

The DUAFIT<sup>®</sup> interphalangeal implant is available in various angles (straight / 0° - 10° - 17°) and lengths (size 1 to 4, corresponding to 11 to 20mm).

Material:

The DUAFIT<sup>®</sup> interphalangeal implant is manufactured from Polyetheretherketone PEEK-OPTIMA<sup>®</sup>, polymer from Invibio<sup>®</sup>, as per ASTM 2026. It does not have any coating.

Single use:

The DUAFIT<sup>®</sup> interphalangeal implant is designed for single use only.

Sterilization:

The DUAFIT<sup>®</sup> interphalangeal implant is supplied sterile, using gamma irradiation.

Place of use:

The DUAFIT<sup>®</sup> interphalangeal implant is indicated for use in a hospital, or outpatient surgery center where sterile field may be created and maintained.

---

**Predicate Devices**

Newdeal K-wire (K022599)  
 Metasurg DigiFuse (K111536)  
 Parcus PEEK CF push-in suture anchor (K102326)  
 WMT Pro-Toe (K120645)  
 Arrowhead Arrow-Lok (K112675)  
 Smith and Nephew Bioraptor 2.3 PK suture anchor (K071586)  
 Merete MetaToe (K100414)  
 Synchro Medical Toegrip (K143477)

---

**Indications for use:**

The DUAFIT<sup>®</sup> interphalangeal implant is intended for fixation of proximal interphalangeal joint arthrodesis of the lesser toes.

Examples include:

- rigid or semi-rigid deformity of the PIP joint
  - revision of failed arthroplasty or arthrodesis
  - 2nd toe shortening
-

---

**Comparison of Technological characteristics**

The technological characteristics of the DUAFIT<sup>®</sup> interphalangeal implant are the same as the characteristics of predicate devices in terms of intended use and design. All these implants have the following features:

- Insertion into bone – the DUAFIT<sup>®</sup> interphalangeal implant and all predicate devices are intended for surgical implantation into bone for longer than 30 days.
  - Tapered barded design in its proximal part - The DUAFIT<sup>®</sup> interphalangeal implant has similar tapered barded design in its proximal part, when compared to the Parcus PEEK CF push-in suture anchor (K102326), Smith and Nephew Bioraptor 2.3 PK suture anchor (K071586), Metasurg DigiFuse implant (K111536) , WMT Pro-Toe (K120645), Merete MetaToe (K100414) and Synchro Toegrip (K133477).
  - Blade design in its distal part - The DUAFIT<sup>®</sup> interphalangeal implant has similar blade design in its distal part, when compared to the Metasurg DigiFuse implant (K111536) and WMT Pro-Toe (K120645). Its shape is also similar to the distal part of the Synchro Toegrip (K133477).
  - Canulated design: The DUAFIT<sup>®</sup> interphalangeal implant is available with a similar canulated design, when compared to the Metasurg DigiFuse implant (K111536).
  - Straight / Angled design: The DUAFIT<sup>®</sup> interphalangeal implant has similar straight and angled design, when compared to the Metasurg DigiFuse implant (K111536), WMT Pro-Toe (K120645) , Arrowhead Arrow-Lok (K112675) and Synchro Toegrip (K133477)
  - Made from PEEK: The DUAFIT<sup>®</sup> interphalangeal implant has exactly the same raw material PEEK-OPTIMA<sup>®</sup>, polymer from Invibio<sup>®</sup>, when compared to the Smith and Nephew Bioraptor 2.3 PK suture anchor (K071586). It has similar raw material, when compared to the Parcus PEEK CF push-in suture anchor (K102326) and Synchro Toegrip (K133477) . The Newdeal / Integra K-wire (K022599), the WMT Pro-Toe (K120645) and Arrowhead Arrow-Lok (K112675) are manufactured from stainless steel 316L, and the Metasurg DigiFuse implant is manufactured from Titanium Alloy TA6V. The Merete MetaToe is manufactured from EndoSorb PLGA bioabsorbable material.
  - Equivalent size range: The DUAFIT<sup>®</sup> interphalangeal implant has similar size range, when compared to the Metasurg DigiFuse implant (K111536), WMT Pro-Toe (K120645) , Arrowhead Arrow-Lok (K112675), Merete MetaToe (K100414) and Synchro Toegrip (K133477).
  - The DUAFIT<sup>®</sup> interphalangeal implant has similar intended use and mechanical properties when compared to the Merete MetaToe EndoSorb (K100414). Results from testing confirmed that the DUAFIT<sup>®</sup> interphalangeal implant is at least equivalent to the predicate device.
-

---

**Substantial  
Equivalence  
Summary**

The DUAFIT<sup>®</sup> interphalangeal implant has similar indications for use and technological characteristics when compared to the predicate devices.

---

**Summary Performance  
Data**

The following tests were performed to demonstrate that the DUAFIT interphalangeal implant is substantially equivalent to other predicate devices:

- Static four-point bending test
- Cyclic four-point bending test.

The results of these studies showed that the DUAFIT interphalangeal implant met the acceptance criteria.

---

**CONCLUSION**

**Based on the evaluations performed, the design and indications of the DUAFIT<sup>®</sup> interphalangeal implant are substantially equivalent to the predicate devices identified in the 510(k) submission. No new materials or processes are used in the development of this implant.**

**In addition, the results of the testing performed by the test lab indicated that the implants performed as expected for each test.**

**The DUAFIT<sup>®</sup> interphalangeal implants are acceptable for the application.**

---



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Center – WO66-G609  
Silver Spring, MD 20993-0002

July 30, 2014

In2Bones SAS  
% Norman F. Estrin, Ph.D.  
Estrin Consulting Group LLC  
9109 Copenhaver Drive  
Potomac, Maryland 20854

Re: K132912

Trade/Device Name: DUAFIT<sup>®</sup> Interphalangeal Implant  
Regulation Number: 21 CFR 888.3040  
Regulation Name: Smooth or threaded metallic bone fixation fastener  
Regulatory Class: Class II  
Product Code: HWC  
Dated: July 3, 2014  
Received: July 9, 2014

Dear Dr. Estrin:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set

Page 2 – Dr. Norman F. Estrin

forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

<http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

**Lori A. Wiggins**

for

Mark N. Melkerson

Director

Division of Orthopedic Devices

Office of Device Evaluation

Center for Devices and

Radiological Health

Enclosure

**Indications for Use**

510(k) Number (if known): K132912

Device Name: DUAFIT<sup>®</sup> Interphalangeal Implant

**Indications For Use:**

The DUAFIT<sup>®</sup> Interphalangeal implant is intended for fixation of proximal interphalangeal joint arthrodesis of the lesser toes.

Examples include:

- Rigid or semi-rigid deformity of the PIP joint
- Revision of failed arthroplasty or arthrodesis
- 2<sup>nd</sup> toe shortening.

Prescription Use   x    
(Part 21 CFR 801 Subpart D)

AND/OR

Over-The-Counter Use \_\_\_\_\_  
(21 CFR 801 Subpart C)

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE OF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

**Elizabeth Frank -S**

(Division Sign-Off)  
Division of Orthopedic Devices  
510(k) Number: K132912

000000