

JUN 20 2014

K140155



CardinalHealth

1500 Waukegan Road
Waukegan, IL 60085

www.cardinalhealth.com

510(k) SUMMARY

Secure-Gard® Surgical and Procedure Masks

Manufacturer: Cardinal Health 200, LLC
1500 Waukegan Road
Waukegan, IL 60085

Regulatory Affairs Contact: Lavenia Ford
1500 Waukegan Road
Waukegan, IL 60085

Telephone Number: (847) 887-3323

Fax Number: (847) 887-2461

Date summary Prepared: May 21, 2014

Trade Name: Secure-Gard® Surgical Mask
Secure-Gard® Procedure Mask

Regulation Number/Device Class: Class II per 21 CFR § 878.4040

Regulation Name: Surgical Apparel

Common Name: Surgical Mask
Procedure Mask

Product Code: FXX

Predicate Device: K111402 –Kimberly-Clark, KC300 Face Mask(s)

Description

The Cardinal Health Secure-Gard® surgical and procedure masks are identified by Regulation 21 CFR 878.4040 with product code FXX.

The Cardinal Health Secure-Gard® surgical and procedure masks are all the same four-layer masks constructed of 1 layer of nonwoven cellulose (inner facing), 1 layer of nonwoven polyolefin melt blown (media), 1 layer of nonwoven polyester/polyethylene blends (media), and 1 layer polyolefin spunbond material (outer facing). The four layers of the mask body are collated and sonically welded around the edges to enclose the filter media. The mask is provided with nonwoven polyolefin ties or polyester-spandex earloops (both attached by ultrasonic welding). A malleable nosepiece is placed within the binding for comfort and individualized fit around the wearer's nose. The surgical masks will be provided with or without an eye shield. Cardinal Health surgical and procedure masks are a single use, disposable device provided non-sterile

Indications for Use

Cardinal Health Secure-Gard® surgical and procedure masks are intended to be worn by operating room personnel and other general healthcare workers to protect both patients and healthcare workers against transfer of microorganisms, blood and body fluids, and airborne particulates. The Cardinal Health Secure-Gard® surgical and procedure masks are single use, disposable devices provided non-sterile.

This submission covers 7 catalog numbers of Secure-Gard® surgical and procedure masks models, see **Table 1** below. Each model is the same four-layer mask constructed of 1 layer of nonwoven cellulose (inner facing), 1 layer of nonwoven polyolefin melt blown (media), 1 layer of nonwoven polyester/polyethylene blends (media), and 1 layer polyolefin spunbond material (outer facing) and has been tested according to ASTM 2100-11 Standard Specification for Performance of Materials Used in Medical Face Masks.

Table 1: Product Description and Catalog Number

Catalog #	Trade Name	Design Features	Color
AT74535	Secure-Gard® surgical mask	Anti-fog foam strip and ties	Teal and Purple
AT74635	Secure-Gard® surgical mask	Anti-fog foam strip, eye shield, and ties	Teal and Purple
AT74635-I	Secure-Gard® surgical mask	Anti-fog foam strip, anti-glare eye shield, and ties	Teal and Purple
AT744235	Secure-Gard® surgical mask	Light weight anti-fog strip and ties	Teal and Purple
AT744335	Secure-Gard® surgical mask	Light weight anti-fog strip with eye shield and ties	Teal and Purple
AT74531	Secure-Gard® procedure mask	Earloops	Teal and Purple
AT74631	Secure-Gard® procedure mask	Eye shield and earloops	Teal and Purple

Device and Predicate Device Technical Characteristics

The Cardinal Health Secure-Gard® surgical and procedure face masks are substantially equivalent to the predicate device Kimberly-Clark KC300 of K111402 in intended use and principles of operation see **Table 3** below. The Cardinal Health Secure-Gard® surgical and procedure masks and predicate masks meet ASTM F2100-11 as shown below in **Table 11**.

Table 3: Comparison of Predicate device and Cardinal Health Secure-Gard® Surgical and Procedure masks

Element of Comparison	Kimberly-Clark (KC300)	Cardinal Health Secure-Gard® Surgical and Procedure Masks
Intended Use	<p>The Kimberly Clark KC300 Face Mask(s) is intended to be worn to protect both the patient and healthcare personnel from transfer of microorganisms, body fluids, and particulate material. These face masks are intended for use in infection control practices to reduce the potential exposure of the wearer to blood and body fluids. The Kimberly Clark KC300 face mask(s) is a single use, disposable device(s), provided non-sterile.</p>	<p>Secure-Gard Surgical Mask Cardinal Health Secure-Gard® surgical masks are intended to be worn by operating room personnel and other general healthcare workers to protect both patients and healthcare workers against transfer of microorganisms, blood and body fluids, and airborne particulates. The Cardinal Health Secure-Gard® surgical masks are single use, disposable devices provided non-sterile</p> <p>Secure-Gard Procedure Mask Cardinal Health Secure-Gard® procedure masks are intended to be worn by operating room personnel and other general healthcare workers to protect both patients and healthcare workers against transfer of microorganisms, blood and body fluids, and airborne particulates. The Cardinal Health Secure-Gard® procedure masks are single use, disposable devices provided non-sterile</p>
Material Composition	<p>4 layer mask made from nonwoven polyester blends and polypropylene materials</p>	<p>Secure-Gard Surgical Mask 4 layer mask made from 1 layer of nonwoven cellulose (inner facing), 1 layer of nonwoven polyolefin melt blown (media), 1 layer of nonwoven polyester/polyethylene blends (media), and 1 layer polyolefin spunbond material (outer facing)</p> <p>Secure-Gard Procedure Mask 4 layer mask made from 1 layer of nonwoven cellulose (inner facing), 1 layer of nonwoven polyolefin melt blown (media), 1</p>

		layer of nonwoven polyester/polyethylene blends (media), and 1 layer polyolefin spunbond material (outer facing)
Specifications and Dimensions	KC300 Surgical Mask 7.0"x3.7" KC300 Procedure Mask 6.9"x3.8"	Secure-Gard Surgical Mask AT74535 – 7"x4" AT74635 – 7 ¼"x4 1/8" AT74635-I – 7 ¼"x4 1/8" AT744235 – 7"x4" AT744335 – 7"x4" Secure-Gard Procedure Mask AT74531 – 7"x3 3/8" AT74631 – 7"x3 3/8" See Appendix I for mask specifications
Mask Style	Pleated	Secure-Gard Surgical Mask Pleated Secure-Gard Procedure Mask Pleated
Design Features	KC300 Surgical Mask Fluidshield Fog-Free KC300 Procedure Mask Fluidshield Fog-Free	Secure-Gard Surgical Mask AT74535 – Anti-fog foam strip and Ties AT74635 – Anti-fog foam strip, Eye shield, and Ties AT74635-I – Anti-fog foam strip, Anti-glare eye shield, and Ties AT744235 – Lightweight anti-fog strip and Ties AT744335 – Lightweight anti-fog strip, Eye shield, and Ties Secure-Gard Procedure Mask AT74531 – Earloops AT74631 – Eye shield and Earloops
Physical Testing	Predicate device was tested according to ASTM F2100-11 Level 3 in previous 510(k) submission K111402.	Secure-Gard Surgical Mask Device was tested in accordance with ASTM F2100-11, and meets Level 3 requirements Secure-Gard Procedure Mask Device was tested in accordance with ASTM F2100-11, and meets Level 3 requirements See Appendix G for all test data
Biocompatibility	Predicate device was tested according ISO 10993	Secure-Gard Surgical Mask Device was tested in accordance to ISO 10993 and passed

		acceptance criteria Secure-Gard Procedure Mask Device was tested in accordance to ISO 10993 and passed acceptance criteria Appendix H for all test data
--	--	--

Summary of Testing

The Cardinal Health Secure-Gard[®] surgical and procedure masks have been tested according to ASTM 2100-11 and standards which comprise ASTM F2100-11, Standard Specification for Performance of Materials Used in Medical Face Masks, see **Table 11** below.

Table 11: Summary of ASTM F2100-11 Testing – Cardinal Health Secure-Gard[®] Surgical and Procedure Masks compared to Predicate

Test	Acceptance Criteria per ASTM F2100-11 Level 3 (AQL = 4.0%)	Predicate device test results K111402 (KC300)	Current device test results	
			ASTM F2100-11 Level 3	Average
ASTM F1862 Synthetic Blood	160 mmHg	Pass	32/32 Pass	Not Applicable
ASTM F2101 BFE	≥ 98%	Pass	32/32 Pass	99.4 - 99.7%
ASTM F2299 PFE at 0.1 micron	≥ 98%	Pass	32/32 Pass	98.6 - 99.1%
Mil-M-36954C Delta P	< 5.0 mmH ₂ O/cm ²	Pass	32/32 Pass	3.1– 3.5 mmH ₂ O/cm ²
CPSC 1610 Flammability	Class 1	Pass	32/32 Pass Class 1	Not Applicable

All results of testing met ASTM F2100-11 Level 3 acceptance criteria. ASTM F2100-11 Level 3 was performed at an AQL of 4% (32 samples tested, accept on 3 failures and reject on 4 failures).

Summary of Testing

Based on the results of the biocompatibility and physical performance testing the Secure-Gard[®] surgical and procedure masks are as safe and as effective for their intended use as the predicate. The Secure-Gard[®] surgical and procedure masks are substantially equivalent to the predicate device, in terms of general intended use performance testing, material composition, configurations/dimensions, and safety and effectiveness.



DEPARTMENT OF HEALTH & HUMAN SERVICES

Public Health Service

Food and Drug Administration
10903 New Hampshire Avenue
Document Control Center - WO66-G609
Silver Spring, MD 20993-000

June 20, 2014

Cardinal Health 200, LLC
Ms. Lavenia Ford
Manager, Regulatory Affairs
1430 Waukegan Road
Waukegan, Illinois 60085

Re: K140155
Trade/Device Name: Secure-Gard® Surgical and Procedure Masks
Regulation Number: 21 CFR 878.4040
Regulation Name: Surgical Mask
Regulatory Class: Class II
Product Code: FXX
Dated: May 21, 2014
Received: May 22, 2014

Dear Ms. Ford,

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address <http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,


Tejasri Purohit-Sheth, M.D.

Tejasri Purohit-Sheth, M.D.
Clinical Deputy Director
DAGRHD/ODE/CDRH FOR

Erin I. Keith, M.S.
Director
Division of Anesthesiology, General Hospital,
Respiratory, Infection Control and
Dental Devices
Office of Device Evaluation
Center for Devices and
Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)
K140155

Device Name
Secure-Gard® surgical and procedure masks

Indications for Use (Describe)

Cardinal Health Secure-Gard® surgical and procedure masks are intended to be worn by operating room personnel and other general healthcare workers to protect both patients and healthcare workers against transfer of microorganisms, blood and body fluids, and airborne particulates. The Cardinal Health Secure-Gard® surgical and procedure masks are single use, disposable devices provided non-sterile.

This submission covers 7 catalog numbers of Secure-Gard® surgical and procedure masks models, see Table 1 below. Each model is the same four-layer mask constructed of 1 layer of nonwoven cellulose (inner facing), 1 layer of nonwoven polyolefin melt blown (media), 1 layer of nonwoven polyester/polyethylene blends (media), and 1 layer polyolefin spunbond material (outer facing) and has been tested according to ASTM F2100-11 Standard Specification for Performance of Materials Used in Medical Face Masks

Table 1: Product Description and Catalog Number

Catalog #	Trade Name	Design Features	Color
AT74535	Secure-Gard® surgical mask	Anti-fog foam strip and ties	Teal and Purple
AT74635	Secure-Gard® surgical mask	Anti-fog foam strip, eye shield, and ties	Teal and Purple
AT74635-I	Secure-Gard® surgical mask	Anti-fog foam strip, anti-glare eye shield, and ties	Teal and Purple
AT744235	Secure-Gard® surgical mask	Light weight anti-fog strip and ties	Teal and Purple
AT744335	Secure-Gard® surgical mask	Light weight anti-fog strip with eye shield and ties	Teal and Purple
AT74531	Secure-Gard® procedure mask	Earloops	Teal and Purple
AT74631	Secure-Gard® procedure mask	Eye shield and earloops	Teal and Purple

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON A SEPARATE PAGE IF NEEDED.

FOR FDA USE ONLY

Concurrence of Center for Devices and Radiological Health (CDRH) (Signature)

Sreekanth Gutala Digitally signed by Sreekanth Gutala -S
DN: cn=US, o=U.S. Government, ou=HHS, ou=FDA, ou=People,
0.9.2342.19200300.100.1.1=2000540490, cn=Sreekanth Gutala -S
Date: 2014.06.19 21:59:43 -04'00'

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."