



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Center – WO66-G609
Silver Spring, MD 20993-0002

Sonoscape Company Limited
% Mr Toki Wu
Regulatory Affairs Manager
Yizhe Bldg., Yuquan Road, Nanshan
Shenzhen 518051
P.R. CHINA

December 19, 2014

Re: K142474
Trade/Device Name: S12 Digital Color Doppler Ultrasound System
Regulation Number: 21 CFR 892.1550
Regulation Name: Ultrasonic pulsed doppler imaging system
Regulatory Class: II
Product Code: IYN, IYO, ITX
Dated: August 29, 2014
Received: December 1, 2014

Dear Mr. Wu:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638 2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, “Misbranding by reference to premarket notification” (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH’s Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

A handwritten signature in black ink that reads "Robert A. Ochs". The signature is written in a cursive style. A faint, semi-transparent "FDA" watermark is visible behind the signature.

Robert Ochs, Ph.D.
Acting Director
Division of Radiological Health
Office of In Vitro Diagnostics
and Radiological Health
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)
K142474

Device Name
S12 Digital Color Doppler Ultrasound System

Indications for Use (Describe)

The SonoScape S12 system is a general-purpose ultrasonic imaging instrument intended for use by a qualified physician for evaluation of Fetal, Abdominal, Pediatric, Small Organ (breast, testes, thyroid), Cephalic (neonatal and adult), Trans-rectal, Trans-vaginal, Peripheral Vascular, Musculo-skeletal (Conventional and Superficial), Cardiac (neonatal and adult), OB/Gyn and Urology.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D) Over-The-Counter Use (21 CFR 801 Subpart C)

PLEASE DO NOT WRITE BELOW THIS LINE – CONTINUE ON A SEPARATE PAGE IF NEEDED.

FOR FDA USE ONLY

Concurrence of Center for Devices and Radiological Health (CDRH) (Signature)

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

Diagnostic Ultrasound Indications for Use Form

System: SonoScape S12
 Diagnostic Ultrasound Pulsed Echo System
 Diagnostic Ultrasound Pulsed Doppler Imaging System

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal	N	N	N		N	N	Note 1	Notes 2,4,5
	Abdominal	N	N	N		N	N	Note 1	Notes 2,4,5
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric	N	N	N		N	N	Note 1	Notes 2,4
	Small Organ (specify)	N	N	N		N	N	Note 1	Notes 2,4,6,7
	Neonatal Cephalic	N	N	N	N	N	N	Note 1	Notes 2,3,4
	Adult Cephalic	N	N	N	N	N	N	Note 1	Notes 2,3,4
	Trans-rectal	N	N	N		N	N	Note 1	Notes 2,4
	Trans-vaginal	N	N	N		N	N	Note 1	Notes 2,4
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)	N	N	N		N	N	Note 1	Notes 2,4
	Musculo-skeletal (Superficial)	N	N	N		N	N	Note 1	Notes 2,4
	Intravascular								
Other (Ob/GYN)	N	N	N		N	N	Note 1	Notes 2,4,5	
Other (Urology)	N	N	N		N	N	Note 1	Notes 2,4	
Cardiac	Cardiac Adult	N	N	N	N	N	N	Note 1	Notes 2,3,4
	Cardiac Pediatric	N	N	N	N	N	N	Note 1	Notes 2,3,4
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)	N	N	N	N	N	N	Note 1	Notes 2,3,4
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel	N	N	N		N	N	Note 1	Notes 2,4
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: C322 Curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
		Ophthalmic	Ophthalmic						
Fetal Imaging & Other	Fetal	P	P	P		P	P	Note 1	Notes 2,4
	Abdominal	P	P	P		P	P	Note 1	Notes 2,4
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric								
	Small Organ (specify)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
	Intravascular								
Other (Ob/GYN)		P	P	P		P	P	Note 1	Notes 2,4
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
	Other (specify)								
Peripheral Vessel	Peripheral vessel								
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix

Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD

Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents

Note 3: TDI Note 4: 3D Note 5: 4D

Note 6: Small Organ: breast, thyroid, testes

Note 7: Elastography

(Division Sign Off)
Division of Radiological Health
Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: C344 Curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal	P	P	P		P	P	Note 1	Notes 2,4
	Abdominal	P	P	P		P	P	Note 1	Notes 2,4
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric								
	Small Organ (specify)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
	Intravascular								
Other (Ob/GYN)		P	P	P		P	P	Note 1	Notes 2,4
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel								
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix

Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD

Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents

Note 3: TDI Note 4: 3D Note 5: 4D

Note 6: Small Organ: breast, thyroid, testes

Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: C354 Curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
		Ophthalmic	Ophthalmic						
Fetal Imaging & Other	Fetal	P	P	P		P	P	Note 1	Notes 2,4
	Abdominal	P	P	P		P	P	Note 1	Notes 2,4
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric								
	Small Organ (specify)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
	Intravascular								
Other (Ob/GYN)		P	P	P		P	P	Note 1	Notes 2,4
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel								
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: C542 Curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
		Ophthalmic	Ophthalmic						
Fetal Imaging & Other	Fetal								
	Abdominal	P	P	P		P	P	Note 1	Notes 2,4
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric	P	P	P		P	P	Note 1	Notes 2,4
	Small Organ (specify)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
	Intravascular								
Other (Ob/GYN)									
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel								
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: VC6-2 Curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)					Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
		B	M	PWD	CWD				
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal	P	P	P		P	P	Note 1	Notes 2,4,5
	Abdominal	P	P	P		P	P	Note 1	Notes 2,4,5
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric								
	Small Organ (specify)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
Intravascular									
Other (Ob/GYN)		P	P	P		P	P	Note 1	Notes 2,4,5
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
	Other (specify)								
Peripheral Vessel	Peripheral vessel								
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix

Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD

Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents

Note 3: TDI Note 4: 3D Note 5: 4D

Note 6: Small Organ: breast, thyroid, testes

Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: C613 Micro-curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal	N	N	N		N	N	Note 1	Notes 2,4
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric	N	N	N		N	N	Note 1	Notes 2,4
	Small Organ (specify)								
	Neonatal Cephalic	N	N	N	N	N	N	Note 1	Notes 2,3,4
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
Intravascular									
Other (Ob/GYN)									
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric	N	N	N	N	N	N	Note 1	Notes 2,3,4
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel								
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: 2P1 Phase Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation								
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B		M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
		B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify	
Ophthalmic	Ophthalmic									
Fetal Imaging & Other	Fetal									
	Abdominal	P	P	P			P	P	Note 1	Notes 2,4
	Intra-operative Specify									
	Intra-operative Neuro									
	Laparoscopic									
	Pediatric									
	Small Organ (specify)									
	Neonatal Cephalic	P	P	P	P	P	P	P	Note 1	Notes 2,3,4
	Adult Cephalic	P	P	P	P	P	P	P	Note 1	Notes 2,3,4
	Trans-rectal									
	Trans-vaginal									
	Trans-urethral									
	Trans-esoph.(non-Card)									
	Musculo-skeletal (Conventional)									
	Musculo-skeletal (Superficial)									
Intravascular										
Other (Ob/GYN)										
Other (Urology)										
Cardiac	Cardiac Adult	P	P	P	P	P	P	P	Note 1	Notes 2,3,4
	Cardiac Pediatric	P	P	P	P	P	P	P	Note 1	Notes 2,3,4
	Intravascular(Cardiac)									
	Trans-esoph.(Cardiac)									
	Intra-cardiac									
	Other (specify)									
Peripheral Vessel	Peripheral vessel									
	Other (specify)									

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD;
 B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: 5P1 Phase Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal								
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric	P	P	P		P	P	Note 1	Notes 2,4
	Small Organ (specify)								
	Neonatal Cephalic	P	P	P	P	P	P	Note 1	Notes 2,3,4
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
	Intravascular								
Other (Ob/GYN)									
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric	P	P	P	P	P	P	Note 1	Notes 2,3,4
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel								
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: L741 Linear Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal								
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric								
	Small Organ (specify)	P	P	P		P	P	Note 1	Notes 2,4,6,7
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)	P	P	P		P	P	Note 1	Notes 2,4
	Musculo-skeletal (Superficial)								
Intravascular									
Other (Ob/GYN)									
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel	P	P	P		P	P	Note 1	Notes 2,4
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD;
 B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: L742 Linear Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal								
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric								
	Small Organ (specify)	P	P	P		P	P	Note 1	Notes 2,4,6
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)	P	P	P		P	P	Note 1	Notes 2,4
	Musculo-skeletal (Superficial)	P	P	P		P	P	Note 1	Notes 2,4
Intravascular									
Other (Ob/GYN)									
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel	P	P	P		P	P	Note 1	Notes 2,4
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: 10L1 Linear Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify
		Ophthalmic	Ophthalmic						
Fetal Imaging & Other	Fetal								
	Abdominal								
	Intra-operative Specify								
	Intra-operative Neuro								
	Laparoscopic								
	Pediatric								
	Small Organ (specify)	P	P	P		P	P	Note 1	Notes 2,4,6
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph.(non-Card)								
	Musculo-skeletal (Conventional)	P	P	P		P	P	Note 1	Notes 2,4
	Musculo-skeletal (Superficial)	P	P	P		P	P	Note 1	Notes 2,4
Intravascular									
Other (Ob/GYN)									
Other (Urology)									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular(Cardiac)								
	Trans-esoph.(Cardiac)								
	Intra-cardiac								
Other (specify)									
Peripheral Vessel	Peripheral vessel	P	P	P		P	P	Note 1	Notes 2,4
	Other (specify)								

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: 6V1 Micro-curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation								
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify	
Ophthalmic	Ophthalmic									
Fetal Imaging & Other	Fetal									
	Abdominal									
	Intra-operative Specify									
	Intra-operative Neuro									
	Laparoscopic									
	Pediatric									
	Small Organ (specify)									
	Neonatal Cephalic									
	Adult Cephalic									
	Trans-rectal		P	P	P		P	P	Note 1	Notes 2,4
	Trans-vaginal		P	P	P		P	P	Note 1	Notes 2,4
	Trans-urethral									
	Trans-esoph.(non-Card)									
	Musculo-skeletal (Conventional)									
	Musculo-skeletal (Superficial)									
	Intravascular									
Other (Ob/GYN)										
Other (Urology)		P	P	P		P	P	Note 1	Notes 2,4	
Cardiac	Cardiac Adult									
	Cardiac Pediatric									
	Intravascular(Cardiac)									
	Trans-esoph.(Cardiac)									
	Intra-cardiac									
	Other (specify)									
Peripheral Vessel	Peripheral vessel									
	Other (specify)									

N = new indication; P = previously cleared by FDA; E = added under this appendix

Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD

Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents

Note 3: TDI Note 4: 3D Note 5: 4D

Note 6: Small Organ: breast, thyroid, testes

Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: EC9-5 Micro-curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation								
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify	
		Ophthalmic	Ophthalmic							
Fetal Imaging & Other	Fetal									
	Abdominal									
	Intra-operative Specify									
	Intra-operative Neuro									
	Laparoscopic									
	Pediatric									
	Small Organ (specify)									
	Neonatal Cephalic									
	Adult Cephalic									
	Trans-rectal		P	P	P		P	P	Note 1	Notes 2,4
	Trans-vaginal		P	P	P		P	P	Note 1	Notes 2,4
	Trans-urethral									
	Trans-esoph.(non-Card)									
	Musculo-skeletal (Conventional)									
	Musculo-skeletal (Superficial)									
Intravascular										
Other (Ob/GYN)										
Other (Urology)		P	P	P		P	P	Note 1	Notes 2,4	
Cardiac	Cardiac Adult									
	Cardiac Pediatric									
	Intravascular(Cardiac)									
	Trans-esoph.(Cardiac)									
	Intra-cardiac									
	Other (specify)									
Peripheral Vessel	Peripheral vessel									
	Other (specify)									

N = new indication; P = previously cleared by FDA; E = added under this appendix

Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD

Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents

Note 3: TDI Note 4: 3D Note 5: 4D

Note 6: Small Organ: breast, thyroid, testes

Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: BCC9-5 Micro-curved Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation								
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify	
		Ophthalmic	Ophthalmic							
Fetal Imaging & Other	Fetal									
	Abdominal									
	Intra-operative Specify									
	Intra-operative Neuro									
	Laparoscopic									
	Pediatric									
	Small Organ (specify)									
	Neonatal Cephalic									
	Adult Cephalic									
	Trans-rectal		P	P	P		P	P	Note 1	Notes 2,4
	Trans-vaginal		P	P	P		P	P	Note 1	Notes 2,4
	Trans-urethral									
	Trans-esoph.(non-Card)									
	Musculo-skeletal (Conventional)									
	Musculo-skeletal (Superficial)									
Intravascular										
Other (Ob/GYN)										
Other (Urology)		P	P	P		P	P	Note 1	Notes 2,4	
Cardiac	Cardiac Adult									
	Cardiac Pediatric									
	Intravascular(Cardiac)									
	Trans-esoph.(Cardiac)									
	Intra-cardiac									
Other (specify)										
Peripheral Vessel	Peripheral vessel									
	Other (specify)									

N = new indication; P = previously cleared by FDA; E = added under this appendix

Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD

Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents

Note 3: TDI Note 4: 3D Note 5: 4D

Note 6: Small Organ: breast, thyroid, testes

Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

Diagnostic Ultrasound Indications for Use Form

Transducer: BCL10-5 Biplane Array

Diagnostic Ultrasound Transducer

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation								
General (TRACK 1 ONLY)	Specific (TRACKS 1 & 3)	B	M	PWD	CWD	Color Doppler	Power (Amplitude) Doppler	Other* Combined	Other* Specify	
Ophthalmic	Ophthalmic									
Fetal Imaging & Other	Fetal									
	Abdominal									
	Intra-operative Specify									
	Intra-operative Neuro									
	Laparoscopic									
	Pediatric									
	Small Organ (specify)									
	Neonatal Cephalic									
	Adult Cephalic									
	Trans-rectal		P	P	P		P	P	Note 1	Notes 2,4
	Trans-vaginal		P	P	P		P	P	Note 1	Notes 2,4
	Trans-urethral									
	Trans-esoph.(non-Card)									
	Musculo-skeletal (Conventional)									
	Musculo-skeletal (Superficial)									
Intravascular										
Other (Ob/GYN)										
Other (Urology)		P	P	P		P	P	Note 1	Notes 2,4	
Cardiac	Cardiac Adult									
	Cardiac Pediatric									
	Intravascular(Cardiac)									
	Trans-esoph.(Cardiac)									
	Intra-cardiac									
Other (specify)										
Peripheral Vessel	Peripheral vessel									
	Other (specify)									

N = new indication; P = previously cleared by FDA; E = added under this appendix
 Note 1: Other Combined includes: B/M; B/PWD; B/THI; M/Color M; B/Color Doppler; B/Color Doppler/PWD; B/Power Doppler/PWD
 Note 2: Tissue Harmonic Imaging. The feature does not use contrast agents
 Note 3: TDI Note 4: 3D Note 5: 4D
 Note 6: Small Organ: breast, thyroid, testes
 Note 7: Elastography

 (Division Sign Off)
 Division of Radiological Health
 Office of *In Vitro* Diagnostic and Radiological Health

510(k) _____ K142474 _____

510(k) Summary

1. Submitter [21 CFR807.92 (a) (1)]

Submitter: SonoScape Company Limited
 Address: Yizhe Building, Yuquan Road, Nanshan, Shenzhen
 518051, P.R.China
 Contact Person: Toki Wu
 Tel: +86 755 26722890
 Fax: +86 755 26722850
 Email: wusq@sonoscape.net
 Date Prepared August 29, 2014

2. Device [21 CFR807.92 (a)(2)]

Trade Name: S12 Digital Color Doppler Ultrasound System
 Common Name: Diagnostic Ultrasound System and Transducers

Classification Regulatory:

	<u>FR Number</u>	<u>Product Code</u>
Ultrasonic Pulsed Doppler Imaging System	892.1550	90-IYN
Ultrasonic Pulsed Echo Imaging System	892.1560	90-IYO
Diagnostic Ultrasound Transducer	892.1570	90-ITX

Classification Panel: Radiology

Device Class: II

3. Predicate Device(s) [21 CFR 807.92(a) (3)]

The identified predicate devices within this submission are as follows:

SonoScape S8 Exp Portable Digital Color Doppler Ultrasound System	K132768
Philips EPIQ Diagnostic Ultrasound System	K132304

4. Device Description [21 CFR 807.92(a) (4)]

This SonoScape S12 Digital Color Doppler Ultrasound System is an integrated preprogrammed color ultrasound imaging system, capable of producing high detail resolution intended for clinical diagnostic imaging applications.

The basic principle is that system transmits ultrasonic energy into patient body and

implements post processing of received echoes to generate onscreen display of anatomic structures and fluid flow within the body.

This system is a Track 3 device that employs a wide array of probes that include linear array, convex array and phased array with a frequency range of 2.0 MHz to 15.0 MHz.

This system consists of a mobile console with keyboard control panel, power supply module, color LCD monitor and optional probes.

This system is a mobile, general purpose, software controlled, color diagnostic ultrasound system. Its basic function is to acquire ultrasound data and to display the image in B-Mode (including Tissue Harmonic Image), M-Mode, TDI, Color-Flow Doppler, Pulsed Wave Doppler, Continued Wave Doppler and Power Doppler, or the combination of these modes, Elastography, 3D/4D.

5. Intended Use [21 CFR 807.92(a) (5)]

The SonoScape S12 system is a general-purpose ultrasonic imaging instrument intended for use by a qualified physician for evaluation of Fetal, Abdominal, Pediatric, Small Organ (breast, testes, thyroid), Cephalic(neonatal and adult), Trans-rectal, Trans-vaginal, Peripheral Vascular, Musculo-skeletal (Conventional and Superficial), Cardiac (neonatal and adult), OB/Gyn and Urology.

6. Comparison with the Predicate Devices [21 CFR 807.92(a) (6)]

S12 Digital Color Doppler Ultrasound System is comparable with and substantially equivalent to the predicate device:

SonoScape S8 Exp Portable Digital Color Doppler Ultrasound System	K132768
Philips EPIQ Diagnostic Ultrasound System	K132304

The basic and main technical features of the subject device S12 are the same as the predicated device S8 Exp (K132768), including Design, Operation Controls, Display Modes, Measurement Items, Cine Loop and Operating and Storage Condition.

For Operation Modes, the essential imaging modes and functions of the Subject Device S12 are identical as the Predicate Device S8 Exp (K132768), including B, M, PW, CW, CFM, DPI, TDI, Color M and 3D/4D Mode, and Tissue Harmonic Image, PIH, Compound Imaging, Panoramic Imaging and Trapezoid Imaging. The Elastography is a special Operation Mode for the Subject Device S12, but already employed by many marketed devices and considered Substantially Equivalent to the Predicate Device Philips EPIQ (K132304) and detailed analysis shown as the following.

There are some differences in the Power Supply and Screen Size, but considered Substantially Equivalent in safety and effectiveness and detailed analysis shown as the

following.

Table 1 Technological Characteristics Comparison

ID	Comparison Items	Subject device SonoScape S12	Predicate Device SonoScape S8 Exp	Remark
1	Design	Based on an embedded Linux operating system.	Based on an embedded Linux operating system.	Same
		Autocorrelation for color processing and FFT for pulse and CW Doppler processing.	Autocorrelation for color processing and FFT for pulse and CW Doppler processing.	Same
		Supporting Linear, Curve linear and Phase array probes from 2 to 15 MHz.	Supporting Linear, Curve linear and Phase array probes from 2 to 15 MHz.	Same
		Cine play back capability	Cine play back capability	Same
		Image file archive	Image file archive	Same
		Software upgrade with USB flash drive.	Software upgrade with USB flash drive.	Same
		Digital Scan Converter 800x600	Digital Scan Converter 800x600	Same
		With full keyboard panel	With full keyboard panel	Same
2	Operation Controls	TGC 8 slider	TGC 8 slider	Same
		Depth Range: 3 to 24.8cm	Depth Range: 3 to 24.8cm	Same
		B Dynamic range control: 20-280dB	B Dynamic range control: 20-280dB	Same
		Gray Scale Control: 1,2,3,4,5, 6,7 (7 optional)	Gray Scale Control: 1,2,3,4,5, 6,7 (7 optional)	Same
		Focal Number: adjustable, max 12	Focal Number: adjustable, max 12	Same
		B persistence:0-95%	B persistence: 0-95%	Same
		PW sweeping speed 2,4,6,8 sec over display	PW sweeping speed 2,4,6,8 sec over display	Same
		PW Wall filter setting: 35 to 750	PW Wall filter setting: 35 to 750	Same
		PW sample volume:0.7 to 21mm	PW sample volume:0.7 to 21mm	Same
		PW angle correction: adjustable, 0 to 72 degree	PW angle correction: adjustable, 0 to 72 degree	Same
		Spectrum baseline: adjustable, such as shift and invert	Spectrum baseline: adjustable, such as shift and invert	Same
		Color ROI setting: trackball and set key to control size and position	Color ROI setting: trackball and set key to control size and position	Same
		Color Wall Filter settings: 35 to 750	Color Wall Filter settings: 35 to 750	Same

ID	Comparison Items	Subject device SonoScape S12	Predicate Device SonoScape S8 Exp	Remark
		Zoom adjustable	Zoom adjustable	Same
		Freeze control: Toggling freeze key	Freeze control: Toggling freeze key	Same
		Cine control: play/stop, loop speed, the start, the end, Frame by Frame	Cine control: play/stop, loop speed, the start, the end, Frame by Frame	Same
3	Operation Mode	B, M, PW, CW, CFM, DPI, TDI, Tissue Harmonic Image, Color M Mode, PIH, Compound Imaging, 3D/4D Mode Panoramic Imaging, Trapezoid Imaging.	B, M, PW, CW, CFM, DPI, TDI, Tissue Harmonic Image, Color M Mode, PIH, Compound Imaging, 3D/4D Mode Panoramic Imaging, Trapezoid Imaging.	SE Analysis1
4	Display Modes	B,CFM,DPI,TDI, 4D: Single, Dual, Quad B+CFM,B+DPI,B+TDI: Dual Live B/M, CFM/M,TDI/M, Steer M: V1/3,V1/2(Dual),V2/3, H1/2,H1/4,01/4 PW,CW:V1/3,V1/2(Dual),V2/3, H1/2,H1/4,01/4	B,CFM,DPI,TDI, 4D: Single, Dual, Quad B+CFM,B+DPI,B+TDI: Dual Live B/M, CFM/M,TDI/M, Steer M: V1/3,V1/2(Dual),V2/3, H1/2,H1/4,01/4 PW,CW:V1/3,V1/2(Dual),V2/3, H1/2,H1/4,01/4	Same
5	Measurement Items	B, CFM, DPI, TDI: Area Ratio, Angle, Volume, Volume L x W x H, Doppler Area, Vascular, Small Part, Ortho, OB/GYN, Cardiac, URO, VHE; M: Distance, Time, Slope, HR, Left Ventricle, Mitral Valve, Aortic Valve; PW/CW: Flow Velocity, Acceleration, Time, Trace, Auto Trace, HR, Vascular, OB/GYN, Cardiac, VE	B, CFM, DPI, TDI: Area Ratio, Angle, Volume, Volume L x W x H, Doppler Area, Vascular, Small Part, Ortho, OB/GYN, Cardiac, URO, VHE; M: Distance, Time, Slope, HR, Left Ventricle, Mitral Valve, Aortic Valve; PW/CW: Flow Velocity, Acceleration, Time, Trace, Auto Trace, HR, Vascular, OB/GYN, Cardiac, VE	Same
6	Cine Loop	Automatic review/ manual review	Automatic review/ manual review	Same
		Review speed can be adjusted	Review speed can be adjusted	Same
7	Operating Condition	Temperature: 10~40°C	Temperature: 10~40°C	Same
		Relative humidity: 30~75%	Relative humidity: 30~75%	Same
		Air pressure: 700hPa ~1060hPa	Air pressure: 700hPa ~1060hPa	Same
8	Storage Condition	Temperature: -20~55°C	Temperature: -20~55°C	Same
		Relative humidity: 20~90%	Relative humidity: 20~90%	Same

ID	Comparison Items	Subject device SonoScape S12	Predicate Device SonoScape S8 Exp	Remark
		Air pressure: 700hPa ~1060hPa	Air pressure: 700hPa ~1060hPa	Same
9	Power Supply	Voltage: 100-240V AC	Voltage: 110-240 VAC	SE Analysis2
		Frequency: 50/60 Hz	Frequency: 50/60 Hz	Same
		Power Consumption: 100-240V AC, 2.7-1.1A	Power Consumption: 110-240V AC, 2.7-1.2A	SE Analysis2
10	Screen Size	15 inch LCD monitor	15 inch Widescreen LCD monitor	SE Analysis3

SE Analysis 1:

According to the following comparison of Elastography technique, clinical applications and functions, the Subject Device S12 has the same Elastography technique and clinical applications and similar functions as the Predicate Device Philips EPIQ (K132304). Therefore the Subject Device S12 can be considered Substantially Equivalent to the Predicate Devices in safety and effectiveness, and no new risk is raised, so the SE is not affected.

Table 2 Elastography Comparison

Comparison Items	Subject device SonoScape S12	Predicate Device Philips EPIQ
Elastography Technique	Strain Elastography: In this technique, an external compression is applied to tissue, and then physical compression results in a 2D display of strain distribution in real-time according to the different elastic properties of body tissue and indicates relative stiffness.	Strain Elastography: In this technique, an external compression is applied to tissue, and then physical compression results in a 2D display of strain distribution in real-time according to the different elastic properties of body tissue and indicates relative stiffness.
Clinical Applications	As an adjunct technique for clinical practice, the Elastography determines if an area of tissue is hard or soft as compared with its surroundings. Furthermore, it provides the Elastography image to discover tumors (stiffer than the surrounding tissue). The Elastography image displays a range of map shades from the softest	Elastography is used clinically as an adjunct technique in a small parts imaging practice to determine if an area of tissue is hard or soft as compared with its surroundings. The elastogram will display a range of map shades from the softest tissue in the image to the stiffest in a given field of view. As a way of discovering tumors

	tissue in the image to the stiffest in a given field of view.	(tumors are stiffer than the surrounding tissue).
Functions	<ul style="list-style-type: none"> • Generating elasticity image. • Perform the Distance and Area (Ellipse or Trace) measurement on either 2D or elasticity image and the measurement will auto populate to the other side, which help to assess and validate the size and location of lesion. • Strain ratio allows calculation of relative strain of two interest regions (lesion, the surrounding tissue) which indicates the stiffness of the lesion indirectly. • Other optimizing parameter and auxiliary function. 	<ul style="list-style-type: none"> • Generating Elastogram. • Choose Ellipse, Trace, or Distance and perform the measurement on either 2D or Elasto first; the measurement will be copied onto the opposite image to validate size and location of lesion. • Strain ratio allows calculation of relative strain of two regions of interest over time. • Other adjusting parameter and auxiliary function.

SE Analysis 2:

Power Supply, Compared with the predicate device, the subject device employs the same power type and has wider input voltage range and some differences in current range. But both them comply with the requirements of IEC60601-1 and include the voltage range in the United States. Therefore they can be considered Substantially Equivalent in safety and effectiveness, and no new risk is raised, so the SE is not affected.

SE Analysis 3:

Screen Size, Compared with the predicate device, the subject device employs the same screen type and has some difference in screen size. But both of them comply with the requirements of IEC60601-1 & IEC60601-1-2 and meet clinical requirements. Therefore they can be considered Substantially Equivalent in safety and effectiveness, and no new risk is raised, so the SE is not affected.

7. Non-Clinical Tests [21 CFR 807.92(b) (1)]

The S12 Digital Color Doppler Ultrasound System has been evaluated for electrical, mechanical, thermal and electromagnetic compatibility safety, biocompatibility and acoustic output.

Laboratory tests were conducted to verify that the S12 system met all design specifications and the S12 system conformed to applicable medical device standards. Phantom test was conducted to verify that the strain Elastography function was effective and Elastography performance met design specifications, including accuracy and repeatability of strainratio measurement and etc.

The S12 system has been designed and manufactured to meet the following standards: IEC 60601-1, IEC 60601-1-2, IEC 60601-2-37, ISO 10993-5, ISO10993-10, UD2, and UD3.

8. Clinical Test [21 CFR 807.92(b) (2)]

No clinical testing was required.

9. Substantially Equivalent Conclusions [21 CFR 807.92(b) (3)]

In accordance with the 21 CFR Part 807 and based on the information provided in this premarket notification, SonoScape Company Limited concludes that S12 Digital Color Doppler Ultrasound System is substantially equivalent to the predicate devices with regard to safety and effectiveness.