



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Center - WO66-G609  
Silver Spring, MD 20993-0002

August 7, 2015

Grandway Technology (shenzhen) Limited  
Mr. Patrick Chow  
General Manager  
Block 7, Zhu Keng Industrial Zone, Ping Shan District,  
Shenzhen, 518118 CN

Re: K151587  
Trade/Device Name: Digital Automatic Wrist Blood Pressure Monitor MD3200  
Regulation Number: 21 CFR 870.1130  
Regulation Name: Non-Invasive Blood Pressure Measurement System  
Regulatory Class: Class II  
Product Code: DXN  
Dated: Not dated  
Received: June 11, 2015

Dear Mr. Patrick Chow,

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the [Federal Register](#).

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in

the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

<http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

**Bram D. Zuckerman -S**

Bram D. Zuckerman, M.D.  
Director  
Division of Cardiovascular Devices  
Office of Device Evaluation  
Center for Devices and Radiological Health

Enclosure

**510(k) Number (if known):**  K151587

**Device Name:**  Digital Automatic Wrist Blood Pressure Monitor MD3200

**Indications For Use:**

The wrist blood pressure monitor is used to carry out non-invasive measurement and monitoring of arterial blood pressure values in human adults. This allows you to quickly and easily measure your systolic and diastolic pressure, and pulse rate through an inflatable cuff wrapped around the wrist, save the measurements and display the development of the measurements.

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE IF NEEDED)

---

Concurrence of CDRH, Office of Device Evaluation (ODE)

**Prescription Use** \_\_\_\_\_  
(Per 21 CFR 801.109)

OR

**Over-The-Counter Use**  X

## **510(k) Summary**

### **1. Submitter Identification**

510(k) Submitter	GRANDWAY TECHNOLOGY (SHENZHEN) LIMITED
Address	Block 7, Zhu Keng Industrial Zone, Ping Shan District, 518118 Shenzhen, PEOPLE'S REPUBLIC OF CHINA
Phone Number	(00852)-2851-6789
Fax Number	(00852)-2851-6278
Contact Person	Mr. Patrick Chow
Date of Submission	<a href="#">10-Jun-2015</a>

### **2. Device Identification**

Trade Name	Digital Automatic Wrist Blood Pressure Monitor MD3200 <a href="#">[Model No.: MD3200]</a>
Common Name	Non-invasive Blood Pressure Measurement System
Classification Name	Non-invasive Blood Pressure Measurement System (CFR 870.1130, Class II, Product Code DXN)

### **3. Predicate Device**

Predicate Device	Digital Automatic Wrist Blood Pressure Monitor SWBPM22 Series
Manufacturer	<a href="#">GRANDWAY TECHNOLOGY (SHENZHEN) LIMITED</a>
510(k) Number	<a href="#">K142088</a>

### **4. Device Description**

Digital Automatic Wrist Blood Pressure Monitor MD3200 is a non-invasive blood pressure measurement system for use by medical professional or at home. It is designed to measure the systolic and diastolic blood pressure, and pulse rate of an individual in each measurement and then display the readings on a digital panel.

The device utilizes the oscillometric methodology, in which an inflatable cuff is wrapped around the wrist of an individual, for blood pressure measurement. This means the monitor

detects your blood's movement through your brachial artery and converts the movement into a digital reading.

The table below illustrates the feature presence in Digital Automatic Wrist Blood Pressure Monitor MD3200.

Model	Blood Pressure Measurement	Pulse Rate Measurement	WHO Classification	Irregular Heartbeat	LCD Type	User × Memory
MD3200	✓	✓	✓	✓	Positive Reflective	2 × 60

## 5. Indication for Use

The wrist blood pressure monitor is used to carry out non-invasive measurement and monitoring of arterial blood pressure values in human adults. This allows you to quickly and easily measure your systolic and diastolic pressure, and pulse rate through an inflatable cuff wrapped around the wrist, save the measurements and display the development of the measurements

## 6. Comparison of Technological Characteristics between New Device and Predicate Devices

Digital Automatic Wrist Blood Pressure Monitor MD3200 is compared to the predicate device, [SWBPM22 Series \(K142088\)](#) in the device comparison table below.

<b>Comparison between Digital Automatic Wrist Blood Pressure Monitor MD3200 and Predicate device</b>			
Item	Predicate Device	Digital Automatic Wrist Blood Pressure Monitor MD3200	Comment
Indication for Use	This device is for use by medical professional or home users. It is intended to measure the systolic and diastolic blood pressure of an adult individual by using a non-invasive technique, in which an inflatable cuff is wrapped around the wrist.	The wrist blood pressure monitor is used to carry out non-invasive measurement and monitoring of arterial blood pressure values in human adults. This allows you to quickly and easily measure your systolic and diastolic pressure, and pulse rate through an inflatable cuff wrapped around the wrist, save the measurements and display the development of the measurements	Equivalent
Measurement Method	Non-invasive, Oscillometric	Non-invasive, Oscillometric	Identical
IHB Detection	Yes	Yes	Identical
Patient Population	Adult	Adult	Identical
BP Measurement Range	Cuff Pressure: 0 - 300 mmHg Systolic Pressure: 50 - 250 mmHg Diastolic Pressure: 30 - 200 mmHg	Cuff Pressure: 0 - 300 mmHg Systolic Pressure: 50 - 250 mmHg Diastolic Pressure: 30 - 200 mmHg	Identical
Number of User	2 independent users	2 independent users	Identical
Memory Space	2 users × 120 memory space	2 users × 60 memory space	Similar

<b>Comparison between Digital Automatic Wrist Blood Pressure Monitor MD3200 and Predicate device</b>			
<b>Item</b>	<b>Predicate Device</b>	<b>Digital Automatic Wrist Blood Pressure Monitor MD3200</b>	<b>Comment</b>
Resolution of Measurement	Blood Pressure: 1 mmHg or 0.1kPa Pulse Rate: 1 beat/ min	Blood Pressure: 1 mmHg Pulse Rate: 1 beat/ min	Similar
Blood Pressure Measurement Accuracy	± 3 mmHg	± 3 mmHg	Identical
Pulse Rate Measurement Range	40 - 180 beats/min	40 - 180 beats/min	Identical
Pulse Rate Measurement Accuracy	± 5 % of the reading	± 5 % of the reading	Identical
Display Type	LCD	LCD	Identical
Power Source	2 × 1.5 V AAA-batteries	2 × 1.5 V AAA-batteries	Identical
Pressurization Mode	Automatic Inflation	Automatic Inflation	Identical
Deflation Mode	Automatic Exhaust/ Deflation	Automatic Exhaust/ Deflation	Identical
Operating Condition	Temperature: +5 to +40 °C Humidity: 15 to 93 % R.H. max Atmospheric Pressure: 700 - 1060 hPa	Temperature: +5 to +40 °C Humidity: 15 to 93 % R.H. max Atmospheric Pressure: 700 - 1060 hPa	Identical
Storage and Transportation Condition	Temperature: -25 to +70 °C Humidity: up to 93% R.H. max Atmospheric Pressure: 700 - 1060 hPa	Temperature: -25 to +70 °C Humidity: up to 93% R.H. max Atmospheric Pressure: 700 - 1060 hPa	Identical
Material	Resistances, capacitance, transistors, amplifiers, pressure sensor, CPU, PCB, cuff ABS button, ABS cabinet, batteries and packaging	Resistances, capacitance, transistors, amplifiers, pressure sensor, CPU, PCB, cuff ABS button, ABS cabinet, batteries and packaging	Identical
Compatibility with Environment and Other Devices	No influence with environment and other device	No influence with environment and other device	Identical
Applicable Standard	- EN 1060-1:1995+A2:2009 - EN 1060-3:1997+A2:2009 - IEC 60601-1:2012 - IEC 60601-1-2:2007 - FCC Part 15 Subpart B - ISO 10993-5:2009 - ISO 10993-10:2010 - IEC 62304:2006 - IEC 81060-2:2009	- ISO 81060-1:2007 - EN 1060-3:1997+A2:2009 - IEC 60601-1:2012 - IEC 60601-1-2:2007 - FCC Part 15 Subpart B - ISO 10993-5:2009 - ISO 10993-10:2010 - IEC 62304:2006 - IEC 81060-2:2013	Equivalent

Digital Automatic Wrist Blood Pressure Monitor MD3200 is a non-invasive measuring device and utilizes the oscillometric methodology to measure the blood pressure reading. The key components of device are: a pressure sensor, an electric valve, an electronic control module, and an electric pump. The electric pump inflate (and deflate) the inflatable cuff automatically according to our designed architecture. The predicate device adopts exactly same methodology and key components for measuring blood pressure.

## 7. Clinical and Non-clinical Tests

### Clinical Test Summary

Testing to insure clinical accuracy of the device in accordance with [ISO 81060-2:2013](#) as documented in Clinical Test report.

One hundred patients (45 males and 55 females) were invited for the study. Standard auscultation method was used as the reference blood pressure monitor measuring in the left wrist. Blood pressure measurements were repeated alternatively with the device and auscultation in the same arm according to the sequence in [ISO 81060-2:2013](#).

### Non-Clinical Test Summary

Digital Automatic Wrist Blood Pressure Monitor MD3200 has performed several non-clinical tests to show that all requirement specifications and standard requirements are met. The tests include the follows:

- ✧ [ISO 81060-1:2007](#)
- ✧ [EN 1060-3:1997+A2:2009](#)
- ✧ [IEC 60601-1:2012](#)
- ✧ [IEC 60601-1-2:2007](#)
- ✧ [FCC Part 15 Subpart B](#)
- ✧ [ISO 10993-5:2009](#)
- ✧ [ISO 10993-10:2010](#)
- ✧ [IEC 62304:2006](#)

All of the clinical and non-clinical testing performed on Digital Automatic Wrist Blood Pressure Monitor MD3200 is same as the predicate device. Therefore, no bench test is conducted to show the performance of Digital Automatic Wrist Blood Pressure Monitor MD3200 is equivalent to the predicate device.

## **8. Conclusion**

Digital Automatic Wrist Blood Pressure Monitor MD3200 has the equivalent intended use and same technological characteristics as the predicate device, [Digital Automatic Wrist Blood Pressure Monitor SWBPM22 Series \(K142088\)](#). Moreover clinical testing has demonstrated that no differences in the technological characteristics and questioning on safety or effectiveness to be raised. Thus, Digital Automatic Wrist Blood Pressure Monitor MD3200 is substantially equivalent to the predicate device.