



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Center – WO66-G609  
Silver Spring, MD 20993-0002

Shenzhen Mindray Bio-Medical Electronics Co., Ltd.  
% Mr. Mark Job  
Responsible Third Party Official  
Regulatory Technology Services LLC  
1394 25<sup>th</sup> Street, NW  
BUFFALO MN 55313

October 2, 2015

Re: K152545  
Trade/Device Name: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System  
Regulation Number: 21 CFR 892.1550  
Regulation Name: Ultrasonic pulsed doppler imaging system  
Regulatory Class: II  
Product Code: IYN, IYO, ITX, LLZ  
Dated: September 4, 2015  
Received: September 8, 2015

Dear Mr. Job:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638 2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, “Misbranding by reference to premarket notification” (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

<http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH’s Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,



For

Robert Ochs, Ph.D.  
Director  
Division of Radiological Health  
Office of In Vitro Diagnostics  
and Radiological Health  
Center for Devices and Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)

**K152545**

Device Name

DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Indications for Use (Describe)

DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System is applicable for adults, pregnant women, pediatric patients and neonates. It is intended for use in fetal, abdominal, pediatric, small organ(breast, thyroid, testes), neonatal and adult cephalic, trans-rectal, trans-vaginal, musculo-skeletal(conventional, superficial), adult and pediatric cardiac, peripheral vessel and urology exam.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

### CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
[PRASStaff@fda.hhs.gov](mailto:PRASStaff@fda.hhs.gov)

*“An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number.”*

**Diagnostic Ultrasound Indications For Use Format**

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: N/A

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General(Track 1 Only)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal	N	N	N		N	N	N	Note 1, 2,3,4,6,7
	Abdominal	N	N	N	N	N	N	N	Note 1, 2,3,4,6,7
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric	N	N	N		N	N	N	Note 1,2,4,7
	Small Organ (Specify**)	N	N	N		N	N	N	Note 1,2,4,7
	Neonatal Cephalic	N	N	N	N	N	N	N	Note 1, 2,4,6,7
	Adult Cephalic	N	N	N	N	N	N	N	Note 1, 2,4,6,7
	Trans-rectal	N	N	N		N	N	N	Note 1, 2,4,6,7
	Trans-vaginal	N	N	N		N	N	N	Note 1, 2,4,6,7
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)	N	N	N		N	N	N	Note 1,2,4,7
	Musculo-skeletal (Superficial)	N	N	N		N	N	N	Note 1, 2,4,7
Intravascular									
Cardiac	Cardiac Adult	N	N	N	N	N	N	N	Note 1, 2,4,5,6,7
	Cardiac Pediatric	N	N	N	N	N	N	N	Note 1, 2,4,5,6,7
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel	N	N	N	N	N	N	N	Note 1, 2,4,6,7
	Other (Specify***)								
N=new indication; P=previously cleared by FDA; E=added under Appendix E									
Additional comments: Combined modes--B+M、PW+B、Color + B、Power + B、PW +Color+ B、Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of In Vitro Diagnostics and Radiological Health (OIR)</b>									

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: 3C5A

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Abdominal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k120699); E=added under Appendix E									
Additional comments: Combined modes--B+M, PW+B, Color + B, Power + B, PW +Color+ B, Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: 6C2

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
Musculo-skeletal (Superficial)									
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k120699); E=added under Appendix E									
Additional comments: Combined modes--B+M, PW+B, Color + B, Power + B, PW +Color+ B, Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: 7L4A

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal	P	P	P		P	P	P	Note 1, 2, 4,7
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric	P	P	P		P	P	P	Note 1, 2, 4,7
	Small Organ (Specify**)	P	P	P		P	P	P	Note 1, 2, 4,7
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)	P	P	P		P	P	P	Note 1, 2, 4,7
Musculo-skeletal (Superficial)	P	P	P		P	P	P	Note 1, 2, 4,7	
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel	P	P	P		P	P	P	Note 1, 2, 4,7
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k120699); E=added under Appendix E									
Additional comments: Combined modes--B+M、PW+B、Color + B、Power + B、PW +Color+ B、Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

Prescription USE (Per 21 CFR 801.109)

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: L14-6NE

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal								
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric	P	P	P		P	P	P	Note 1,2, 4,7
	Small Organ (Specify**)	P	P	P		P	P	P	Note 1,2, 4,7
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)	P	P	P		P	P	P	Note 1,2, 4,7
	Musculo-skeletal (Superficial)	P	P	P		P	P	P	Note 1,2, 4,7
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel	N	N	N		N	N	N	Note 1,2, 4,7
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k150204); E=added under Appendix E									
Additional comments: Combined modes--B+M、PW+B、Color + B、Power + B、PW +Color+ B、Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

Prescription USE (Per 21 CFR 801.109)

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: V11-3

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Abdominal								
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Trans-vaginal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
Musculo-skeletal (Superficial)									
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel								
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k150204); E=added under Appendix E									
Additional comments: Combined modes--B+M, PW+B, Color + B, Power + B, PW +Color+ B, Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: V11-3B

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Abdominal								
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Trans-vaginal	P	P	P		P	P	P	Note 1, 2, 4,6,7
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
Musculo-skeletal (Superficial)									
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel								
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k150204); E=added under Appendix E									
Additional comments: Combined modes--B+M, PW+B, Color + B, Power + B, PW +Color+ B, Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: P4-2

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal	P	P	P	P	P	P	P	Note 1, 2,4,6,7
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic								
	Adult Cephalic	P	P	P	P	P	P	P	Note 1, 2,4,6,7
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
Musculo-skeletal (Superficial)									
Intravascular									
Cardiac	Cardiac Adult	P	P	P	P	P	P	P	Note 1, 2,4,5,6,7
	Cardiac Pediatric	P	P	P	P	P	P	P	Note 1, 2,4,5,6,7
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel								
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k120699); E=added under Appendix E									
Additional comments: Combined modes--B+M, PW+B, Color + B, Power + B, PW +Color+ B, Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: P7-3

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal	P	P	P	P	P	P	P	Note 1, 2,6
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic	P	P	P	P	P	P	P	Note 1, 2,6
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
Musculo-skeletal (Superficial)									
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric	P	P	P	P	P	P	P	Note 1, 2,5,6
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel								
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k120699); E=added under Appendix E									
Additional comments: Combined modes--B+M、PW+B、Color + B、Power + B、PW +Color+ B、Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

Prescription USE (Per 21 CFR 801.109)

**002-10**

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: D7-2E

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal	P	P	P		P	P	P	Note 1, 2, 3, 4, 6
	Abdominal	P	P	P		P	P	P	Note 1, 2, 3, 4, 6
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel								
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k150204); E=added under Appendix E									
Additional comments: Combined modes--B+M、PW+B、Color + B、Power + B、PW +Color+ B、Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

Prescription USE (Per 21 CFR 801.109)

**002-11**

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: CW5s

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal								
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
Intravascular									
Cardiac	Cardiac Adult								
	Cardiac Pediatric								
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel				P				
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k150204); E=added under Appendix E									
Additional comments: Combined modes--B+M、PW+B、Color + B、Power + B、PW +Color+ B、Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

Prescription USE (Per 21 CFR 801.109)

**002-12**

System: DC-60/DC-60S/DC-60 Exp/DC-55 Diagnostic Ultrasound System

Transducer: CW2s

Intended Use: Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:

Clinical Application		Mode of Operation							
General (Track 1)	Specific (Track 1 & 3)	B	M	PWD	CWD	Color Doppler	Amplitude Doppler	Combined (specify)	Other (specify)
Ophthalmic	Ophthalmic								
Fetal Imaging & Other	Fetal								
	Abdominal								
	Intra-operative (Specify*)								
	Intra-operative (Neuro)								
	Laparoscopic								
	Pediatric								
	Small Organ (Specify**)								
	Neonatal Cephalic								
	Adult Cephalic								
	Trans-rectal								
	Trans-vaginal								
	Trans-urethral								
	Trans-esoph. (non-Card.)								
	Musculo-skeletal (Conventional)								
	Musculo-skeletal (Superficial)								
Intravascular									
Cardiac	Cardiac Adult					P			
	Cardiac Pediatric								
	Intravascular (Cardiac)								
	Trans-esoph. (Cardiac)								
	Intra-cardiac								
Peripheral vessel	Peripheral vessel								
	Other (Specify***)								
N=new indication; P=previously cleared by FDA(k120699); E=added under Appendix E									
Additional comments: Combined modes--B+M, PW+B, Color + B, Power + B, PW +Color+ B, Power + PW +B									
*Intraoperative includes abdominal, thoracic, and vascular etc.									
**Small organ-breast, thyroid, testes.									
***Other use includes Urology.									
Note 1: Tissue Harmonic Imaging. The feature does not use contrast agents.									
Note 2: Smart3D									
Note 3:4D(Real-time 3D)									
Note 4: iScape									
Note5: TDI									
Note6: Color M									
Note7: Biopsy Guidance									
Note8: Elastography									
(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE NEEDED)									
<b>Concurrence of CDRH, Office of Device Evaluation(ODE)</b>									

Prescription USE (Per 21 CFR 801.109)

**002-13**

# 510(K) SUMMARY

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirements of SMDA 1990 and 21 CFR §807.92(c).

The assigned 510(k) number is: K152545.

## **1. Submitter:**

Shenzhen Mindray Bio-medical Electronics Co., LTD

Mindray Building, Keji 12th Road South, Hi-tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. China

Tel: +86 755 8188 5640

Fax: +86 755 2658 2680

## **Contact Person:**

Yang Zhaohui

Shenzhen Mindray Bio-medical Electronics Co., LTD

Mindray Building, Keji 12th Road South, Hi-tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. China

**Date Prepared:** July 27, 2015

## **2. Device Name:** DC-60/DC-60S/DC-60 EXP/DC-55 Diagnostic Ultrasound System

### **Classification**

Regulatory Class: II

Review Category: Tier II

21 CFR 892.1550 Ultrasonic Pulsed Doppler Imaging System (IYN)

21 CFR 892.1560 Ultrasonic Pulsed Echo Imaging System (IYO)

21 CFR 892.1570 Diagnostic Ultrasound Transducer (ITX)

21 CFR 892.2050 Picture Archiving and Communications System(LLZ)

## **3. Device Description:**

DC-60/DC-60S/DC-60 EXP/DC-55 is a software controlled, ultrasonic diagnostic system. Its function is to acquire and display ultrasound data in B-Mode, M-Mode, PW-Mode, CW-Mode, Color-Mode, Power/Dirpower Mode, THI or the combined mode (i.e. B/M-Mode, B/PW-mode, B/PW/Color).

This system is a Track 3 device that employs an array of probes that include linear array and convex array with a frequency range of approximately 2.0 MHz to 10.0 MHz.

The differences between these 4 models are shown as followed table:

Product Model	B-Profile	B-Histogram
DC-60	√	√
DC-60 Exp	√	×
DC-60S	×	×
DC-55	×	√

Note: B histogram and B-Profile are the only two differences between the three models. They have exactly the same hardware and software configuration, mechanical features and appearance.

#### **4. Intended Use:**

DC-60/DC-60S/DC-60 EXP/DC-55 Diagnostic Ultrasound System is applicable for adults, pregnant women, pediatric patients and neonates. It is intended for use in fetal, abdominal, pediatric, small organ(breast, thyroid, testes), neonatal and adult cephalic, trans-rectal, trans-vaginal, musculo-skeletal(conventional, superficial), adult and pediatric cardiac, peripheral vessel and urology exam.

#### **5. Comparison with Predicate Devices:**

DC-60/DC-60S/DC-60 EXP/DC-55 Diagnostic Ultrasound System is comparable with and substantially equivalent to these predicate devices:

Predicate Device	Manufacturer	Model	510(k) Control Number
1	Mindray	DC-70	K150204
2	Mindray	DC-T6	K120699
3	Mindray	DC-8	K150080

DC-60/DC-60S/DC-60 EXP/DC-55 has the same technological characteristics, is comparable in key safety and effectiveness features, and has the same intended uses and basic operating modes as the predicate devices. All systems transmit ultrasonic energy into patients and perform post processing of received echoes to generate onscreen display of anatomic structures and fluid flow within the body. All systems allow for specialized measurements of structures and flow, and calculations.

- Subject device DC-60/DC-60S/DC-60 EXP/DC-55 has the same intended uses as the predicated device DC-70(K150204)

<b>Subject Device</b> <b>DC-60/DC-60S/DC-60 EXP/DC-55</b>	<b>Predicate device</b> <b>DC-70(K1502041)</b>
The DC-60/DC-60S/DC-60 EXP/DC-55 Diagnostic Ultrasound System is applicable for adults, pregnant women, pediatric patients and neonates. It is intended for use in fetal, abdominal, pediatric, small organ(breast, thyroid, testes), neonatal and adult cephalic, trans-rectal, trans-vaginal, musculo-skeletal (conventional, superficial), adult and pediatric cardiac, peripheral vessel and urology exams.	The DC-70 Diagnostic Ultrasound System is applicable for adults, pregnant women, pediatric patients and neonates. It is intended for use in fetal, abdominal, pediatric, small organ(breast, thyroid, testes), neonatal and adult cephalic, trans-rectal, trans-vaginal, musculo-skeletal (conventional, superficial), adult and pediatric cardiac, peripheral vessel and urology exams.

- All of the patient contact material of the DC-60/DC-60S/DC-60 EXP/DC-55 are the same as that of the predicated device DC-8(K150080).
- The acoustic power levels of DC-60/DC-60S/DC-60 EXP/DC-55 are below the limits of FDA, which is the same as the predicated device DC-70(K1502041).
- DC-60/DC-60S/DC-60 EXP/DC-55 is designed in compliance with the FDA recognized electrical and physical safety standard, which is the same as the predicated device DC-70(K150204).
- DC-60/DC-60S/DC-60 EXP/DC-55 has the same imaging modes as the predicated device DC-70(K150204): B, M, Color Doppler, PWD, CWD, Amplitude Doppler, Anatomical M Mode and combined mode.

- DC-60/DC-60S/DC-60 EXP/DC-55 has some special functions. All of them are identical as the predicated device DC-70(K150204).
- DC-60/DC-60S/DC-60 EXP/DC-55 has the same capacity in term of making comments and body marks on the images, reporting patient exam results as the predicated device DC-70(K150204).
- DC-60/DC-60S/DC-60 EXP/DC-55 has similar probes as the predicated device:

<b>Subject device</b> <b>DC-60/DC-60S/DC-60 EXP/DC-55</b>	<b>Predicated device</b>
3C5A	3C5A (K120699)
6C2	6C2 (K120699)
7L4A	7L4A (K120699)
P4-2	P4-2 (K120699)
P7-3	P7-3 (K120699)
V11-3	V11-3E (K150204)
V11-3B	V11-3BE (K150204)
L14-6NE	L14-6NE (K150204)
D7-2E	D6-2E (K150204)
CW2s	CW2s (K150080)
CW5s	CW5s (K150204)

- DC-60/DC-60S/DC-60 EXP/DC-55 has the same measurement and calculation functions as the predicated device DC-70(K150204).

## **6. Non-clinical Tests:**

DC-60/DC-60S/DC-60 EXP/DC-55 Diagnostic Ultrasound System has been evaluated for acoustic output, biocompatibility, cleaning and disinfection effectiveness as well as thermal, electrical and mechanical safety, and has been designed to conform with applicable medical safety standards. This device has been tested and evaluated under the following standards:

- UD 2 Acoustic Output Measurement Standard for Diagnostic Ultrasound Equipment.
- AAMI / ANSI ES60601-1 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.

- IEC 60601-1-2 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral standard: Electromagnetic compatibility - Requirements and tests (Edition 3)
- IEC 60601-2-37 Medical electrical equipment - Part 2-37: Particular requirements for the basic safety and essential performance of ultrasonic medical diagnostic and monitoring equipment
- ISO14971 Medical devices - Application of risk management to medical devices
- ISO 10993-1 Biological evaluation of medical devices -- Part 1: Evaluation and testing within a risk management process
- IEC 62366 Medical devices - Application of usability engineering to medical devices
- IEC 62304 Medical device software - Software life cycle processes

These non-clinical tests relied on in this premarket notification submission can support the determination of substantial equivalence of the subject device.

## **7. Clinical Studies**

Not applicable. The subject of this submission, DC-60/DC-60S/DC-60 EXP/DC-55 Diagnostic Ultrasound System, does not require clinical studies to support substantial equivalence.

## **Conclusion:**

Intended uses and other key features are consistent with traditional clinical practices, FDA guidelines and established methods of patient examination. The design, development and quality process of the manufacturer confirms with 21 CFR 820, ISO 9001 and ISO 13485 quality systems. The device conforms to applicable medical device safety standards. Therefore, the DC-60/DC-60S/DC-60 EXP/DC-55 Diagnostic Ultrasound System is substantially equivalent with respect to safety and effectiveness to devices currently cleared for market.