



Food and Drug Administration  
10903 New Hampshire Avenue  
Document Control Center – WO66-G609  
Silver Spring, MD 20993-0002

FUJIFILM SonoSite, Inc.  
% Mr. Mark Job  
Responsible Third Party Official  
Regulatory Technology Services LLC  
1394 25th Street, NW  
BUFFALO MN 55313

January 20, 2016

Re: K153626  
Trade/Device Name: SonoSite Edge II Ultrasound System  
Regulation Number: 21 CFR 892.1550  
Regulation Name: Ultrasonic pulsed doppler imaging system  
Regulatory Class: II  
Product Code: IYN, IYO, ITX  
Dated: January 13, 2016  
Received: January 14, 2016

Dear Mr. Job:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638 2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, “Misbranding by reference to premarket notification” (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

<http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH’s Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely yours,

A handwritten signature in black ink that reads "Robert Ochs". The signature is written in a cursive style with a prominent "R" and "O".

Robert Ochs, Ph.D.  
Director  
Division of Radiological Health  
Office of In Vitro Diagnostics  
and Radiological Health  
Center for Devices and Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)

~~TBD~~ K153626

Device Name

FUJIFILM SonoSite Edge II Ultrasound System

Indications for Use (Describe)

The FUJIFILM SonoSite Edge II Ultrasound System is a general purpose ultrasound system intended for use by qualified physicians and healthcare professionals for evaluation by ultrasound imaging or fluid flow analysis of the human body. Specific clinical applications and exam types include:

Ophthalmic  
Fetal - OB/GYN  
Abdominal  
Pediatric  
Small Organ (breast, thyroid, testicle, prostate)  
Neonatal Cephalic  
Adult Cephalic  
Trans-vaginal  
Musculo-skeletal (Conventional)  
Musculo-skeletal (Superficial)  
Cardiac Adult  
Cardiac Pediatric  
Trans-esophageal (cardiac)  
Peripheral Vessel

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

### CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
PRAStaff@fda.hhs.gov

*"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."*

Table 1.3-1: Diagnostic Ultrasound Indications for Use Form – FUJIFILM SonoSite Edge II Ultrasound System

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	N/A						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic	N	N	N		N	B+M; B+PWD; B+CD	1-5
Fetal	N	N	N		N	B+M; B+PWD; B+CD	1-5
Abdominal	N	N	N		N	B+M; B+PWD; B+CD	1-5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	N	N	N		N	B+M; B+PWD; B+CD	1-5
Small Organ (breast, thyroid, testicles, prostate)	N	N	N		N	B+M; B+PWD; B+CD	1-5
Neonatal Cephalic	N	N	N		N	B+M; B+PWD; B+CD	1-3,5
Adult Cephalic	N	N	N		N	B+M; B+PWD; B+CD	1-3,5
Trans-rectal							
Trans-vaginal	N	N	N		N	B+M; B+PWD; B+CD	1,5
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)	N	N	N		N	B+M; B+PWD; B+CD	1,2,4,5
Musculo-skel. (Superfic.)	N	N	N		N	B+M; B+PWD; B+CD	1,4,5
Intra-luminal							
Other (spec.)							
Cardiac Adult	N	N	N	N	N	B+M; B+PWD; B+CWD; B+CD	1-5
Cardiac Pediatric	N	N	N	N	N	B+M; B+PWD; B+CWD; B+CD	1-5
Trans-esophageal (card.)	N	N	N	N	N	B+M; B+PWD; B+CWD; B+CD	1,3
Other (spec.)							
Peripheral vessel	N	N	N		N	B+M; B+PWD; B+CD	1-5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

Prescription Use (Per 21 CFR 801.109)

Table 1.3-2 Diagnostic Ultrasound Indications for Use Form – C11x/8-5 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	C11x/8-5 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal							
Abdominal	P	P	P		P	B+M; B+PWD; B+CD	1,5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,5
Small Organ (breast, thyroid, testicles, prostate)							
Neonatal Cephalic	P	P	P		P	B+M; B+PWD; B+CD	1,5
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)							
Musculo-skel. (Superfic.)							
Intra-luminal							
Other (spec.)							
Cardiac Adult							
Cardiac Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,5
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	P	P	P		P	B+M; B+PWD; B+CD	1,5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K133454.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-3 Diagnostic Ultrasound Indications for Use Form – rC60xi/5-2 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	rC60xi/5-2 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal	N	N	N		N	B+M; B+PWD; B+CD	1,2,4,5
Abdominal	N	N	N		N	B+M; B+PWD; B+CD	1,2,4,5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	N	N	N		N	B+M; B+PWD; B+CD	1,2,4,5
Small Organ (breast, thyroid, testicles, prostate)							
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)	N	N	N		N	B+M; B+PWD; B+CD	1,2,4,5
Musculo-skel. (Superfic.)							
Intra-luminal							
Other (spec.)							
Cardiac Adult							
Cardiac Pediatric							
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	N	N	N		N	B+M; B+PWD; B+CD	1,2,4,5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.

2: Tissue Harmonic Imaging (THI)

3: Tissue Doppler Imaging (TDI)

4: Steep Needle Profiling (Sono MBe)

5: Multi-beam Imaging (SonoMB) in B-Mode

Prescription Use (Per 21 CFR 801.109)

Table 1.3-4 Diagnostic Ultrasound Indications for Use Form – HFL38xi/13-6 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	HFL38xi/13-6 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal							
Abdominal	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Small Organ (breast, thyroid, testicles, prostate)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Musculo-skel. (Superfic.)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-luminal							
Other (spec.)							
Cardiac Adult	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Cardiac Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.

2: Tissue Harmonic Imaging (THI)

3: Tissue Doppler Imaging (TDI)

4: Steep Needle Profiling (Sono MBe)

5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K152209.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-5 Diagnostic Ultrasound Indications for Use Form – HFL50x/15-6 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	HFL50x/15-6 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal							
Abdominal	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Small Organ (breast, thyroid, testicles, prostate)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Musculo-skel. (Superfic.)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-luminal							
Other (spec.)							
Cardiac Adult							
Cardiac Pediatric							
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K133454.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-6 Diagnostic Ultrasound Indications for Use Form – HSL25x/13-6 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	HSL25x/13-6 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Fetal							
Abdominal	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Small Organ (breast, thyroid, testicles, prostate)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Musculo-skel. (Superfic.)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-luminal							
Other (spec.)							
Cardiac Adult	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Cardiac Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K152209.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-7 Diagnostic Ultrasound Indications for Use Form – ICTx/8-5 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	ICTx/8-5 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal	P	P	P		P	B+M; B+PWD; B+CD	1,5
Abdominal							
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric							
Small Organ (breast, thyroid, testicles, prostate)							
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal	P	P	P		P	B+M; B+PWD; B+CD	1,5
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)							
Musculo-skel. (Superfic.)							
Intra-luminal							
Other (spec.)							
Cardiac Adult							
Cardiac Pediatric							
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel							
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K133454.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-8 Diagnostic Ultrasound Indications for Use Form – L25x/13-6 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	L25x/13-6 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Fetal							
Abdominal	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Small Organ (breast, thyroid, testicles, prostate)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Musculo-skel. (Superfic.)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-luminal							
Other (spec.)							
Cardiac Adult	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Cardiac Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.

2: Tissue Harmonic Imaging (THI)

3: Tissue Doppler Imaging (TDI)

4: Steep Needle Profiling (Sono MBe)

5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K152209.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-9 Diagnostic Ultrasound Indications for Use Form - L38xi/10-5 Transducer, L38xi/10-5 Armored Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	L38xi/10-5 MHz Transducer, L38xi/10-5 MHz Armored Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal							
Abdominal	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Small Organ (breast, thyroid, testicles, prostate)	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)							
Musculo-skel. (Superfic.)							
Intra-luminal							
Other (spec.)							
Cardiac Adult	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Cardiac Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	P	P	P		P	B+M; B+PWD; B+CD	1,4,5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked "P" were previously cleared in 510(k) K152209.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-10 Diagnostic Ultrasound Indications for Use Form – P10x/8-4 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	P10x/8-4 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal	P	P	P		P	B+M; B+PWD; B+CD	1,5
Abdominal	P	P	P		P	B+M; B+PWD; B+CD	1,5
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	P	P	P		P	B+M; B+PWD; B+CD	1,5
Small Organ (breast, thyroid, testicles, prostate)	P	P	P		P	B+M; B+PWD; B+CD	1,5
Neonatal Cephalic	P	P	P		P	B+M; B+PWD; B+CD	1,5
Adult Cephalic	P	P	P		P	B+M; B+PWD; B+CD	1,5
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)	P	P	P		P	B+M; B+PWD; B+CD	1,5
Musculo-skel. (Superfic.)							
Intra-luminal							
Other (spec.)							
Cardiac Adult	P	P	P	P	P	B+M; B+PWD; B+CWD; B+CD	1,3,5
Cardiac Pediatric	P	P	P	P	P	B+M; B+PWD; B+CWD; B+CD	1,3,5
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	P	P	P		P	B+M; B+PWD; B+CD	1,5
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.

2: Tissue Harmonic Imaging (THI)

3: Tissue Doppler Imaging (TDI)

4: Steep Needle Profiling (Sono MBe)

5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K133454.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-11 Diagnostic Ultrasound Indications for Use Form – P11x/10-5 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	P11x/10-5 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal							
Abdominal							
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	P	P			P	B+M; B+CD	1
Small Organ (breast, thyroid, testicles, prostate)							
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)							
Musculo-skel. (Superfic.)							
Intra-luminal							
Other (spec.)							
Cardiac Adult							
Cardiac Pediatric							
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	P	P			P	B+M; B+CD	1
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K133454.**

Prescription Use (Per 21 CFR 801.109)

Table 1.3-12 Diagnostic Ultrasound Indications for Use Form – rP19x/5-1 Transducer, rP19x/5-1 Armored Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	rP19x/5-1 MHz Transducer, rP19x/5-1 MHz Armored Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic	N	N	N		N	B+M; B+PWD; B+CD	1-3
Fetal	N	N	N		N	B+M; B+PWD; B+CD	1-3
Abdominal	N	N	N		N	B+M; B+PWD; B+CD	1-3
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric	N	N	N		N	B+M; B+PWD; B+CD	1-3
Small Organ (breast, thyroid, testicles, prostate)	N	N	N		N	B+M; B+PWD; B+CD	1-3
Neonatal Cephalic	N	N	N		N	B+M; B+PWD; B+CD	1-3
Adult Cephalic	N	N	N		N	B+M; B+PWD; B+CD	1-3
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)							
Musculo-skel. (Superfic.)							
Intra-luminal							
Other (spec.)							
Cardiac Adult	N	N	N	N	N	B+M; B+PWD; B+CWD; B+CD	1-3
Cardiac Pediatric	N	N	N	N	N	B+M; B+PWD; B+CWD; B+CD	1-3
Trans-esophageal (card.)							
Other (spec.)							
Peripheral vessel	N	N	N		N	B+M; B+PWD; B+CD	1-3
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

Prescription Use (Per 21 CFR 801.109)

Table 1.3-13 Diagnostic Ultrasound Indications for Use Form – TEExi/8-3 Transducer

<b>System:</b>	FUJIFILM SonoSite Edge II Ultrasound System						
<b>Transducer:</b>	TEExi/8-3 MHz Transducer						
<b>Intended Use:</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body as follows:						
<b>Clinical Application</b>	<b>Mode of Operation</b>						
	B	M	PWD	CWD	Color Doppler	Combined (Spec.)	Other (Spec.)
Ophthalmic							
Fetal							
Abdominal							
Intra-operative (Abdominal organs and vascular)							
Intra-operative (Neuro.)							
Laparoscopic							
Pediatric							
Small Organ (breast, thyroid, testicles, prostate)							
Neonatal Cephalic							
Adult Cephalic							
Trans-rectal							
Trans-vaginal							
Trans-urethral							
Trans-esoph. (non-Card.)							
Musculo-skel. (Convent.)							
Musculo-skel. (Superfic.)							
Intra-luminal							
Other (spec.)							
Cardiac Adult							
Cardiac Pediatric							
Trans-esophageal (card.)	P	P	P	P	P	B+M; B+PWD; B+CWD; B+CD	1,3
Other (spec.)							
Peripheral vessel							
Other (spec.)							

N= new indication; P= previously cleared by FDA; E= added under this appendix

**Additional Comments:**

- 1: Includes imaging to assist in the placement of needles and catheters in vascular or other anatomical structures and imaging guidance for peripheral nerve block procedures. Color Doppler includes Power/Velocity/Variance. M-Mode includes color M-Mode.
- 2: Tissue Harmonic Imaging (THI)
- 3: Tissue Doppler Imaging (TDI)
- 4: Steep Needle Profiling (Sono MBe)
- 5: Multi-beam Imaging (SonoMB) in B-Mode

**All items marked “P” were previously cleared in 510(k) K152209.**

Prescription Use (Per 21 CFR 801.109)

## 510(K) Summary

This summary of safety and effectiveness is provided as part of this Premarket Notification in compliance with 21 CFR, Part 807, Subpart E, Section 807.92.

### 1) Submitter's name, address, telephone number, contact person:

FUJIFILM SonoSite, Inc.  
21919 30<sup>th</sup> Drive SE  
Bothell, WA 98021-3904

**Corresponding Official:** Patricia Liao  
Manager, Regulatory Affairs  
**E-mail:** [Patricia.Liao@sonosite.com](mailto:Patricia.Liao@sonosite.com)  
**Telephone:** (425) 951-6870  
**Facsimile:** (425) 951-1201  
**Date prepared:** December 9, 2015

### 2) Name of the device, including the trade or proprietary name if applicable, the common or usual name, and the classification name, if known:

#### Common/ Usual Name

Diagnostic Ultrasound System with Accessories

#### Proprietary Name

SonoSite Edge II Ultrasound System (*subject to change*)

#### Classification Names

Name	FR Number	Product Code
Ultrasonic Pulsed Doppler Imaging System	892.1550	90-IYN
Ultrasonic Pulsed Echo Imaging System	892.1560	90-IYO
Diagnostic Ultrasound Transducer	892.1570	90-ITX

### 3) Identification of the predicate or legally marketed device:

SonoSite Edge Ultrasound System                      K133454  
SonoSite X-Porte Ultrasound System                K152209

### 4) Device Description:

The SonoSite Edge II Ultrasound System is a portable laptop style, full featured, general purpose, software controlled, diagnostic ultrasound system used to acquire and display high-resolution, real-time ultrasound data through multiple imaging modes. Edge II is a custom fabricated digital electronic design that readily lends itself to be configured for specific ultrasound imaging applications through different system feature selections. Edge II can operate on either battery or AC power.

**5) Intended Use:**

The FUJIFILM SonoSite Edge II Ultrasound System is a general purpose ultrasound system intended for use by qualified physicians and healthcare professionals for evaluation by ultrasound imaging or fluid flow analysis of the human body. Specific clinical applications and exam types include:

- Ophthalmic
- Fetal - OB/GYN
- Abdominal
- Pediatric
- Small Organ (breast, thyroid, testicle, prostate)
- Neonatal Cephalic
- Adult Cephalic
- Trans-vaginal
- Musculo-skeletal (Conventional)
- Musculo-skeletal (Superficial)
- Cardiac Adult
- Cardiac Pediatric
- Trans-esophageal (cardiac)
- Peripheral Vessel

**6) Technological Characteristics:**

SonoSite Edge and X-Porte Ultrasound Systems are both Track 3 devices that employ the same fundamental scientific technology. A comparison table is provided below.

Feature	SonoSite Edge II Ultrasound System (This submission)	SonoSite Edge Ultrasound System (K133454)	SonoSite X-Porte Ultrasound System (K152209)
<b>Intended Use</b>	Diagnostic ultrasound imaging or fluid flow analysis of the human body	Diagnostic ultrasound imaging or fluid flow analysis of the human body	Diagnostic ultrasound imaging or fluid flow analysis of the human body
<b>Indications for Use</b>	Ophthalmic Fetal – OB/GYN Abdominal  Pediatric Small Organ (breast, thyroid, testicle, prostate) Neonatal Cephalic Adult Cephalic  Trans-Vaginal Musculo-skeletal (Conventional) Musculo-skeletal (Superficial) Cardiac Adult Cardiac Pediatric Trans-esophageal (cardiac) Peripheral Vessel Needle guidance	Ophthalmic Fetal – OB/GYN Abdominal Intraoperative (abdominal organs and vascular) Intra-operative (Neuro.) Pediatric Small Organ (breast, thyroid, testicle, prostate) Neonatal Cephalic Adult Cephalic Trans-Rectal Trans-Vaginal Musculo-skeletal (Conventional) Musculo-skeletal (Superficial) Cardiac Adult Cardiac Pediatric Trans-esophageal (cardiac) Peripheral Vessel Needle guidance	Ophthalmic Fetal - OB/GYN Abdominal Intraoperative (abdominal organs and vascular)  Pediatric Small Organ (breast, thyroid, testicle, prostate) Neonatal Cephalic Adult Cephalic  Trans-Vaginal Musculo-skeletal (Conventional) Musculo-skeletal (Superficial) Cardiac Adult Cardiac Pediatric Trans-esophageal (cardiac) Peripheral Vessel Needle guidance
<b>Transducer Types</b>	Linear Array Curved Linear Array Intracavitary Phased Array  Trans-esophageal	Linear Array Curved Linear Array Intracavitary Phased Array Static Probes Trans-esophageal	Linear Array Curved Linear Array Intracavitary Phased Array  Trans-esophageal
<b>Transducer Frequency</b>	1.0 – 15.0 MHz	1.0 – 15.0 MHz	1.0 – 15.0 MHz

Feature	SonoSite Edge II Ultrasound System (This submission)	SonoSite Edge Ultrasound System (K133454)	SonoSite X-Porte Ultrasound System (K152209)
<b>Modes of Operation</b>	B-mode Grayscale Imaging Tissue Harmonic Imaging M-mode Color M-Mode  Color Power Doppler Zoom Combination Modes Pulsed Wave (PW) Doppler Continuous Wave (CW) Doppler SonoHD2 Noise Reduction SonoMB/MBe Image Compounding Steered CW Doppler Velocity Color Doppler Tissue Doppler Imaging (TDI)	B-mode Grayscale Imaging Tissue Harmonic Imaging M-mode Color M-Mode  Color Power Doppler Zoom Combination Modes Pulsed Wave (PW) Doppler Continuous Wave (CW) Doppler SonoHD2 Noise Reduction SonoMB/MBe Image Compounding Steered CW Doppler Velocity Color Doppler Tissue Doppler Imaging (TDI)	B-mode Grayscale Imaging Tissue Harmonic Imaging M-mode  Simultaneous M-Mode Color Power Doppler Zoom Combination Modes Pulsed Wave (PW) Doppler Continuous Wave (CW) Doppler SonoHD2 Noise Reduction SonoMB/MBe Image Compounding Steered CW Doppler Velocity Color Doppler Tissue Doppler Imaging (TDI)
<b>PW Doppler</b>	Available	Available	Available
<b>CW Doppler</b>	Available	Available	Available
<b>Patient Contact Materials</b>	<b>Transducers:</b>  Cycology  Epoxy paste adhesive Polyethylene (PE) Ionomer Polyetheretherketone (PEEK) Polycarbonate Polysulfone UDEL P1700 Polyurethane Poly-Vinyl-Chloride (PVC) Silicone RTV Adhesive Silicone Rubber Urethane <b>Needle Guides:</b> Acetal copolymer Acrylonitrile-butadien-styrene (ABS)	<b>Transducers:</b> Acrylonitrile-butadien-styrene (ABS) Cycology Dow Medical Adhesive, Type A Epoxy paste adhesive Polyethylene (PE) Ionomer Polyetheretherketone (PEEK) Polycarbonate Polysulfone UDEL P1700 Polyurethane Poly-Vinyl-Chloride (PVC) Silicone RTV Adhesive Silicone Rubber Urethane <b>Needle Guides:</b> Acetal copolymer Acrylonitrile-butadien-styrene (ABS)	<b>Transducers:</b> Acrylonitrile-butadien-styrene (ABS) Cycology Dow Medical Adhesive, Type A Epoxy paste adhesive Polyethylene (PE) Ionomer Polyetheretherketone (PEEK)  Polysulfone UDEL P1700 Polyurethane Poly-Vinyl-Chloride (PVC) Silicone RTV Adhesive Silicone Rubber Urethane <b>Needle Guides:</b> Acetal copolymer Acrylonitrile-butadien-styrene (ABS)
<b>System Characteristics</b>	<b>Edge II:</b> Beamformer 128/128 using SA (configurable) Hand held display and control Single 12.1" Liquid Crystal Display (LCD) 256 gray shades on LCD  2 USB ports  Dimensions: 12.8"(W) x 12.1" (L) x 2.5"(H)  Weight: 9.0 lbs  System operates via battery or AC power Battery life: 1.5 - 4 hour operation	<b>Edge:</b> Beamformer 128/128 using SA (configurable) Hand held display and control Single 12.1" Liquid Crystal Display (LCD) 256 gray shades on LCD  2 USB ports  Dimensions: 12.9"(W) x 12.4" (L) x 2.5"(H)  Weight: 8.5 lbs  System operates via battery or AC power Battery life: 1.5 - 4 hour operation	<b>X-Porte (stand configuration):</b> Beamformer 128/128 using SA (configurable) 12.1" Capacitive touch screen interface 19" LED LCD HD monitor 256 gray shades on LED LCD  6 USB 2.0 ports  Stand Base Dimensions: 26.4" L x 21.2" W Stand Height (max): 64" (monitor up) Stand Height (min): 42.2" (monitor down)  Weight: 149.35 lbs (fully configured w/ 3 transducers)  System operates via battery or AC power Battery life: 1 hour operational - 3

Feature	SonoSite Edge II Ultrasound System (This submission)	SonoSite Edge Ultrasound System (K133454)	SonoSite X-Porte Ultrasound System (K152209)
	<p>per charge</p> <p>100 – 240V options, 50/60 Hz, 15VDC output</p> <p>Various obstetrical, cardiac, volume, M-mode, PW and CW Doppler measurement and calculation packages</p> <p>ECG acquisition and display capabilities CW/PW Doppler Audio Spectral Doppler Audio and image storage on removable media</p> <p>Wireless 802.11 (b/g/n) support for image transfer</p>	<p>per charge</p> <p>100 – 240V options, 50/60 Hz, 15VDC output</p> <p>Various obstetrical, cardiac, volume, M-mode, PW and CW Doppler measurement and calculation packages</p> <p>ECG acquisition and display capabilities CW/PW Doppler Audio Spectral Doppler Audio and image storage on removable media</p> <p>Wireless 802.11 (b/g/n) support for image transfer</p>	<p>days idle</p> <p>Input: 100 – 240 VAC, 50/60 Hz Output 1: 24VDC output, 275 W max Output 2: 100-240VAC, 50-60 Hz (AC Printer)</p> <p>Various obstetrical, cardiac, volume, M-mode, PW and CW Doppler measurement and calculation packages</p> <p>ECG acquisition and display capabilities CW/PW Doppler Audio Spectral Doppler Audio and image storage on removable media Measurement on Recalled Images.</p> <p>Wireless 802.11 (a/b/g/n) support for image transfer</p> <p><b>X-Porte (desktop configuration):</b></p> <p>Same software features/capabilities as the stand configuration. Does not have the stand, touch panel interface, DVR, and mobile power unit.</p> <p>Weight: 32.80 lbs (w/ 1 transducer)</p> <p>AC power only. 100 – 240V options, 50/60 Hz</p>
<b>510(k) Track</b>	Track 3	Track 3	Track 3

## 7) Determination of Substantial Equivalence:

### Summary of Non-Clinical Tests:

The SonoSite Edge II Ultrasound System has been evaluated for electrical, thermal, mechanical, and EMC safety. Additionally, cleaning/disinfection, biocompatibility, and acoustic output have been evaluated, and the device has been found to conform to applicable mandatory medical device safety standards. Assurance of quality was established by employing the following elements of product development but were not limited to: Design Phase Reviews, Risk Assessment, Requirements Development, and Verification and Validation.

The SonoSite Edge II Ultrasound System is designed to comply with the following FDA recognized standards.

Reference No.	Title
ISO 10993-1	AAMI / ANSI / ISO 10993-1:2009/(R)2013, Biological evaluation of medical devices – Part 1: Evaluation and testing within a risk management process
IEC 60601-1	AAMI / ANSI ES60601-1:2005/(R)2012 and A1:2012,, C1:2009/(R)2012 and A2:2010/(R)2012 (Consolidated Text) Medical electrical equipment - Part 1: General requirements for basic safety and essential performance (IEC 60601-1:2005, MOD)
IEC 60601-1-2	AAMI / ANSI / IEC 60601-1-2:2007(R)2012, Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance - Collateral standard: Electromagnetic compatibility - Requirements and tests (Edition 3)
IEC 60601-1-6	IEC 60601-1-6 Edition 3.1 2013-10, Medical electrical equipment – Part 1-6: General requirements for basic safety and essential performance - Collateral standard: Usability
IEC 60601-2-37	IEC 60601-2-37:2007 Edition 2.0 2007-08, Medical electrical equipment – Part 2-37: Particular requirements for the basic safety and essential performance of ultrasonic medical diagnostic and monitoring equipment
IEC 62304	AAMI / ANSI / IEC 62304:2006, Medical device software - Software life cycle processes
IEC 62359	IEC 62359 Edition 2.0 2010-10-10, Ultrasonics – Field characterization – Test methods for the determination of thermal and mechanical indices related to medical diagnostic ultrasonic fields [Including: Technical corrigendum 1 (2011)]
ISO 14971	ISO 14971:2007, Medical devices - Application of risk management to medical devices
NEMA UD 2-2004	Acoustic Output Measurement Standard for Diagnostic Ultrasound Equipment

### Summary of Clinical Tests:

The SonoSite Edge II Ultrasound System and transducers, subject of this submission, did not require clinical studies to support the determination of substantial equivalence.

## 8) Conclusion:

Intended uses and other key features are consistent with traditional clinical practice and FDA guidance. The SonoSite Edge II system and predicates meet FDA requirements for Track 3 devices, share indications for use, have biosafety equivalence, and conform to applicable electromedical device safety standards. FUJIFILM SonoSite, Inc. believes that the SonoSite Edge II Ultrasound System is substantially equivalent with regard to safety and effectiveness to the predicate devices.