



Food and Drug Administration
10903 New Hampshire Avenue
Document Control Center - WO66-G609
Silver Spring, MD 20993-0002

June 5, 2017

Contec Medical Systems Co., Ltd.
% Ms. Diana Hong
General Manager
Mid-Link Consulting Co., Ltd.
P.O. Box 120-119
Shanghai, 200120 CN

Re: K170954

Trade/Device Name: Portable ECG Monitor
Regulation Number: 21 CFR 870.2340
Regulation Name: Electrocardiograph
Regulatory Class: Class II
Product Code: DPS
Dated: March 22, 2017
Received: March 31, 2017

Dear Ms. Diana Hong:

This letter corrects our substantially equivalent letter of May 30, 2017.

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the [Federal Register](#).

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-

related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to

<http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely,

A handwritten signature in black ink, appearing to read "M. D. Zuckerman", is written over a large, light blue "FDA" watermark.

for
Bram D. Zuckerman, M.D.
Director
Division of Cardiovascular Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)

K170954

Device Name

Portable ECG Monitor

Indications for Use (Describe)

The device is a handheld, personal electrocardiograph unit, which can measure electrical activities of the heart easily and conveniently. It is immediately available at any time to manually record transient cardiac events, suitable for home health care use, which can detect, display and store ECG signal, and if possible, provide average heart rate message after ECG measurement. The users can use it themselves to check their heart condition.

It is suitable for the adult users, who suffers from cardiovascular diseases, or the adult people who are caring about their heart working conditions during their daily life. This device is not intended for use as a conventional diagnostic tool, but use as a healthcare tool which can provide doctor the recorded data as references.

The product is not a conventional diagnostic tool

Type of Use (Select one or both, as applicable)

☒ Prescription Use (Part 21 CFR 801 Subpart D)

☐ Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRAStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

K170954

Page 1 of 4

Summary

This 510(k) Summary is being submitted in accordance with requirements of SMDA 1990 and Title 21, CFR Section 807.92.

The assigned 510(k) Number: _____

1. Date of Preparation: 03/24/2017

Sponsor Identification

Contec Medical Systems Co., Ltd

No.112 Qinhuang West Street, Economic & Technical Development Zone,
Qinhuangdao, Hebei, 066004, China.

Establishment Registration Number: 3006979678

Contact Person: Xueyong Li

Position: Quality Manager

Tel: 86 355 8015489

Fax: 86 355 8015490

Email: lxy1011@163.com

Designated Submission Correspondent

Ms. Diana Hong (Primary Contact Person)

Mr. Cindy Wang (Alternative Contact Person)

Mid-Link Consulting Co., Ltd

P.O. Box 120-119, Shanghai, 200120, China

Tel: +86-21-22815850,

Fax: 240-238-7587

Email: info@mid-link.net

Identification of Proposed Device

Trade Name: Portable ECG Monitor

Common Name: Electrocardiograph

Model: PM10

Regulatory Information

Classification Name: Electrocardiograph

Classification: Class II

Product Code: DPS

Regulation Number: 21 CFR870.2340

Review Panel: Cardiovascular

Indication for use

The device is a handheld, personal electrocardiograph unit, which can measure electrical activities of the heart easily and conveniently. It is immediately available at any time to manually record transient cardiac events, suitable for home health care use, which can detect, display and store ECG signal, and if possible, provide average heart rate message after ECG measurement. The users can use it themselves to check their heart condition.

It is suitable for the adult users, who suffers from cardio-vascular diseases, or the adult people who are caring about their heart working conditions during their daily life. This device is not intended for use as a conventional diagnostic tool, but use as a healthcare tool which can provide doctor the recorded data as references.

The product is not a conventional diagnostic tool.

Device Description

The Portable ECG Monitor is intended to record and store Lead I ECG signals, and display heart rate for home health care use. It is composed of host and USB cable, powered by Built-in large capability rechargeable lithium battery.

The device can be connected with PC via USB, with mobile phone via Bluetooth protocol. The function of software PC and mobile phone includes sample mode and time setting, upload case, case review, QRS interval and VEB measurement, print, etc.

Identification of Predicate Device

510(k) Number: K093872
Product Name: Handheld ECG Monitor
Model Name: MD100

Non-Clinical Test Conclusion

Non clinical tests were conducted to verify that the proposed device met all design specifications as was Substantially Equivalent (SE) to the predicate device. The test results demonstrated that the proposed device complies with the following standards:

IEC 60601-1: 2005 + CORR. 1:2006 + CORR. 2:2007 + AM1:2012, Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.

IEC 60601-1-2: 2014, Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral Standard: Electromagnetic disturbances – Requirements and tests.

ISO 60601-2-47:2012, Medical Electrical Equipment -- Part 2-47: Particular Requirements For The Basic Safety And Essential Performance Of Ambulatory Electrocardiographic Systems.

Clinical Test Conclusion

No clinical study is included in this submission.

Substantially Equivalent (SE) Comparison

Table 1 Comparison of Technology Characteristics

Item	Proposed Device(s)	Predicate Device(s)
Product Code	DPS	Same
Regulation Number	870.2340	Same
Indication for use	The device is a handheld, personal electrocardiograph unit, which can measure electrical activities of the heart easily and conveniently. It is immediately available at any time to manually record transient cardiac events, suitable for home health care use, which can detect, display and store ECG signal, and if possible, provide average heart	Similar

	rate message after ECG measurement. The users can use it themselves to check their heart condition. It is suitable for the adult users, who suffers from cardio-vascular diseases, or the adult people who are caring about their heart working conditions during their daily life. This device is not intended for use as a conventional diagnostic tool, but use as a healthcare tool which can provide doctor the recorded data as references. The product is not a conventional diagnostic tool.	
Lead	Lead I	Same
Recording mode	automatic	Same
Measurement parameters	Heart rate, ECG waveform	Same
Display	LCD	Same
HR measurement range	30bpm~300bpm	Similar
HR measurement accuracy	±1bpm or 1%	Similar
Power supply	Battery	Same
Electrical safety	The proposed device was tested to demonstrated to comply with IEC 60601-1	Same
EMC	The proposed device was tested to demonstrated to comply with IEC 60601-1-2	Same
Patient Contact Material	Metal electrode	Same

Substantially Equivalent (SE) Conclusion

Based on the comparison and analysis above, the proposed devices are determined to be Substantially Equivalent (SE) to the predicate devices.