



July 8, 2018

Shen Wei USA Inc.
Cheryl Reep
Compliance Director
33278 Central Ave Ste 102
Union City, California 94587

Re: K173766

Trade/Device Name: PF Nitrile Exam Glove with pH coating, Black-Gray Color, Tested for Use with
Chemo Drugs
Regulation Number: 21 CFR 880.6250
Regulation Name: Patient Examination Glove
Regulatory Class: Class I
Product Code: LZC, LZA
Dated: June 19, 2018
Received: July 3, 2018

Dear Cheryl Reep:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's

requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/>) and CDRH Learn (<http://www.fda.gov/Training/CDRHLearn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<http://www.fda.gov/DICE>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Elizabeth F. Claverie -S

For Tina Kiang, Ph.D.
Acting Director
Division of Anesthesiology,
General Hospital, Respiratory,
Infection Control, and Dental Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

DEPARTMENT OF HEALTH AND HUMAN SERVICES
Food and Drug Administration

Form Approved: OMB No. 0910-0120

Expiration Date: January 31, 2017

See PRA Statement below.

Indications for Use

510(k) Number (if known)

K173766

Device Name

Powder-Free Nitrile Examination Glove with pH coating, Black-Gray Color, Tested for Use with Chemotherapy Drugs

Indications for Use (Describe)

A patient examination glove is a disposable device intended for medical purposes that is worn on the examiner's hand or finger to prevent contamination between patient and examiner. These gloves were tested for use with chemotherapy drugs as per ASTM D6978-05 (Reapproved 2013) Standard Practice for Assessment of Medical Gloves to Permeation by Chemotherapy Drugs. This device is single use only.

Chemotherapy Drug and Concentration	Breakthrough Detection Time in Minutes
1. Carmustine (BCNU), 3.3mg/ml	27.1
2. Cisplatin, 1.0mg/ml	>240
3. Cyclophosphamide (Cytosan), 20mg/ml	>240
4. Dacarbazine (DTIC) 10g mg/ml	>240
5. Doxorubicin Hydrochloride, 2.0 mg/ml	>240
6. Etoposide (Toposar), 20mg/ml	>240
7. Fluorouracil 50mg/ml	>240
8. Paclitaxel (Taxol), 6.0mg/ml	>240
9. Thiotepa, 10mg/ml	27.4

*Please note that Carmustine (BCNU) has low permeation time of 27.1 minutes and Thiotepa has permeation time of 27.4 minutes.

WARNING: Do not use with Carmustine and Thiotepa.

Type of Use (Select one or both, as applicable)

 Prescription Use (Part 21 CFR 801 Subpart D) Over-The-Counter Use (21 CFR 801 Subpart C)**CONTINUE ON A SEPARATE PAGE IF NEEDED.**

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."