



November 30, 2018

Better Care Plastic Technology Co., Ltd
% Kathy Liu
Project Manager
Hongray USA Medical Products Inc
3973 Schaefer Avenue, Chino, CA 91810, USA

Re: K182600

Trade/Device Name: Powder Free Nitrile Examination Glove, Tested for Use with Chemotherapy
Drugs (Blue)

Regulation Number: 21 CFR 880.6250

Regulation Name: Non-Powdered Patient Examination Glove

Regulatory Class: Class I

Product Code: LZA, LZC

Dated: September 16, 2018

Received: September 21, 2018

Dear Kathy Liu:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal

statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/CombinationProducts/GuidanceRegulatoryInformation/ucm597488.htm>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/MedicalDevices/DeviceRegulationandGuidance/>) and CDRH Learn (<http://www.fda.gov/Training/CDRHLearn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<http://www.fda.gov/DICE>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Clarence W. Murray III -S

For Tina Kiang, Ph.D.
Acting Director
Division of Anesthesiology,
General Hospital, Respiratory,
Infection Control, and Dental Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)

K182600

Device Name

Powder Free Nitrile Examination Gloves, Tested for Use with Chemotherapy Drugs (Blue)

Indications for Use (Describe)

Powder Free Nitrile Examination Gloves, Tested for Use with Chemotherapy Drugs (Blue) is a disposable device intended for medical purposes that is worn on the examiner's hand to prevent contamination between patient and examiner.

Gloves have been tested for use with chemotherapy drugs using ASTM D6978 and will be labeled with a statement of compliance and a summary of the testing results.

Chemotherapy Drug Permeation

The following chemicals have been tested with these gloves:

Chemotherapy Drug	Minimum Breakthrough Detection Time (Minutes)
Carmustine(BCNU) (3.3 mg/ml)	11.0
Cisplatin (1mg/ml)	>240
Cyclophosphamide (20mg/ml)	>240
Dacarbazine (DTIC) (10mg/ml)	>240
Doxorubicin Hydrochloride (2mg/ml)	>240
Etoposide (Toposar) (20mg/ml)	>240
Fluorouracil (50mg/ml)	>240
Methotrexate (25mg/ml)	>240
Paclitaxel (Taxol) (6mg/ml)	>240
Thiotepa (THT) (10mg/ml)	28.8
Bleomycin (15.0mg/ml)	>240
Busulfan (6.0mg/ml)	>240
Carboplatin, (10.0mg/ml)	>240
Chloroquine, (50.0mg/ml)	>240
Cyclosporin, (100.0mg/ml)	>240
Cytarabine, (100.0mg/ml)	>240
Daunorubicin, (5.0mg/ml)	>240
Docetaxel, (10.0mg/ml)	>240
Epirubicin (Ellence), (2.0mg/ml)	>240
Fludarabine, (25.0mg/ml)	>240
Gemcitabine (Gemzar) (38.0mg/ml)	>240
Idarubicin, (1.0mg/ml)	>240
Ifosfamide, (50.0mg/ml)	>240
Irinotecan, (20.0mg/ml)	>240
Mechlorethamine HCl, (1.0mg/ml)	>240
Melphalan, (5.0mg/ml)	>240
Mitomycin C, (0.5mg/ml)	>240
Mitoxantrone, (2.0mg/ml)	>240
Oxaliplatin, (2.0mg/ml)	>240
Paraplatin, (10.0mg/ml)	>240
Retrovir, (10.0mg/ml)	>240
Rituximab, (10.0mg/ml)	>240
Topotecan HCL, (1.0mg/ml)	>240
Trisonex, (1.0mg/ml)	>240
Velcade (Bortezomib), (1.0mg/ml)	>240
Vincristine, (1.0mg/ml)	>240

* Please note that the following drugs have extremely low permeation times:

Carmustine: 11.0 minutes and Thiotepa: 28.8 minutes

Type of Use (Select one or both, as applicable)

☐ Prescription Use (Part 21 CFR 801 Subpart D)

☒ Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

Better Care Plastic Technology Co., Ltd

Fuqian Xi Road, West district of Shenze Industrial Base, Shenze County,
Hebei Province, CHINA 050000

510(K) SUMMARY

The assigned 510(K) numbers: K182600

Date Prepared: November 18, 2018

1. Owner's Identification:

Mrs. Zhu Chunyan

Better Care Plastic Technology Co., Ltd.

Fuqian Xi Road, West district of Shenze Industrial Base, Shenze County, Hebei Province, CHINA
050000

Tel:86-311-66179653

Fax: 86-311-66179653

Contact: Ms. Kathy Liu, Project Manager or Ms Monica Yu

Hongray USA Medical Products Inc.

Address: 3973 Schaefer Avenue, Chino, CA 91810, USA

Tel:909-590-1611

Fax: 909-673-8347

2. Name of the Device:

Trade / Product Name: Powder Free Nitrile Examination Gloves, Tested for Use with
Chemotherapy Drugs (Blue)

Common Name: Exam Gloves

Classification Name: Non-powdered patient Examination

Classification Regulation: 21 CFR 880.6250

Product Code: LZA, LZA

Classification Panel: General Hospital

Device Class: Class I

3. Predicate Device Information:

Central Medicare Sdn. Bhd.

Blue Non Sterile Powder Free Nitrile Examination Gloves Tested for Use with
Chemotherapy Drugs (K173942)

4. Device Description:

Powder Free Nitrile Examination Gloves, Tested for Use with Chemotherapy Drugs (Blue)
are Class I Patient Examination Gloves and Specialty Chemotherapy Gloves. They are
ambidextrous and come in different sizes—Extra Small, Small, Medium, Large and Extra
Large.

Gloves meet the specification of ASTM D6319-10(2015) and have been tested for
resistance to permeation by chemotherapy drugs as per ASTM D6978-05(2013).

Better Care Plastic Technology Co., Ltd

Fuqian Xi Road, West district of Shenze Industrial Base, Shenze County,
Hebei Province, CHINA 050000

510(K) SUMMARY

5. Indications for Use:

Powder Free Nitrile Examination Gloves, Tested for Use with Chemotherapy Drugs (Blue) is a disposable device intended for medical purposes that is worn on the examiner's hand to prevent contamination between patient and examiner.

Gloves have been tested for use with chemotherapy drugs using ASTM D6978-05(2013) and will be labeled with a statement of compliance and a summary of the testing results.
Chemotherapy Drug Permeation

The following chemicals have been tested with these gloves:

Chemotherapy Drug	Minimum Breakthrough Detection Time (Minutes)
Carmustine(BCNU) (3.3 mg/ml)	11.0
Cisplatin (1mg/ml)	>240
Cyclophosphamide (20mg/ml)	>240
Dacarbazine (DTIC) (10mg/ml)	>240
Doxorubicin Hydrochloride (2mg/ml)	>240
Etoposide (Toposar) (20mg/ml)	>240
Fluorouracil (50mg/ml)	>240
Methotrexate (25mg/ml)	>240
Paclitaxel (Taxol) (6mg/ml)	>240
Thiotepa (THT) (10mg/ml)	28.8
Bleomycin (15.0mg/ml)	>240
Busulfan (6.0mg/ml)	>240
Carboplatin, (10.0mg/ml)	>240
Chloroquine, (50.0mg/ml)	>240
Cyclosporin, (100.0mg/ml)	>240
Cytarabine, (100.0mg/ml)	>240
Daunorubicin, (5.0mg/ml)	>240
Docetaxel, (10.0mg/ml)	>240
Epirubicin (Ellence), (2.0mg/ml)	>240
Fludarabine, (25.0mg/ml)	>240
Gemcitabine (Gemzar) (38.0mg/ml)	>240
Idarubicin, (1.0mg/ml)	>240
Ifosfamide, (50.0mg/ml)	>240
Irinotecan, (20.0mg/ml)	>240
Mechlorethamine HCl, (1.0mg/ml)	>240
Melphalan, (5.0mg/ml)	>240

Better Care Plastic Technology Co., Ltd

Fuqian Xi Road, West district of Shenze Industrial Base, Shenze County,
Hebei Province, CHINA 050000

510(K) SUMMARY

Mitomycin C, (0.5mg/ml)	>240
Mitoxantrone, (2.0mg/ml)	>240
Oxaliplatin, (2.0mg/ml)	>240
Paraplatin, (10.0mg/ml)	>240
Retrovir, (10.0mg/ml)	>240
Rituximab, (10.0mg/ml)	>240
Topotecan HCL, (1.0mg/ml)	>240
Trisonex, (1.0mg/ml)	>240
Velcade (Bortezomib), (1.0mg/ml)	>240
Vincristine, (1.0mg/ml)	>240

* Please note that the following drugs have extremely low permeation times: Carmustine: 11.0 minutes and Thiotepa: 28.8 minutes

6. Technological Characteristic Comparison Table:

The proposed device will be known as Powder Free Nitrile Examination Gloves, Tested for Use with Chemotherapy Drugs (Blue).

The following tables are summaries of the technological characteristics, biocompatibility and testing for use with chemotherapy drugs of the proposed and predicate devices.

General Comparison Table:

	Proposed Device K182600	Predicate Device K173942	Comparison
Trade Name	Powder Free Nitrile Examination Gloves, Tested for Use with Chemotherapy Drugs (Blue)	Blue Non Sterile Powder Free Nitrile Examination Gloves Tested for Use with Chemotherapy Drugs	Similar
Product Code	LZA, LZC	LZA, LZC	Same
Regulation Number	21 CFR 880.6250	21 CFR 880.6250	Same
Class	I	I	Same

Better Care Plastic Technology Co., Ltd

Fuqian Xi Road, West district of Shenze Industrial Base, Shenze County,
Hebei Province, CHINA 050000

510(K) SUMMARY

Indications for Use	Powder Free Nitrile Examination Gloves, Tested for Use with Chemotherapy Drugs (Blue) is a disposable device intended for medical purposes that is worn on the examiner's hand to prevent contamination between patient and examiner.	Blue Non Sterile Powder Free Nitrile Examination Gloves Tested for Use with Chemotherapy Drugs is a disposable device intended for medical purposes that is worn on the examiner's hands to prevent contamination between patient and examiner.	Same
Powder or Powder Free	Powder Free	Powder Free	Same
Design Feature	Ambidextrous	Ambidextrous	Same
Color	Blue	Blue	Similar
Labeling Information	Single-use indication, powder free, device name, glove size, quantity, Nitrile Examination Gloves, Non Sterile	Single-use indication, powder free, device name, glove size, quantity, Nitrile Examination Gloves, Non Sterile	Same
Chemotherapy Drug Permeation Claim	See below comparison table	See below comparison table	

Dimensions and Performance Comparison Table:

Technological Characteristics	Proposed Device K182600	Predicate Device K173942	Comparison
Length	Minimum 230mm	Minimum 230mm	Same
Palm Width (size) (mm)			
XS	70±10	70±10	Same
S	80±10	80±10	Same
M	95±10	95±10	Same

Better Care Plastic Technology Co., Ltd

Fuqian Xi Road, West district of Shenze Industrial Base, Shenze County,
Hebei Province, CHINA 050000

510(K) SUMMARY

L	110±10	110±10	Same
XL	120±10	120±10	Same
Thickness(mm)			
Finger	Minimum 0.05	0.10±0.03	Similar
Palm	Minimum 0.05	0.08±0.03	Similar
Tensile Strength, Before Aging	14MPa, min	14MPa, min	Same
Ultimate Elongation, Before Aging	500%, min	500%, min	Same
Tensile Strength, After Accelerated Aging	14MPa, min	14MPa, min	Same
Ultimate Elongation, After Accelerated Aging	400%, min	400%, min	Same
Freedom from holes	In accordance with ASTM D 5151-06, following ASTM D6319-10, G-1, AQL 2.5	In accordance with ASTM D 5151-06, following ASTM D6319-10, G-1, AQL 2.5	Same
Powder-Content	≤ 2 mg per glove	≤ 2 mg per glove	Similar
10993-10:2010 Skin Irritation Study	Under the conditions of the study, not an irritant	Under the conditions of the study, not an irritant	Same
10993-10:2010 Maximization Sensitization Study	Under the conditions of the study, not a sensitizer	Under the conditions of the study, not a sensitizer	Same
10993-5:2009 Cytotoxicity Test		Under the conditions of this study, not a cytotoxic potential	

Chemotherapy Permeation Comparison:

Tested Chemotherapy Drug and Concentration	Minimum BDT (Minutes)		Comparison
	Proposed Device K182600	Predicate Device K173942	
Carmustine(BCNU) (3.3 mg/ml)	11.0	12.4	Similar
Cisplatin (1mg/ml)	>240	>240	Same
Cyclophosphamide (20mg/ml)	>240	>240	Same
Dacarbazine (DTIC)	>240	>240	Same

Better Care Plastic Technology Co., Ltd

Fuqian Xi Road, West district of Shenze Industrial Base, Shenze County,
Hebei Province, CHINA 050000

510(K) SUMMARY

(10mg/ml)			
Doxorubicin Hydrochloride (2mg/ml)	>240	>240	Same
Etoposide (Toposar) (20mg/ml)	>240	>240	Same
Fluorouracil (50mg/ml)	>240	>240	Same
Methotrexate (25mg/ml)	>240	>240	Same
Paclitaxel (Taxol) (6mg/ml)	>240	>240	Same
Thiotepa (THT) (10mg/ml)	28.8	24.4	Similar
Bleomycin (15.0mg/ml)	>240	>240	Same
Busulfan (6.0mg/ml)	>240	>240	Same
Carboplatin, (10.0mg/ml)	>240	>240	Same
Chloroquine, (50.0mg/ml)	>240	>240	Same
Cyclosporin, (100.0mg/ml)	>240	>240	Same
Cytarabine, (100.0mg/ml)	>240	>240	Same
Daunorubicin, (5.0mg/ml)	>240	>240	Same
Docetaxel, (10.0mg/ml)	>240	>240	Same
Epirubicin (Ellence), (2.0mg/ml)	>240	>240	Same
Fludarabine, (25.0mg/ml)	>240	>240	Same
Gemcitabine (Gemzar) (38.0mg/ml)	>240	>240	Same
Idarubicin, (1.0mg/ml)	>240	>240	Same
Ifosfamide, (50.0mg/ml)	>240	>240	Same
Irinotecan, (20.0mg/ml)	>240	>240	Same
Mechlorethamine HCl, (1.0mg/ml)	>240	>240	Same
Melphalan, (5.0mg/ml)	>240	>240	Same
Mitomycin C, (0.5mg/ml)	>240	>240	Same
Mitoxantrone, (2.0mg/ml)	>240	>240	Same
Oxaliplatin, (2.0mg/ml)	>240	>240	Same
Paraplatin, (10.0mg/ml)	>240	>240	Same
Retrovir, (10.0mg/ml)	>240	>240	Same
Rituximab, (10.0mg/ml)	>240	>240	Same
Topotecan HCL, (1.0mg/ml)	>240	>240	Same
Trisonex, (1.0mg/ml)	>240	>240	Same
Velcade (Bortezomib), (1.0mg/ml)	>240	>240	Same
Vincristine, (1.0mg/ml)	>240	>240	Same

Better Care Plastic Technology Co., Ltd

Fuqian Xi Road, West district of Shenze Industrial Base, Shenze County,
Hebei Province, CHINA 050000

510(K) SUMMARY

7. Summary Non-Clinical Testing

Non-clinical tests were conducted to verify that the proposed device met all design specifications. The test results demonstrated that the proposed device complies with the following standards:

- ISO 10993-10:2010 Biological Evaluation of Medical Devices - Part 10: Tests For Irritation And Skin Sensitization.
- ISO 10993-5:2009 Biological Evaluation of Medical Devices - Part 5: Tests For In Vitro Cytotoxicity
- ASTM D6124-06 (Reapproved 2017), Standard Test Method for Residual Powder on Medical Gloves
- ASTM D5151-06 (Reapproved 2015), Standard Test Method for Detection of Holes in Medical Gloves.
- ASTM D6319-10(Reapproved 2015), Standard Specification for Nitrile Examination Gloves For Medical Application.
- ASTM D6978-05 (Reapproved 2013), Assessment of Reissuance of Medical Gloves to Permeation by Chemotherapy Drugs.

8. Summary of Clinical Testing

N/A

9. Conclusion:

The conclusions drawn from the nonclinical tests demonstrate that the proposed device is as safe, as effective, and performs as well as or better than the legally marketed predicated device.