



June 7, 2019

Meta Biomed Co., Ltd.  
% April Lee  
Consultant  
Withus Group Inc  
106 Superior  
Irvine, California 92620

Re: K190503  
Trade/Device Name: CeraSeal  
Regulation Number: 21 CFR 872.3820  
Regulation Name: Root Canal Filling Resin  
Regulatory Class: Class II  
Product Code: KIF  
Dated: March 11, 2019  
Received: March 19, 2019

Dear April Lee:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email ([DICE@fda.hhs.gov](mailto:DICE@fda.hhs.gov)) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Srinivas Nandkumar, Ph.D.  
Assistant Director  
DHT1B: Division of Dental Devices  
OHT1: Office of Ophthalmic, Anesthesia,  
Respiratory, ENT and Dental Devices  
Office of Product Evaluation and Quality  
Center for Devices and Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)  
K190503

Device Name  
CeraSeal

Indications for Use (Describe)

- Permanent obturation of the root canal following vital pulp-extirpation
- Permanent obturation of the root canal following removal of infected or necrotic pulp and placement of intracanal dressings

CeraSeal is suitable for use in the single cone and lateral condensation technique.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

### CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
[PRASStaff@fda.hhs.gov](mailto:PRASStaff@fda.hhs.gov)

*"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."*



Address  
270, Osongsaengmyeong 1-ro,  
Osong-eup, Heungdeok-gu,  
Cheongju-si, Chungcheongbuk-do,  
Korea

Tel  
+82-43-216-0433  
Fax  
+82-43-217-1988

METABIOMED CO.LTD  
E-mail  
info@meta-biomed.com  
http://www.meta-biomed.com

## 510(k) Summary

K190503

### Submitter

META BIOMED CO., LTD.  
Suk Song Oh  
270, Osongsaengmyeong 1-ro, Osong-eup,  
Heungdeok-gu, Cheongju-si, Chungcheongbuk-do,  
South Korea  
Email: ef1459@metabiogw.bizmeka.com  
Phone: +82-43-230-8841  
Fax: +82-43-217-1983

### Official Correspondent

Withus Group Inc  
April Lee  
106 Superior,  
Irvine, CA 92620  
USA  
Email: withus6664@gmail.com  
Phone: 1-909-274-9971  
Fax: 1-909-460-8122

### Device Information

- Trade Name: CeraSeal
- Classification Name: resin, root canal filling
- Product Code: KIF
- Panel: Dental
- Regulation Number: 21 CFR 872.3820
- Device Class: Class II
- Date prepared: 02/20/2019

### Predicate Devices:

- K080917, iRoot SP manufactured by Innovative BioCeramix Inc.

### Device Description

CeraSeal is used for root canal filling after the removal of infected pulp tissue in the root canal. CeraSeal is in the form of a flowable paste and can be immediately applied inside the root canal using a disposable tip, material, and a thickening agent. This product characteristically cures slowly by absorbing the ambient water inside the root canal. It is white and aesthetic.

### Indication for Use

- Permanent obturation of the root canal following vital pulp-extirpation
- Permanent obturation of the root canal following removal of infected or necrotic pulp and placement of intracanal dressings

CeraSeal is suitable for use in the single cone and lateral condensation technique.

## Non-clinical Testing

The following testing was conducted on our subject device:

- Biocompatibility Tests according to ISO 10993-1:2009, ISO 10993-5:2009, ISO 10993-10:2010, ISO 10993-11:2006.
- Performance tests such as visual, capacity, package, flow, setting time, film thickness, solubility and radio-opacity according to ISO 6876:2012.
- Shelf Life test: ISO 6876 tests (extraneous matter, package, flow, setting time, film thickness and Solubility)

## Summary of Technological Characteristics:

The subject device and the predicate device have the same intended use and have the similar technological characteristics and are made of similar materials. They encompass the same range of physical and chemical properties. The subject device and predicate devices are packaged in similar material and use similar methods of application.

The subject device is different from the predicate devices in raw materials, however, the test results provided in this submission supports that it is substantially equivalent to the predicate device.

	Subject Device	Predicate Device
Manufacturer	Meta Biomed Co., Ltd.	Innovative BioCeramix, Inc.
Device Name	CeraSeal	iRoot SP
510(k) Number	NA	K080917
Classification Name	resin, root canal filling	resin, root canal filling
Product Code	KIF	KIF
Regulation Number	21 CFR 872.3820	21 CFR 872.3820
Indications for use	<ul style="list-style-type: none"> <li>● Permanent obturation of the root canal following vital pulp-extirpation</li> <li>● Permanent obturation of the root canal following removal of infected or necrotic pulp and placement of intracanal dressings</li> </ul> <p>CeraSeal is suitable for use in the single cone and lateral condensation technique.</p>	<ul style="list-style-type: none"> <li>● Permanent obturation of the root canal following vital pulp-extirpation</li> <li>● Permanent obturation of the root canal following removal of infected or necrotic pulp and placement of intracanal dressings</li> </ul> <p>iRoot SP is suitable for use in the single cone and lateral condensation technique.</p>

Raw Material	<p>Insoluble, radiopaque material based on a calcium silicate compound containing:</p> <p>Thickening agent</p> <p>Zirconium dioxide</p> <p>Calcium silicates</p> <p>1, 3 Propanediol</p>	<p>Insoluble, radiopaque material based on a calcium silicate compound containing:</p> <p>Thickening agent</p> <p>Zirconium oxide filler</p> <p>Calcium silicates</p> <p>Calcium phosphate monobasic</p> <p>Calcium hydroxide</p>
Principle of Operation	identical	<p>iRoot SP is a convenient premixed ready-to-use injectable white hydraulic cement paste developed for permanent root canal filling and sealing applications. iRoot SP is an insoluble, radiopaque material which requires the presence of water to set and harden. iRoot SP is packaged in a pre-loaded syringe and is supplied with disposable tips.</p>
Performance Standard Conformance	Conformed to ISO 6876	Conformed to ISO 6876
Biocompatibility	Yes	Yes
Use	Prescription / Hospital	Prescription / Hospital
Delivery Forms	Single Paste	Single Paste
Sterility	Non-sterile	Non-sterile
Shelf Life	2 years	2 years

**Conclusion:**

Based on documentation supplied with this submission, conclusions drawn from the testing results demonstrate that the subject device is substantially equivalent to our legally marketed predicate device.