



Canon Medical Systems Corporation
% Orlando Tadeo, Jr.
Sr. Manager, Regulatory Affairs
Canon Medical Systems USA, Inc.
2441 Michelle Drive
TUSTIN CA 92780

July 23, 2019

Re: K191662

Trade/Device Name: Vantage Orian 1.5T, MRT-1550, V4.5
Regulation Number: 21 CFR 892.1000
Regulation Name: Magnetic resonance diagnostic device
Regulatory Class: Class II
Product Code: LNH
Dated: June 20, 2019
Received: June 21, 2019

Dear Mr. Tadeo:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for

devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

For

Thalia T. Mills, Ph.D.
Director
Division of Radiological Health
OHT7: Office of In Vitro Diagnostics
and Radiological Health
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)

K191662

Device Name

Vantage Orian 1.5T, MRT-1550, V4.5

Indications for Use (Describe)

Vantage Orian 1.5T systems are indicated for use as a diagnostic imaging modality that produces cross-sectional transaxial, coronal, sagittal, and oblique images that display anatomic structures of the head or body. Additionally, this system is capable of non-contrast enhanced imaging, such as MRA.

MRI (magnetic resonance imaging) images correspond to the spatial distribution of protons (hydrogen nuclei) that exhibit nuclear magnetic resonance (NMR). The NMR properties of body tissues and fluids are:

- Proton density (PD) (also called hydrogen density)
- Spin-lattice relaxation time (T1)
- Spin-spin relaxation time (T2)
- Flow dynamics
- Chemical Shift

Depending on the region of interest, contrast agents may be used. When interpreted by a trained physician, these images yield information that can be useful in diagnosis.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRAStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(k) SUMMARY

K191662

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirements of Safe Medical Device Act 1990 and 21 CFR § 807.92

1. CLASSIFICATION and DEVICE NAME

Classification Name:	Magnetic Resonance Diagnostic Device
Regulation Number:	90-LNH (Per 21 CFR § 892.1000)
Trade Proprietary Name:	Vantage Orian 1.5T, MRT-1550, V4.5
Model Number:	MRT-1550

2. SUBMITTER'S NAME

Canon Medical Systems Corporation
1385 Shimoishigami
Otawara-Shi, Tochigi-ken, Japan 324-8550

3. OFFICIAL CORRESPONDENT

Naofumi Watanabe
Senior Manager, Regulatory Affairs and Vigilance
Canon Medical Systems Corporation

4. CONTACT PERSON, U.S. AGENT and ADDRESS**Contact Person**

Orlando Tadeo, Jr.
Sr. Manager, Regulatory Affairs
Canon Medical Systems USA, Inc.
2441 Michelle Drive, Tustin, CA 92780
Phone: (714) 669-7459
Fax: (714) 730-1310
otadeo@us.medical.canon

Official Correspondent/U.S. Agent

Paul Biggins
Senior Director, Regulatory Affairs
Canon Medical Systems USA, Inc.
2441 Michelle Drive, Tustin, CA 92780
Phone: (714) 730-7808
Fax: (714) 730-1310
pbiggins@us.medical.canon

MANUFACTURING SITE

Canon Medical Systems Corporation
1385 Shimoishigami
Otawara-shi, Tochigi 324-8550, Japan

5. ESTABLISHMENT REGISTRATION

9614698

6. DATE PREPARED

June 20, 2019

7. DEVICE NAME

Vantage Orian 1.5T, MRT-1550, V4.5

8. TRADE NAME

Vantage Orian 1.5T, MRT-1550, V4.5

9. CLASSIFICATION NAME

Magnetic Resonance Diagnostic Device (MRDD)

10. CLASSIFICATION PANEL

Radiology

11. DEVICE CLASSIFICATION

Class II (per 21 CFR 892.1000, Magnetic Resonance Diagnostic Device)

12. PRODUCT CODE

90-LNH

13. PREDICATE DEVICE

System	Predicate Device
	Vantage Orian 1.5T, MRT-1550, V4.5
Marketed By	Canon Medical Systems USA
510(k) Number	K182282
Clearance Date	October 19, 2018

14. REASON FOR SUBMISSION

Modification of a cleared device

15. SUBMISSION TYPE

Special 510(k) Premarket Notification

16. DEVICE DESCRIPTION

The Vantage Orian (Model MRT-1550) is a 1.5 Tesla Magnetic Resonance Imaging (MRI) System. The Vantage Orian uses 1.4 m short and 4.0 tons light weight magnet. It includes the Canon Pianissimo™ and Pianissimo Zen technology (scan noise reduction technology). The design of the gradient coil and the whole body coil of the Vantage Orian provides the maximum field of view of 55 x 55 x 50 cm. The Model MRT-1550/AS, AT, AW, AX includes the standard gradient system. This system is based upon the technology and materials of previously marketed Canon Medical Systems MRI systems and is intended to acquire and display cross-sectional transaxial, coronal, sagittal, and oblique images of anatomic structures of the head or body. The Vantage Orian MRI System is comparable to the current 1.5T Vantage Orian MRI System (K182282), cleared October 19, 2018 with the following modifications.

17. SUMMARY OF CHANGE(S)

- **Magnet (For MRT-1550/AS, MRT-1550/AT, MRT-1550/AW, MRT-1550/AX):**
The Vantage Orian 1.5T Encore Program is a field upgrade program designed to utilize the currently installed 1.5T magnet (OR76) of Vantage, Atlas and Vantage Titan 1.5T systems, and convert the existing system to a Vantage Orian 1.5T MR system. The Vantage Orian 1.5T Encore Program designation will be MRT-1550/AS, AT, AW, AX.

	Orian	Orian Upgrade from Vantage/Atlas/Vantage Titan
Magnet	OR200 magnet	Used OR76 magnet on site (Same magnet performance as brand-new OR76 magnet)
System label	MRT-1550/AC, MRT-1550/AD, MRT-1550/AG, MRT-1550/AH, MRT-1550/AK, MRT-1550/AL, MRT-1550/AO, MRT-1550/AP (Predicate Devices)	MRT-1550/AS, MRT-1550/AT, MRT-1550/AW, MRT-1550/AX, (Subject Devices)

List of the systems with OR76 magnet which are included in the Encore Orian Program

510(k) number	Device name	Model Number
K032490	EXCELART Vantage AGV	MRT-1503/P2
	EXCELART Vantage XGV	MRT-1503/P3
K060003	EXCELART Vantage ZGV	MRT-1503/P5
K063361	EXCELART Vantage Atlas-X	MRT-1503/S3
	EXCELART Vantage Atlas-Z	MRT-1503/S5
K080038	Vantage Titan	MRT-1504/S3
K112003	Vantage Titan HSR	MRT-1504/A5
K120638	Vantage Titan	MRT-1504/U4
K122613	Vantage Titan	MRT-1504/U5
K150427	Vantage Titan	MRT-1510/3A*

*Internal documentation determined that a new 510(k) was not required to market this model number.

18. SAFETY PARAMETERS

Item	Subject Device: Vantage Orian 1.5T, MRT-1550, V4.5	Predicate Device: Vantage Orian 1.5T, MRT-1550, V4.5	Notes
Static field strength	1.5T	1.5T	Same
Operational Modes	Normal and 1st Operating Mode	Normal and 1st Operating Mode	Same
i. Safety parameter display	SAR, dB/dt	SAR, dB/dt	Same
ii. Operating mode access requirements	Allows screen access to 1st level operating mode	Allows screen access to 1st level operating mode	Same
Maximum SAR	4W/kg for whole body (1st operating mode specified in IEC 60601-2-33: 2010+A1:2013+A2:2015)	4W/kg for whole body (1st operating mode specified in IEC 60601-2-33: 2010+A1:2013)	
Maximum dB/dt	1st operating mode specified in IEC 60601-2-33: 2010+A1:2013+A2:2015	1st operating mode specified in IEC 60601-2-33: 2010+A1:2013	
Potential emergency condition and means provided for shutdown	Shutdown by Emergency Ramp Down Unit for collision hazard for ferromagnetic objects	Shutdown by Emergency Ramp Down Unit for collision hazard for ferromagnetic objects	Same

19. IMAGING PERFORMANCE PARAMETERS

No change from the previous predicate submission, K182282.

20. INDICATIONS FOR USE

Vantage Orian 1.5T systems are indicated for use as a diagnostic imaging modality that produces cross-sectional transaxial, coronal, sagittal, and oblique images that display anatomic structures of the head or body. Additionally, this system is capable of non-contrast enhanced imaging, such as MRA.

MRI (magnetic resonance imaging) images correspond to the spatial distribution of protons (hydrogen nuclei) that exhibit nuclear magnetic resonance (NMR). The NMR properties of body tissues and fluids are:

- Proton density (PD) (also called hydrogen density)
- Spin-lattice relaxation time (T1)
- Spin-spin relaxation time (T2)
- Flow dynamics
- Chemical Shift

Depending on the region of interest, contrast agents may be used. When interpreted by a trained physician, these images yield information that can be useful in diagnosis.

21. SUMMARY OF DESIGN CONTROL ACTIVITIES

Hazard analysis has been performed and documentation is included in this submission. The test methods used are the same as those submitted in the previously cleared submissions of the predicate device, Vantage Orian 1.5T, MRT-1550, V4.5 (K182282). A declaration of conformity with design controls is included in this submission.

22. SAFETY

This device is designed and manufactured under the Quality System Regulations as outlined in 21 CFR § 820 and ISO 13485 Standards.

This device is based upon the same technologies, materials and software as the predicate device. Risk activities were conducted in concurrence with established medical device development standards and guidance. Additionally, testing was done in accordance with applicable recognized consensus standards published by the International Electrotechnical Commission (IEC) for medical devices and the National Electrical Manufacturers Association (NEMA):

LIST OF APPLICABLE STANDARDS

- IEC60601-1 (2005), Amd.1 (2012)
- IEC60601-1-2 (2014)
- IIEC60601-1-6 (2010), Amd.1 (2013)
- IEC60601-2-33 (2010), Amd.1 (2013), Amd.2 (2015)
- IEC60825-1 (2007)
- IEC62304 (2006)
- IEC62366 (2007), Amd.1 (2014)
- NEMA MS 1 (2008)
- NEMA MS 2 (2008)
- NEMA MS 3 (2008)
- NEMA MS 4 (2010)
- NEMA MS 5 (2010)

23. TESTING

Risk analysis and verification/validation testing conducted through bench testing are included in this submission which demonstrate that the system requirements have been met. Additionally, image quality testing was completed which demonstrated that the subject device meets predetermined acceptance criteria

Software Documentation for a Moderate Level of Concern, per the FDA guidance document, "Guidance for the Content of Premarket Submissions for Software Contained in Medical Devices Document" issued on May 11, 2005, is also included as part of this submission.

24. SUBSTANTIAL EQUIVALENCE

Canon Medical Systems Corporation believes that the Vantage Oriion 1.5T, MRT-1550, V4.5 Magnetic Resonance Imaging (MRI) System is substantially equivalent to the previously cleared predicate device referenced in this submission.

Canon Medical Systems Corporation believes that the changes incorporated into the Vantage Oriion 1.5T, MRT-1550, V4.5 are substantially equivalent to the previously cleared predicate devices.

25. CONCLUSION

The modifications incorporated into the Vantage Oriion 1.5T, MRT-1550, V4.5 do not change the indications for use or the intended use of the device. Based upon bench testing, successful completion of software validation and application of risk management and design controls, it is concluded that the subject device is safe and effective for its intended use.