



January 14, 2020

Globus Medical Inc.
Kelly Baker, PhD
Senior Vice President, Regulatory and Clinical Affairs
2560 General Armistead Ave.
Audubon, Pennsylvania 19403

Re: K192115
Trade/Device Name: SABLE™ Expandable Spacer
Regulation Number: 21 CFR 888.3080
Regulation Name: Intervertebral Body Fusion Device
Regulatory Class: Class II
Product Code: MAX
Dated: December 13, 2019
Received: December 16, 2019

Dear Dr. Baker:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part

801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Brent Showalter, PhD
Assistant Director (Acting)
DHT6B: Division of Spinal Devices
OHT6: Office of Orthopedic Devices
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)

K192115

Device Name

SABLE™ Expandable Spacer

Indications for Use (Describe)

SABLE™ Expandable Spacer is an interbody fusion device intended for use in patients with degenerative disc disease (DDD) at one or two contiguous levels of the lumbosacral spine (L2-S1). DDD is defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies. These patients should be skeletally mature and have had at least six (6) months of non-operative treatment. In addition, these patients may have up to Grade 1 spondylolisthesis or retrolisthesis at the involved level(s). The spacer is to be filled with autograft and/or allogenic bone graft comprised of cancellous and/or corticocancellous bone and is to be used with supplemental fixation.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASTaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(k) Summary: SABLE™ Expandable Spacer

Company: Globus Medical Inc.
2560 General Armistead Ave.
Audubon, PA 19403
610-930-1800

Contact: Kelly Baker, Ph.D.
Senior Vice President, Regulatory and Clinical Affairs

Date Prepared: August 5, 2019

Device Name: SABLE™ Expandable Spacer

Common Name: Intervertebral Body Fusion Device

Classification: Per 21 CFR as follows:
§888.3080 Intervertebral Body Fusion Device
Product Code(s): MAX
Regulatory Class: II, Panel Code: 87

Primary Predicate: RISE® Spacer (K113447)

Additional Predicates: ALTERA® Spacer (K140411)
SUSTAIN® Spacers (K181357)
NeoFuse Ti3D Interbody (K170318)

Purpose:

The purpose of this submission is to request clearance for the SABLE™ Expandable Spacer.

Device Description:

The SABLE™ Expandable Spacer is an expandable lumbar interbody fusion device used to provide structural stability in skeletally mature individuals following discectomy. The device is available in various heights and geometric options to fit the anatomical needs of a wide variety of patients.

SABLE™ Spacers are manufactured from titanium alloy. The endplates are additively manufactured from titanium alloy powder and an internal component is manufactured from radiolucent PEEK polymer. The drive screw is manufactured from cobalt chromium alloy.

Indications for Use:

SABLE™ Expandable Spacer is an interbody fusion device intended for use in patients with degenerative disc disease (DDD) at one or two contiguous levels of the lumbosacral spine (L2-S1). DDD is defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies. These patients should be skeletally mature and have had at least six (6) months of non-operative treatment. In addition, these patients may have up to Grade 1 spondylolisthesis or retrolisthesis at the involved level(s). The spacer is to be filled with autograft and/or allogenic bone graft comprised of cancellous and/or corticocancellous bone and is to be used with supplemental fixation.

Performance Data:

Mechanical testing (static and dynamic compression and compression-shear, and subsidence) was conducted in accordance with the “Guidance for Industry and FDA Staff, Class II Special Controls Guidance Document: Intervertebral Fusion Device,” June 12, 2007, ASTM F2077, and ASTM F2267 to demonstrate substantial equivalence to the predicate devices. Bacterial endotoxin testing (BET) was conducted in accordance with ANSI/AAMI ST-72:2011.

Technological Characteristics:

Subject implants have the same technological characteristics as the predicate devices including design, intended use, material composition, function, and range of sizes.

Basis of Substantial Equivalence:

Subject interbody spacers have been found to be substantially equivalent to the predicate devices with respect to technical characteristics, performance, and intended use. The information provided within this premarket notification supports substantial equivalence of the subject spacers to the predicate devices.