

July 2, 2020

Little Doctor Electronic (Nantong) Co.,Ltd % Iris Du RA manager Little Doctor Electronic (Nantong)Co.,Ltd No.14, Zhenxingxilu Road, Chongchuan Area Nantong, 226000 CHINA

Re: K193259

Trade/Device Name: Aneroid Sphygmomanometer Model NT-20, NT-201 Aneroid sphygmomanometer with stethoscope, Model NT-50 Regulation Name: Blood Pressure Cuff Regulatory Class: Class II Product Code: DXQ, LDE Dated: May 27, 2020 Received: June 1, 2020

Dear Iris Du:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <u>https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems</u>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<u>https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance</u>) and CDRH Learn (<u>https://www.fda.gov/training-and-continuing-education/cdrh-learn</u>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<u>https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice</u>) for more information or contact DICE by email (<u>DICE@fda.hhs.gov</u>) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

LT Stephen Browning Assistant Director Division of Cardiac Electrophysiology, Diagnostics and Monitoring Devices Office of Cardiovascular Devices Office of Product Evaluation and Quality Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number *(if known)* K193259

Device Name

Aneroid Sphygmomanometer Model NT-20, NT-201 Aneroid Sphygmomanometer with Stethoscope, Model NT-50

Indications for Use (Describe)

The device is intended to be used by medical professionals or in the home for the measurement of systolic and diastolic pressure. The device is intended to be manually attached to a patient and manually inflated along with a manual method for detecting Korotkoff sounds.

	both, as applicable)	e or	(Select one	of Use	Type
--	----------------------	------	-------------	--------	------

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services Food and Drug Administration Office of Chief Information Officer Paperwork Reduction Act (PRA) Staff *PRAStaff@fda.hhs.gov*

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

Sponsor:Little Doctor Electronic(Nantong)Co.,Ltd

Subjective device: Aneroid sphygmomanometer Model NT-20, NT-201

Aneroid sphygmomanometer with stethoscope Model NT-50

Chapter 6. 510(K) Summary

1.submitter

- Company name: Little Doctor Electronic(Nantong)Co.,Ltd
- Address:No.14, Zhenxingxilu Road, Chongchuan Area, Nantong, Jiangsu, 226000, China
- Postal Code:226000
- TEL:86-0513-80580127
- Contact person: Xinhua Pan (General Manager)

Iris Du(RA Manager)

• E-mail:dulh@lordmed.com

2.Subject Device Information

- Model: NT-20, NT-201, NT-50
- Common name: Blood Pressure Kit(Blood Pressure Cuff)
- Trade name: Aneroid sphygmomanometer Model NT-20, NT-201/ Aneroid sphygmomanometer with stethoscope, Model NT-50
- Product Code: DXQ,LDE
- Regulation name: 21CFR 870.1120 (Blood pressure cuff)/21CFR 870.1875(Stethoscope)
- Regulation class: II
- Review Panel: Cardiovascular

3.Predicate device

 RH non-Automated Blood Pressure Meter, MODEL Max02 and RH non-Automated Blood Pressure Meter with stethoscope.MODEL Max0201,Max0202,Max0203,(K190902)

Sponsor:Little Doctor Electronic(Nantong)Co.,Ltd

Subjective device: Aneroid sphygmomanometer Model NT-20, NT-201

Aneroid sphygmomanometer with stethoscope Model NT-50

4.Intended use

The device is intended to be used by medical professionals or in the home for the measurement of systolic and diastolic pressure. The device is intended to be manually attached to a patient and manually inflated along with a manual method for detecting Korotkoff sounds.

5.Device description

Aneroid sphygmomanometer Model NT-20, NT-201/ Aneroid sphygmomanometer with stethoscope, Model NT-50 are a manual non-invasive aneroid sphygmomanometer which respectively uses an inflation cuff wrapped around the upper arm. The cuff is inflated and deflated by a manual inflation bulb. Besides a manometer (Aneroid gauge), the accessories include cuff, inflation bulb, instruction manual and carrying case. Model NT-50 also includes a stethoscope. It is conjunction with stethoscope when use.

This is subject device same as the predicate device(K190902) in terms of PVC or latex inflation bulb and the PVC or latex bladder. Besides, the material of added the optional various size of cuff (nylon or cotton cuff for this application) and the aluminum manometer are identical to the predicate device (K190902).

6. Substantial equivalence comparison

It is substantially equivalence to the predicate device (K190902) RH non-Automated Blood Pressure Meter, MODEL Max02 and RH non-Automated Blood Pressure Meter with stethoscope MODEL Max0201,Max0202,Max0203 with respect to indication for use, device description, and technical description.

All comparison table for applied device are as following, and the substantial equivalence determination is based on the 510(K) Substantial Equivalence Decision-Making Process Flowchart which includes the comparison and discussion of indication for use, technology, and performance specifications.

Sponsor:Little Doctor Electronic(Nantong)Co.,Ltd

Subjective device: Aneroid sphygmomanometer Model NT-20, NT-201

Aneroid sphygmomanometer with stethoscope Model NT-50

The new device column includes the following device: Aneroid sphygmomanometer Model NT-20, NT-201/ Aneroid sphygmomanometer with stethoscope, Model NT-50

Comparison item	Subject device: Aneroid	Predicate device : BH non-Automated Blood	Result of
	sphygmomanometer Mode NT-20, NT-201/ Aneroid sphygmomanometer with	pressure meter Model Max 02,RH non-Automated Blood pressure meter	
	stethoscope, Model NT-50	with stethoscope, Max0201,Max0202,Max02 03	
Applicant	Little Doctor Electronic (Nantong)Co.,Ltd	Wenzhou Renhua Instruments Co.,Ltd	
510(K) number	Applying	К190902	-
Regulation number	21 CFR 870.1120	21 CFR 870.1120	Same
Product code	DXQ.LDE	DXQ.LDE	Same
Classificatio n	Class II	Class II	Same
Intended use	The device is intended to be used by medical professional or at home for the measurement of systolic and diastolic pressure by detecting korotkoff sounds.	The device is intended to be used by medical professional or at home for the measurement of systolic and diastolic pressure by detecting korotkoff sounds.	Same
Over-the-co unter use	Yes	Yes	Same

Sponsor:Little Doctor Electronic(Nantong)Co.,Ltd

Subjective device: Aneroid sphygmomanometer Model NT-20, NT-201

Target population	New born, Infants, children, young adults,adults	New born,Infants,children , young adults,adults	Same
Where used	Hospital, home, office, and ambulance, etc.	Hospital, home, office, and ambulance, etc.	Same
Anatomical sites	Upper arm(leg for child)	Upper arm(leg for child)	Same
Measureme nt Method	Ausculatory Korotkoff sounds method	Ausculatory Korotkoff sounds method	Same
Inflation	Manual by inflation bulb	Manual by inflation bulb	Same
Deflation	Manual deflation via valve	Manual deflation via valve	Same
Display	Aneroid Manometer	Aneroid Manometer	Same
The monitor scale	From 0 to 300mmHg with a minimum interval of 2 mmHg	From 0 to 300mmHg with a minimum interval of 2 mmHg	Same
Design	The device comprised tubing attached to a cuff with an integrated inflatable bladder that is wrapped around the patient's limb and secured by hook and loop closure.	The device comprised tubing attached to a cuff with an integrated inflatable bladder that is wrapped around the patient's limb and secured by hook and loop closure.	Same

Aneroid sphygmomanometer with stethoscope Model NT-50

Sponsor:Little Doctor Electronic(Nantong)Co.,Ltd

Subjective device: Aneroid sphygmomanometer Model NT-20, NT-201

Design of	One type option:	Three types option:	Similar
stethoscope	Single head	Single head	
		Dual head	
		Rappaport	
Materials	The manometer	The manometer	Same
	aluminum and stainless:	aluminum and stainless:	
	steel materials.	steel materials.	
	The tubing ,inflation	The tubing ,inflation	
	bulb:PVC or latex.	bulb:PVC or latex.	
	Cuff:Nylon cloth or cotton	Cuff:Nylon cloth or cotton	
	cloth for outside layer.	cloth for outside layer.	
	Cuff bladder:PVC or nature	Cuff bladder:PVC or nature	
	latex	latex	
Accuracy	Pressure : ±3mmHg of	Pressure : ±3mmHg of	Same
	reading	reading	
Compatibility	It can be used from 50° F to	It can be used from 50 $^\circ\mathrm{F}$ to	Same
with	104° F(10° C to 40° C)	104°F (10°C to 40°C) and	
environment	and 15%~85% RH	15%~85% RH humidity .	
	humidity .		
Cuff size	185mm*55mm	20"×5.5"(510mm*140mm)	Similar
	255mm*75mm	21.7"×6.3"(550mm*160m	
	345mm*110mm	m)	
	500mm*140mm	24.4"×6.9"(620mm*175m	
	540mm*145mm	m)	
	615mm*175mm	28.3"×8.3"(720mm*210m	
	780mm*220mm	m)	
		13.4"×4.15"(340mm*105m	
		m)	
		10.2"×3"(260mm*75mm)	
1			1

Aneroid sphygmomanometer with stethoscope Model NT-50

Sponsor:Little Doctor Electronic(Nantong)Co.,Ltd

Subjective device: Aneroid sphygmomanometer Model NT-20, NT-201

Cuff circumference	Fits arm circumferences 100mm-660mm,the standard cuff should be available for use in measuring a child's leg blood pressure and for children with larger arms.	Fits arm circumferences 8.7"to 17.3" (220mm-440cm),the standard cuff should be available for use in measuring a child's leg blood pressure and for children with larger arms.	Similar
Cuff bladder Size	40mm*80mm 60mm*120mm 80mm*150mm 120mm*220mm 145mm*315mm 180mm*370mm	8.7"×4.7"(220mm*120mm) 11.8"×5.9"(300mm*150m m) 13.4"×6.7"(340mm*170m m) 6.9"×3.3"(175mm*85mm) 7.5"×2"(190mm*50mm)	Similar
Contents(with accessories)	Aneroid gauge,Arm cuff, inflation bulb, and instruction manual, stethoscope(option)	Aneroid gauge,Arm cuff, inflation bulb, and instruction manual, stethoscope(option)	Same
Biocompatibili ty	Biocompatible as requirement of ISO 10993-1 ISO 10993-5 ISO 10993-10	Biocompatible as requirement of ISO 10993-1 ISO 10993-5 ISO 10993-10	Same
performance	Compatible as requirement of ISO 81060-1	Compatible as requirement of ISO 81060-1	Same

Aneroid sphygmomanometer with stethoscope Model NT-50

Sponsor:Little Doctor Electronic(Nantong)Co.,Ltd

Subjective device: Aneroid sphygmomanometer Model NT-20, NT-201

Aneroid sphygmomanometer with stethoscope Model NT-50

Discussion:

Same in details:

- Compared with the predicate device(K190902) RH non-Automated Blood pressure meter Model Max 02,RH non-Automated Blood pressure meter with stethoscope, Max02 01, Max0202, Max0203, the subject device Aneroid sphygmomanometer Model NT-20, NT-201/ Aneroid sphygmomanometer with stethoscope, Model NT-50 are same as the predicate device(K190902) in terms of material s of the inflation bulb, tubing, cuff and manometer.
- Predicate device (K190902) and subject device NT-20, NT-201, NT-50 have same classification, indication for use, target population, measurement method, product physical design and operation, performance parameter ranges, mechanical safety, anatomical sites, operation principles and etc. The structure of the predicate device (K190902) are same as the subject device , which all have manometer, bulb, tube, various size of cuff and stethoscope(optional). The predicate device and subject device all have single head stethoscope.

Difference in details:

The different sizes of the cuffs and bladders,cuff circumstance are provided in order to accommodate target population. All performance have been tested in this submission and do not raised any safety or effectiveness issue.All performance specification was verified to comply with the ISO 81060-1 standard.

Sponsor:Little Doctor Electronic(Nantong)Co.,Ltd

Subjective device: Aneroid sphygmomanometer Model NT-20, NT-201

Aneroid sphygmomanometer with stethoscope Model NT-50

7. Non-clinical testing

Non-clinical testing included biocompatibility and bench testing. The tests listed in the ISO 10993 and ISO 81060-1, were included. Results of testing were acceptable.

8. Clinical testing

No clinical study is included in this submission.

Conclusion

RH non-automated blood pressure meter Model Max 02, non-automated blood pressure meter with stethoscope Model Max0201, Max 0202,Max0203(K190902) and subject device have the same intended use and similar technological characteristics. Moreover, information contained in this submission supplied demonstrates that any difference in their characteristic do not raise any new questions of safety or effectiveness. Thus, aneroid sphymomanometer Model NT-20, NT-201/ aneroid sphymomanometer with stethoscope, Model NT-50 are equivalent to the predicate device(K190902).