SUMMARY OF SAFETY AND EFFECTIVENESS DATA (SSED)

I. GENERAL INFORMATION

Device Generic Name: Next generation sequencing oncology panel,

somatic or germline variant detection system

Device Trade Name: FoundationOne® Liquid CDx (F1 Liquid CDx)

Device Procode: PQP

Applicant's Name and Address: Foundation Medicine, Inc.

150 Second Street Cambridge, MA 02141

Date(s) of Panel Recommendation: None

Premarket Approval Application (PMA) Number: P190032/S010

Date of FDA Notice of Approval: June 8, 2023

The original Premarket Approval (PMA) (190032) for FoundationOne® Liquid CDx (F1LCDx) was approved on August 26, 2020 as a companion diagnostic for *BRCA1* and *BRCA2* alterations in metastatic castration-resistant prostate cancer (mCRPC) patients who may benefit from treatment with RUBRACA® (rucaparib) and *EGFR* activating mutations (Exon 19 deletions and L858R substitution mutation) in patients with advanced and metastatic non-small cell lung cancer (NSCLC) who may benefit from treatment with IRESSA® (gefitinib), TAGRISSO® (osimertinib), and TARCEVA® (erlotinib). Subsequently, additional PMA supplements were approved for expanding the indications for use of F1LCDx since its original approval. See Section VII for more details.

The current supplement was submitted to expand the indication for the FoundationOne Liquid CDx test as a companion diagnostic for the indication listed in the table below.

New Indication Being Sought in this PMA supplement submission.

Tumor Type	Biomarker(s) Detected	Therapy
Colorectal cancer	BRAF V600E alteration	BRAFTOVI® (encorafenib) in
(CRC)	BRAF VOOUE alteration	combination with cetuximab

II. <u>INDICATIONS FOR USE</u>

FoundationOne Liquid CDx is a qualitative next generation sequencing based *in vitro* diagnostic test that uses targeted high throughput hybridization-based capture technology to detect and report substitutions, insertions and deletions (indels) in 311 genes, rearrangements in eight (8) genes, and copy number alterations in three (3) genes. FoundationOne Liquid CDx utilizes circulating cell-free DNA (cfDNA) isolated from

plasma derived from anti-coagulated peripheral whole blood of cancer patients collected in FoundationOne Liquid CDx cfDNA blood collection tubes included in the FoundationOne Liquid CDx Blood Sample Collection Kit. The test is intended to be used as a companion diagnostic to identify patients who may benefit from treatment with the targeted therapies listed in Table 1 in accordance with the approved therapeutic product labeling.

Table 1: Companion diagnostic indications

Tumor Type	Biomarker(s) Detected	Therapy	
Non-small cell lung	ALK Rearrangements	ALECENSA® (alectinib)	
cancer (NSCLC)	EGFR Exon 19 deletions and	EGFR tyrosine kinase	
	EGFR Exon 21 L858R	inhibitors approved by	
	substitution	FDA*	
	EGFR Exon 20 insertions	EXKIVITY® (mobocertinib)	
	MET single nucleotide variants	TABRECTA® (capmatinib)	
	(SNVs) and indels that lead to		
	MET exon 14 skipping		
	ROS1 fusions**	ROZLYTREK®	
		(entrectinib)	
Prostate cancer	BRCA1, BRCA2, and ATM	LYNPARZA® (olaparib)	
	alterations		
	BRCA1, BRCA2 alterations	RUBRACA® (rucaparib)	
Breast Cancer	PIK3CA mutations C420R,	PIQRAY® (alpelisib)	
	E542K, E545A, E545D [1635G>T		
	only], E545G, E545K, Q546E,		
	Q546R, H1047L, H1047R, and		
	H1047Y		
Solid Tumors	NTRK1/2/3 fusions**	ROZLYTREK®	
		(entrectinib)	
Colorectal cancer	BRAF V600E alteration	BRAFTOVI ® (encorafenib)	
(CRC)		in combination with	
		cetuximab	

^{*}For the most current information about the therapeutic products in this group, go to: https://www.fda.gov/medical-devices/in-vitro-diagnostics/list-cleared-or-approved-companion-diagnostic-devices-in-vitro-and-imaging-tools#Group Labeling

Additionally, FoundationOne Liquid CDx is intended to provide tumor mutation profiling to be used by qualified health care professionals in accordance with professional guidelines in oncology for patients with solid malignant neoplasms.

A negative result from a plasma specimen does not mean that the patient's tumor is negative for genomic findings. Patients who are negative for the mutations listed in Table 1 (see **Note for NTRK1/2/3 and ROSI fusions) should be reflexed to routine biopsy and

their tumor mutation status confirmed using an FDA-approved tumor tissue test, if feasible.

**Note: when considering eligibility for ROZLYTREK® based on the detection of *NTRK1/2/3* and *ROS1* fusions, testing using plasma specimens is only appropriate for patients for whom tumor tissue is not available for testing.

Genomic findings other than those listed in Table 1 of the intended use statement are not prescriptive or conclusive for labeled use of any specific therapeutic product.

FoundationOne Liquid CDx is a single-site assay performed at Foundation Medicine, Inc. in Cambridge, MA.

III. <u>CONTRAINDICATIONS</u>

There are no known contraindications.

IV. WARNINGS AND PRECAUTIONS

The warnings and precautions can be found in the FoundationOne Liquid CDx labeling.

V. <u>DEVICE DESCRIPTION</u>

The FoundationOne Liquid CDx (F1LCDx) assay is performed exclusively as a laboratory service using circulating cell-free DNA (cfDNA) isolated from plasma derived from anti-coagulated peripheral whole blood from patients with solid malignant neoplasms. The assay employs a single DNA extraction method to obtain cfDNA from plasma from whole blood. Extracted cfDNA undergoes whole-genome shotgun library construction and hybridization-based capture of 324 cancer-related genes. All coding exons of 309 genes are targeted; select intronic or non-coding regions are targeted in three genes (refer to Table 2 for the complete list of genes reported by F1LCDx).

Hybrid-capture selected libraries are sequenced with deep coverage using the NovaSeq 6000 platform. Sequence data are processed using a custom analysis pipeline designed to detect genomic alterations, including base substitutions and indels in 311 genes, copy number variants in three genes, and genomic rearrangements in eight genes. A subset of targeted regions in 75 genes is baited for increased sensitivity.

Table 2: Genomic Regions in which Variants are Reported by F1LCDx¹

ABL1 [Exons 4-9]	ACVR1B	AKT1 [Exon 3]	AKT2		ALK [Exons 20-29, Introns 18,19]		AMER1 (FAM123B)	APC	AR
ARAF [Exons 4, 5, 7, 11, 13, 15, 16]	ARFRP1	ARID1A	ASXL1	ATM	ATR	ATRX	AURKA	AURKB	AXINI
AXL	BAP1	BARD1	BCL2	BCL2L1	BCL2L2	BCL6	BCOR	BCORL1	BCR* [Introns 8, 13,

									14]
BRAF [Exons 11- 18, Introns 7-10]	BRCA1 [Introns 2, 7, 8, 12, 16, 19, 20]	BRCA2 [Intron 2]	BRD4	BRIP1	BTG1	BTG2	BTK [Exons 2, 15]	C11orf30 (EMSY)	C17orf39 (GID4)
CALR	CARD11	CASP8	CBFB	CBL	CCND1	CCND2	CCND3	CCNE1	CD22
CD70	CD74* [Introns 6-8]	CD79A	CD79B	CD274 (PD-L1)	CDC73	CDH1	CDK12	CDK4	CDK6
CDK8	CDKN1A	CDKN1B	CDKN2A	CDKN2B	CDKN2C	CEBPA	СНЕКІ	CHEK2	CIC
CREBBP	CRKL	CSF1R	CSF3R	CTCF	CTNNA1	CTNNB1 [Exon 3]	CUL3	CUL4A	CXCR4
CYP17A1	DAXX	DDR1	DDR2 [Exons 5, 17, 18]	DIS3	DNMT3A	DOTIL	EED	EGFR [Introns 7, 15, 24-27]	EP300
ЕРНА3	ЕРНВ1	ЕРНВ4	ERBB2	ERBB3 [Exons 3, 6,7,8, 10, 12,20, 21, 23,24, 25]	ERBB4	ERCC4	ERG	ERRF11	ESR1 [Exons 4-8]
ETV4* [Intron 8]	ETV5* [Introns 6,7]	ETV6* [Introns 5,6]	EWSR1* [Introns 7-13]	EZH2 [Exons 4,16, 17, 18]	EZR* [Introns 9 - 11]	FAM46C	FANCA	FANCC	FANCG
FANCL	FAS	FBXW7	FGF10	FGF12	FGF14	FGF19	FGF23	FGF3	FGF4
FGF6	FGFR1 [Introns 1,5, Intron17]	FGFR2 [Intron 1, Intron 17]	FGFR3 [Exons 7, 9 (alternative designation exon 10),14, 18, Intron 17]	FGFR4	FH	FLCN	FLTI	FLT3 [Exons 14, 15, 20]	FOXL2
FUBP1	GABRA6	GATA3	GATA4	GATA6	GNA11 [Exons 4, 5]	GNA13	GNAQ [Exons 4, 5]	GNAS [Exons 1, 8]	GRM3
GSK3B	H3F3A	HDAC1	HGF	HNF1A	HRAS [Exons 2, 3]	HSD3B1	ID3	IDH1 [Exon 4]	IDH2 [Exon 4]
IGF1R	IKBKE	IKZF1	INPP4B	IRF2	IRF4	IRS2	JAK1	JAK2 [Exon 14]	JAK3 [Exons 5, 11, 12, 13, 15, 16]
JUN	KDM5A	KDM5C	KDM6A	KDR	KEAP1	KEL	KIT [Exons 8, 9, 11, 12, 13, 17, Intron 16]	KLHL6	KMT2A (MLL) [Introns 6, 8- 11, Intron 7]
KMT2D (MLL2)	KRAS	LTK	LYN	MAF	MAP2K1 (MEK1) [Exons 2, 3]	MAP2K2 (MEK2) [Exons 2-4, 6, 7]	MAP2K4	MAP3K1	MAP3K13
MAPK1	MCL1	MDM2	MDM4	MED12	MEF2B	MEN1	MERTK	MET	MITF
MKNK1	MLH1	MPL [Exon 10]	MRE11A	MSH2 [Intron 5]	MSH3	MSH6	MSTIR	MTAP	MTOR [Exons 19, 30, 39 40, 43-45, 47, 48, 53, 56]
MUTYH	MYB* [Intron 14]	MYC [Intron 1]	MYCL (MYCL1)	MYCN	MYD88 [Exon 4]	NBN	NF1	NF2	NFE2L2
NFKBIA	NKX2-1 (TTF-1)	NOTCH1	NOTCH2 [Intron 26]	NOTCH3	NPM1 [Exons 4-6, 8, 10]	NRAS [Exons 2, 3]	NSD3 (WHSC1L1)	NT5C2	NTRK1 [Exons 14,15, Introns 8-11]
NTRK2 [Intron 12]	NTRK3 [Exons 16,	NUTM1* [Intron 1]	P2RY8	PALB2	PARK2	PARP1	PARP2	PARP3	PAX5

	17]								
PBRM1	PDCD1 (PD- 1)	PDCD1L G2 (PD-L2)	PDGFRA [Exons 12, 18, Introns 7, 9, 11]	PDGFRB [Exons 12- 21, 23]	PDK1	PIK3C2B	PIK3C2G	PIK3CA [Exons 2, 3, 5-8, 10, 14, 19, 21 (Coding Exons 1, 2, 4-7, 9, 13, 18, 20)]	PIK3CB
PIK3R1	PIM1	PMS2	POLD1	POLE	PPARG	PPP2R1A	PPP2R2A	PRDM1	PRKAR1A
PRKCI	PTCH1	PTEN	PTPN11	PTPRO	QKI	RAC1	RAD21	RAD51	RAD51B
RAD51C	RAD51D	RAD52	RAD54L	RAF1 [Exons 3, 4, 6, 7, 10, 14, 15, 17, Introns 4-8]	RARA [Intron 2]	RB1	RBM10	REL	RET [Introns 7, 8, Exons 11, 13-16, Introns 9-11]
RICTOR	RNF43	ROS1 [Exons 31, 36-38, 40, Introns 31-35]	RPTOR	RSPO2* [Intron 1]	SDC4* [Intron 2]	SDHA	SDHB	SDHC	SDHD
SETD2	SF3B1	SGK1	SLC34A2* [Intron 4]	SMAD2	SMAD4	SMARCA4	SMARCB1	SMO	SNCAIP
SOCS1	SOX2	SOX9	SPEN	SPOP	SRC	STAG2	STAT3	STK11 (LKB1)	SUFU
SYK	TBX3	TEK	TERC* {ncRNA}	TERT* {Promoter}	TET2	TGFBR2	TIPARP	TMPRSS2* [Introns 1-3]	TNFAIP3
TNFRSF14	TP53	TSC1	TSC2	TYRO3	U2AF1	VEGFA	VHL	WHSC1	WTI
XPO1	XRCC2	ZNF217	ZNF703						

¹As part of its FDA-approved intended use, the F1LCDx assay interrogates 324 genes, including 309 genes with complete exonic (coding) coverage and 15 genes with only select non-coding coverage (indicated with an *).

The reporting of rearrangements and copy number alterations are restricted to those genes included in Table 3, below.

Table 3: Genes for which copy number alterations and rearrangements are reported for tumor profiling by F1LCDx

Alteration Type	Genes			
Copy Number Alterations	BRCA1, BRCA2, ERBB2			
Rearrangements	ALK, BRCA1, BRCA2, NTRK1, NTRK2, NTRK3			

The test report includes variants reported in the following levels:

Level 1: Companion Diagnostics (CDx)

Clinical evidence should be presented from a prospectively designed clinical trial. Results can also be presented from a retrospective clinical bridging study demonstrating that the clinical endpoints are preserved using plasma samples in trials where enrollment was based on tissue test results. For follow-on markers, a clinical concordance study demonstrating non-inferiority to the original FDA-approved cfDNA-based companion diagnostic device (refer to Li, Meijuan. Statistical Methods for Clinical Validation of Follow-On Companion Diagnostic Devices via an External Concordance Study. Statistics in Biopharmaceutical Research. 8: 35-363, 2016) is required. In addition to the clinical

validation, analytical validation for each specific Level 1 CDx biomarker should be presented.

Level 2: cfDNA Biomarkers with Strong Evidence of Clinical Significance in cfDNA For a Level 2 claim of cfDNA biomarkers with strong evidence of clinical significance, clinical validation needs to be from evidence presented with FDA-approved liquid biopsy companion diagnostic biomarkers for the specific tumor type at the biomarker or variant level. Such claims should also be supported by analytical performance for each biomarker from at least limit of detection (LoD), precision/reproducibility, and accuracy studies.

Level 3A: Biomarkers with Evidence of Clinical Significance in Tissue Supported by Strong Analytical Validation Using cfDNA and Concordance Between cfDNA and Tissue.

Clinical evidence can be provided from tissue-based companion diagnostics. This should also be supported by analytical validation (LoD, precision, analytical accuracy, and concordance study to a tissue-based test) for the specific tumor type at the biomarker or variant level, using a representative approach for SNVs and indels. Evidence evaluating concordance between cfDNA- and tissue-samples for FDA-approved tissue markers should be demonstrated using an FDA-approved tissue test or a validated tissue test.

Level 3B: Biomarkers with Evidence of Clinical Significance in Tissue Supported by Analytical Validation Using cfDNA

Clinical evidence can be provided from tissue-based companion diagnostics, with analytical validation supported by a representative approach for SNVs and indels from key analytical studies (such as LoD, accuracy, and precision).

Level 4: Other Biomarkers with Potential Clinical Significance

Biomarkers not categorized into Levels 1, 2, or 3 can be included under Level 4 for informational purposes or to be used to direct patients toward clinical trials for which they may be eligible. Such claims can be supported by clinical rationale for inclusion in the panel. Such rationale could also include peer-reviewed publications for genes/variants in tissue, variant information from well curated public databases, or *in vitro* preclinical models. Analytical validation should be supported by a representative approach for SNVs and indels from key analytical studies (such as LoD, accuracy, and precision).

FoundationOne® Liquid CDx cfDNA Blood Specimen Collection Kit Contents The test includes a blood specimen collection kit, which is sent to ordering laboratories. The shipping kit contains the following components:

- Specimen preparation and shipping instructions
- Two FoundationOne® Liquid CDx cfDNA Blood Collection Tubes (8.5 mL nominal fill volume per tube)
- Return shipping label

Instruments

The F1LCDx assay is intended to be performed with the serial number-controlled instruments indicated in Table 4, below. All instruments are qualified by Foundation Medicine, Inc. (Foundation Medicine or FMI) under Foundation Medicine's Quality System.

Table 4: Instruments for use with the F1LCDx assay

Table 4: Their uniones for use with the FILEDX assay					
Instrument					
Illumina NovaSeq 6000					
Thermo Scientific Kingfisher Flex DW 96					
Hamilton STARlet STAR Liquid Handling Workstation					

Test Process

All assay reagents including blood collection tubes included in the F1LCDx assay process are qualified by Foundation Medicine and are compliant with the medical device Quality System Regulation (QSR).

A. Specimen Collection and Preparation

Whole blood specimens are collected in F1LCDx cfDNA Blood Collection Tubes (BCT) provided as a component of the F1LCDx specimen collection kit. Prior to cfDNA isolation, the plasma is collected from whole blood by centrifugation, which separates the plasma from the buffy coat (white blood cells) and red blood cells. The plasma layer is removed from the buffy coat to avoid contamination of cellular DNA into the plasma sample. A residual volume of plasma remains in the tube to avoid disturbing the buffy coat. A second spin of the separated plasma at high-speed further pellets cell debris and protein.

B. DNA Extraction

Following the separation of plasma from whole blood, cfDNA is isolated from plasma using the KingFisher Flex Magnetic Particle Processor, which uses an efficient and automated method to purify cfDNA. The KingFisher Instrument uses magnetic rods to move nucleic acid through purification phases of binding, washing, and elution to yield high purity cfDNA. After isolating cfDNA, the Agilent 4200 TapeStation is used to quantify cfDNA.

C. Library Construction

Library Construction (LC) begins with the normalization of cfDNA. The samples are purified, using AMPure XP Beads (Agencourt). Solid-phase reversible immobilization (SPRI) purification is used subsequent to library construction with the NEBNext kits (NEB), including mixes for end repair with blunt-end and 5'- phosphorylate the cfDNA fragments using T4 Polynucleotide Kinase and T4 DNA Polymerase. This step prepares the 3'-end for dA-addition while also preparing the 5'-end of the DNA fragment for ligation. Second, dA-addition will incorporate a single dAMP to the 3'-end of the End-Repaired material. After dA-addition, a universal Y-adaptor is ligated

onto each end of the DNA fragment using a DNA ligase. These steps are performed in 96-well plates (Eppendorf) on a liquid handling workstation (Hamilton STAR) using the "with-bead" protocol to maximize reproducibility and library yield. Dual-indexed (Foundation Medicine customized six base pair barcodes) sequencing libraries are PCR amplified with a high-fidelity DNA polymerase (HiFiTM, Kapa) for ten cycles, SPRI purified and quantified by PicoGreen fluorescence assay (Invitrogen). Process matched control (PMC) is prepared and added to the plate with other cfDNA samples at the beginning of LC.

On May 25, 2022, F1LCDx was approved for a change in the design of the primers used for LC. Specifically, a change was made to the regions of the primer that hybridize to the universal Y-adaptors during the PCR amplification process, which resulted in increased amplification efficiency and reduced the minimum recommended cfDNA input level to 20ng.

D. Hybrid Capture

Hybrid Capture (HC) begins with the normalization of each library from 500 ng to 2000 ng. Solution hybridization is performed using a >50-fold molar excess of a pool of individually synthesized 5'-biotinylated DNA 120 base pair oligonucleotides (Integrated DNA Technology) for baits. The baits target regions from 324 cancer- related genes including all coding exons of 309 genes and only select introns or non- coding regions in 15 genes. Baits were designed by appointing overlapping 120 bp DNA sequence intervals covering target exons (60 bp overlap) and introns (20 bp overlap), with a minimum of three baits per target; single nucleotide polymorphism (SNP) targets were allocated one bait each. Intronic baits were filtered for repetitive elements as defined by the University of California at Santa Cruz (UCSC) Genome Repeat Masker track. Hybrid selection of targets demonstrating reproducibly low coverage was boosted by increasing the number of baits for these targets.

Upon completion of the pre-capture normalization, blocking DNA (adaptor block, Cot, Salmon Sperm DNA) is added to the sequencing library and the mixture is lyophilized in a 96-well plate. The library is then re-suspended in nuclease-free water, heat denatured at 95°C for 5 minutes, temperature ramps from 95°C to 68°C to anneal blocking DNA, and then the samples are incubated at 68°C for a minimum of 5 minutes before the addition of the bait set reagent. After a 20-24-hour incubation, the library-bait duplexes are captured on paramagnetic MyOneTM streptavidin beads (Invitrogen) and off-target library is removed by washing one time with Saline Sodium Citrate (SSC) at 25°C and four times with SSC at 55°C. The PCR master mix is added to directly amplify the captured library from the washed beads. After amplification, the samples are SPRI purified and quantified by PicoGreen.

E. Sequencing

Sequencing on the Illumina NovaSeq 6000 platform employs on-board cluster generation (OBCG) using patterned flow cell (FC) technology to generate monoclonal clusters via ExAmp from a single DNA template. The clusters are then sequenced using sequencing by synthesis (SBS) chemistry. The NovaSeq system is capable of sequencing up to two flow cells at a time. During OBCG, a single DNA template is introduced into each of the primer substrate layered nanowells of the flow cell, where the template is immediately and rapidly amplified by ExAmp. This rapid amplification prevents other DNA templates from binding, ensuring a monoclonal cluster is formed in each nanowell. The procedure allows for fixed size and spacing of the clusters which results in improved and more accurate resolution.

A growing nucleotide chain is created on the flow cell by incorporating fluorescently labeled, 3'-blocked deoxynucleoside triphosphates (dNTPs). After excitation by a laser, the camera captures the emission color of the incorporated, fluorescently labeled nucleotide. The 3'-block is then removed, reverting the nucleotide to its natural form, which allows the polymerase to add another base to the growing double strand of DNA. With each successive SBS cycle, a new fluorescently labeled 3'- blocked dNTP is added. SBS allows for millions of discrete clusters of clonal copies of DNA to be sequenced in parallel.

F. Sequence Analysis

Sequence data are analyzed using mainly proprietary software developed by Foundation Medicine. External tools used include: 1) BWA (Burrows-Wheeler Aligner) v0.7.17, for aligning sequence reads to the genomic reference, 2) SAMtools v1.6 for utility operations, 3) Picard tools v1.56 for metrics calculations, and 4) Biopython for the pairwise2 sequence alignment module.

Reads from each Illumina flow cell are demultiplexed (sorted into sets of reads deriving from distinct samples), and their fragment barcodes (FBCs) are extracted and encoded into the read names. For each sample, read pairs with matching, valid FBCs are aligned and processed together to: 1) identify clusters of reads originating from the same original fragment; 2) merge overlapping read pairs into single reads, where possible; and 3) generate consensus reads representing all information in the set of reads for each cluster, encoding positions with mismatches (errors) with base quality 20. The consensus reads are then aligned to the reference genome to generate the 'consensus' binary alignment map (BAM).

For the detection of short variants (e.g., substitutions and small indels) in each target region of interest, a de novo assembly is performed. This is done using proprietary software to generate a de Bruijn graph including all k-mers in reads mapping to a particular locus. The graph is parsed to identify paths that

originate and terminate in reference nodes from the locus. Increased k-mer sizes may be used to account for ambiguities, cycles, and other problematic regions within the graph. The result of the graph traversal is a set of candidate variants. For each variant, there is a set of k-mers supporting the variant and a set of k-mers that would support the reference or another variant at the location.

Each candidate variant is then scanned against reads in the locus to identify which reads support either the candidate variant or a different variant or reference at the location. The cluster membership of the supporting reads is then assessed to determine which clusters show unambiguous support for the variant and which have conflicting assignments, indicating that the variant may have arisen as an error in sequencing or library preparation. The final variant calls are made based on a model that takes into account the coverage at the location, the number of supporting read clusters and their redundancy level, and the number of error-containing clusters.

G. Report Generation

Approved results are annotated by automated software with CDx relevant information and are merged with patient demographic information and any additional information provided by Foundation Medicine as a professional service prior to approval and release by the laboratory director or designee.

H. Internal Process Controls

Process Control

Each assay run includes a control sample run in duplicate. The control sample contains a pool of eleven HapMap cell lines and is used as a positive mutation detection control. Hundred different germline SNPs present across the entire targeted region are required to be detected by the analysis pipeline.

Sensitivity Control

The HapMap control pool used as the positive control is prepared to contain variants at 0.1%, 10% mutant allele frequency (MAF) which must be detected by the analysis pipeline to ensure expected sensitivity for each run.

Negative Control

Samples are barcoded molecularly at the library construction (LC) stage. Only reads with a perfect molecular barcode sequence are incorporated into the analysis. The Analysis Pipeline includes an algorithm that analyzes the SNP profile of each specimen to identify potential contamination that may have occurred prior to molecular barcoding.

I. Classification Criteria for CDx Biomarkers Detected by F1LCDx

1. BRCA1 and BRCA2 alterations to identify patients eligible for rucaparib in prostate cancer:

The CDx classification criteria and the list of *BRCA1/BRCA2* missense mutations for rucaparib, based on the trial prespecifications are described in Table 5 and Table 6; however, not all the missense mutations listed below were observed in the TRITON2, and PROfound clinical studies.

Table 5: Classification Criteria for Deleterious Tumor BRCA Variants

Qualification Criteria	Sequence Classification	Methodology
A BRCA1 or BRCA2	Protein truncating mutations	Sequence analysis identifies premature
alteration that includes		stop codons anywhere in the gene
any of the sequence		coding region, except, 3' of and
classifications		including BRCA2 K3326*
	Splice site mutations	Sequence analysis identifies variant
		splice sequences at intron/exon
		junctions -/+ 2bp of exon starts/ends
	Homozygous deletions	Sequence analysis identifies deletions
		in both gene alleles of ≥1 exon in size
	Large protein truncating	Sequence analysis identifies protein
	rearrangements	truncating rearrangements
	Deleterious missense mutations	Curated list

Table 6: Deleterious BRCA Missense Alterations in rucaparib

Table 6. Deleterious BKCA Wilssense Afterations in Tucapario								
RRC	741 Alter	ations (Pr	BRCA2 Alterations (Protein					
BRCA1 Alterations (Protein Change)				Change)				
M1V	C44Y	R71T	R1699W	G1770V	M1V	R2336P	T2722R	
M1T	C44F	R71M	R1699Q	M1775K	M1T	R2336L	D2723H	
M1R	C47S	S770L	G1706R	M1775R	M1R	R2336H	D2723G	
M1I	C47Y	R1495T	G1706E	C1787S	M1I	T2412I	G2724W	
M18T	C47F	R1495M	A1708E	G1788V	D23N	R2602T	G2748D	
L22S	C61S	R1495K	S1715R	P1812A	D23Y	W2626C	A2911E	
I26N	C61G	E1559K	S1722F	A1823T	S142N	I2627F	E3002K	
T37K	C61Y	E1559Q	V1736A	V1833M	S142I	R2659T	R3052W	
C39R	C64R	T1685A	G1738R	W1837R	V159M	R2659K	D3095G	
C39G	C64G	T1685I	G1738E	V1838E	V211I	E2663V	D3095E	
C39Y	C64Y	D1692N	K1759N		V211L	S2670L	N3124I	
C39W	C64W	M1689R	L1764P		Y600C	I2675V	N3187K	
H41R	R71G	D1692H	I1766N		K1530N	T2722K		
C44S	R71K	D1692Y	I1766S					

2. ATM, BRCA1 and BRCA2 alterations to identify patients eligible for olaparib in mCRPC:

Table 7: Rules Applied to the Aforementioned Genes:

	ned to the Aloremo	cittofica defies.	
Qualification Criteria	Sequence Classification	Methodology	Comments
A gene alteration that includes any of	Protein truncating mutations	Sequence analysis identifies premature stop codons	Does not include VUS.
the sequence classifications		anywhere in the gene coding region, except 3' of and	Includes mutations on the canonical transcript only
Classifications		including BRCA2 K3326*	for genes ATM, BRCA1, and BRCA2.
	Splice site mutations	Sequence analysis identifies variant splice sequences at intron/exon junctions -/+ 2bp of exon starts/ends	Does not include VUS. Includes indels that extend through ±2bp from the intron/exon junction. Includes mutations on the canonical transcript only for genes <i>ATM</i> , <i>BRCA1</i> , and <i>BRCA2</i> .
	Homozygous deletions	Sequence analysis identifies deletions in both gene alleles of ≥1 exon in size	Does not include VUS Only reported for <i>BRCA1</i> and <i>BRCA2</i> . Not reported for <i>ATM</i> .
	Large protein truncating rearrangements	Sequence analysis identifies protein truncating rearrangements	Does not include VUS
	Deleterious missense mutations	Curated list	Protein effects from list of missense mutations on the canonical transcript only for genes <i>ATM</i> , <i>BRCA1</i> , and <i>BRCA2</i> .

Alterations reported are limited to those within the alteration-calling capabilities of FMI as of March 2, 2020. ATM missense mutations were identified from the ClinVar database. Should the calling capabilities expand, additional alterations that meet the above criteria may also be reported, per FDA approval.

Table 8. List of Deleterious Missense Mutations by Protein Effect, Implemented on the Respective Canonical Transcript.

BRCA1		BRCA2		ATM		
Protein Effect (PE)	FMI Annotated PE	Protein Effect (PE)	FMI Annotated PE	Protein Effect (PE)	FMI Annotated PE	
MIV	MIV	MIR	MIR	MIT	MIT	
MlI	MlI	MlI	MlI	R2032K	R2032K	
C6IG	C6IG	VI59M	VI59M	R2227C	R2227C	
C64Y	C64Y	V211L	V211L	R2547 S2549del	R2547 S2549del	
R7IG	R7IG	V211I	V211I	G2765S	G2765S	
R7IK	R7IK	R2336P	R2336P	R2832C	R2832C	
RI495M	RI495M	R2336H	R2336H	S2855	S2855	

BRCA1		BRCA2		ATM		
Protein Effect (PE)	FMI Annotated PE	Protein Effect (PE)	FMI Annotated PE	Protein Effect (PE)	FMI Annotated PE	
				V2856delinsR1	V2856delinsRl	
					S2855 V2856>R1	
EI559K	EI559K			R3008C	R3008C	
DI692N	DI692N			R3008H	R3008H	
DI692H	DI692H			[VUS from Jan 20	016 HRR* List to be	
D1092П	D1092П			Excluded]		
RI699W	RI699W			V2424G	V2424G	
AI708E	AI708E			[Excluded from Jan 2016 HRR List]		
G1788V	GI788V			K750K	splice site 2250G>A	

HRR = Homologous Recombination Repair genes

Intronic Variants

Gene	Chr	Position	Ref	Alt	dbSNP	FMI Protein Effect
ATM	chr11	108128198	T	G	rs730881346	[Variant Not Called by FMI]
ATM	chr11	108214102	AGTGA	A	rs730881295	splice site 8418+5_8418+8delGTGA
						or
						splice site 8418+1 8418+4delGTGA

- 3. *CDx classification criteria for EGFR alterations:*
 - Base substitutions resulting in *EGFR* L858R
 - In-frame deletions occurring within *EGFR* Exon 19
- 4. ALK rearrangements to identify patients eligible for treatment with ALECENSA® (alectinib):

CDx positivity for an *ALK* rearrangement is based on the following variant classification criteria:

- The *ALK* rearrangement must have pathogenic driver status (FMI driver status of "known" or "likely")
- AND the disease type must be NSCLC
- AND one of the following two conditions must hold:
 - 1. The partner gene is *EML4*, or
 - 2. The ALK breakpoint occurs within ALK intron 19
- 5. SNVs and indels that lead to MET exon 14 skipping to identify patients eligible for treatment with TABRECTA® (capmatinib):

A SNV or indel in *MET* shall be considered to result in skipping of exon 14 if one or more of the following criteria are met:

- 1. Deletions greater than or equal to 5 bp that affect positions -3 to -30 in the intronic region immediately adjacent to the splice acceptor site at the 5' boundary of *MET* exon 14.
- 2. Indels affecting positions -1 or -2 at the splice acceptor site of the 5' boundary of *MET* exon 14.

- 3. Base substitutions and indels affecting positions 0, +1, +2, or +3 at the splice donor site of the 3' boundary of MET exon 14.
- 6. Biomarker Rules for Rearrangements that Lead to NTRK1, NTRK2, or NTRK3 Fusions:

Rearrangements in *NTRK1*, *NTRK2*, or *NTRK3* shall be considered CDx biomarker positive, that is, to lead to a *NTRK1*, *NTRK2*, or *NTRK3* RNA fusion, if the following criterion is met:

• In-strand rearrangement events that may lead to an *NTRK1*, *NTRK2* or *NTRK3* RNA fusion with a previously reported or novel partner gene in which the kinase domain is not disrupted. This also includes rearrangement events that result in reciprocal fusions (*NTRK* may be on either the 5' or the 3' end of the detected fusion).

In this regard out-of-strand events are considered as non-fusion rearrangements and are classified as CDx biomarker negative. Intragenic fusions in which genomic rearrangement events are wholly internal to the *NTRK1*, *NTRK2*, or *NTRK3* genes (i.e., *NTRK1-NTRK1*, *NTRK2-NTRK3*, *NTRK3-NTRK3* events) are also considered biomarker negative. Unidentified partners (encoded as N/A) or LINC non-coding partners are also considered CDx biomarker negative.

- 7. Biomarker Rules for Rearrangements that Lead to ROS1 Fusions: Rearrangements in ROS1 shall be considered CDx biomarker positive, i.e., to lead to ROS1 RNA fusion, if the following condition is met:
 - In-strand rearrangement events that may lead to a *ROS1* RNA fusion with another protein coding gene in which the *ROS1* kinase domain is not disrupted. *ROS1* must be on the 3' end of the detected fusion.

In this regard, out-of-strand events are considered as non-fusion rearrangements and are classified as CDx biomarker negative. Intragenic fusions in which genomic rearrangement events are wholly internal to the *ROS1* (i.e., *ROS1-ROS1* events) are also considered biomarker negative. Unidentified partners (encoded as N/A) or LINC non-coding partners are also considered CDx biomarker negative. *ROS1* fusions with novel partners are required to be in frame.

- 8. EGFR exon 20 insertions to identify NSCLC patients eligible for treatment with EXKIVITY:
 - CDx positivity for *EGFR* exon 20 insertions is determined if the following criteria were met:
 - Any in-frame insertions affecting amino acids 762 775 (inclusive) in EGFR exon 20

9. BRAF V600E alteration to identify metastatic CRC patients eligible for treatment with BRAFTOVI (encorafenib) in combination with cetuximab:

CDx classification criteria for BRAF alterations:

• Base alterations resulting in *BRAF* V600E

VI. ALTERNATIVE PRACTICES AND PROCEDURES

There are no FDA-approved CDx alternatives using cfDNA isolated from plasma for the detection of *BRAF* V600E alterations to identify patients with mCRC eligible for treatment with BRAFTOVI (encorafenib) in combination with cetuximab.

However, *therascreen* BRAF V600E RGQ PCR Kit (QIAGEN GmbH) is an FDA-approved CDx for the detection of *BRAF* V600E mutation in mCRC to identify patients eligible for BRAFTOVI (encorafenib) in combination with cetuximab with formalin-fixed, paraffin-embedded (FFPE) tissue specimens (P190026).

There are FDA-approved alternatives for the detection of select CDx and tumor profiling genetic alterations using either cfDNA isolated from plasma samples or FFPE tissue specimens. For additional details see FDA List of Cleared or Approved Companion Diagnostic Devices at: https://www.fda.gov/media/119249/download. Each alternative has its own advantages and disadvantages. A patient should fully discuss these alternatives with his/her physician to select the method that best meets expectations and lifestyle.

VII. MARKETING HISTORY

The F1LCDx assay was FDA-approved on August 26, 2020, and subsequently commercialized in the United States. The F1LCDx assay has been marketed in the United States, the European Union, and in several other foreign countries since the approval. On September 21, 2022, the companion diagnostic indication for F1LCDx to identify patients with ovarian cancer harboring *BRCA1* or *BRCA2* alterations for treatment with RUBRACA® (rucaparib) was removed. The approved PMA supplements that affected the intended use are listed in Table 9.

Table 9. Marketing History

Submission No.	Date of Approval	Biomarker/Update	Indication	Drug
October 26,		ALK Rearrangements	NSCLC	ALECENSA® (alectinib)
P200006	2020	PIK3CA alterations	Breast Cancer	PIQRAY® (alpelisib)
P200016	November 6, 2020	BRCA1, BRCA2, and ATM alterations	Prostate Cancer	LYNPARZA® (olaparib)
P190032/S001	July 15, 2021	MET single nucleotide variants (SNVs) and indels that lead to MET exon 14 skipping	NSCLC	TABRECTA® (capmatinib)

Submission No.	Date of Approval	Biomarker/Update	Indication	Drug
P190032/S004	December 22,	NTRK1/2/3 fusions	Solid Tumors	ROZLYTREK [®]
P190032/8004	2022	ROS1 fusions	NSCLC	(entrectinib)
P190032/S008	December 19, 2022	EGFR Exon 19 deletions and EGFR Exon 21 L858R alteration	NSCLC	EGFR tyrosine kinase inhibitors approved by FDA
P190032/S005	May 3, 2023	EGFR Exon 20 insertions	NSCLC	EXKIVITY® (mobocertinib)

VIII. POTENTIAL ADVERSE EFFECTS OF THE DEVICE ON HEALTH

Failure of the device to perform as expected or failure to correctly interpret test results may lead to incorrect F1LCDx assay results, and subsequently, inappropriate patient management decisions. Patients with false positive CDx biomarker results may undergo treatment with one of the therapies listed in the intended use statement without clinical benefit and may experience adverse reactions associated with the therapy. Patients with false negative results may not be considered for treatment with the indicated targeted therapy. There is also a risk of delayed results, which may lead to delay of treatment with the indicated therapy. For the specific adverse events related to the approved therapeutics, please see approved drug product labels.

For the specific adverse events that occurred in the clinical study, please see the FDA approved package insert for BRAFTOVI (encorafenib) in combination with cetuximab which is available at Drugs@FDA.

IX. SUMMARY OF NONCLINICAL STUDIES

A. <u>Laboratory Studies</u>

The evidence in support of the analytical performance of F1LCDx in detecting *BRAF* V600E alteration is presented in this section. Analytical accuracy/concordance and precision near the LoD studies were conducted to support the indication for *BRAF* V600E alteration using clinical samples.

For F1LCDx platform-level validation (P190032), due to the challenges with obtaining sufficient volume of clinical specimens, analytical performance characteristics were established for some of the studies using contrived samples, which consisted of enzymatically sheared cell line DNA spiked into human plasma from healthy donors, extracted according to the assay's standard procedure, and diluted with cfDNA isolated from healthy donor plasma. A contrived sample functional characterization (CSFC) study was conducted to demonstrate comparable performance of sheared cell line DNA samples spiked into plasma as compared to cfDNA isolated from plasma specimens obtained from cancer positive intended use patient specimens. These contrived samples were used to establish the LoD for short variants evaluated in the platform-level LoD establishment study. For information regarding the platform-level validation, refer to Section IX.A. in P190032 Summary of Safety and Effectiveness Data.

1. Analytical Accuracy/Concordance

An analytical accuracy/concordance study was performed to demonstrate the concordance between F1LCDx and an externally validated NGS assay (evNGS) for the detection of *BRAF* V600E alteration. For this study, 50 frozen plasma samples were identified from clinical trial assay-positive (CTA+) patients enrolled in the ARRAY-818-302 (BEACON) trial and 153 residual cfDNA samples including 40 biomarker positives and 113 biomarker negatives were sourced from FMI's clinical archives. Of the 153 residual cfDNA samples, seven (7) were excluded due to LC/HC QC failure or evNGS ctDNA QC failure. Of the 50 frozen plasma samples from CTA+ patients, seven (7) were excluded from the analysis due to diluted DNA concentration being out of acceptable range or evNGS ctDNA QC failure.

Analytical concordance of F1LCDx for detecting *BRAF* V600E alteration was determined with 189 samples tested by F1LCDx assay (Table 10). Since archived specimens were selected based on a previous version of the device and confirmed by the evNGS assay, calculation of percent agreement (PPA) and negative percent agreement (NPA) is presented adjusted for the enrichment of BRAF V600E positives in the concordance evaluation sample cohort. Adjusted PPA has a point estimate of 92.31% with a 95% two-sided CI of (78.45%, 100.00%). Adjusted NPA has a point estimate of 100.00% with a 95% two-sided CI of (96.53%, 100.00%). For informational purposes, unadjusted PPA, NPA, PPV (positive predictive value) and NPV (negative predictive value) are also displayed.

Table 10. Concordance summary for BRAF V600E alteration by F1LCDx and the evNGS

THE TOTAL	Table 10. Concordance summary for Divit voode afteration by File Dx and the CVINGS					
		evNGS				
		BRAF V600E alteration positive	BRAF V600E alteration negative	Total	PPV/NPV (Unadjusted) (95% CI¹)	
	BRAF V600E alteration positive	81	0	81	PPV: 100% (95.47%, 100%)	
F1LCDx	BRAF V600E alteration negative	1*	107	108	NPV: 99.07% (94.94%, 99.84%)	
	Total	82	107	189		
	PPA/NPA (Unadjusted) (95% CI¹)	PPA: 98.78% (93.41%, 99.78%)	NPA: 100% (96.53%, 100%)			

¹Calculated with Wilson 2-sided 95% CI

The one (1) sample that was *BRAF* V600E alteration negative by F1LCDx and BRAF V600E alteration positive by the evNGS was discordant due to low supporting reads (4) and variant allele fraction (VAF), 0.0544% VAF in F1LCDx, which did not pass F1LCDx positive calling threshold.

^{*}This sample has very low supporting reads (4) and variant allele fraction (0.000544) in F1LCDx, which did not pass F1LCDx positive calling threshold.

The lowest VAF F1LCDx test result observed in the accuracy study was 1.02% VAF; therefore, accuracy of F1LCDx was not demonstrated for samples with variants below this VAF level. A limitation is included in the device labeling to address the uncertainty of the accuracy of F1LCDx for detecting *BRAF* V600E alteration with VAFs below those evaluated in the analytical concordance study (refer to Section XII.C. below). Although accuracy is not supported by direct comparison to orthogonal method, the clinical trial included specimens below the LoD and efficacy was demonstrated in specimens below the LoD supporting the conclusion that positives below the LoD are true positives for BRAF.

2. Analytical Sensitivity

a. Limit of Blank (LoB)

The LoB of F1LCDx was evaluated in the platform LoB study for PMA P190032 (refer to Section IX.A.3.a. in the Summary of Safety and Effectiveness Data for P190032).

A supplemental LoB study was performed for F1LCDx to support the updated LC input range (20-60ng) by collecting whole blood samples from 44 healthy donors and preparing two plasma cfDNA replicates per donor for a total of 88 cfDNA sample replicates. Additionally, one matched gDNA replicate per donor was isolated from buffy coat and mechanically fragmented for F1LCDx testing to obtain non-tumor variant (e.g., germline) information to support the LoB analysis. One cfDNA replicate was excluded from the analysis due to failure at the DNA extraction step. *BRAF* V600E alterations were not observed in any of the replicates, and therefore, the false positive rate was 0% and the LoB for *BRAF* V600E alteration is zero.

b. Limit of Detection (LoD)

The LoD of *BRAF* V600E alteration was not established as part of this PMA supplement. However, the LoD for single nucleotide variants (SNVs) in contrived samples was established as part of the LoD study for PMA P190032 (refer to Section IX.A.3.b in the Summary of Safety and Effectiveness Data for P190032). In this study, the median LoD for short variants was determined to be 0.40% VAF in the enhanced sensitivity region of the bait set, which includes BRAF, and the LoD for *BRAF* V600E alteration was determined to be 0.33% VAF using the empirical hit rate approach. The LoD was confirmed with clinical samples in the precision study (refer to Section IX.A.3, below). The confirmed LoD was 0.70%.

3. Precision and Reproducibility

a. Within-Laboratory (Intermediate) Precision
A precision study was conducted using two (2) clinical CRC samples harboring
BRAF V600E alteration, however just one sample was close to the LoD for
BRAF V600E (~2x, the second sample was ~16x LoD).

Repeatability including intra-run performance (run on the same plate under the same conditions) and reproducibility including inter-run performance (run on different plates under different conditions) were assessed and compared in duplicates across two (2) different sequencers and three (3) different reagent lots, across multiple days of performance by multiple operators.

The results for the precision study for the clinical sample are summarized in Tables 11 and 12, below.

Table 11. Reproducibility results for *EGFR* exon 20 insertions

BRAF V600E alteration	Previously Established LoD VAF ¹		Concordant /Total (n/N)	Reproducibility (%) 95% CI ²	Fold LoD
1799T>A	0.33%	0.7%	24/24	100% (86.2, 100)	2.12x

¹0.33% VAF represents the VAF evaluated in the platform LoD establishment study using contrived samples (See Section IX.A.2.b)

Table 12. Repeatability results for *EGFR* exon 20 insertions

BRAF V600E alteration	Previously Established LoD VAF ¹		Concordant /Total (n/N)	Repeatability (%) 95%CI ²	Fold LoD
1799T>A	0.33%	0.7%	12/12	100% (75.75, 100)	2.12x

¹0.33% VAF represents the VAF evaluated in the platform LoD establishment study using contrived samples (See Section IX.A.2.b)

B. Animal Studies

No animal studies were conducted using the F1LCDx assay.

C. Additional Studies

None

X. SUMMARY OF PRIMARY CLINICAL STUDIES

The reasonable assurance of safety and effectiveness for F1LCDx for detection of *BRAF* V600E alteration in patients with mCRC who may benefit from treatment with BRAFTOVI (encorafenib) in combination with cetuximab was established through a clinical bridging study using clinical plasma specimens from patients enrolled in the BEACON (ARRAY-818-302) trial, as well as *BRAF* V600E alteration-negative tissue-matched plasma samples from commercial sources. Data from this clinical study were the basis for the PMA supplement approval decision.

A summary of the clinical study is presented below.

²Calculated with Wilson 2-sided 95% CI

²Calculated with Wilson 2-sided 95% CI

A. Study Design

1. BEACON (ARRAY-818-302) Study Design

The BEACON trial is a Phase 3 multicenter, randomized, open-label, 3-arm study designed to evaluate the efficacy and safety of encorafenib + cetuximab + binimetinib (referred to as the Triplet arm) and encorafenib + cetuximab (referred to as the Doublet arm) versus Investigator's choice of either irinotecan/cetuximab or FOLFIRI/cetuximab (Control arm) in patients with *BRAF* V600E mutation mCRC whose disease had progressed after 1 or 2 prior regimens in the metastatic setting. The primary endpoints of the BEACON study were overall survival (OS) (Triplet arm vs. Control arm) and confirmed overall response rate (ORR) by Blinded independent central review (BICR) per Response Evaluation Criteria in Solid Tumors (RECIST) v1.1 (Triplet arm vs. Control arm). The secondary endpoints of the BEACON study were OS and ORR by BICR per RECIST v1.1 of the Doublet Arm compared against the Control Arm. This SSED presents analysis from the Doublet Arm against the Control Arm, in order to align with NDA labelling.

Patients were enrolled into the BEACON trial based on tumor tissue samples using either a local laboratory developed test (LDT), followed by confirmation of the local LDT result via a central clinical trial assay (CTA) test or using the CTA for enrollment directly.

2. Clinical Bridging Study Design

A clinical bridging study was conducted to evaluate: 1) the concordance between the F1LCDx assay and the CTA for the detection of *BRAF* V600E alteration for patient samples from the Doublet and Control Arms, and 2) the clinical validity of F1LCDx in identifying mCRC patients with *BRAF* V600E alteration who may be eligible for treatment with BRAFTOVI (encorafenib) in combination with cetuximab.

Plasma samples from the BEACON trial patients were collected by the therapeutic investigational sites per the study protocol and study documents and shipped to the central testing laboratories. All patients with available plasma samples from the Doublet and Control Arms (i.e., patient population that supported the BRAFTOVI (encorafenib) in combination with cetuximab approval) from the BEACON trial were tested by F1LCDx as part of this clinical bridging study. Additionally, tissue and plasma-matched CTA-negative samples procured from commercial sources were also included in the clinical bridging study.

3. Clinical Inclusion and Exclusion Criteria

The inclusion/exclusion criteria for the retrospective testing of plasma samples in the clinical bridging study were:

Sample inclusion criteria:

• Samples from enrolled patients from Doublet and Control Arms with informed consent provided in the BEACON trial

- Samples from commercial sources must have a valid CTA negative result from tissue
- Specimens in frozen plasma
- Samples must meet F1LCDx operational testing requirements

Sample exclusion criteria:

- Tissue, other liquid samples
- Samples that do not meet F1LCDx operational testing requirements

Specimens included in the clinical bridging study were tested according to the standard testing protocol for the F1LCDx assay test with a minimum recommended cfDNA input of ≥ 20 ng for the library construction step.

4. Follow-up Schedule

The F1LCDx clinical bridging study involved only retrospective testing of plasma samples; as such, no additional patient follow-up was conducted.

5. Clinical Endpoints

This SSED presents analysis from the Doublet Arm against the Control Arm (which was the secondary endpoint), in conjunction with the corresponding NDA (210496) (See Section D. Safety and Effectiveness Results).

The Endpoints were the following:

- OS, defined as the time from randomization to death due to any cause, of Doublet Arm vs. Control Arm
- Confirmed ORR (by BICR) per RECIST, v1.1 of Doublet Arm vs. Control Arm.

B. Accountability of PMA Cohort

A total of 561 samples, including 441 plasma samples from the BEACON clinical trial and 120 commercial BRAF V600E negative plasma samples, were identified for the clinical bridging study. Out of the 561 samples, 37 CTA unevaluable samples (31 from the clinical trial and 6 procured) and one CTA positive procured sample were excluded from the clinical bridging analysis, resulting a total of 523 samples included in the bridging analysis.

Among the 441 samples from the Doublet and Control Arms from the BEACON trial, 8 samples were CTA-negative and were removed from the positive population in the bridging analysis; 31 samples were CTA unevaluable and therefore were excluded from the clinical bridging analysis. Out of the remaining 402 CTA+ clinical trial samples, 42 CTA+ samples did not have a plasma sample available for F1LCDx testing, 32 failed the F1LCDx QC metrics, resulting in a total of 328 *BRAF* V600E-positive samples that had both CTA and F1LCDx evaluable results.


Among the 120 tissue and plasma-matched procured samples, 1 sample was CTA+, and 6 samples were CTA unevaluable, 14 failed the F1LCDx QC metrics, resulting in 99

procured BRAF V600E-negative samples that had F1LCDx evaluable results. Additionally, 6 out of 8 CTA- samples from the clinical trial that had F1LCDx evaluable results were included in the negative population in the analysis.

For the clinical bridging analysis, the CTA+ set contains 402 CTA+ (328 F1LCDx-evaluable; F1LCDx-evaluable refers to samples with valid F1LCDx testing results) clinical trial samples, and the CTA-negative set consists of 121 CTA- samples, where 113 CTA- samples (99 F1LCDx-evaluable) are from procured samples and 8 CTA- (6 F1LCDx-evaluable) samples are from the clinical trial, resulting in a total of 105 (99+6) CTA- and F1LCDx-evaluable samples.

The accountability for all samples reported in this study are detailed in Figure 1 below.

Figure 1. Sample Processing Flow Chart

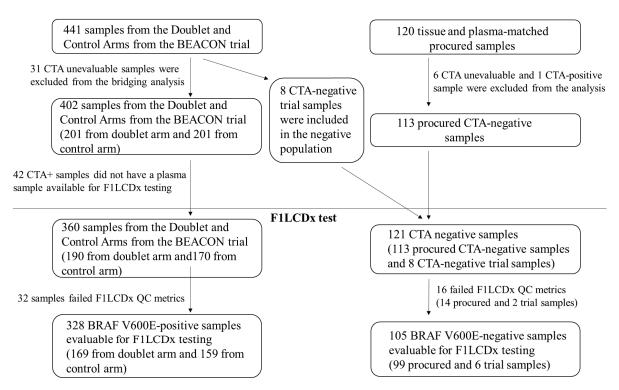

C. Study Population Demographics and Baseline Parameters

The demographics and baseline disease characteristics were similar between F1LCDx-evaluable and F1LCDx-unevaluable, and CTA-positive patients (Table 13).

Table 13. Comparison of baseline demographic and clinical characteristics between the

CDx-evaluable patients and the CDx-unevaluable patients

Baseline characteristic	CTA-positive	F1LCDx- evaluable	F1LCDx- unevaluable	p-value comparing the two subsets*
AGE				0.17
Mean	59.43	59.79	57.82	
Min	27.00	27.00	29.00	
Q1	53.00	53.00	50.25	
Median	61.00	61.00	59.50	
Q3	68.00	68.25	65.75	
Max	91.00	91.00	80.00	
SD		12.00	11.80	
SEX				1
Female	212 (52.74%)	173 (52.74%)	39 (52.7%)	
Male	190 (47.26%)	155 (47.26%)	35 (47.3%)	
ECOG				0.13
0	197 (49.00%)	154 (46.95%)	43 (58.11%)	
1	201 (50.00%)	171 (52.13%)	30 (40.54%)	
2	4 (1.00%)	3 (0.91%)	1 (1.35%)	
LOC TUMOR				
(Location of primary tumor)				0.38
Both Sides	28 (6.97%)	24 (7.32%)	4 (5.41%)	
Left Colon	137 (34.08%)	109 (33.23%)	28 (37.84%)	
Right Colon	213 (52.99%)	178 (54.27%)	35 (47.3%)	
Unknown Colon	24 (5.97%)	17 (5.18%)	7 (9.46%)	
INVOL_ORGAN (Involvement of ≥3 organs)				0.12
No No	217 (53.98%)	171 (52.13%)	46 (62.16%)	0.12
Yes	185 (46.02%)	157 (47.87%)	28 (37.84%)	
LIVER METAS	105 (10.0270)	127 (17.0770)	20 (37.0170)	
(Presence of liver metastases)				0.24
No	170 (42.29%)	134 (40.85%)	36 (48.65%)	7.2
Yes	232 (57.71%)	194 (59.15%)	38 (51.35%)	
PRI_TUMOR (Primary tumor removed)	- ()	(0.36
Completely Resected	228 (56.72%)	190 (57.93%)	38 (51.35%)	0.50
Partially Resected or	228 (30.7270)	190 (37.9370)	36 (31.3370)	
Unresected	174 (43.28%)	138 (42.07%)	36 (48.65%)	
PRE_THERAPY				
(Previous lines of therapy)				0.91
1	264 (65.67%)	216 (65.85%)	48 (64.86%)	
2	137 (34.08%)	111 (33.84%)	26 (35.14%)	
3	1 (0.25%)	1 (0.3%)	0 (0%)	
HIGH_MICRO (High microsatellite instability)				0.23
N/A	76	40	36	
No	295 (90.49%)	263 (91.32%)	32 (84.21%)	

Baseline characteristic	CTA-positive	F1LCDx- evaluable	F1LCDx- unevaluable	p-value comparing the two subsets*
Yes	31 (9.51%)	25 (8.68%)	6 (15.79%)	
BASE_CARCI (Baseline				
carcinoembryonic antigen level				
>5 µg/liter)				0.00
N/A	1	0	1	
No	100 (24.94%)	72 (21.95%)	28 (38.36%)	
Yes	301 (75.06%)	256 (78.05%)	45 (61.64%)	
BASE_CPRO				
(Baseline C-reactive protein				0.69
level >10 mg/liter)				
N/A	6	5	1	
No	244 (61.62%)	197 (60.99%)	47 (64.38%)	
Yes	152 (38.38%)	126 (39.01%)	26 (35.62%)	
IRINOTECAN STATUS				
(Irinotecan status)				0.37
No	196 (48.76%)	156 (47.56%)	40 (54.05%)	
Yes	206 (51.24%)	172 (52.44%)	34 (45.95%)	

^{*}p-value was from nonparametric Mann-Whitney Test for continuous measures, and Fisher-Freeman-Halton Test for categorical measures between the CDx-evaluable and CDx-unevaluable sets

D. Safety and Effectiveness Results

1. Safety Results

The safety with respect to treatment with BRAFTOVI (encorafenib) in combination with cetuximab was addressed during the review of the supplemental NDA and is not addressed in detail in this Summary of Safety and Effectiveness Data. The evaluation of safety was based on the analysis of adverse events (AEs), clinical laboratory evaluations, physical examinations, and vital signs. No adverse events were reported in connection with the bridging study used to support this PMA supplement, as the study was performed retrospectively using banked samples.

Please refer to Drugs@FDA for complete safety information on BRAFTOVI (encorafenib) in combination with cetuximab.

2. Effectiveness Results

a. Concordance Analysis with enrollment CTA

Since patients in the BEACON trial were enrolled based on testing tumor tissue samples using either a local LDT followed by central CTA confirmation or using CTA for enrollment directly, a study was conducted to evaluate the concordance between the LDT and CTA results. Data supported that prescreened LDT and CTA results are highly concordant. Therefore, CTA+ population and CTA+LDT+ double positive population are consistent and the impact of the prescreening with the LDT is negligible. Direct bridging from the CTA to the F1LCDx ignoring the pre-screening by LDT was performed in this clinical study

to support the intended use of the F1LCDx. As described above, a total of 523 samples (402 CTA+ samples from the clinical trial and 121 CTA- procured or clinical trial samples) were included in the clinical bridging study. Concordance of the F1LCDx assay with the enrolling CTA was demonstrated with the CDx-evaluable population (328 CTA+ and 105 CTA- samples) and summarized in Table 14, below.

Table 14. Concordance for *BRAF* V600E mutation between F1LCDx in

plasma and the CTA in tissue

Diamina and the C111 in tissue							
			CTAs				
		Detected	Not Detected	Total			
	Detected	286	3	289			
E1L CD _w	Not Detected Unevaluable	42	102	144			
FILCDX	Unevaluable	74	16	90			
	Total	402	121	523			
Agreeme	nt Statistics	PPA: 87.2% (286/328)	NPA: 97.1% (102/105)				
Excluding CDx-		95% CI ¹ : (83.1%, 90.4%)	05% CI ¹ · (01 0% 00 0%)				
Unevalua	ble Results	7570 C1 : (65.170, 70.470)	7570 C1 . (71.770, 77.070)				
Percent U	J nevaluable	18.4% (74/402)	13.2% (16/121)				
1							

¹Calculated with Wilson 2-sided 95% CI

The PPA was 87.2% (286/328)) with 95% two-sided CI (83.1%, 90.4%) and NPA was 97.1% (102/105) with 95% CI (91.9%, 99.0%) after excluding CDx-unevaluable results. Since patients were enrolled and initially tested by LDT and/or central CTA, the PPV and NPV were calculated using the PPA and NPA, after adjusting for the prevalence of *BRAF* V600E alteration among the intention-to-treat (ITT) population. The prevalence estimate used in the adjusted agreement was 10% and 15%. In this analysis, F1LCDx demonstrated an adjusted PPV of 77.2% with 95% two-sided CI (59.4%, 100%) and NPV of 98.6% with 95% two-sided CI (98.2%, 98.9%) at 10% prevalence. At the estimated prevalence of 15%, F1LCDx demonstrated an adjusted PPV of 84.3% with 95% two-sided CI (69.9%, 100%) and NPV of 97.7% with 95% two-sided CI (97.1%, 98.3%).

The discordance between the CTA and F1LCDx among *BRAF* V600E positive patients in the primary analysis were evaluated. Of the 42 CTA-positive/F1LCDx-negative samples, 27 had no *BRAF* V600E alteration detected by the F1LCDx pipeline. Fifteen (15) samples had an *BRAF* V600E alteration detected by F1LCDx but were filtered out for failing the pipeline's quality threshold. All 42 CTA-positive/F1LCDx-negative samples were from patients in the Doublet and Control arms of the BEACON trial that had clinical outcome data (refer to Section X.D.2.b, below). Note that all 3 CTA- negative /F1LCDx-positive samples were from patients in the Control arm of the BEACON trial that had clinical outcome data.

Based on the PPA between F1LCDx and the enrolling CTAs, which were predominately tissue-based tests, as shown in Table 15, F1LCDx may miss a

proportion of patients with mCRC with *BRAF* V600E alteration who may derive benefit from BRAFTOVI (encorafenib) in combination with cetuximab. Therefore, reflex testing using tissue specimens to an FDA approved tissue test will be required, if feasible, if the plasma test is negative.

b. Bridging clinical outcome from CTA to F1LCDx

The clinical efficacy of BRAFTOVI (encorafenib) in combination with cetuximab in the clinical trial was measured by OS and ORR with either confirmed complete response (CR) or partial response (PR) based on BICR per RECIST v1.1 of the Doublet Arm compared against the Control Arm. Only clinical samples with clinical outcome data were used in this part of the study analysis.

The ORR for the Doublet Arm and Control Arm as well as the ORR difference are reported in Table 15 for the following subpopulations: CTA+, F1LCDx+|CTA+, F1LCDx-|CTA+, and F1LCDx unevaluable |CTA+. Table 15 also summarizes the median overall survival (OS) by the Kaplan-Meier method for each arm as well as the log(hazard ratio) [log(HR)] with 95% two-sided CI for each of the aforementioned subpopulations. The ORR differences between the Doublet and Control Arms were 18.41% (12.74%, 24.55%), 17.06%

(10.51%, 24.22%), 17.39% (-2.39%, 37.14%) and 25.74% (9.73%, 43.10%) for CTA+, F1LCDx+|CTA+, F1LCDx-|CTA+, and F1LCDx unevaluable |CTA+ population, respectively.

Table 15. Efficacy by *BRAF* V600E alteration status in biomarker

subgroups

	CTA+	F1LCDx+ CTA+	F1LCDx- CTA+	F1LCDx unevaluable CTA+
# Total	402	286	42	74
ORR for Doublet Arm	19.90%	18.49%	17.39%	28.13%
ORR for Control Arm	1.49%	1.43%	0.00%	2.38%
ORR Difference (95% two-sided CI)*	18.41% [12.74%, 24.55%]	17.06% [10.51%, 24.22%]	17.39% [-2.39%, 37.14%]	25.74% [9.73%, 43.10%]
Median OS (months) for Doublet Arm	9.49	7.62	NA [§]	18.89
Median OS (months) for Control Arm	5.88	5.38	12.16	7.16

	CTA+	F1LCDx+ CTA+	F1LCDx- CTA+	F1LCDx unevaluable CTA+
log(HR) (95% two-sided CI)	-0.51 [-0.76, - 0.26]	-0.47 [-0.75, - 0.19]	-2.72 [-4.71, - 0.74]	-0.44 [-1.23, 0.34]

^{*}CI was calculated using the Newcombe method.

The F1LCDx-positive intended use population consists of both F1LCDx-positive | CTA-positive and F1LCDx-positive | CTA-negative subjects and the final drug efficacy in F1LCDx-positive can be estimated as a weighted average of drug efficacy in F1LCDx-positive | CTA-positive and drug efficacy in F1LCDx-positive | CTA-negative with weight Pr(CTA+| F1LCDx +) and 1- Pr(CTA+| F1LCDx +) respectively. The bridging efficacy analysis to evaluate final drug efficacy in F1LCDx-positive intended use population was performed for log (HR) and ORR difference, separately (Table 16).

The ORR difference between the Doublet and Control Arms in F1LCDx-positive subjects were 13.18% (95% CI: 6.86% -19.50%), 14.34% (95% CI: 8.48% - 20.21%), 15.12% (95% CI: 9.48% - 20.76%), 15.90% (95% CI: 10.42% -21.38%) and 17.06% (95% CI: 11.67% - 22.46%) when a range of ORR values for the F1LCDx-positive | CTA-negative subjects were assumed as 0%, 30%, 50%, 70%, and 100% of the observed ORR in the F1LCDx-positive | CTA-positive subjects, assuming that the prevalence of CTA-positive subjects in the intended use population is 10%.

The log (HR) between the Doublet Arm and Control Arm in F1LCDx-positive subjects ranges from -0.36 (95% CI: -0.61, -0.12) to -0.47 (95% CI: -0.70, -0.24) when the log(HR) values for the F1LCDx-positive | CTA-negative subjects were assumed to be between the worst-case scenario (no drug efficacy in F1LCDx-positive | CTA-negative subjects) and the best case scenario (full drug efficacy in F1LCDx-positive | CTA-negative subjects as that in F1LCDx-positive | CTA-positive subjects) and we also assume that the prevalence of CTA-positive subjects in the intended use population is 10%...

The efficacy for the F1LCDx+ population was also calculated assuming that the prevalence of CTA-positive subjects in the intended use population is 15% and the results are showed in Table 16.

Table 16. Estimated Efficacy for the F1LCDx+ Population

	$\widehat{\delta}_{\mathit{CDx+}}$ with 95% CI [log(HR)]	$\widehat{\delta}_{CDx+}$ with 95% CI (ORR difference)		
	prev = 10%			
c*=0%	-0.36 [-0.61, -0.12]	13.18 [6.86, 19.50]		

[§]The estimated median OS is NA due to the small number of events in this group (3 events).

	$\widehat{\delta}_{\mathit{CDx}+}$ with 95% CI [log(HR)]	$\widehat{\delta}_{\mathit{CDx}^+}$ with 95% CI (ORR difference)
c=30%	-0.40 [-0.63, -0.16]	14.34 [8.48, 20.21]
c=50%	-0.42 [-0.65, -0.18]	15.12 [9.48, 20.76]
c=70%	-0.44 [-0.67, -0.21]	15.90 [10.42, 21.38]
c=100%	-0.47 [-0.70, -0.24]	17.06 [11.67, 22.46]
	prev = 15%	
c=0%	-0.40 [-0.65, -0.15]	14.39 [8.20, 20.58]
c=30%	-0.42 [-0.67, -0.17]	15.19 [9.25, 21.14]
c=50%	-0.43 [-0.68, -0.19]	15.73 [9.90, 21.55]
c=70%	-0.45 [-0.69, -0.21]	16.26 [10.52, 22.01]
c=100%	-0.47 [-0.71, -0.23]	17.06 [11.37, 22.76]

^{*}c is the ratio of efficacy between F1LCDx+|CTA- and F1LCDx+|CTA+ populations.

Samples that were positive by F1LCDx for *BRAF* V600E alteration in the clinical bridging study had VAFs as low as 0.11%. Since F1LCDx does not have a pre-specified VAF cut-off for positivity, but rather is based on quality threshold based on select metrics for each variant position, it is possible that patients with VAFs below the limit of detection (0.70%) and below that which was enrolled in the trial (e.g., 0.11%) can be reported as positive. The clinical effectiveness of F1LCDx has not been demonstrated for patients with mCRC that have *BRAF* V600E alteration with VAF below 0.11%. A limitation addressing the uncertainty of the clinical effectiveness of the device for *BRAF* V600E alteration with VAFs below those evaluated in the clinical study is included as a limitation to the device (also refer to Section XII. C. below).

c. Sensitivity Analysis

Sensitivity analyses with regard to missing values were conducted to evaluate the robustness of the ORR estimates considering F1LCDx unevaluable patients enrolled in the BEACON trial. Samples were considered missing if the samples were not tested, if they were tested but returned an invalid result, or if they did not satisfy the F1LCDx cfDNA minimum input requirement (i.e., \geq 20ng).

Amongst all CTA-positive NDA patients, 18.4% did not have a F1LCDx result (74/402). To evaluate the impact of the F1LCDx unevaluable population, the distribution of patients for baseline covariates and disease characteristics was compared among the CTA-positive population, the F1LCDx-evaluable/CTA-positive subpopulation, and F1LCDx-unevaluable/CTA-positive subpopulation. A multiple imputation method was utilized to account for patients with missing or non-evaluable F1LCDx results (n=74).

The imputed ORR difference between the Doublet and Control Arms by BICR was estimated to range from 14.54% (95% CI: 8.27%, 20.81%) to 18.91(95% CI: 12.49%, 25.34%) when a range of ORR values for the F1LCDx-

unevaluable/CTA-positive subjects were assumed as 0%, 30%, 50%, 70%, and 100% of the observed ORR in the F1LCDx-positive /CTA-positive subjects, which is similar to the ORR difference for the CTA-positive population based on the observed data [18.41% (95% CI: 12.74%, 24.55%)]. The imputed log (HR) was estimated to range from -0.41% (95% CI: -0.66%, -0.15%) to -0.53 (95% CI: -0.82%, -0.24%) when a range of log (HR) for the F1LCDx-unevaluable/CTA-positive subjects were assumed as 30%, 50%, 70%, and 100% of the observed log (HR) in the F1LCDx-positive /CTA-positive subjects, which is similar to the log (HR) for the CTA-positive population based on the observed data [-0.51 (95% CI: -0.76, -0.26)]. Thus, the sensitivity analysis demonstrated the robustness of the clinical efficacy estimate. The sensitivity analysis results demonstrate that the estimated drug efficacy in the F1LCDx-positive set of the primary bridging studies for ORR difference and log (HR) remain robust to missing F1LCDx results.

3. Pediatric Extrapolation

In this premarket application, existing clinical data was not leveraged to support approval of a pediatric patient population.

E. Financial Disclosure

The Financial Disclosure by Clinical Investigators regulation (21 CFR 54) requires applicants who submit a marketing application to include certain information concerning the compensation to, and financial interests and arrangement of, any clinical investigator conducting clinical studies covered by the regulation. The pivotal clinical study included one (1) investigator which was a full-time employee of the sponsor and had disclosable financial interests/arrangements as defined in 21 CFR 54.2(a), (b), (c) and (f) and described below:

- Compensation to the investigator for conducting the study where the value could be influenced by the outcome of the study: [0]
- Significant payment of other sorts: [0]
- Proprietary interest in the product tested held by the investigator: [1]
- Significant equity interest held by investigator in sponsor of covered study: [0]

The applicant has adequately disclosed the financial interest/arrangements with clinical investigators. Statistical analyses were conducted by FDA to determine whether the financial interests/arrangements had any impact on the clinical study outcome. The information provided does not raise any questions about the reliability of the data.

XI. PANEL MEETING RECOMMENDATION AND FDA'S POST-PANEL ACTION

In accordance with the provisions of section 515(c)(3) of the act as amended by the Safe Medical Devices Act of 1990, this PMA was not referred to the Molecular and Clinical Genetics Panel, an FDA advisory committee, for review and recommendation because

the information in the PMA substantially duplicates information previously reviewed by this panel.

XII. CONCLUSIONS DRAWN FROM PRECLINICAL AND CLINICAL STUDIES

A. Effectiveness Conclusions

The effectiveness of F1LCDx to identify *BRAF* V600E alteration in patients with mCRC who may benefit from treatment with BRAFTOVI (encorafenib) in combination with cetuximab was demonstrated through clinical bridging studies using specimens from patients enrolled into the BEACON trial. The data from the analytical validation and clinical bridging studies support the reasonable assurance of safety and effectiveness of the F1LCDx assay when used in accordance with the indications for use. Data from the BEACON trial show that patients with mCRC harboring *BRAF* V600E alteration received benefit from treatment with BRAFTOVI (encorafenib) in combination with cetuximab and support the addition of the CDx indication to F1LCDx.

B. Safety Conclusions

The risks of the device are based on data collected in the analytical and clinical validation studies conducted to support sPMA approval, as described above. The F1LCDx assay is an *in vitro* diagnostic test, which involves testing of cfDNA extracted from blood or plasma.

Failure of the device to perform as expected or failure to correctly interpret test results may lead to incorrect test results, and subsequently, inappropriate patient management decisions in cancer treatment. Patients with false positive results may undergo treatment with one of the therapies listed in Table 1 of the intended use statement without clinical benefit and may experience adverse reactions associated with the therapy. Patients with false negative results may not be considered for treatment with the indicated therapy. There is also a risk of delayed results, which may lead to delay of treatment with indicated therapy.

C. Benefit-Risk Determination

The probable benefit of the F1LCDx assay in identifying patients with mCRC with *BRAF* V600E alteration for treatment with BRAFTOVI (encorafenib) in combination with cetuximab was demonstrated through clinical bridging studies using specimens from patients enrolled into the BEACON (ARRAY-818-302) trial.

For patients with mCRC with *BRAF* V600E alteration positive status, the ORR difference for the CTA-positive patients was 18.41% (95% CI: 12.74%, 24.55%), and the log(HR) was -0.51(95% CI: -0.76, -0.26). Clinical outcome for patients with mCRC with *BRAF* V600E alteration positive status by the CTA and F1LCDx indicated an ORR difference of 17.06% (95% CI:10.51%, 24.22%) and log(HR) of -0.47 (95% CI: -0.75, -0.19), which were comparable to the ORR difference and log (HR) in the CTA-positive population and provides evidence of a meaningful clinical benefit in this population. The observed ORR difference and log(HR) for the F1LCDx *BRAF* V600E alteration positive patients supports probable benefit of

F1LCDx in selecting *BRAF* V600E alteration positive mCRC patients for treatment with BRAFTOVI (encorafenib) in combination with cetuximab.

There is potential risk associated with the use of this device, mainly due to 1) false positive, false negatives, or failure to provide a result, and 2) incorrect interpretation of test results by the user. The risks of the F1LCDx assay are associated with the potential mismanagement of patients resulting from false results of the test. Patients who are determined to be false positive by the test may be exposed to a drug that is not beneficial which may lead to adverse events or may have delayed access to treatments that could be more beneficial. A false negative result may prevent a patient from accessing a potentially beneficial drug.

The risks of false results are partially mitigated by the analytical and clinical performance of the device, as summarized above, including the analytical accuracy, and clinical concordance and bridging efficacy studies. In addition, the risks of false negative results are partially mitigated by a recommendation that those patients whose plasma generates a negative result for those alterations included in Table 1, including BRAF V600E alteration, should have their tumor mutation status verified by using an FDA-approved tumor tissue test, if feasible. Additional factors to consider in determining probable risks and benefits for F1LCDx included: the availability of alternative tests. Of note, there are no FDA-approved CDx alternatives using cfDNA isolated from plasma for the detection of BRAF V600E alteration to identify patients with mCRC for treatment with BRAFTOVI (encorafenib) in combination with cetuximab, and this device may meet an unmet clinical need; however, there is an FDA-approved tissue test available for the identification of mCRC patients with BRAF V600E mutation for treatment with BRAFTOVI (encorafenib) in combination with cetuximab. The advantage of F1LCDx over the FDA-approved tissue test, is that it offers a non-invasive method to obtain DNA from the cancer, rather than a tissue biopsy and may meet an unmet need for patients who cannot otherwise provide a biopsy. Another factor to consider in the benefit-risk profile is that this device would provide additional potential benefit to the patient by profiling 324 cancer related genes. However, given uncertainties that remain based on the analytical and clinical validation data, the following limitations are included in the device labeling:

- The analytical accuracy for the FoundationOne Liquid CDx assay for detection of BRAF V600E alteration has not been demonstrated for samples with <1.02% VAF
- BRAFTOVI (encorafenib) in combination with cetuximab efficacy has not been established in patients with *BRAF* V600E alteration <0.11 % VAF tested with FoundationOne Liquid CDx

The overall clinical and analytical validation data support that for F1LCDx, and the indications noted in the intended use statement, the probable benefits outweigh the probable risks.

1. Patient Perspective

This submission did not include specific information on patient perspectives for this device.

In conclusion, given the available information above, the data support that for the selection of patients with mCRC with *BRAF* V600E alteration for treatment with BRAFTOVI (encorafenib) in combination with cetuximab, the probable benefits outweigh the probable risks.

D. Overall Conclusions

The data in this application support the reasonable assurance of safety and effectiveness of this device when used in accordance with the indications for use. Data from the clinical bridging study support the performance of F1LCDx as an aid for the identification of *BRAF* V600E alteration in patients with mCRC for whom BRAFTOVI (encorafenib) in combination with cetuximab may be indicated.

XIII. CDRH DECISION

CDRH issued an approval order on June 8, 2023. The final conditions of approval cited in the approval order are described below.

FMI will provide the following information in a post-approval report within 6 months of approval of this PMA supplement:

- FMI will submit a list of the cumulative changes and in sufficient detail acceptable to FDA, made between the currently deployed genomics platform, which includes analytical pipeline software version v3.22.0 (AP v3.22.0), and the AP versions used in the analytical and clinical validation studies in this supplement.
- FMI will submit a detailed description of the validation activity conducted to support the version change, including the associated risk assessments for each change, and the rationale, acceptable to FDA, that the validation performed supports reasonable assurance that the modification has not affected the performance or raised new concerns regarding the safety and effectiveness of the device.
- FMI will provide evidence, acceptable to FDA, that performance expectations with the currently deployed genomics platform, including AP v3.22.0 are representative of the performance in the analytical and clinical validation studies in this supplement. Such evidence may include regression testing using the clinical and analytical datasets to perform *in silico* reanalysis of the results obtained in the analytical and clinical validation studies in this supplement and confirmation that there is little or no deviation in the quality metrics for each of the samples to support the accuracy and precision of the assay remains the same.

Foundation Medicine, Inc. agreed to implement alternative controls to address violations of the current good manufacturing practice requirements of the Quality System regulations found at Title 21, Code of Federal Regulations, Part 820 identified at the manufacturing facility of the cfDNA blood collection tubes used with the FoundationOne Liquid CDx assay. FDA subsequently approved a variance plan on August 26, 2020 that met the requirements set forth in 21 C.F.R. 820.1(e)(2).

XIV. APPROVAL SPECIFICATIONS

Directions for use: See device labeling.

Hazards to Health from Use of the Device: See Indications, Contraindications, Warnings and Precautions, and Adverse Events in the device labeling.

Post-approval Requirements and Restrictions: See approval order.

XV. <u>REFERENCES</u>

None.