

510(k) Summary

[As required by 21 CFR 807.92]

510(k) Number K023369

Date Prepared: 10/07/2002

<i>Submitter</i>	<i>Contact Person</i>
Beckman Coulter, Inc Diagnostics Division 1000 Lake Hazeltine Drive Chaska, MN 55318	Denise Thompson Regulatory Affairs Specialist Phone: 952-368-1202 Fax: 952-368-7610

General Information

Trade Name	Access® Total T4
Common Name	Total thyroxine assay
Classification Name	Total thyroxine test system (21 CFR 862.1700)

Device Description

The Access® Total T4 assay consists of the reagent pack and calibrators. Consumables required for the assay include substrate and wash buffers.

Intended Use

The Access Total T4 assay is a paramagnetic particle, chemiluminescent immunoassay for the quantitative determination of total thyroxine (T4) levels in human serum and plasma using the Access® Immunoassay Systems.

Substantial Equivalence Comparison

The Access Total T4 assay is substantially equivalent to the previously cleared Access Total T4 assay (K926475). Both assays utilize the same methodology, are the same product type, and are quantitative. The subject and predicate assays include the same components. The only difference between the two assays is that the sample type has been expanded to include human plasma samples.

Supporting Data

To demonstrate substantial equivalence, a method correlation study, using paired plasma and serum samples, was conducted using the Access Total T4 assay. Study results demonstrate good correlation between the plasma and serum samples.

Conclusion

The information provided in this submission supports a substantial equivalence determination, and therefore 510(k) premarket notification clearance of the Access Total T4 assay.



DEPARTMENT OF HEALTH & HUMAN SERVICES

Food and Drug Administration
2098 Gaither Road
Rockville MD 20850

DEC 09 2002

Ms. Denise Thompson
Regulatory Affairs Specialist
Beckman Coulter, Inc.
1000 Lake Hazeltine Drive
Chaska, MN 55318-1084

Re: k023369
Trade/Device Name: Access[®] Total T4 on the Access[®] Immunoassay Systems
Regulation Number: 21 CFR 862.1700
Regulation Name: Total thyroxine test system
Regulatory Class: Class II
Product Code: KLI; JIS
Dated: October 7, 2002
Received: October 8, 2002

Dear Ms. Thompson:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to such additional controls. Existing major regulations affecting your device can be found in Title 21, Code of Federal Regulations (CFR), Parts 800 to 895. In addition, FDA may publish further announcements concerning your device in the Federal Register.

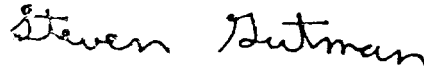
Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Parts 801 and 809); and good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820).

Page 2 –

This letter will allow you to begin marketing your device as described in your Section 510(k) premarket notification. The FDA finding of substantial equivalence of your device to a legally marketed predicate device results in a classification for your device and thus, permits your device to proceed to the market.

If you desire specific information about the application of labeling requirements to your device, or questions on the promotion and advertising of your device, please contact the Office of In Vitro Diagnostic Device Evaluation and Safety at (301) 594-3084. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21CFR Part 807.97). Other general information on your responsibilities under the Act may be obtained from the Division of Small Manufacturers, International and Consumer Assistance at its toll-free number (800) 638-2041 or (301) 443-6597 or at its Internet address <http://www.fda.gov/cdrh/dsma/dsmamain.html>.

Sincerely yours,

A handwritten signature in black ink that reads "Steven Gutman". The signature is written in a cursive style with a large, prominent "S" at the beginning.

Steven I. Gutman, M.D., M.B.A.
Director
Office of *In Vitro* Diagnostic Device
Evaluation and Safety
Center for Devices and
Radiological Health

Enclosure

Indications Statement

510(k) Number: K023369

Device Name:

Access® Total T4 on the Access® Immunoassay Systems

Indications:

The Access Total T4 assay provides in vitro quantitative measurement of total thyroxine (T4) levels in human serum or plasma. The Access Total T4 assay is indicated for use with patients where an evaluation of their thyroid status is desired.

Jan Cooper
(Division Sign-Off)
Division of Clinical Laboratory Devices
510(k) Number K023369

(PLEASE DO NOT WRITE BELOW THIS LINE-CONTINUE ON ANOTHER PAGE OF NEEDED)

Concurrence of CDRH, Office of Device Evaluation (ODE)

Rx ✓

OTC —