



October 2, 2020

Sk bioland Co., Ltd.  
% Sanglok Lee  
Manager  
WISE COMPANY Inc.  
RM #303, The Sky Balley-303, 142, Gasan Digital 1-Ro  
Geumcheon-gu, Seoul 08507  
Republic of Korea

Re: K200623  
Trade/Device Name: OssGuide  
Regulation Number: 21 CFR 872.3930  
Regulation Name: Bone Grafting Material  
Regulatory Class: Class II  
Product Code: NPL  
Dated: September 1, 2020  
Received: September 3, 2020

Dear Sanglok Lee:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email ([DICE@fda.hhs.gov](mailto:DICE@fda.hhs.gov)) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

for Srinivas Nandkumar, Ph.D.  
Director  
DHT1B: Division of Dental Devices  
OHT1: Office of Ophthalmic, Anesthesia,  
Respiratory, ENT and Dental Devices  
Office of Product Evaluation and Quality  
Center for Devices and Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)  
K200623

Device Name  
OssGuide

### Indications for Use (Describe)

OssGuide is recommended for:

- Simultaneous use of GBR-membrane (OssGuide) and implants;
- Augmentation around implants placed in immediate extraction sockets;
- Augmentation around implants placed in delayed extraction sockets;
- Localized ridge augmentation for later implantation;
- Alveolar ridge reconstruction for prosthetic treatment;
- Aid regeneration of bone defects after root resection, cystectomy, removal of retained teeth;
- Guided bone regeneration in dehiscence defects

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

### CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
PRAStaff@fda.hhs.gov

*"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."*

# *SK bioland Co., Ltd.*

162, Gwahaksaneop 3-ro, Ochang-eup, Cheongwon-gu,  
Cheongju-si, Chungcheongbuk-do, 28125 Republic of Korea



---

## 510(k) Summary – K200623

### 01. Date of Submission: October 2, 2020

### 02. Applicant

Company name: SK bioland Co., Ltd.  
Address: 162, Gwahaksaneop 3-ro, Ochang-eup, Cheongwon-gu, Cheongju-si, Chungcheongbuk-do,  
Republic of Korea  
TEL: +82 43 240 8633  
FAX: +82 43 240 8699  
Email: yjchae@sk.com

### 03. Submission Correspondent

Sanglok, Lee  
Wise COMPANY Inc.  
#303, 142, Gasan digital 1-ro, Geumcheon-gu, Seoul, Korea  
TEL: +82 70 8812 3619 / +82 2 831 3615  
FAX: +82 50 4031 3619  
Email: info@wisecompany.org

### 04. Proposed Device Identification

Trade Name: OssGuide  
Common Name: Absorbable collagen membrane  
Classification Name: Barrier, Animal Source, Intraoral  
Product Code: NPL  
Panel: Dental  
Regulation Number: 21 CFR 872.3930  
Device Class: Class II

### 05. Indication for use

OssGuide is recommended for:

- Simultaneous use of GBR-membrane (OssGuide) and implants;
- Augmentation around implants placed in immediate extraction sockets;
- Augmentation around implants placed in delayed extraction sockets;
- Localized ridge augmentation for later implantation;
- Alveolar ridge reconstruction for prosthetic treatment;
- Aid regeneration of bone defects after root resection, cystectomy, removal of retained teeth;
- Guided bone regeneration in dehiscence defects

### 06. Predicate devices

510(k) Number: K050446  
Device Name: BIO-GIDE®  
Manufacturer: Ed. Geistlich Soehne Ag fuer Chemische Industrie

### 07. Device Description

*OssGuide* (Absorbable collagen membrane) is made of purified collagen without further cross-linking or chemical treatment. The collagen is extracted from veterinary certified pigs and is carefully purified to avoid antigenic reactions. The *OssGuide* is a collagen membrane obtained by a standardized controlled

# SK bioland Co., Ltd.

162, Gwahaksaneop 3-ro, Ochang-eup, Cheongwon-gu,  
Cheongju-si, Chungcheongbuk-do, 28125 Republic of Korea



manufacturing process. The *OssGuide* has a porous and fibrous microstructure that prevents the ingrowth of fibrous tissue into the bone defect.

*OssGuide* is packed by a packaging material composed of a transparent side and an aluminium side and is easy to use by open system of easy peel type. It is easy to check the *OssGuide*, because one side of packaging is transparent closed by double packing system in order to protect *OssGuide* from microbial formation and moisture penetration. Then this packaged *OssGuide* is sterilized by gamma irradiation. The packaging contains the *OssGuide* and one Tyvek material template.

The surgical procedure consists of placing of an occlusive physical barrier between the connective tissue and the bone defect to prevent the migration of the epithelial and connective tissue cells into the defect. The periodontal defect or bone defect is exposed by a mucoperosteal flap and basic surgical procedures (e.g. curettage) are performed. The clinician should perform thorough debridement and efficient planing of the defect. Space-making material such as autologous bone or bone substitute may be used to fill the defect. ***OssGuide*** can be placed either dry or hydrated. ***OssGuide*** can be trimmed to the size and shape of the defect in a dry or wet state using scissors. A piece of sterile Tyvek may be used as a template for membrane sizing and trimming. ***OssGuide*** should overlap the walls of the defect by at least 2 mm to allow complete bone contact and to prevent gingival connective tissue invasion below the material. The membrane may be secured in place with sutures, tacks, or by means of sufficient contact with the mucoperiosteal flap to avoid displacement and micro motion during healing. To facilitate proper healing, ensure that there is tension-free primary closure of the soft tissue flap covering the membrane.

*OssGuide* a bioresorbable scaffold that is eventually resorbed and replaced by host tissue. *OssGuide* is substantially resorbed within 24 weeks after implantation.

The ***OssGuide*** is available to the United States market in 5 sizes as shown below. They are supplied sterile and intended for single use only.

| Model No. | Size      | Packaging (ea/box) |
|-----------|-----------|--------------------|
| TG-1      | 15 x 20mm | 1                  |
| TG-2      | 20 x 30mm | 1                  |
| TG-3      | 30 x 40mm | 1                  |
| TG-4      | 15 x 30mm | 1                  |
| TG-5      | 25 x 25mm | 1                  |

## 08. Non-Clinical Test Conclusion

The determination of substantial equivalence was based on an assessment of non-clinical performance data obtained from *in vitro* characterization studies, an *in vivo* animal study, biocompatibility testing.

*In vitro* product characterization testing was performed to demonstrate substantial equivalence of the subject device to its predicate device in biochemical, physicochemical, mechanical properties. The test results demonstrated that the proposed device complies with the following standards:

- Physicochemical Characterization per ASTM F2212
- Tensile strength testing
- Suture pull-out strength testing

# SK bioland Co., Ltd.

162, Gwahaksaneop 3-ro, Ochang-eup, Cheongwon-gu,  
Cheongju-si, Chungcheongbuk-do, 28125 Republic of Korea



Validation of packaging, shelf life, transport and sterilization are performed as following:

- Validation of Primary and Secondary barrier packaging per ISO 11607-2
- Validation of shelf life per ASTM F1980 using age accelerated and real-time aged samples
- Validation of transport packaging per ASTM D4169
- Validation of sterilization parameters per ISO 11137

Biocompatibility of the subject device was evaluated in accordance with ISO 10993-1 as followings:

- Cytotoxicity per ISO 10993-5
- Sensitization per ISO 10993-10
- Irritation per ISO 10093-10
- Subchronic Toxicity per ISO 10993-11
- Genotoxicity per ISO 10993-3
- Implantation per ISO 10993-6
- Material mediated pyrogenicity per USP

Results indicate that the device's biocompatibility profile is acceptable.

The pyrogenicity of **OssGuide** was tested *in vivo* using the Rabbit Pyrogen Test as per USP <151>, and the test article was judged as nonpyrogenic. As a pyrogenicity sample testing plan, the endotoxin (LAL) testing will be conducted on every batch of **OssGuide** products.

The manufacturing control of animal tissue components is performed as following:

- Controls on sourcing, collection, and handling per ISO 22442-2
- Viral Inactivation per ISO 22442-3

Animal study was conducted in the alveolar bone defect of beagle dogs to evaluate the *in vivo* performance and degradation of the **OssGuide**, supporting substantial equivalence. The study was conducted on 20 animals followed for 4, 8, 12 and 24 weeks. When comparing **OssGuide** with the predicate device, Bio-Gide with Bio-Oss bone graft material as a positive control, it showed sufficient GBR effect for the alveolar bone regeneration on the alveolar bone defect of beagle dogs. **OssGuide** maintained their contour until 8 weeks, lost most of its integrity after 12 weeks, and disappeared by biodegradation at 24 weeks.

## 09. Substantially Equivalent Conclusion

Table 1: Substantial Equivalence Comparison

| Product Name      | SUBJECT Device<br>OssGuide   | PREDICATE Device<br>BIO-GIDE® (K050446) | Equivalence Discussion   |
|-------------------|------------------------------|---|--|
| Product code      | NPL                          | NPL                                     | <b>Identical</b>   |
| Regulatory class  | Class II                     | Class II                                | <b>Identical</b>   |
| Regulation Number | 21 CFR 872.3930              | 21 CFR 872.3930                         | <b>Identical</b>   |
| Intended use      | OssGuide is recommended for; | Bio-Gide® is recommended for;           | <b>Equivalent</b><br>The indications for the subject device are a subset |

# SK bioland Co., Ltd.

162, Gwahaksaneop 3-ro, Ochang-eup, Cheongwon-gu,  
Cheongju-si, Chungcheongbuk-do, 28125 Republic of Korea



|                      |   |   |   |
|----------------------|---|---|---|
|                      | <ul style="list-style-type: none"> <li>- Simultaneous use of GBR-membrane (OssGuide) and implants;</li> <li>- Augmentation around implants placed in immediate extraction sockets;</li> <li>- Augmentation around implants placed in delayed extraction sockets;</li> <li>- Localized ridge augmentation for later implantation;</li> <li>- Alveolar ridge reconstruction for prosthetic treatment;</li> <li>- Aid regeneration of bone defects after root resection, cystectomy, removal of retained teeth;</li> <li>- Guided bone regeneration in dehiscence defects</li> </ul> | <ul style="list-style-type: none"> <li>- Simultaneous use of GBR-membrane (BIO-GIDE) and implants;</li> <li>- Augmentation around implants placed in immediate extraction sockets;</li> <li>- Augmentation around implants placed in delayed extraction sockets;</li> <li>- Localized ridge augmentation for later implantation;</li> <li>- Alveolar ridge reconstruction for prosthetic treatment;</li> <li>- Aid regeneration of bone defects after root resection, cystectomy, removal of retained teeth;</li> <li>- Guided bone regeneration in dehiscence defects; and</li> <li>- Guided tissue regeneration procedures in periodontal defects.</li> </ul> | of the indications for the predicate devices.   |
| Mode of Action       | OssGuide functions as a barrier when applied between bone graft material and soft tissue. The membrane serves as a bioresorbable scaffold that is eventually remodeled, resorbed, and replaced by host tissue.  | Bio-Gide® functions as a barrier when applied between bone graft material and soft tissue. The membrane serves as a bioresorbable scaffold that is eventually remodeled, resorbed, and replaced by host tissue.   | <b>Identical</b>  |
| Operating Principles | Cell-Occlusive<br>Implantable<br>Resorbable<br>Biocompatible  | Cell-Occlusive<br>Implantable<br>Resorbable<br>Biocompatible  | <b>Identical</b>  |
| Material             | Purified collagen   | Purified collagen   | <b>Identical</b>  |
| Collagen source      | Porcine pericardium   | Porcine peritoneum  | <b>Equivalent</b><br><br>Using the tissue extracted from porcine is the same, but the parts of tissue used are different. |
| Form                 | Membrane  | Membrane  | <b>Identical</b>  |
| Structure            | Bilayer structure   | Bilayer structure   | <b>Identical</b>  |
| Color                | White to off-white  | White to off-white  | <b>Identical</b>  |
| Product Size         | 15x20mm, 20x30mm, 30x40mm, 15x30mm, 25x25mm   | 13x25mm, 25x25mm, 40x50mm   | <b>Equivalent</b><br><br>Both devices are provided in clinically relevant sizes for intra-oral surgical procedures        |

# *SK bioland Co., Ltd.*

162, Gwahaksaneop 3-ro, Ochang-eup, Cheongwon-gu,  
Cheongju-si, Chungcheongbuk-do, 28125 Republic of Korea



---

|                 |                                    |                                    |                  |
|-----------------|------------------------------------|------------------------------------|------------------|
| Resorption time | Substantially resorbed by 24 Weeks | Substantially resorbed by 24 Weeks | <b>Identical</b> |
| Sterilization   | Gamma Irradiation                  | Gamma Irradiation                  | <b>Identical</b> |
| Singe Use/Reuse | Single use only                    | Single use only                    | <b>Identical</b> |
| Packaging       | Double pouch pack                  | Double blister pack                | <b>Identical</b> |
| Shelf -life     | 36 months                          | 36 months                          | <b>Identical</b> |

The proposed device, OssGuide, is determined to be Substantially Equivalent (**SE**) to the predicate devices, BIO-GIDE® (K050446) in respect of safety and effectiveness.