

June 2, 2023

Fresenius Kabi % Keith Dunn Director Regulatory Affairs Fresenius Kabi LLC, USA 3 Corporate Dr Suite 300 Lake Zurich, Illinois 60047

Re: K220301

Trade/Device Name: Volumat<sup>™</sup> Line Regulation Number: 21 CFR 880.5440 Regulation Name: Intravascular Administration Set Regulatory Class: Class II Product Code: FPA Dated: May 3, 2023 Received: May 3, 2023

Dear Keith Dunn:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <a href="https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm">https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm</a> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the <u>Federal Register</u>.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's

requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <a href="https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products">https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products</a>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <u>https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems</u>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<u>https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance</u>) and CDRH Learn (<u>https://www.fda.gov/training-and-continuing-education/cdrh-learn</u>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<u>https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice</u>) for more information or contact DICE by email (<u>DICE@fda.hhs.gov</u>) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Porsche Bennet

For Dr. David Wolloscheck, Ph.D. Assistant Director DHT3C: Division of Drug Delivery and General Hospital Devices, and Human Factors OHT3: Office of GastroRenal, ObGyn, General Hospital and Urology Devices Office of Product Evaluation and Quality Center for Devices and Radiological Health

Enclosure

## **Indications for Use**

510(k) Number *(if known)* K220301

Device Name Volumat<sup>TM</sup> Line

Indications for Use (Describe)

IV Administration Set for the infusion of parenteral fluids and medications from a container into the patient's vascular system through a vascular access device with Agilia VP MC/Volumat MC Agilia pump or gravity only.

Transfusion Set for the infusion of blood and blood derivatives from a container into the patient's vascular system through a vascular access device with Agilia VP MC/Volumat MC Agilia pump or gravity only.

Type of Use (Select one or both, as applicable)	
Rescription Use (Part 21 CFR 801 Subpart D)	Over-The-Counter Use (21 CFR 801 Subpart C)

#### CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

#### \*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\*

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services Food and Drug Administration Office of Chief Information Officer Paperwork Reduction Act (PRA) Staff *PRAStaff@fda.hhs.gov* 

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."



# 510(k) SUMMARY- K220301

1. Date Prepared:	June 2, 2023
2. Submitter Information	
Name:	Fresenius Kabi
Address:	Fresenius Kabi AG 61346 Bad Homburg Germany
Contact Person:	Keith Dunn Director, Regulatory Affairs Fresenius Kabi USA, LLC Three Corporate Drive, Lake Zurich, IL 60047 USA
	<b>Phone:</b> (224) 817-2430 <b>Fax:</b> (847) 550 2960 <b>E-mail:</b> keith.dunn@fresenius-kabi.com
Secondary Contact Person:	Jason Ma Sr. Manager, Regulatory Affairs Fresenius Kabi USA, LLC Three Corporate Drive, Lake Zurich, IL 60047 USA
	Phone: (224) 817-4100 Fax: (847) 550 2960 E-mail: Jason.ma01@fresenius-kabi.com

# 3. Device Name and Classification

Device Trade Name: Common Name: Classification Name: Regulation Number: Regulatory Class:	Volumat <sup>TM</sup> Line I.V. Administration Set Intravascular administration set 21 CFR 880.5440 Class II
Product Code:	FPA
4. Predicate Device Device Trade Name:	Intravascular Administration Sets



Common Name: Classification Name: Regulatory Class: Product Code: 510(k) Number: Intravascular Administration Set 21 CFR 880.5440 Class II FPA K203609

#### **5.** Device Description

The Volumat <sup>TM</sup> Line is available for dedicated use with the Volumat MC Agilia Infusion Pump and the Agilia VP MC Pump, or they can administer parenteral fluids and medications by gravity flow. Both the Volumat MC Agilia Infusion Pump and the Volumat I.V. Administration Sets were cleared under K121613, and the Agilia VP MC Pump was cleared under K210073. The Volumat <sup>TM</sup> Line components include a spike, drip chamber, roller clamp, upstream/downstream clamp, backcheck valve, pump segment, upstream filter, needle free port, rotating male luer lock, luer lock connector, Yconnector, tubing, and burette.

Set Number	Description
M46441960	Volumat Line IV Administration Set with spike, air vent, drip chamber, roller clamp, pump segment, downstream clamp, needle-free port, and rotating male luer
M46441985	Volumat Line IV Administration Set with non-siliconized spike, air vent, drip chamber, roller clamp, pump segment, downstream clamp, downstream needle-free port, rotating male luer
M46442460	Volumat Line IV Administration Set with spike, air vent, drip chamber, roller clamp, pump segment, 1.2micron filter (neutrally charged), downstream clamp, downstream needle-free port, rotating male luer
M46443160	Volumat Line Transfusion Set with spike, roller clamp, drip chamber with 200micron filter, roller clamp, pump segment, downstream clamp, downstream needle-free port, rotating male luer
M46443450	Volumat Line IV Administration Set with spike, air vent, drip chamber, roller clamp, pump segment, downstream clamp, downstream needle-free port, rotating male luer
M46444160	Volumat Line IV Administration Set with spike, air vent, drip chamber, roller clamp, pump segment, 0.2micron filter downstream clamp, downstream needle-free port, rotating male luer
M46444175	Volumat Line IV Administration Set with non-siliconized spike, air vent, drip chamber, roller clamp, pump segment, 0.2micron filter, downstream clamp, downstream needle-free port, rotating male luer,
	Volumat Line IV Administration Set with spike, air vent spike, upstream clamp, (5) air vent
M46445260	burette, 150 mL burette with 15 $\mu$ m filter and upstream needle-free port, drip
	chamber, roller clamp, pump segment, downstream clamp, downstream needle-free port, rotating male luer
M46445270	Volumat Line IV Administration Set with spike, air vent spike, upstream clamp, air vent



Three Corporate Drive Lake Zurich, Illinois 60047 T 847-550-2300 T 888-391-6300 www.fresenius-kabi.us

Set Number	Description
	burette, 150 mL burette with 15µm filter and upstream needle-free port, drip chamber, roller clamp, pump segment, downstream clamp, downstream needle-free port, rotating male luer
M46445475	Volumat Line IV Administration Set with non-siliconized spike, air vent, drip chamber, upstream clamp, backcheck valve, upstream, needle-free port, roller clamp, pump segment, downstream clamp, downstream needle-free port, rotating male luer
M46445660	Volumat Line IV Administration Set with spike, air vent, drip chamber, upstream clamp, backcheck valve, upstream needle-free port, roller clamp, pump segment, downstream clamp, downstream needle-free port, rotating male luer
M46445670	Volumat Line IV Administration Set with spike, air vent, drip chamber, upstream clamp, backcheck valve, 2 upstream needle-free ports, roller clamp, pump segment, downstream clamp, downstream needle-free port, rotating male luer
M46445690	Volumat Line IV Administration Sets with spike, air vent, drip chamber, upstream clamp, backcheck valve, upstream needle-free ports, roller clamp, pump segment, downstream clamp, 2 downstream needle-free ports, rotating male luer
M46445780	Volumat Line IV Administration Set with spike, air vent, drip chamber, upstream clamp, backcheck valve, upstream needle-free port, roller clamp, pump segment, downstream clamp, 2 downstream needle-free ports, rotating male luer
M46445860	Volumat Line IV Administration Set with spike, air vent, drip chamber, upstream clamp, backcheck valve, upstream needle-free port, roller clamp, pump segment, 0.2micron filter, downstream clamp, downstream needle-free port, rotating male luer

# 6. Principle of Operation

The range of sets provides options for intermittent or continuous delivery of parenteral fluids (solutions, colloids, parenteral nutrition) and medications (including but not limited to diluted drugs, chemotherapy) through clinically accepted intravenous (I.V.) routes of administration and transfusions of blood and blood derivative products.

The device may be used for adult patients. The devices M46445260 and M46445270 may be used for pediatric patients.

# 7. Indication for Use/ Intended Use

#### **Indication for Use:**

I.V. Administration Set for the infusion of parenteral fluids and medications from a container into the patient's vascular system through a vascular access device with Agilia VP MC/Volumat MC Agilia pump or gravity only.

Transfusion set for the infusion of blood and blood derivatives from a container into the patient's vascular system through a vascular access device with Agilia VP MC/Volumat MC Agilia pump or gravity only.



#### Intended Use:

The Volumat <sup>TM</sup>Line is intended for the administration of drugs, solutions and blood products.

# 8. Comparison of the Technological Characteristics with the Predicate Device

The technological characteristics of the subject device are substantially equivalent to those of the predicate device in regard to the following technological characteristics:

- Principle of operation and conditions of use of the subject device are the same as those of the predicate device.
- Materials of the proposed device do not raise new questions of safety and effectiveness, as demonstrated by performance testing and biocompatibility evaluation.
- Physical specifications of the subject device are equivalent to those of the predicate device. Both devices have a needle free access port. The subject device incorporates a neutral displacement needleless access port with the same intended use

A comparison between the predicate device and the subject device is provided in Table 1 and Table 2.

Technological Characteristics	Volumat <sup>TM</sup> Line (Subject Device)	Baxter Administration Set K203609 (Predicate Device)	Assessment of Differences
Indication for Use	I.V. Administration Set for the infusion of parenteral fluids and medications from a container into the patient's vascular system through a vascular access device with Agilia VP MC/Volumat MC Agilia pump or gravity only. Transfusion set for the infusion of blood and blood derivatives from a container into the patient's vascular system through a	For the administration of fluids from a container into the patients' vascular system through a vascular access device.	Similar. Both I.V. administration sets are used to administer fluids from a container to a patient. However, the Subject Device interoperates with infusion systems that have been cleared in other 510(k) submissions including K121613 and K210073.

#### Table 1: Summary of Substantial Equivalence Comparison—Primary Sets



Three Corporate Drive Lake Zurich, Illinois 60047 T 847-550-2300 T 888-391-6300 www.fresenius-kabi.us

Technological Characteristics	Volumat <sup>™</sup> Line (Subject Device)	Baxter Administration Set K203609 (Predicate Device)	Assessment of Differences
	vascular access device with Agilia VP MC/Volumat MC Agilia pump or gravity only.		
Operating Mechanism	The Volumat administration sets deliver the intravenous infusions via an infusion pump through a pumping mechanism ("cassette") or by gravity flow.	The Baxter administration sets can be used to deliver solutions for gravity or pump by infusion of I.V. fluids.	Similar, both devices can be used with an infusion pump or by gravity flow. The predicate device does not have a pumping mechanism ("cassette"). Difference has been verified through performance testing including flow rate accuracy under various environmental conditions (temperature, pressure, humidity) which demonstrate equivalence.
Sterile	Yes	Yes	Same
Non-Pyrogenic	Yes	Yes	Same
Single Use	Yes	Yes	Same
Length	75-115 inches	69-133.5 inches	Similar. The sets included in the subject device range in length from 75 inches to 115 inches which is within the range of the predicate device. Difference tested according to ISO 8536-4 and flow rate accuracy to demonstrate the subject device performance.
Priming Volume	20-29 mL	6.1 to 21.2 mL	Similar. The Volumat set requires more priming volume than the predicate. Bench testing confirmed that the differences in priming volume do not impact safety or effectiveness. Difference tested to ISO 8536-4 to demonstrate
Internal Tube Diameter	0.098 to 118 inches	0.102 to 133 inches	equivalence.



Three Corporate Drive Lake Zurich, Illinois 60047 T 847-550-2300 T 888-391-6300 www.fresenius-kabi.us

Technological Characteristics	Volumat <sup>™</sup> Line (Subject Device)	Baxter Administration Set K203609 (Predicate Device)	Assessment of Differences
External Tube Diameter	1.61 inches	1.42 to 2.09 inches	Similar internal and external diameter and approximate range of the predicate device. Difference tested to ISO 8536-8 has demonstrated equivalence.
	Co	omponents	
Spike Drip chamber	Yes Yes	Yes Yes	Different. The subject device includes an additional
Backcheck Valve	Yes	Yes	pumping segment,
Pump Segment Downstream Filter	Yes Yes	No No	downstream filter, and burette, however, no significant difference between the subject and
Needle Free Port	Yes	Yes	predicate administration sets
Male Luer	Yes	Yes	that would raise new issues
Roller Clamp	Yes	Yes	of safety or efficacy.
Connector Luer Lock	Yes	Yes	The differences have been
Burette	Yes	No	The differences have been verified in various aspects to demonstrate the subject device's safety and performance including: Biocompatibility testing according to ISO 10993 collateral standards, Microbial Ingress Testing, and Particulate Testing; Performance Testing according to ISO 8536-4, ISO 80369-20, ISO 8536-8 and ISO 8436-14.

#### Table 2: Summary of Substantial Equivalence Comparison—Transfusion Sets

Technological Characteristics	Modified Volumat I.V. Administration Set (Subject Device)	Baxter Administration Set K203609 (Predicate Device)	Assessment of Differences
Indication for Use	Transfusion set for infusion pumps (Agilia VP MC pumps/Volumat	For the administration of fluids from a	Similar. Slight difference but both the subject device



Three Corporate Drive Lake Zurich, Illinois 60047 T 847-550-2300 T 888-391-6300 www.fresenius-kabi.us

Technological Characteristics	Modified Volumat I.V. Administration Set (Subject Device)	Baxter Administration Set K203609 (Predicate Device)	Assessment of Differences
	MC Agilia pump or gravity only.)	container into the patients' vascular system through a vascular access device.	and predicate device include a blood transfusion set. However, the Subject Device interoperates with infusion systems that have been cleared in other 510(k) submissions including K121613 and K210073.
Operating Mechanism	The Volumat administration sets deliver the transfusion via an infusion pump through a pumping mechanism ("cassette") or by gravity flow.	The Baxter administration sets can be used to deliver solutions for gravity or pump by infusion of I.V. fluids.	Similar, both devices can be used with an infusion pump or by gravity flow. The predicate device does not have a pumping mechanism ("cassette"). Difference has been verified through performance testing including flow rate accuracy under various environmental conditions (temperature, pressure, humidity) which demonstrate equivalence.
Sterile	Yes	Yes	Same
Non-Pyrogenic	Yes	Yes	Same
Single Use	Yes	Yes	Same
Length	110 inches	79 inches	Similar. The transfusion set included in the subject device is longer than the predicate device. Testing did not raise any new issues of safety or efficacy. Difference tested according to ISO 8536-4 and flow rate accuracy to demonstrate the
Priming Volume	33 ml	6.1 to 21.2 mL	subject device performance. Similar. The Volumat set requires more priming volume than the predicate. Bench testing confirmed that the differences in priming volume does not impact safety or effectiveness.



Three Corporate Drive Lake Zurich, Illinois 60047 T 847-550-2300 T 888-391-6300 www.fresenius-kabi.us

Technological Characteristics	Modified Volumat I.V. Administration Set (Subject Device)	Baxter Administration Set K203609 (Predicate Device)	Assessment of Differences
			Difference tested to ISO 8536-4 to demonstrate equivalence.
Internal Tube Diameter	0.118 inches	0.102 to 133 inches	Similar internal diameter and approximate range of the
External Tube Diameter	1.61 inches	1.42 to 2.09 inches	predicate device.
			Difference tested to ISO 8536-8 has demonstrated equivalence.
	Com	ponents	
Spike	Yes	Yes	Different. The subject device
Drip chamber	Yes	Yes	includes a pump segment,
Pump Segment	Yes	No	however, no significant
Needle Free Port	Yes	Yes	difference between the
Male Luer	Yes	Yes	administration sets that
Y-Connector	Yes	Yes	would raise new issues of safety or efficacy.
			The differences have been verified in various aspects to demonstrate the subject device's safety and performance including: Biocompatibility testing according to ISO 10993 collateral standards, Microbial Ingress Testing, and Particulate Testing; Performance Testing according to ISO 8536-4, ISO 80369-20, ISO 8536-8 and ISO 8436-14.

#### Table 3: Administration/Transfusion Set Material Comparison Table

Component	Modified Volumat I.V. Administration Set (Subject Device)	Baxter Administration Set K203609 (Predicate Device)	Assessment of Differences
Spike	Polystyrene	Acrylonitrile Butadiene Styrene	Similar, testing to biocompatibility and performance standards demonstrated equivalence



Three Corporate Drive Lake Zurich, Illinois 60047 T 847-550-2300 T 888-391-6300 www.fresenius-kabi.us

Component	Modified Volumat I.V. Administration Set (Subject Device)	Baxter Administration Set K203609 (Predicate Device)	Assessment of Differences
Drip chamber	Polystyrene, styrene- butadiene-copolymer, Polyamide	Polyvinyl Chloride	Different, testing to biocompatibility and performance standards demonstrated equivalence
Backcheck Valve	Methylmethacrylate- Acrylonitrile-Butadiene- Styrene)	Polymethyl methacrylate (acrylic) Silicone Rubber	Similar, testing to biocompatibility and performance standards demonstrated equivalence
Pump Segment	Methylmethacrylate- Acrylonitrile-Butadiene- Styrene	N/A – predicate device does not have a pump segment	Testing to biocompatibility and performance standards demonstrated equivalence
Downstream Filter	Polyacryl, polyethersulfone	N/A – predicate device does not have a downstream filter	Testing to biocompatibility and performance standards demonstrated equivalence
Needle Free Port	Methylmethacrylate- Acrylonitrile-Butadiene- Styrene, Copolyester	Polyester	Similar, Testing to biocompatibility and performance standards demonstrated equivalence
Rotating Male Luer Lock	Polyvinylchloride, Acrylonitrile butadiene styrene	Acrylonitrile butadiene styrene	Similar, Testing to biocompatibility and performance standards demonstrated equivalence
Y-Connector	Methylmethacrylate- Acrylonitrile-Butadiene- Styrene	Polyvinyl chloride	Different, testing to biocompatibility and performance standards demonstrated equivalence
Tubing	Polyvinyl chloride, Polyurethane, Polypropylene	Polyvinyl chloride	Similar, Testing to biocompatibility and performance standards demonstrated equivalence
Burette	Acrylonitrile-Butadiene- Styrene, Polyvinylchloride, Copolyester, Polyethylene, Versapor, Stainless steel, Polyamid, Polycarbonate	N/A – predicate device does not have a burette	Different, testing to biocompatibility and performance standards demonstrated equivalence

# 9. Substantial Equivalence Discussion

Intended Use/Indication for Use—Discussion of Differences



Both the subject and predicate device have the same intended use and similar indication for use. Both I.V. administration sets are used to administer fluids with infusion pumps as well as for gravity flow.

#### **Technological Characteristics—Discussion of Differences**

Note: All performance testing / design control activities has been conducted and has confirmed that the different technological characteristics of the proposed devices do not raise different questions of safety and effectiveness. See performance testing below for standards/methods used to evaluate these technological characteristic differences.

- Both the subject and predicate device can be used with an infusion pump or by gravity flow. The subject device delivers the intravenous infusions via an infusion pump through a pumping mechanism ("cassette") or by gravity flow, and the predicate device does not have a pumping mechanism ("cassette"). However, both devices have a pump set interface that was verified through flow rate testing.
- The subject device ranges in length from 75 inches to 115 inches, which is within the length range of the predicate device. Difference tested according to ISO 8536-4 and flow rate accuracy to demonstrate the subject device performance.
- The subject device carries more priming volume than the predicate. Bench testing confirmed that the differences in priming volume do not impact safety or effectiveness. Difference tested to ISO 8536-4 to demonstrate equivalence.
- The subject device includes an additional downstream filter and burette. However, no significant difference between the subject and predicate administration sets that would raise new questions of safety and efficacy. The differences have been verified in various aspects to demonstrate the subject device's safety and performance including Biocompatibility testing according to ISO 10993 collateral standards, Microbial Ingress Testing, and Particulate Testing: Performance Testing to ISO 8536-4, ISO 80369-20, ISO 8536-8.

#### **Conclusion on Substantial Equivalence**

The subject device has the same intended use and equivalent indications fruse as the predicate device. It also has similar technological characteristics to the predicate, and the descriptive as well as the performance information provided within this premarket notification demonstrates that:

• any differences do not raise different questions of safety and effectiveness compared to the predicate device; and



• the proposed device is at least as safe and effective as the legally marketed predicate device.

Based on the comparison of the intended use and the technological characteristics, the subject device is substantially equivalent to the predicate device.

## **10.Performance Testing**

#### **Performance Testing—Bench**

Functional performance bench testing was conducted to demonstrate that the Volumat I.V. Administration Set performs as intended.

The following performance testing was conducted to support the substantial equivalence determination:

- ISO 8536-8
  - Leakage Testing
  - Tensile Strength Testing
  - Storage Tube Volume Testing
- ISO 80369-20
  - Luer Fittings Leakage Testing
  - Stress Cracking
  - Resistance to Separation from Axial Loading
  - Resistance to Separation from Unscrewing
  - Resistance to Overriding
- ISO 8536-4
  - Closure-piercing Device Testing
  - Air-inlet Device Testing
  - Tubing Testing
  - Fluid Filter Testing
  - Flow Rate Testing of the Infusion Pump
  - Drip Chamber Testing
  - Clamp Opening and Closing
  - Protective Cap Testing
  - Chemical Compatibility Testing
- ISO 8536-14
  - Clamps and flow regulators Testing
- ISO 11607-1 and ASTM D4169
  - Sterile barrier and packaging systems and simulated shipping



- USP<788> Particulate Matter in Injections
  - Particulate Testing
- Pump Segment Compatibility Testing with Infusion Pump
- Operation under Temperature, Pressure and Humidity
- Microbial Ingress Testing

# **11.Biocompatibility Testing**

Following the FDA Guidance: "Use of International Standard ISO 10993-1, Biological evaluation of medical devices-Part 1: Evaluation and testing within a risk management process", the tests selected were for prolonged, blood path indirect, externally communicating devices. The following biocompatibility tests were successfully conducted on the Volumat I.V. Administration Set:

- Acute Systemic Toxicity
- Hemolysis
- Intracutaneous Reactivity
- Sensitization
- Cytotoxicity
- Pyrogens
- Subacute/subchronic toxicity
- Targeted Chemical Analysis for DEHP

## **12. Sterilization Validation**

Sterilization was achieved by ethylene oxide and meets the requirements of ISO 11135:2020, which is equivalent to ISO 11135 (2014-7) + AMD 1 (2018-10). The ethylene oxide sterilization method achieves a Sterilization Assurance Level (SAL) of  $10^{-6}$ .

# **13. Clinical Testing**

No clinical testing was performed as this device does not require clinical studies to demonstrate substantial equivalence with the predicate device.

## 14. Conclusion

The differences between the predicate and the subject device do not raise any new or different questions of safety or effectiveness. The subject device, Volumat<sup>TM</sup>



Line is substantially equivalent to the predicate device, I.V. Administration Set, cleared under K203609.