

LG Electronics Inc.
% Jonghyun Kim
Chief Consultant
GMS Consulting
4th Floor, Digital Cube, 34, Sangamsan-ro
Seoul, Mapo-gu 03909
REPUBLIC OF KOREA

Re: K223423 March 1, 2023

Trade/Device Name: 32HQ713D Regulation Number: 21 CFR 892.2050

Regulation Name: Medical image management and processing system

Regulatory Class: Class II

Product Code: PGY Dated: February 1, 2023 Received: February 3, 2023

# Dear Jonghyun Kim:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <a href="https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm">https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm</a> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the <u>Federal Register</u>.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part

801; medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <a href="https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products">https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products</a>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <a href="https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems">https://www.fda.gov/medical-device-problems</a>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<a href="https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance">https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance</a>) and CDRH Learn (<a href="https://www.fda.gov/training-and-continuing-education/cdrh-learn">https://www.fda.gov/training-and-continuing-education/cdrh-learn</a>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<a href="https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice">https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice">https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice</a>) for more information or contact DICE by email (<a href="DICE@fda.hhs.gov">DICE@fda.hhs.gov</a>) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Jessica Lamb, Ph.D.

Assistant Director

Imaging Software Team

DHT8B: Division of Radiological Imaging Devices

and Electronic Products

OHT8: Office of Radiological Health Office of Product Evaluation and Quality Center for Devices and Radiological Health

Enclosure

# DEPARTMENT OF HEALTH AND HUMAN SERVICES Food and Drug Administration

# **Indications for Use**

510(k) Number (if known)

Form Approved: OMB No. 0910-0120

Expiration Date: 06/30/2020 See PRA Statement below.

K223423
Device Name 32HQ713D
Indications for Use ( <i>Describe</i> ) This Medical Monitor is indicated for use in displaying radiological images (including full-field digital mammography and digital breast tomosynthesis) for review, analysis, and diagnosis by trained medical practitioners.
Type of Use (Select one or both, as applicable)
Prescription Use (Part 21 CFR 801 Subpart D) Over-The-Counter Use (21 CFR 801 Subpart C)
CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

# \*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\*

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services Food and Drug Administration Office of Chief Information Officer Paperwork Reduction Act (PRA) Staff PRAStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

# 510(k) Summary

[As Required by 21 CFR 807.92]

# 1. Date Prepared [21 CFR 807.92(a)(a)]

Nov 11, 2022

#### 2. Submitter's Information [21 CFR 807.92(a)(1)]

• Name of Sponsor: LG Electronics Inc.

- Address: 222, LG-ro, Cheongho-ri, Jinwi-myeon, Pyeongtaek-si,

Gyeonggi-do, 17709, Republic of Korea

Contact Name: Jinhwan Jun / Chief Research Engineer

Telephone No.: +82-31-8066-5641Email Address: jinhwan.jun@lge.com

Name of Manufacturer:
 LG Electronics Inc.

- Address: 168, Suchul-daero, Gumi-si, Gyeongsangbuk-do, 39368,

Republic of Korea

# 3. Trade Name, Common Name, Classification [21 CFR 807.92(a)(2)]

Trade Name: 32HQ713DCommon Name: Medical Monitor

Classification:

Classification Name	Picture archiving and communications system
Classification Number	21 CFR 892.2050
Product Code	PGY
Device Class	п
Review Panel	Radiology

# 4. Identification of Predicate Device(s) [21 CFR 807.92(a)(3)]

The identified predicate devices within this submission are shown as follow;

#### **Primary Predicate Device**

• 510(k) Number: K172738

• Applicant: EIZO Corporation

Classification Name: Picture archiving and communications system

• Trade Name: RX560

#### **Predicate Device**

510(k) Number: K201777Applicant: LG Electronics

Classification Name: Picture archiving and communications system

• Trade Name: 31HN713D

# 5. Description of the Device [21 CFR 807.92(a)(4)]

The Medical monitor is intended to provide high resolution color and grayscale medical imaging for PACS and Radiology system. This Medical Monitor is intended to be used by trained medical practitioners for displaying, reviewing, and analysis of medical images

#### 6. Indications for use [21 CFR 807.92(a)(5)]

This Medical Monitor is indicated for use in displaying radiological images (including full-field digital mammography and digital breast tomosynthesis) for review, analysis, and diagnosis by trained medical practitioners.

# 7. Technological Characteristics (Equivalence to Predicate Device) [21 CFR 807.92(a)(6)]

The table below presents comparisons between the subject device (32HQ713D) and the legally marketed predicate devices (K172738 and K201777):

[Table 1. Comparison of Proposed Device to Primary Predicate Device]

	Proposed Device	Primary Predicate Device
K Number	Not known	K172738
Manufacturer	LG Electronics Inc.	EIZO Corporation
Model Name	32HQ713D	RX560
Classification Name	Picture archiving and communications	Picture archiving and communications
	system	system
Classification Number	21 CFR 892.2050	21 CFR 892.2050
Indications for Use	This Medical Monitor is indicated for use	This product is intended to be used in
	in displaying radiological images	displaying and viewing digital images,
	(including full-field digital mammography	including standard and multiframe digital
	and digital breast tomosynthesis) for	mammography, for review, analysis and
	review, analysis, and diagnosis by trained	diagnosis by trained medical
	medical practitioners.	practitioners. It is specially designed for
		breast tomosynthesis applications.
Display Technology	The 32HQ713D has a LB315WR6-SSZ1	This is a color LCD monitor for viewing
	module for displaying. The a LB315WR6-	digital images, including standard and
	SSZ1 is a 31.5" TFT Liquid Crystal Display	multiframe digital mammography, for
	module with LED Backlight unit and one	review, analysis and diagnosis. The color
	30pin eDP port and 1ch-eDP interface.	panel employs in-plane switching (IPS)
	This module supports 3840x2160 Pixel	technology allowing wide viewing angles
	mode and can display color driven by 10bit drivers. The LCD module includes	and the matrix size (or resolution) is
		2048x2560 pixels.
Power Consumption	built-in converter for Backlight MAX. 120W	87 W / Less than 1 W
rower consumption	Off Mode ≤ 0.3W	07 W / Less than I W
Screen size	730.70 x 425.20 mm	337.9 mm x 422.4 mm
LCD Screen	TFT LCD	TFT LCD
Pixel Pitch	0.18159 x 0.18159 mm	0.165 x 0.165 mm
Resolution	3,840 x 2,160 pixels	2,048 x 2,560
Horizontal Frequency	30 kHz to 135kHz	31 kHz to 135 kHz
Vertical Frequency	24 Hz to 61 Hz	59 Hz to 61 Hz
Input video signals	DVI IN x 1,	DVI x 1
promise angular	DP IN x 1	DP IN x 1
	DP OUT x 1	DP OUT x 1

The comparison table shows that the subject device (32HQ713D) has the similar indications for use the primary predicate device. Although the devices have some different technological characteristics (screen size, pixel pitch, Resolution, horizontal/vertical frequency and input video signals), these differences do not make the subject device less safe and reliable, so the subject device fits for diagnostic use as the predicate device does. There are no significant differences in the technological characteristics of the subject device. All the differences between the subject and predicate device do not raise different questions of safety and effectiveness. It is substantially equivalent to a predicate device in indications for use and technology characteristics.

[Table 2. Comparison of Proposed Device to Predicate Device]

	Proposed Device	Predicate Device
K Number	Not known	K201777
Manufacturer	LG Electronics Inc.	LG Electronics Inc.
Model Name	32HQ713D	31HN713D
Classification Name	Picture archiving and communications	Picture archiving and communications
	system	system
Classification Number	21 CFR 892.2050	21 CFR 892.2050
Indications for Use	This Medical Monitor is indicated for use	This Medical Monitor is indicated for use
	in displaying radiological images	in displaying radiological images
	(including full-field digital mammography	(including full-field digital mammography
	and digital breast tomosynthesis) for	and digital breast tomosynthesis) for
	review, analysis, and diagnosis by trained	review, analysis, and diagnosis by trained
	medical practitioners.	medical practitioners.
Display Technology	The 32HQ713D has a LB315WR6-SSZ1	The 31HN713D has a LB310FTM module
	module for displaying. The a LB315WR6-	for displaying. The LB310FTM is a 31"
	SSZ1 is a 31.5" TFT Liquid Crystal Display	TFT Liquid Crystal Display module with
	module with LED Backlight unit and one	LED Backlight unit. This module supports
	30pin eDP port and 1ch-eDP interface.	4,200 x 2,800 pixels and can display
	This module supports 3840x2160 Pixel	color driven by 10bit drivers.
	mode and can display color driven by 10bit drivers. The LCD module includes	
Power Consumption	built-in converter for Backlight  MAX. 120W	MAX. 150W
Fower Consumption	Off Mode ≤ 0.3W	Sleep Mode ≤ 0.5W
	On Mode 2 0.5W	Off Mode ≤ 0.3W
Screen size	730.70 x 425.20 mm	676.9 x 459.7 mm
LCD Screen	TFT LCD	TFT LCD
Pixel Pitch	0.18159 x 0.18159 mm	0.1554 x 0.1554 mm
Resolution	3,840 x 2,160 pixels	4,200 x 2,800 pixels
Horizontal Frequency	30 kHz to 135kHz	30 kHz to 135kHz
Vertical Frequency	24 Hz to 61 Hz	56 Hz to 61 Hz
Input video signals	DVI IN x 1,	DisplayPort x 2
	DP IN x 1	HDMI x 1
	DP OUT x 1	

The comparison table shows that the subject device (32HQ713D) has the similar indications for use the predicate device. Although the devices have some different technological characteristics (screen size, pixel pitch, Resolution, horizontal/vertical frequency and input video signals), these differences do not make the subject device less safe and reliable, so the subject device fits for diagnostic use as the predicate device does. There are no significant differences in the technological characteristics of the subject device. All the differences between the subject and predicate device do not raise different questions of safety and effectiveness. It is substantially equivalent to a predicate device in indications for use and technology characteristics.

#### 8. Non-Clinical Test summary

1) Electrical Safety and Electromagnetic Compatibility

The test results demonstrated that the proposed device complies with the following standards:

- Electrical Basic Safety and Essential Performance requirements in accordance with IEC 60601-1:2015+COPR. 1:2006+COPR. 2:2007 + A1:2012 +A2:2020
- Electromagnetic Compatibility Testing in accordance with IEC 60601-1-2:2014
- Medical Electrical Equipment Part 1-6: General Requirements for Basic Safety and Essential Performance - Collateral Standard: Usability in accordance with IEC 60601-1-6:2010+A1:2013+A2:2020

#### 2) Software Validation

The 32HQ713D contains MODERATE level of concern software. The software was designed and developed according to a software development process and was verified and validated.

The software information is provided in accordance with FDA guidance: The content of premarket submissions for software contained in medical devices, on May 11, 2005.

#### 3) Guidance

Display Devices for Diagnostic Radiology – Guidance for Industry and Food and Drug Administration Staff, issued on October 2, 2017

According to the guidance, we test following performance items:

	Measurements	Description	SE Note
a.	Spatial resolution	Measurements of the transfer of information from the image data to the luminance fields at different spatial frequencies of interest typically done by reporting the modulation transfer function. Non-isotropic resolution properties should be characterized properly by providing two-dimensional measurements or measurements along at least two representative axes.	As demonstrated through the function of the resolution for the TG18 test pattern and the comparison of MTF measurements, the specification of the proposed device is equivalent to the predicate device.
b.	Pixel defects	Measurements (count, types (e.g., sub-pixel or entire pixel, always-on, always-off), and locations (map) of pixel defects. This is typically provided as a tolerance limit. Pixel defects can interfere with the visibility of small details in medical images.	As demonstrated through the measurements of the pixel defects, the specification of the proposed device is an equivalent predicate device.
C.	Artifacts	Evaluate for image artifacts such as ghosting and/or image sticking from displaying a fixed test pattern for a period of time.	Demonstrated through the measurements of the Using 5x5 mosaic pattern, 64Gray / 127 Gray judgment, the specification of the proposed

	Measurements	Description	SE Note
			device is equivalence to that of the predicate device.
d.	Temporal response	Measurements of the temporal behavior of the display in responding to changes in image values from frame to frame. Since these transitions are typically not symmetric, rise and fall time constants are needed to characterize the system. Slow displays can alter details and contrast of the image when large image stacks are browsed or in video, panning, and zooming modes.	Demonstrated through the measurements of the rise and fall time constants for 5–95% and 40–60% luminance transitions, the specification of the proposed device is equivalent to that of the predicate device.
e.	Luminance	Measurements of the maximum and minimum luminance that the device outputs as used in the application under recommended conditions and the achievable values if the device is set to expand the range to the limit.	Demonstrated through the function of the maximum measured, minimum, achievable, and recommended luminance values analysis, the specification of the proposed device is equivalence to that of the predicate device.
f.	Conformance to a grayscale-to-luminance function	Measurements of the mapping between image values and the luminance output following a target model response for 256 or more levels.	Demonstrated through the function of the mapping between image values and the luminance output following a target model response for 256 or more levels of analysis, the specification of the proposed device is equivalent to that of the predicate device.
g.	Luminance at 30° and 45° in diagonal, horizontal, and vertical directions at center and four corners	Measurements of the luminance response at off- normal viewing related to the target model for the luminance response.	Demonstrated through the function of luminance response for luminance response, the specification of the proposed device is equivalency to that of the predicate device.
h.	Luminance uniformity or Mura test	Measurements of the uniformity of the luminance across the display screen.	As demonstrated through the measurements of the uniformity of the luminance across the display screen, the specification of the proposed device is an equivalence predicate device.
i.	Stability of luminance and chromaticity response with temperature and time of operation (on-time)	Measurements of the change in luminance and chromaticity response with temperature and use time.	As demonstrated through the measurements, there is no display off, and the user can see the display without

	Measurements	Description	SE Note
			temperature on use time; the specification of the proposed device is an equivalence predicate device.
j.	Spatial noise	Measurements of the spatial noise level as represented by the noise power spectrum using an appropriate ratio of camera and display pixels. Spatial noise and resolution affect the way images are presented to the viewer and can alter features that are relevant to the interpretation process of the physician or radiologist.	Demonstrated through the function of the noise levels for the TG18 test pattern and recommended luminance values analysis, the specification of the proposed device is equivalence to that of the predicate device.
k.	Reflection coefficient	Measurements of the reflection coefficients of the display device. Specular and diffuse reflection coefficients can be used as surrogates for the full bidirectional reflection distribution function.	As demonstrated through the measurements of the reflection Coefficient (Rd), the specification of the proposed device is equivalent predicate device.
I.	Veiling glare or small-spot contrast	Measurements of the contrast obtained for small targets.	Demonstrated through the function of the contrast obtained for the TG18 test pattern and recommended luminance values analysis, the specification of the proposed device is equivalence to that of the predicate device.
m.	Color tracking	Chromaticity at different luminance levels of primary colors as indicated by the color coordinates in an appropriate units system (e.g., CIE u'v') and the color gamut enveloped by the primary colors.	As demonstrated through the measurements of the sRGB xy coverage, the specification of the proposed device is equivalent to that of the predicate device.
n.	Gray tracking	Chromaticity at different luminance levels of gray shades, including the white point, as indicated by the color coordinates in an appropriate units system	As demonstrated through the measurements of the sRGB UV coverage, the specification of the proposed device is equivalent to that of the predicate device.

According to the above test results, there are no significant performance differences between 32HQ713D and the predicate devices that would adversely affect the use of the product. It has substantially equivalent performance compared to the predicate devices.

510(k) Summary **7 / 7** 32HQ713D

# **Clinical Test Summary:**

No clinical studies were considered necessary and performed.

# 9. Conclusion [21 CFR 807.92(b)(3)]

In according with the Federal Food & Drug and cosmetic Act, 21 CFR Part 807, and based on the information provided in this premarket notification LG Electronics, concludes that the 32HQ713D is substantially equivalent in safety and effectiveness to the predicate devices as described herein.