

June 8, 2023

TermoSalud % Aubrey Thompson Regulatory Consultant Hoy and Associates Regulatory Consulting 1830 Bonnie Way Sacramento, California 95825

Re: K230659

Trade/Device Name: Zionic Aesthetic Regulation Number: 21 CFR 878.4400 Regulation Name: Electrosurgical Cutting And Coagulation Device And Accessories Regulatory Class: Class II Product Code: PBX Dated: March 8, 2023 Received: March 9, 2023

Dear Aubrey Thompson:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal

statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <u>https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems</u>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<u>https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance</u>) and CDRH Learn (<u>https://www.fda.gov/training-and-continuing-education/cdrh-learn</u>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<u>https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice</u>) for more information or contact DICE by email (<u>DICE@fda.hhs.gov</u>) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Ма	Mark		Digitally signed by Mark Trumbore -S	
Τrι	ımbo	ore - S	5 Date: 2023.06.08 15:25:02 -04'00'	

Mark Trumbore, Ph.D. Assistant Director DHT4A: Division of General Surgery Devices OHT4: Office of Surgical and Infection Control Devices Office of Product Evaluation and Quality Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number *(if known)* K230659

Device Name Zionic Aesthetic

Indications for Use (Describe)

The Zionic Aesthetic is intended to provide topical heating for the purpose of elevating tissue temperature for treatment of selected medical conditions such as temporary relief of pain, muscle spasms, and increase in local circulation.

The massage device provided is intended to provide a temporary reduction in the appearance of cellulite.

Type of Use (Select one or both, as applicable)	
☐ Prescription Use (Part 21 CFR 801 Subpart D)	Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services Food and Drug Administration Office of Chief Information Officer Paperwork Reduction Act (PRA) Staff *PRAStaff@fda.hhs.gov*

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

This 510(K) Summary of safety and effectiveness for the ZIONIC AESTHETIC is submitted in accordance with the requirements of the SMDA 1990 and following guidance concerning the organization and content of a 510(K) summary.

Applicant	TermoSalud
Address	Ataulfo Friera Tarfe, 8 -33211 Gijón, Spain
Contact Person	Aubrey Thompson, Regulatory Consultant
Contact Information	aubreythompson@hoyregulatory.com (323)533-8994
Preparation Date	June 8, 2023
Device Trade Name	ZIONIC AESTHETIC
Common Name Regulation Number	Massager, vacuum, radio frequency induced heat 878.4400
Product Code Regulatory Class	PBX II
Legally Marketed Predicate Device	Nuera Tight RF (K200359)

Device Description:

The ZIONIC ASTHETIC is a resistive radiofrequency device which output energy can reach up to 200W. It has two different electrodes with different sizes, to adapt the treatment to the different body areas. The electrodes roll to perform a massage while applying the radiofrequency energy.

Indications for use:

The ZIONIC AESTHETIC equipment is indicated to provide topical heating for the purpose of elevating tissue temperature for the treatment of selected medical conditions such as temporary relief of pain, muscle spasms, and increase in local circulation.

The massage device is intended to provide a temporary reduction in the appearance of cellulite.

Substantial Equivalence—Indications for Use:

Subject Device – K230659	Predicate Device -K200359	Comparison
The ZIONIC AESTHETIC is intended:	The NuEra Tight RF Family is intended:	Same
 to provide topical heating for the purpose of elevating 	 to provide topical heating for the purpose of elevating 	
tissue temperature for the treatment of selected medical	tissue temperature for the treatment of selected	
conditions such as temporary relief of pain, muscle spasms,	medical conditions such as temporary relief of pain, muscle	
and increase local circulation;	spasms, and increase local circulation;	
• The massage device is intended to provide a temporary	• to provide, with a massage device, a temporary reduction in	
reduction in the appearance of cellulite.	the appearance of cellulite.	

Technical Comparison	ZIONIC AESTHETIC	Nuera Tight RF – K200359	Comparison
Modes of action	Monopolar radiofrequency	Monopolar and bipolar	Same. The Nuera Tight RF device
		radiofrequency	contains an additional bipolar
			handpiece that the Zionic does not.
Mechanism of action	Electromagnetic waves	Electromagnetic waves	Same
	penetrate in exposed tissue	penetrate in exposed tissue	
	and produce heat, increasing	and produce heat, increasing	
	the temperature on a certain	the temperature on a certain	
	part of the body for therapeutic	part of the body for	
	purposes.	therapeutic	
		purposes.	
Clinical use	Prescription Use	Prescription Use	Same
Electrical Protection	Class I type BF	Class I type BF	Same
User Interface	Touchscreen	Touchscreen	Same
Firmware Controlled	Yes	Yes	Same
Type of energy	Radiofrequency waves	Radiofrequency waves	Same
Temperature Control	Yes	Yes	Same
Frequency	470kHz	470 kHz; 1 MHz; 2 MHz; 4	ZIONIC AESTHETIC working frequency
		MHz ; 6 MHz	is identical to one of the working
			frequencies of the predicate device
			Nuera Tight RF.
Output RF Power	Max 200 W	Max 250 W	Nearly identical. ZIONIC AESTHETIC
			maximum power is 20% lower than
			NueraTight RF, which does not
			generate safety or performance issues,
			as an output power of 200W is enough
			to achieve the desired temperature for
			the effectiveness of the treatment, and
			lowering the output power makes the

			device safer.
Voltage	115VAC (with external	100-240VAC	Same
	autotransformer) or 230VAC		
	Handpieces		
	ZIONIC AESTHETIC	Nuera Tight	
Transfer technology	Monopolar Resistive	Resistive and capacitive monopolar and bipolar	Predicate device includes both transfer modes, monopolar and bipolar, resistive and capacitive.
Electrode area	57/90mm	20/30/40/60/80/100mm	Very similar- the Nuera Tight device includes more variety of electrode sizes, with lower contact surface that are not included with the Zionic.

Performance Testing

Verification and validation activities were successfully completed and establish that the ZIONIC AESTHETIC control unit performs as intended. Testing included the following:

- IEC 60601-1:2005 + Corr.1:2006 + A1:2012; Medical Electrical Equipment Part 1: General Requirements For Basic Safety And Essential Performance;
- IEC 60601-1-2:2014 + A1:2020; Medical Electrical Equipment Part 1-2: General Requirements For Basic Safety And Essential Performance Collateral Standard: Electromagnetic Disturbances Requirements And Tests;
- IEC 60601-2-2:2017; Medical Electrical Equipment Part 2-2: Particular Requirements For The Basic Safety And Essential Performance Of High Frequency Surgical Equipment And High Frequency Surgical Accessories;
- IEC 62304:2006+A1:2015; Medical Device Software Life Cycle Processes;
- EN ISO 14971:2019+A11:2021; Medical Devices Application Of Risk Management To Medical Devices

Software verification and validation testing was conducted, and documentation provided in accordance with FDA's Guidance or the Content of Premarket Submissions for Software Contained in Medical Devices.

In addition, tests were performed to demonstrate that the device can maintain a temperature of 40 treatment C for 10 minutes of treatment for every electrode used. Performance testing was performed to demonstrate the accuracy of the output power, output frequency, and output voltage.

Clinical Evidence – N/A. No clinical studies were conducted as part of this submission.

Conclusion

The ZIONIC AESTHETIC and NueraTight RF are identical in their indications for use and nearly identical in their overall design. The key specification to compare for monopolar radiofrequecy transfer technology is the output power, which is slightly lower in the ZIONIC AESTHETIC. This difference in the maximum output power does not generate safety or performance issues, but increases the safety of the device. Both devices underwent thermal testing to show that they can maintain the desired temperature during treatment. Tests for the Zionic Aesthetic included two patients and three test sites – abdomen, thighs, and arms—and a temperature of 40° C (+2 °C) was maintained for 10 minutes of treatment. Test performed on the predicate device showed the same performance. The device maintained a temperature of 40°C (+2°C) for 15 minutes when tested on 3 different test sites and 2 patients. It can be concluded, therefore, that performance between the ZIONIC AESTHETIC and the predicate device are substantially equivalent and therefore the two devices are substantially equivalent.