

June 23, 2023

Misonix, LLC, a Bioventus Company John Salerno Vice President of Regulatory Affairs and Quality Assurance 1938 New Highway Farmingdale, New York 11735

Re: K231117

Trade/Device Name: neXus Ultrasonic Surgical Aspirator System

Regulatory Class: Unclassified

Product Code: LFL, GEI Dated: May 26, 2023 Received: May 26, 2023

Dear John Salerno:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the <u>Federal Register</u>.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for

K231117 - John Salerno Page 2

devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reportingcombination-products); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reportingmdr-how-report-medical-device-problems.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (https://www.fda.gov/medicaldevices/device-advice-comprehensive-regulatory-assistance) and CDRH Learn (https://www.fda.gov/training-and-continuing-education/cdrh-learn). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatoryassistance/contact-us-division-industry-and-consumer-education-dice) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Mark

Trumbore -S

Digitally signed by Mark Trumbore -S Date: 2023.06.23 07:57:22 -04'00'

Mark Trumbore, Ph.D. Assistant Director DHT4A: Division of General Surgery Devices OHT4: Office of Surgical and Infection Control Devices Office of Product Evaluation and Ouality Center for Devices and Radiological Health

Enclosure

DEPARTMENT OF HEALTH AND HUMAN SERVICES Food and Drug Administration

Indications for Use

Form Approved: OMB No. 0910-0120

Expiration Date: 06/30/2023 See PRA Statement below.

510(k) Number *(if known)* K231117

Device Name

neXus Ultrasonic Surgical Aspirator System

Indications for Use (Describe)

The Misonix Inc. neXus® Ultrasonic Surgical Aspirator System is intended for the fragmentation, emulsification and aspiration of both soft and hard (i.e., bone) tissue.

The indications for use for the Standard Handpiece in combination with BoneScalpel® and SonicOne® OR probe kit accessory configurations, the SonaStar® Long and Short handpieces in combination with SonaStar® probe kit accessory configurations, the BoneScalpel Access® handpiece with BoneScalpel Access® probe kit accessory configurations, and the SonaStar Elite Handpiece with probe kit accessory configurations are listed below.

Standard Handpiece with BoneScalpel Probe Kit

Indicated for use in the fragmentation, emulsification and aspiration of soft and hard (e.g., bone) tissue in the following surgical specialties:

- Neurosurgery
- · Gastrointestinal and Affiliated Organ Surgery
- Urological Surgery
- Plastic and Reconstructive Surgery
- General Surgery
- Orthopedic Surgery
- Gynecology

External genitalia - condyloma - benign tumors (lipomas, fibromas, and leiomyomas) - malignant primary and metastatic tumors of all types and the following cystic lesions: Bartholin's cysts, Vestibular adenitis, Inclusion cysts, Sebaceous cysts

Abdominal area - any abnormal growth, cystic or solid, benign or malignant, involving the ovary, fallopian tube, uterus, or the supporting structures of the uterus except as contraindicated for uterine fibroids.

• Thoracic Surgery

Limited pulmonary resection such as segmentectomies, nonanatomical subsegmentectomies and metastatectomies.

• Wound Care

The neXus Ultrasonic Surgical Aspirator is also indicated for use in the debridement of wounds, such as, but not limited to, burn wounds, diabetic ulcers, bedsores and vaginal ulcers, soft tissue debridement and cleansing of the surgical site in applications in which, in the physician's judgment would require the use of an ultrasonic aspirator with sharp debridement.

Standard Handpiece with SonicOne Probe Kits

Indicated for use in the fragmentation, emulsification and aspiration of soft and hard tissue (i.e. bone) in the following surgical specialty:

• Wound Care

The neXus Ultrasonic Surgical Aspirator is also indicated for use in the debridement of wounds, such as, but not limited to, burn wounds, diabetic ulcers, bedsores and vaginal ulcers, soft tissue debridement and cleansing of the surgical site in applications in which, in the physician's judgment would require the use of an ultrasonic aspirator with sharp debridement.

• Plastic and Reconstructive Surgery

neXus SonaStar Handpieces with SonaStar Probe Kits

Indicated for use in the fragmentation, emulsification and aspiration of both soft and hard (i.e., bone) tissue in the following surgical specialties:

Neurosurgery

- Gastrointestinal and Affiliated Organ Surgery including removal of benign or malignant tumors or other unwanted tissue, including hepatic parenchyma, in open or laparoscopic procedures, hepatic resection, tumor resection, lobectomy or trisegmentectomy, or removal of tissue during liver allotransplantation and donor hepatectomy
- Urological Surgery including removal of renal parenchyma during nephrectomy or partial nephrectomy
- Plastic and Reconstructive Surgery
- General Surgery including removal of benign or malignant tumors or other unwanted tissue in open or minimally invasive general surgical procedures
- Orthopedic Surgery
- Gynecological Surgery except as contraindicated for uterine fibroids.
- Thoracic Surgery
- Laparoscopic Surgery including removal of hepatic parenchyma in laparoscopic hepatic resection, lobectomy or trisegmentectomy, in laparoscopic donor hepatectomy or laparoscopic cholecystectomy or laparoscopic pancreatic jejunostomy, or pancreatectomy, or laparoscopic appendectomy, laparoscopic colon resection or laparoscopic partial gastrectomy
- Thoracoscopic Surgery

The SonaStar Handpieces may also be combined with electrosurgery using optional RF surgery interface components.

Bone Scalpel Access Handpiece with BoneScalpel Access Probe Kits Indicated for use in the fragmentation and aspiration of soft and hard (e.g., bone) tissue in the following surgical specialties:

- Neurosurgery
- · Gastrointestinal and Affiliated Organ Surgery
- Urological Surgery
- Plastic and Reconstructive Surgery
- General Surgery
- Orthopedic Surgery
- Gynecology

External genitalia - condyloma - benign tumors (lipomas, fibromas, and leiomyomas) - malignant primary and metastatic tumors of all types and the following cystic lesions: Bartholin's cysts, Vestibular adenitis, Inclusion cysts, Sebaceous cysts Abdominal area - any abnormal growth, cystic or solid, benign or malignant, involving the ovary, fallopian tube, uterus, or the supporting structures of the uterus except as contraindicated for uterine fibroids.

Thoracic Surgery

Limited pulmonary resection such as segmentectomies, nonanatomical subsegmentectomies and metastatectomies.

SonaStar Elite Handpiece with SonaStar Elite Probe Kits

Indicated for use in the fragmentation and aspiration of soft and hard (e.g., bone) tissue in the following surgical specialties:

- Neurosurgery
- Gastrointestinal and Affiliated Organ Surgery including removal of benign or malignant tumors or other unwanted tissue, including hepatic parenchyma, in open or laparoscopic procedures, hepatic resection, tumor resection, lobectomy or trisegmentectomy, or removal of tissue during liver allotransplantation and donor hepatectomy
- Urological Surgery including removal of renal parenchyma during nephrectomy or partial nephrectomy
- Plastic and Reconstructive Surgery
- General Surgery including removal of benign or malignant tumors or other unwanted tissue in open or minimally invasive general surgical procedures
- Orthopedic Surgery
- Gynecological Surgery except as contraindicated for uterine fibroids.
- Thoracic Surgery
- Laparoscopic Surgery including removal of hepatic parenchyma in laparoscopic hepatic resection, lobectomy or trisegmentectomy, in laparoscopic donor hepatectomy or laparoscopic cholecystectomy or laparoscopic pancreatic jejunostomy, or pancreatectomy, or laparoscopic appendectomy, laparoscopic colon resection or laparoscopic partial gastrectomy
- Thoracoscopic Surgery

The system may also be combined with electrosurgery using optional RF surgery interface components.

Type of Use (Select one or both, as applicable)		
Prescription Use (Part 21 CFR 801 Subpart D)	Over-The-Counter Use (21 CFR 801 Subpart C)	

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services Food and Drug Administration Office of Chief Information Officer Paperwork Reduction Act (PRA) Staff PRAStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

1. SUBMITTER

Applicant: Misonix

1938 New Highway

Farmingdale, NY 11735

Contact: John Salerno

Vice President of Regulatory Affairs and Quality

Assurance Misonix

1938 New Highway

Farmingdale, NY 11735

Main: 631-694-9555 Direct: 631-927-9123

john.salerno@bioventus.com

Submission Correspondent:

John Salerno

Vice President of Regulatory Affairs and Quality

Assurance

Misonix

1938 New Highway

Farmingdale, NY 11735

Main: 631-694-9555 Direct: 631-927-9123

john.salerno@bioventus.com

Date Prepared: June 22, 2023

2. DEVICE

Device Trade Name: neXus Ultrasonic Surgical Aspirator System

Device Common Name: Instrument, ultrasonic surgical Review Panel: General and Plastic Surgery

Classification Name Unclassified Regulatory Class: Unclassified

Product Code: LFL, GEI

3. PREDICATE DEVICE

Predicate Device: neXus Ultrasonic Aspirator System with SonaStar Elite Handpiece (K221235)

The predicate device has not been subject to a design-related recall.

4. DEVICE DESCRIPTION

The neXus Ultrasonic Surgical Aspirator System is intended for fragmentation, emulsification and aspiration of both soft and hard (i.e.bone) tissue. The system includes a generator housed inside the console. A reusable handpiece is plugged directly into the front panel of the console.

The generator and handpiece are compatible with various single use disposable "probes" which are selected and attached to the handpiece by the end user. An irrigation system provides sterile irrigant to the operative site. An aspiration system removes the fragmented, emulsified material and waste liquids from the operative site.

Accessories include a wireless footswitch, various probe tip combinations, sterilization trays, probe covers, assembly & disassembly wrenches, irrigation & aspiration tubing sets, disposable electrocautery cable, and waste collection canisters.

5. INDICATIONS FOR USE

Indications for Use Statement:

The Misonix Inc. neXus Ultrasonic Surgical Aspirator System is intended for the fragmentation, emulsification and aspiration of both soft and hard (i.e., bone) tissue.

The indications for use for the Standard Handpiece in combination with BoneScalpel® and SonicOne® OR probe kit accessory configurations, the SonaStar® long and short handpieces in combination with SonaStar® probe kit accessory configurations, the BoneScalpel AccessTM handpiece with BoneScalpel AccessTM probe kit accessory configurations, and the SonaStar Elite Handpiece with probe kit accessory configurations are listed below.

Standard Handpiece with BoneScalpel Probe Kits

Indicated for use in the fragmentation, emulsification and aspiration of soft and hard (e.g., bone) tissue in the following surgical specialties:

- Neurosurgery
- Gastrointestinal and Affiliated Organ Surgery
- Urological Surgery
- Plastic and Reconstructive Surgery
- General Surgery
- Orthopedic Surgery

• Gynecology

External genitalia - condyloma - benign tumors (lipomas, fibromas, and leiomyomas) - malignant primary and metastatic tumors of all types and the following cystic lesions: Bartholin's cysts, Vestibular adenitis, Inclusion cysts, Sebaceous cysts

Abdominal area - any abnormal growth, cystic or solid, benign or malignant, involving the ovary, fallopian tube, uterus, or the supporting structures of the uterus except as contraindicated for uterine fibroids.

• Thoracic Surgery

Limited pulmonary resection such as segmentectomies, nonanatomical subsegmentectomies and metastatectomies.

• Wound Care

The neXus Ultrasonic Surgical Aspirator is also indicated for use in the debridement of wounds, such as, but not limited to, burn wounds, diabetic ulcers, bedsores and vaginal ulcers, soft tissue debridement and cleansing of the surgical site in applications in which, in the physician's judgment would require the use of an ultrasonic aspirator with sharp debridement.

Standard Handpiece with SonicOne Probe Kits

Indicated for use in the fragmentation, emulsification and aspiration of soft and hard tissue (i.e. bone) in the following surgical specialty:

• Wound Care

The neXus Ultrasonic Surgical Aspirator is also indicated for use in the debridement of wounds, such as, but not limited to, burn wounds, diabetic ulcers, bedsores and vaginal ulcers, soft tissue debridement and cleansing of the surgical site in applications in which, in the physician's judgment would require the use of an ultrasonic aspirator with sharp debridement.

• Plastic and Reconstructive Surgery

neXus SonaStar Handpieces with SonaStar Probe Kits

- Indicated for use in the fragmentation, emulsification and aspiration of both soft and hard (i.e., bone) tissue in the following surgical specialties:
- Neurosurgery -Gastrointestinal and Affiliated Organ Surgery including removal of benign or malignant tumors or other unwanted tissue, including hepatic parenchyma, in open or laparoscopic procedures, hepatic resection, tumor resection, lobectomy or trisegmentectomy, or removal of tissue during liver allotransplantation and donor hepatectomy
- **Urological Surgery** including removal of renal parenchyma during nephrectomy or partial nephrectomy

- Plastic and Reconstructive Surgery
- **General Surgery** including removal of benign or malignant tumors or other unwanted tissue in open or minimally invasive general surgical procedures
- Orthopedic Surgery
- **Gynecological Surgery** except as contraindicated for uterine fibroids.
- Thoracic Surgery
- Laparoscopic Surgery including removal of hepatic parenchyma in laparoscopic hepatic resection, lobectomy or trisegmentectomy, in laparoscopic donor hepatectomy or laparoscopic cholecystectomy or laparoscopic pancreatic jejunostomy, or pancreatectomy, or laparoscopic appendectomy, laparoscopic colon resection or laparoscopic partial gastrectomy
- Thoracoscopic Surgery

The SonaStar Handpieces may also be combined with electrosurgery using optional RF surgery interface components.

Bone Scalpel Access Handpiece with BoneScalpel Access Probe Kits

Indicated for use in the fragmentation and aspiration of soft and hard (e.g.: bone) tissue in the following surgical specialties:

- Neurosurgery
- Gastrointestinal and Affiliated Organ Surgery
- Urological Surgery
- Plastic and Reconstructive Surgery
- General Surgery
- Orthopedic Surgery
- Gynecology

External genitalia - condyloma - benign tumors (lipomas, fibromas, and leiomyomas) - malignant primary and metastatic tumors of all types and the following cystic lesions: Bartholin's cysts, Vestibular adenitis, Inclusion cysts, Sebaceous cysts

Abdominal area - any abnormal growth, cystic or solid, benign or malignant, involving the ovary, fallopian tube, uterus, or the supporting structures of the uterus except as contraindicated for uterine fibroids.

Thoracic Surgery

Limited pulmonary resection such as segmentectomies, nonanatomical subsegmentectomies and metastatectomies.

SonaStar Elite Handpiece with SonaStar Elite Probe Kits

Indicated for use in the fragmentation and aspiration of soft and hard (e.g., bone) tissue in the following surgical specialties:

- Neurosurgery
- Gastrointestinal and Affiliated Organ Surgery including removal of benign or
 malignant tumors or other unwanted tissue, including hepatic parenchyma, in open or
 laparoscopic procedures, hepatic resection, tumor resection, lobectomy or
 trisegmentectomy, or removal of tissue during liver allotransplantation and donor
 hepatectomy
- **Urological Surgery** including removal of renal parenchyma during nephrectomy or partial nephrectomy
- Plastic and Reconstructive Surgery
- **General Surgery** including removal of benign or malignant tumors or other unwanted tissue in open or minimally invasive general surgical procedures
- Orthopedic Surgery
- **Gynecological Surgery** except as contraindicated for uterine fibroids.
- Thoracic Surgery
- Laparoscopic Surgery including removal of hepatic parenchyma in laparoscopic hepatic resection, lobectomy or trisegmentectomy, in laparoscopic donor hepatectomy or laparoscopic cholecystectomy or laparoscopic pancreatic jejunostomy, or pancreatectomy, or laparoscopic appendectomy, laparoscopic colon resection or laparoscopic partial gastrectomy
- Thoracoscopic Surgery

The system may also be combined with electrosurgery using optional RF surgery interface components.

6. SUBSTANTIAL EQUIVALENCE

Comparison of Indications

The indications for use are identical.

Technological Comparisons

The table below compares the key technological features of the subject devices to the predicate device (K221235, Misonix, a Bioventus Company, neXus Ultrasonic Aspirator System with SonaStar Elite Handpiece).

Table 1: Technological Comparison

	Predicate Device (K221235)	Subject Device
510(k) Number	K221235	TBD
Company	Misonix, a Bioventus company	Misonix, a Bioventus company
Device Name	neXus Ultrasonic Surgical Aspirator System	neXus Ultrasonic Surgical Aspirator System
Product Code	LFL	LFL
	Subsequent product code: GEI	Subsequent product code: GEI
Indications for Use	See Section 5	See Section 5
Handpieces	neXus® Standard Handpiece [100-21-0001]	Identical
	neXus® SonaStar® Short Handpiece [100-24-0001]	Identical
	neXus® SonaStar® Long Handpiece [100-25-0001]	Identical
	neXus® BoneScalpel Access™ Handpiece [100-22-0001]	Identical
	SonaStar Elite (SSE) Handpiece [100-26-0001]	Identical
Principle of Operation	Handpiece-probe assembly vibrate at resonant frequency resulting from the conversion of the electrical drive signal from the neXus console into mechanical vibrations by the piezo stack.	Identical
Materials of	Front Driver: Titanium alloy	Identical
Construction	Piezo ceramics: PZT (lead zirconate titanate)	Identical
	Housing: PPSU	Identical
	Cable: Electrical conductors, electrical shield, silicone jacket	Identical
Reusable Accessories	100-60-0000 Standard Handpiece Counter Wrench	Identical
	100-61-0000 Standard Handpiece Torque Wrench	Identical
	100-62-0000 SonaStar Handpiece Counter Wrench	Identical
	100-63-0000 Handpiece Torque Wrench	Identical

Predicate Device (K221235)	Subject Device
100-64-0000 Access Elite Counter Wrench	Identical
100-70-0000 Standard Sterilization Tray	Identical
100.71.0000	Identical
100-71-0000 SonaStar Sterilization Tray	Identical
100-72-0000 BoneScalpel Access Sterilization Tray	
	Identical
100-73-0000 SonaStar Elite Sterilization Tray	Identical
100-21-0002 Standard Front Housing (BoneScalpel)	Identical
100-21-0003 Standard Front Housing (SonicOne)	Identical
100-24-0002 SonaStar Long Housing (Short)	Identical
100-24-0003 SonaStar Short Housing (Short)	Identical
100-24-0004 SonaStar Lap Housing (Short)	Identical
100-25-0002 SonaStar Mid Housing (Long)	Identical
100-25-0003 SonaStar Front Housing (Long)	Identical
100-25-0002 SonaStar Mid Housing (Long)	Identical
100-26-0003 SonaStar Elite Front Housing Smooth	

	Predicate Device (K221235)	Subject Device
Reprocessing	Cleaning and Sterilize as per 100-21-1000	Identical
(Cleaning & Sterilization or Reusable	Standard Handpiece IFU	
Accessories)	Clean and Sterilize as per 100-22-1000 BoneScalpel Access Handpiece IFU	Identical
	Clean and Sterilize as per 100-24-1000	Identical
	SonaStar Handpiece IFU	
	Clean and Sterilize as per 100-26-1000 SonaStar Elite Handpiece IFU	Identical
Probes & Probe Tip	S	
Probes cleared in K190160	110-31-1110 BoneScalpel 10mm, Blunt Blade	Identical
	110-31-1120 BoneScalpel 20mm, Blunt Blade	Identical
	110-31-1125 BoneScalpel 25mm, Blunt Blade	Identical
	110-31-1121 BoneScalpel 20mm, Uni-lateral Serrated Blade	Identical
	110-31-2110 BoneScalpel 10mm MIS, Blunt Blade	Identical
	110-31-2120 BoneScalpel 20mm MIS, Blunt Blade	Identical
	150-32-2120: BoneScalpel Access MIS 20mm Blade & Tubeset	Identical
	110-31-1210 BoneScalpel Micro Hook Shaver	Identical
	110-31-1220 BoneScalpel Macro Hook Shaver	Identical
	110-31-1230 BoneScalpel Diamond Shaver	Identical
	110-31-2210 BoneScalpel Micro Hook MIS Shaver	Identical
	110-31-5501 Standard Decompression Kit (20mm Blade, microhook shaver & tubeset)	Identical
	130-35-1411 SonaStar 1.1 mm Precision Short Tip & Tubeset	Identical

	Predicate Device (K221235)	Subject Device
	130-35-1416	Identical
	SonaStar 1.6 mm Micro Short Tip & Tubeset	
	130-35-1419	Identical
	SonaStar 1.9 mm Standard Short Tip and Tubeset	
	130-33-2411	Identical
	SonaStar 1.1 mm Precision Long Curved Tip & Tubeset	
	130-33-2416	Identical
	SonaStar 1.6 mm Micro Long Curved Tip & Tubeset	
	130-33-2419	Identical
	SonaStar 1.9 mm Standard Long Curved Tip & Tubeset	
	130-35-1499 SonaStar 1.9 mm Standard Short Tip & Tubeset	Identical
	130-33-3419	Identical
	SonaStar Deep Access, 1.9 mm Standard Long Tip & Tubeset	
	130-35-1220	Identical
	SonaStar Cylindrical Shaver Short Tip with Aspiration & Tubeset	
	130-33-2210	Identical
	Micro Hook Shaver Long Tip with Aspiration and Tubeset	
	120-31-10X1	Identical
	SonicOne Hatched Probe & Tubeset	
	120-31-13X2 SonicOne SonicVac & Tubeset	Identical
	12931-13C2 SonicOne SharpVac & Tubeset	Identical
	120-31-10R1	Identical
	SonicOne Cylindrical Probe & Tubeset	
Probes cleared in	150-32-2120:	Identical
K212060	BoneScalpel Access MIS 20mm Blade & Tubeset	
	150-32-2120:	Identical
	BoneScalpel Access MIS Micro Hook Shaver & Tubeset	
	130-35-5502	Identical
	SonaStar Shaver, Aspirator & Tubeset Kit	
	120-31-13C3	Identical
	SonicOne Curette Tip Excel Kit	
Probes cleared in K221235	130-26-2416 SonaStar Elite 1.6mm Micro Long Tip & Tubeset	Identical

	Predicate Device (K221235)	Subject Device
	130-26-2498	Identical
	SonaStar Elite 1.6mm Notched Long Tip & Tubeset	
	130-26-2419	Identical
	SSE 1.9mm Standard Long Tip & Tubeset	
	130-26-2499	Identical
	SonaStar Elite 1.9mm Notched Long Tip & Tubeset	
Cleaning and Sterilization of Sterile Single Use Devices	EtO Sterilization Identical Sterilization Cycles	Identical
Shelf-Life Sterile Single Use Devices	3 Years Identical Sterile Barrier Packaging	Identical
Electrosurgery	,	-
Electrosurgery	100-29-0000	Identical
Accessory	SonaStar Monopolar Handswitch Cable	
Console		
P/N	100-10-0000	Identical
	neXus Console	
Classification	Class 1	Identical
	Type BF Applied Part	
Power Input Voltage	100-240 VAC	Identical
Power Input Current	5 A max	Identical
Power Input Frequency	50/60Hz	Identical
Ground Leakage	500 μA (max.)	Identical
Functions	Vibration	Identical
	Irrigation	Identical
	Aspiration	Identical
Vibration System	Continuous Wave	Identical Pulsed Wave, NEW
	Frequency:	Frequency:
	22.0-24.5 kHz / Amplitude: up to 355 microns	Identical
	34.5-37.0 kHz / Amplitude: up to 206 microns	Identical
Irrigation pump	Peristaltic pump	Identical

	Predicate Device (K221235)	Subject Device
Irrigation Flow	Standard Handpiece	Standard Handpiece
Rate	Min: 12-18 ml/min – Max: 67-85 ml/min	Identical
	Flush: 67-85 ml/min	
	SonaStar Long and Short Handpieces	SonaStar Long and Short Handpieces
	Min: 1-3 ml/min – Max: 9-14 ml/min	Identical
	Flush: 25-29ml/min	
	Bone Scalpel Access Handpiece	Bone Scalpel Access Handpiece
	Min: 1-3 ml/min – Max: 67-85 ml/min	Identical
	Flush: 67-85 ml/min	
	SonaStar Elite Handpiece	SonaStar Elite Handpiece
	Min: 1-3 ml/min – Max: 17-23 ml/min	Identical
	Flush: 25-29ml/min	
Vacuum Pump	Dual Vacuum Head Pump, 24V	Identical
Vacuum	Min: 2.0 inHg or lower	Identical
Specification	Max: 25 inHg	Identical
	Vacuum Sleep Mode: 0 inHg	Identical
Footswitch	100-50-0000	Identical
	neXus Wireless Footpedal	
	100-51-0000	Identical
	neXus Wired Footpedal	
Console Display	neXus Console with touch screen graphical user interface	Identical

		T
Console GUI	Standard Handpiece	Standard Handpiece
	DEFAULT SETTINGS	DEFAULT SETTINGS
	Vibration: 70	Identical
	Irrigation: 70	Identical
	Aspiration: Disabled	Identical
	Lap Endo Mode: Disabled	Identical
	USER SETTINGS	USER SETTINGS
	Vibration:	Vibration:
	Incremental adjustment = 5	Identical
	Range = 5 to 100%	Identical
	Irrigation:	Irrigation:
	Incremental adjustment = 5	Identical
	Range = 5 to 100%	Identical
	Aspiration:	Aspiration:
	Incremental adjustment = 5	Identical
	Range = 5 to 100%	Identical
	SonaStar Handpieces	SonaStar Handpieces
	DEFAULT SETTINGS	DEFAULT SETTINGS
	Vibration: 50	Identical
	Irrigation: 50	Identical
	Aspiration: 50	Identical
	Lap Endo Mode: Off	Lap Endo Mode: Identical
	USER SETTINGS	USER SETTINGS
	Vibration:	Vibration:
	Incremental adjustment = 5	Identical
	Range = 5 to 100%	Identical
	Irrigation:	Irrigation:
	Incremental adjustment = 5	Identical
	Range = 5 to 100%	Identical
	Aspiration:	Aspiration:
	Incremental adjustment = 5	Identical5
	Range = 5 to 100%	Identical
	BoneScalpel Access Handpiece	BoneScalpel Access Handpiece
	DEFAULT SETTINGS	DEFAULT SETTINGS
	Vibration: 70	Identical
	Irrigation: 70	Identical
	Aspiration: 50	Identical
	Lap Endo Mode: On	Lap Endo Mode: Identical
		•
	USER SETTINGS	USER SETTINGS
	Vibration:	Vibration:
	Incremental adjustment = 5	Identical
	Range = 5 to 100%	Identical
	Irrigation:	Irrigation:
	Incremental adjustment = 1	Identical
	Range = 1 to 20%	Identical
	1 to 2070	Tacifficat

	Predicate Device (K221235)	Subject Device
	Incremental adjustment = 5	Identical
	Range = 20 to 100%	Identical
	Aspiration:	Aspiration:
	Incremental adjustment = 5	Identical
	Range = 5 to 100%	Identical
	SonaStar Elite Handpiece	SonaStar Elite Handpiece
	DEFAULT SETTINGS	DEFAULT SETTINGS
	Vibration: 60	Identical
	Irrigation: 25	Identical
	Aspiration: 60	Identical
	Lap Endo Mode: Off	Lap Endo Mode: Identical
		DTC (DYNAMIC TISSUE RESPONSE): Off
	USER SETTINGS	USER SETTINGS
	Vibration:	Vibration:
	Incremental adjustment = 5	Incremental adjustment = 5
	Range = 5 to 100%	Range = 5 to 100%
	Irrigation:	Irrigation:
	Incremental adjustment = 5	Incremental adjustment = 5
	Range = 5 to 100%	Range = 5 to 100%
	Aspiration:	Aspiration:
	Incremental adjustment = 5	Incremental adjustment = 5
	Range = 5 to 100%	Range = 5 to 100%
		DTC (DYNAMIC TISSUE RESPONSE):
		Incremental adjustment = 1
		Range = 1 to 6
Console Software	v1.9 RC2	v1.10 RC0
Dimensions	11.5" H x 16" W x 17" D	Identical
wo/Cart	292mm H x406 mm W x 432mm D	
Weight wo/Cart	45 lbs. 20.4 kg	Identical

7. PERFORMANCE DATA

Biocompatibility Testing

Not applicable. There have been no modifications related to patient contacting materials, therefore the biocompatibility testing submitted under K221235 remains valid.

Electrical safety and electromagnetic compatibility (EMC)

No re-testing required. The change to the neXus console is considered a minor software update only. The safety and performance of the device has been validated and verified through software testing in accordance with IEC62304. There have been no device modifications related to the construction of the console or handpiece, therefore the electrical safety and EMC testing submitted under K221235 remains valid.

Software Verification and Validation Testing

Software verification and validation testing was conducted and a summary of testing provided. The software classification for this device is unchanged as a major level of concern. The following verification tests were conducted:

- DTC DSP pulse mode verification
- DTC GUI verification
- Fault detection and response test using the SonaStar Elite handpiece
- neXus test and calibration (Regression testing)
- System performance verification (Regression testing)
- SonaStar Elite GUI verification (Regression testing)

Bench Testing

The following tests were conducted to support the claim of substantial equivalence:

- Tissue removal test using the SonaStar Elite handpiece
- DTC mode vibration test using the SonaStar Elite handpiece

8. CONCLUSION

Subject device modifications do not raise new questions of safety or effectiveness. Technological comparison to the predicate and performance testing on the subject device demonstrates substantial equivalence to the predicate device for the requested intended use.