



August 9, 2024

Suzhou Ideal Medical Co., Ltd.
% Jarvis Wu
Senior Consultant
Shanghai SUNGO Management Consulting Co., Ltd.
14th Floor, Dongfang Building, 1500# Century Ave
Shanghai, Shanghai 200122
China

Re: K240079

Trade/Device Name: Electric Wheelchair (Panda 10); Electric Wheelchair (Panda 12)
Regulation Number: 21 CFR 890.3860
Regulation Name: Powered Wheelchair
Regulatory Class: Class II
Product Code: ITI
Dated: July 19, 2024
Received: July 19, 2024

Dear Jarvis Wu:

We have reviewed your section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (the Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database available at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Additional information about changes that may require a new premarket notification are provided in the FDA guidance documents entitled "Deciding When to Submit a 510(k) for a Change to an Existing Device" (<https://www.fda.gov/media/99812/download>) and "Deciding When to Submit a 510(k) for a Software Change to an Existing Device" (<https://www.fda.gov/media/99785/download>).

Your device is also subject to, among other requirements, the Quality System (QS) regulation (21 CFR Part 820), which includes, but is not limited to, 21 CFR 820.30, Design controls; 21 CFR 820.90, Nonconforming product; and 21 CFR 820.100, Corrective and preventive action. Please note that regardless of whether a change requires premarket review, the QS regulation requires device manufacturers to review and approve changes to device design and production (21 CFR 820.30 and 21 CFR 820.70) and document changes and approvals in the device master record (21 CFR 820.181).

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR Part 803) for devices or postmarketing safety reporting (21 CFR Part 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR Part 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR Parts 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,


Tushar Bansal -S

for Heather Dean, PhD

Assistant Director, Acute Injury Devices Team

DHT5B: Division of Neuromodulation and

Physical Medicine Devices
OHT5: Office of Neurological and
Physical Medicine Devices
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

Indications for Use

Submission Number (if known)

K240079

Device Name

Electric Wheelchair (Panda 10);
Electric Wheelchair (Panda 12)

Indications for Use (Describe)

It is a motor driven, indoor and outdoor transportation vehicle with the intended use to provide mobility to a disabled or elderly person limited to a seated position.

Type of Use (Select one or both, as applicable)

☐

Prescription Use (Part 21 CFR 801 Subpart D)

☒

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(K) Summary

K240079

Preparation date: July 19, 2024

A. Applicant:

SUZHOU IDEAL MEDICAL CO., LTD.

Address: No.88 Huachi Street, Suzhou Industrial Park, Suzhou (215000), Jiangsu, China.

Contact Person: Hu Qi

Tel: +86-13912803601

Submission Correspondent: Primary contact: Mr. Jarvis Wu Title: Senior Consultant

Shanghai SUNGO Management Consulting Co., Ltd. 14th Floor, Dongfang Building, 1500#

Century Ave., Shanghai 200122, China

Tel: +86-21-58817802

Email: jiawei.wu@sungoglobal.com

Secondary contact: Mr. Raymond Luo Title: Technical Director

Shanghai SUNGO Management Consulting Co., Ltd. 14th Floor, Dongfang Building, 1500#

Century Ave., Shanghai 200122, China

Tel: +86-21-68828050

Email: zxfda@sungoglobal.com

B. Device:

Trade Name: Electric Wheelchair Panda 10, Panda 12

Common Name: Powered wheelchair

Regulatory Information

Classification Name: Powered wheelchair

Classification: Class II.

Product code: ITI

Regulation Number: 890.3860

Review Panel: Physical Medicine

C. Predicate device:

510K number: K220747

Device Name: Power Wheelchair Model: N5515B

Zhejiang Innuovo Rehabilitation Devices Co.,Ltd

D. Indications for use of the device:

It is a motor driven, indoor and outdoor transportation vehicle with the intended use to provide mobility to a disabled or elderly person limited to a seated position.

E. Device Description:

This Electric Wheelchair, models: Panda 10 and Panda 12, is a motor driven, indoor and outdoor transportation vehicle, which is a device for assisting action handicapped people and

disabled people to move. It is suitable for disabled people with mobility difficulties and elderly people.

The device consists of front wheel, drive wheel, frame, controller, motor, armrest, backrest, seat cushion, pedal, battery box and charger.

The device is powered by Li-ion Battery pack with 9.7 Km range, which can be recharged by an off-board battery charger that can be plugged into an AC socket outlet (100-240V, 50/60Hz) when the device is not in use.

The patient can activate the controller handle (joystick) to control the speed and direction of the wheelchair movement. In addition, when the patient releases the joystick, the joystick will return back to the central position and the wheelchair will be automatically stopped soon due to automatic electromagnetic brake system. Once the joystick is activated again move to other position, the wheelchair will be re-energized.

F. Non-Clinical Test Conclusion

Non-clinical tests were conducted to verify that the proposed device met all design specifications as was Substantially Equivalent (SE) to the predicate device. The test results demonstrated that the proposed device complies with the following standards:

- ISO 10993-5: 2009 Biological Evaluation of Medical Devices -- Part 5: Tests For In Vitro Cytotoxicity
- ISO 10993-10: 2010 Biological Evaluation of Medical Devices - Part 10: Tests For Irritation And Skin Sensitization
- ISO 7176-1: 2014, Wheelchairs - Part 1: Determination of static stability
- ISO 7176-2:2017, Wheelchairs - Part 2: Determination of dynamic stability of Powered Wheelchairs
- ISO 7176-3: 2012, Wheelchairs - Part 3: Determination of effectiveness of brakes
- ISO 7176-4, Third edition 2008-10-01, Wheelchairs - Part 4: Energy consumption of electric wheelchairs and wheelchairs for determination of theoretical distance range
- ISO 7176-5, Second edition 2008-06-01, Wheelchairs - Part 5: Determination of overall dimensions, mass and manoeuvring space
- ISO 7176-6: 2018, Wheelchairs - Part 6: Determination of maximum speed, acceleration and deceleration of Powered Wheelchairs
- ISO 7176-7, Wheelchairs - Part 7: Measurement of seating and wheel dimensions
- ISO 7176-8:2014, Wheelchairs -Part 8: Requirements and test methods for static, impact and fatigue strengths
- ISO 7176-9:2009, Wheelchairs - Part 9: Climatic tests for Powered Wheelchairs
- ISO 7176-10:2008, Wheelchairs - Part 10: Determination of obstacle-climbing ability of electrically powered wheelchairs
- ISO 7176-11:2012 Wheelchairs - Part 11: Test dummies.
- ISO 7176-13, First edition 1989-08-01, Wheelchairs - Part 13: Determination of coefficient of friction of test surfaces

- ISO 7176-14:2008, Wheelchairs - Part 14: Power and control systems for electrically powered wheelchairs and wheelchairs - Requirements and test methods
- ISO 7176-15:1996, Wheelchairs - Part 15: Requirements for information disclosure, documentation and labeling
- ISO 7176-21:2009 Wheelchairs - Part 21: Requirements and test methods for electromagnetic compatibility of electrically powered wheelchairs and wheelchairs, and battery chargers
- ISO 7176-25:2013 Wheelchairs - Part 25: Batteries and chargers for powered wheelchairs
- ISO 16840-10:2021 Wheelchairs - Part 16: Resistance to ignition of postural support devices.

G. Clinical Test Conclusion

No clinical study is included in this submission.

H. Comparison with predicate Device

Table 1 General Comparison

Elements of Comparison	Subject Device	Predicate Device (K220747)	Remark
Manufacturer	SUZHOU IDEAL MEDICAL CO., LTD.	Zhejiang Innuovo Rehabilitation Devices Co.,Ltd	-
Common or Usual name	Power Wheelchair	Power Wheelchair	Same
Model(s)	Panda 10 & Panda 12	N5515B	--
Indications for use	It is a motor driven, indoor and outdoor transportation vehicle with the intended use to provide mobility to a disabled or elderly person limited to a seated position.	It is a motor driven, indoor and outdoor transportation vehicle with the intended use to provide mobility to a disabled or elderly person limited to a seated position.	S.E.
Use condition	indoor and outdoor use	indoor and outdoor use	S.E
Number of wheels	4,including two front wheels and two rear Wheels	4,including two front wheels and two rear Wheels	S.E
Function of wheels	Front wheels: driven wheels suitable for rotation, acceleration, retrograde. Rear wheels: driving wheels to control the speed and direction	Front wheels: driven wheels suitable for rotation, acceleration, retrograde. Rear wheels: driving wheels to control the speed and direction	S.E
Movement control	By Joystick control	By Joystick control	S.E

method			
Driving system	Direct drive on the rear wheels	Direct drive on the rear wheels	S.E
Brake system	Automatic electromagnetic brake system	Automatic electromagnetic brake system	S.E
Braking distance	$\leq 1.5 \text{ m}$	$\leq 1.5 \text{ m}$	S.E
Maximum safe operational incline degree	10°	9 °	S.E
Armrest	Mesh Fabric(Polyester & Spandex)	PU	Analysis
Battery charger	Off-board charger Input: 100-240V, 50/60Hz, 1.5A, Output: 24 Vdc, 2A;	Off-board charger Input: 100-240V, 50/60Hz, 1.5A, Output: 24 Vdc, 2A;	Analysis
Main frame material	Aluminum Alloy	Carbon fiber material	Analysis
Back cushion	Mesh Fabric(Polyester & Spandex)	Polyester fabric	Analysis
Seat cushion	Mesh Fabric(Polyester & Spandex)	rubber patch cloth and Oxford fabric	Analysis
Overall Dimension (length*width*height)	Panda 10 model: 1000*630*880mm Panda 12 model: 1065*630*930mm	940*610*960mm	Analysis
Folded Dimension (length*width*height)	Panda 10 model:730*630*410mm Panda 10 model:770*630*410mm	720*310*610mm	Analysis
Front wheel size/type	Panda 10 model: 7" x 2"/PU Solid tire Panda 12 model: 8" x 2"/PU Solid tire	7" x 1.75"/PU Solid tire	Analysis

Rear wheel size/type	Panda 10 model: 10" x 2.4"/PU Solid tire Panda 12 model: 12" x 2.3"/ Rubber Pneumatic tire	8.5"x 2"/ PU Solid tire	Analysis
Max speed forward	Up to adjustable 6.8 km/h (1.9 m/s),	Up to 6 adjustable km/h (1.6 m/s)	S.E
Max Speed backward	Less than 3.2 km/h (0.9 m/s)	Less than 3 km/h (0.5 m/s)	S.E
Max loading weight	120Kg	136kg	S.E
Battery	Lithium 24V10Ah	Li-ion battery pack; rechargeable, 24 VDC 12Ah	S.E
Maximum distance of travel on the fully charged battery	9.7km	15 km	Analysis
Motor	Brushless DC motor; 24VDC;	Brushless DC motor; 24VDC;	S.E
	220W; 2pcs	250W; 2pcs	
Electronic controller	Brushless controller	Brushless dual-drive rocker controller	S.E
Turning Radius	1070 mm	900 mm	S.E
Maximum obstacle climbing	20 mm	40 mm	Analysis

I. Difference analysis

The design and technological characteristics of the Electric Wheelchair is similar to the predicates chosen. There are minor differences between the devices including Armrest, Seat & Back cushion material, Main frame material, Overall Dimension, Folded Dimension, Wheel size/type, Maximum distance of travel on the fully charged battery and Turning Radius. All of the parameter with difference have been tested according to ISO7176 series standards and the test records support its safety and effectiveness. There is no deleterious effect on safety and effectiveness due to the minor differences do not influence the intended use of the device. Therefore, the proposed Wheelchair is substantially equivalent (SE) to the Power Wheelchair

(K220747).

**Table 2 Safety
comparison**

Item	Proposed Device	Predicate Devices	Results
Biocompatibility	All user directly contacting materials are compliance with ISO10993-5, ISO10993-10 and ISO10993-23 requirements. (Having long-term skin contact of greater than 30 days)	All user directly contacting materials are compliance with ISO10993-5 and ISO10993-10 requirements.	S.E.
EMC	ISO7176-21 & IEC 60601-1-2	ISO7176-21	S.E.
Performance	ISO7176 series	ISO7176 series	S.E.
Label and labeling	Conforms to FDA Regulatory	Conforms to FDA Regulatory	S.E.

**Table 3 Safety
comparison**

Item	Proposed Device	Predicate Devices	Results
ISO7176-1	The Static stability has been determined after the testing according to the ISO 7176-1, and test results meet its design specification.	The Static stability has been determined after the testing according to the ISO 7176-1, and test results meet its design specification.	S.E.
ISO7176-2	The dynamic stability has been determined after the testing according to the ISO 7176-2, and test results meet its design specification.	The dynamic stability has been determined after the testing according to the ISO 7176-2, and test results meet its design specification.	S.E.
ISO7176-3	The effectiveness of brakes has been determined after the testing according to the ISO 7176-3, and test results meet its design specification.	The effectiveness of brakes has been determined after the testing according to the ISO 7176-3, and test results meet its design specification.	S.E.
ISO7176-4	The theoretical distance range has been determined after the testing according to the ISO 7176-4, and test results meet its design specification.	The theoretical distance range has been determined after the testing according to the ISO 7176-4, and test results meet its design specification.	S.E.

ISO7176-5	The dimensions, mass has been determined after the testing according to the ISO 7176-5.	The dimensions, mass has been determined after the testing according to the ISO 7176-5.	S.E.
ISO7176-6	The dimensions, mass has been determined after the testing according to the ISO 7176-6.	The dimensions, mass has been determined after the testing according to the ISO 7176-6.	S.E.
ISO7176-7	The seating and wheel dimensions has been determined after the testing according to the ISO 7176-7.	The seating and wheel dimensions has been determined after the testing according to the ISO 7176-7.	S.E.
ISO7176-8	All test results meet the requirements in Clause 4 of ISO 7176-8.	All test results meet the requirements in Clause 4 of ISO 7176-8.	S.E.
ISO7176-9	The test results shown that the device under tests could continue to function according to manufacturer's specification after being subjected to each of the tests specified in Clause 8 of ISO 7176-9.	The test results shown that the device under tests could continue to function according to manufacturer's specification after being subjected to each of the tests specified in Clause 8 of ISO 7176-9.	S.E.
ISO7176-10	The obstacle-climbing ability of device has been determined after the testing according to the ISO 7176-10.	The obstacle-climbing ability of device has been determined after the testing according to the ISO 7176-10.	S.E.
ISO7176-11	The test dummies used in the testing of ISO 7176 series are meet the requirements of ISO 7176-11.	The test dummies used in the testing of ISO 7176 series are meet the requirements of ISO 7176-11.	S.E.
ISO7176-13	The coefficient of friction of test surfaces has been determined, which could be used in other 7176 series tests involved.	The coefficient of friction of test surfaces has been determined, which could be used in other 7176 series tests involved.	S.E.
ISO7176-14	All test results meet the requirements in Clause 7, 8, 9, 10, 11, 12, 13, 14, 15, 17 of ISO 7176-14.	All test results meet the requirements in Clause 7, 8, 9, 10, 11, 12, 13, 14, 15, 17 of ISO 7176-14.	S.E.
ISO7176-15	The test results shown that information disclosure, documentation and labelling of device meet the requirements of ISO 7176-15.	The test results shown that information disclosure, documentation and labelling of device meet the requirements of ISO 7176-15.	S.E.

ISO 16840-10	The performance of resistance to ignition meet the requirements of ISO 16840-10	The performance of resistance to ignition meet the requirements of ISO 7176-16.	S.E.
ISO 7176-21	The EMC performance results meet the requirements of ISO 7176-21.	The EMC performance results meet the requirements of ISO 7176-21.	S.E.
ISO7176-25	The performance of batteries and charger of device meet the Requirements in Clause 5 and 6 of ISO 7176-25.	The performance of batteries and charger of device meet the Requirements in Clause 5 and 6 of ISO 7176-25.	S.E.

J. Conclusion

The conclusion drawn from the nonclinical tests demonstrates that the subject device in 510(K) submission, Electric Wheelchair, Models: Panda 10 & Panda 12, is as safe, as effective, and performs as well as or better than the legally marketed predicate device cleared under K220747.