



Shenzhen Mindray Bio-medical Electronics Co., LTD.

April 22, 2024

% Tang Jing

Technical manager Regulatory Affairs, Technical Regulation Department
Mindray Building, Keji 12th Road South,
Hi-tech Industrial Park, Nanshan
Shenzhen, Guangdong 518057
CHINA

Re: K240115

Trade/Device Name: Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nuewa I8W, Nuewa I8, Nuewa I8 Exp, Nuewa I8S, Nuewa I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nuewa IY Diagnostic Ultrasound System

Regulation Number: 21 CFR 892.1550

Regulation Name: Ultrasonic Pulsed Doppler Imaging System

Regulatory Class: Class II

Product Code: IYN, IYO, ITX

Dated: March 21, 2024

Received: March 22, 2024

Dear Tang Jing:

We have reviewed your section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (the Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database available at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Additional information about changes that may require a new premarket notification are provided in the FDA guidance documents entitled "Deciding When to Submit a 510(k) for a Change to an Existing Device"

(<https://www.fda.gov/media/99812/download>) and "Deciding When to Submit a 510(k) for a Software Change to an Existing Device" (<https://www.fda.gov/media/99785/download>).

Your device is also subject to, among other requirements, the Quality System (QS) regulation (21 CFR Part 820), which includes, but is not limited to, 21 CFR 820.30, Design controls; 21 CFR 820.90, Nonconforming product; and 21 CFR 820.100, Corrective and preventive action. Please note that regardless of whether a change requires premarket review, the QS regulation requires device manufacturers to review and approve changes to device design and production (21 CFR 820.30 and 21 CFR 820.70) and document changes and approvals in the device master record (21 CFR 820.181).

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR Part 803) for devices or postmarketing safety reporting (21 CFR Part 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR Part 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR Parts 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,


Yanna S. Kang -S

Yanna Kang, Ph.D.
Assistant Director
Mammography and Ultrasound Team
DHT8C: Division of Radiological Imaging
and Radiation Therapy Devices
OHT8: Office of Radiological Health
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)

K240115

Device Name

Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nuewa I8W, Nuewa I8, Nuewa I8 Exp, Nuewa I8S, Nuewa I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nuewa IY Diagnostic Ultrasound System

Indications for Use (Describe)

Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nuewa I8W, Nuewa I8, Nuewa I8 Exp, Nuewa I8S, Nuewa I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nuewa IY Diagnostic Ultrasound System is applicable for adults, pregnant women, pediatric patients and neonates. It is intended for use in fetal, abdominal, Intra-operative (includes abdominal, thoracic, and vascular), Laparoscopic, pediatric, small organ (breast, thyroid, testes), neonatal and adult cephalic, trans-rectal, trans-vaginal, musculo-skeletal (conventional, superficial), Thoracic/Pleural, adult and pediatric cardiac, trans-esoph. (Cardiac), peripheral vessel, and urology exams.

Modes of operation include: B, M, PWD, CWD, Color Doppler, Amplitude Doppler, Combined mode(B+M, PW+B, Color+B, Power+B, PW+Color+B, Power+PW+B), Tissue Harmonic Imaging, Smart3D, 4D(Real-time 3D), iScape View, TDI, Color M, Strain Elastography, Contrast imaging (Contrast agent for LVO), V Flow, STE, STQ, Contrast imaging (Contrast agent for Liver).

This device is a general purpose diagnostic ultrasound system intended for use by qualified and trained healthcare professionals for ultrasound imaging, measurement, display and analysis of the human body and fluid, which is intended to be used in a hospital or medical clinic.

Type of Use (Select one or both, as applicable)

☒ Prescription Use (Part 21 CFR 801 Subpart D)

☐ Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRAStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(K) SUMMARY

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirements of SMDA 1990 and 21 CFR § 807.92(c).

The assigned 510(k) number: K240115

1. **Submitter**

Shenzhen Mindray Bio-medical Electronics Co., LTD
Mindray Building, Keji 12th Road South, Hi-tech Industrial Park, Nanshan,
Shenzhen, 518057, P. R. China

Tel: +86 755 8188 6129

Fax: +86 755 2658 2680

Contact Person

Tang Jing

Shenzhen Mindray Bio-medical Electronics Co., LTD
Mindray Building, Keji 12th Road South, Hi-tech Industrial Park,
Nanshan, Shenzhen, 518057, P. R. China

Date Prepared: January 8, 2024

2. **Device Name**

Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY,
Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva
I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva
IY Diagnostic Ultrasound System

Classification

Regulatory Class: II

Review Category: Tier II

21 CFR 892.1550 Ultrasonic Pulsed Doppler Imaging System (IYN)

21 CFR 892.1560 Ultrasonic Pulsed Echo Imaging System (IYO)

21 CFR 892.1570 Diagnostic Ultrasound Transducer (ITX)

3. **Predicate devices**

Resona I8W series Diagnostic Ultrasound System is comparable with and substantially equivalent to the predicate devices listed below. Resona I9 is the main

predicate devices.

Device	Manufacturer	Model	Device Class	Product Code	510K Number
1. Main predicate device	Mindray	Resona I9	II	IYN, IYO, ITX	K210699
2. Reference device	Mindray	Resona R9	II	IYN, IYO, ITX	K222928
3. Reference device	Mindray	MX7	II	IYN, IYO, ITX	K212900
4. Reference device	Mindray	TEX20	II	IYN, IYO, ITX	K220242
5. Reference device	Mindray	TE7	II	IYN, IYO, ITX	K203391
6. Reference device	Mindray	DC-80A	II	IYN, IYO, ITX	K201693
7. Reference device	Mindray	ZS3	II	IYN, IYO, ITX	K192410
8. Reference device	Mindray	Consona N9	II	IYN, IYO, ITX	K221300
9. Reference device	Mindray	Voluson E10	II	IYN, IYO, ITX	K181985

The result shows the conformance of subject device to the predicate devices.

Regulation name and code

21 CFR 892.1550 Ultrasonic Pulsed Dopple Imaging System

(IYN)

21 CFR 892.1560 Ultrasonic Pulsed Echo Imaging System (IYO)

21 CFR 892.1570 Diagnostic Ultrasound Transducer (ITX)

4. Device Description:

The Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System is a general purpose, mobile, software controlled, ultrasonic diagnostic system. Its function is to acquire and display ultrasound images in Modes of operation include: B, M, PWD, CWD, Color Doppler, Amplitude Doppler, Combined mode(B+M, PW+B, Color+B, Power+B, PW+Color+B, Power+PW+B), Tissue Harmonic Imaging, Smart3D, 4D(Real-time 3D), iScape View, TDI, Color M, Strain Elastography, Contrast imaging (Contrast agent for LVO), V Flow, STE, STQ, Contrast imaging (Contrast agent for Liver).

The Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System can also measure anatomical structures and offer analysis packages to provide information based on which the competent health care professionals can make the diagnosis.

5. Indications for Use:

Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System is applicable for adults, pregnant women, pediatric patients and neonates. It is intended for use in fetal, abdominal, Intra-operative (includes abdominal, thoracic, and vascular), Laparoscopic, pediatric, small organ (breast, thyroid, testes), neonatal and adult cephalic, trans-rectal, trans-vaginal, musculo-skeletal (conventional, superficial), Thoracic/Pleural, adult and pediatric cardiac, trans-esoph. (Cardiac), peripheral vessel, and urology exams.

Modes of operation include: B, M, PWD, CWD, Color Doppler, Amplitude Doppler, Combined mode(B+M, PW+B, Color+B, Power+B, PW+Color+B, Power+PW+B), Tissue Harmonic Imaging, Smart3D, 4D(Real-time 3D), iScape View, TDI, Color M, Strain Elastography, Contrast imaging (Contrast agent for LVO), V Flow, STE, STQ, Contrast imaging (Contrast agent for Liver).

This device is a general purpose diagnostic ultrasound system intended for use by qualified and trained healthcare professionals for ultrasound imaging, measurement, display and analysis of the human body and fluid, which is intended to be used in a hospital or medical clinic.

6. Comparison with Predicate Devices:

Subject device Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System is comparable with and substantially equivalent to these predicate devices mentioned in 3. *Predicate* Devices with regards to intended use, imaging modes, features and functions and technological characteristics.

- All systems transmit ultrasonic energy into patients, perform post processing of received echoes to generate onscreen display of anatomic structures and fluid flow within the body. All systems allow for specialized measurements of structures and flow, as well as calculations.
- Subject device Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System has the same intended uses as the predicated device Resona I9 (K210699).

- The patient contact materials of the transducers and needle-guided brackets of subject device Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System are the same to the predicate devices.
- The acoustic power levels of Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System are below the limits of FDA, which are the same as the predicated device Resona I9 (K210699).
- Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System is designed in compliance with the FDA recognized electrical and physical safety standards, which are the same as the predicated device Resona I9 (K210699).
- Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System has the equivalent features and functions as the predicated devices. Among these features, the USAT, HRI+, HiFR CEUS, CEUS Chrono-Parametric Mode, TCMR, 3D-Print Format, Ultra-Micro Angiography, DICOM Pediatric SR, DICOM Urology SR and Biopsy Grid supported in proposed Resona I8W series has been cleared in predicate device Resona R9(K222928).
- Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System has the equivalent features and functions as the predicated devices. Among these features, the Smart Bladder and ClamAV supported in proposed Resona I8W series has been cleared in predicate device Consona N9 (K221300).
- Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System has the equivalent features and functions as the predicated devices. Among these features, the IOTA supported in proposed Resona I8W series has been cleared in predicate device Voluson E10 (K181985).

7. Non-clinical Tests:

Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona

I8 Easi, Eagus I8 Easi, Nuewa I8W, Nuewa I8, Nuewa I8 Exp, Nuewa I8S, Nuewa I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nuewa IY Diagnostic Ultrasound System has been evaluated for acoustic output, biocompatibility, cleaning and disinfection effectiveness as well as thermal, electrical and mechanical safety, and this device has been designed to conform with applicable medical safety standards.

This device has been tested and evaluated under the following standards:

- NEMA UD 2-2004 (R2009), acoustic output measurement standard for diagnostic ultrasound equipment revision 3.
- ANSI AAMI ES60601-1:2005/(R)2012 & A1:2012, C1:2009/(R)2012 & A2:2010/(R)2012 (Cons. Text) [Incl. AMD2:2021] Medical electrical equipment - Part 1: General requirements for basic safety and essential performance (IEC 60601-1:2005, MOD) [Including Amendment 2 (2021)].
- IEC 60601-1-2 Edition 4.1 2020-09 CONSOLIDATED VERSION, Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests.
- IEC TR 60601-4-2 Edition 1.0 2016-05 Medical electrical equipment - Part 4-2: Guidance and interpretation - Electromagnetic immunity: performance of medical electrical equipment and medical electrical systems
- IEC 60601-1-6 Edition 3.2 2020-07 CONSOLIDATED VERSION, Medical electrical equipment - Part 1-6: General requirements for basic safety and essential performance - Collateral standard: Usability.
- IEC 60601-2-37 Edition 2.1 2015, Medical electrical equipment - Part 2-37: Particular requirements for the basic safety and essential performance of ultrasonic medical diagnostic and monitoring equipment.
- ISO 14971 Third Edition 2019-12, Medical devices - Application of risk management to medical devices.
- ISO 10993-1 Fifth edition 2018-08, biological evaluation of medical devices - part 1: evaluation and testing within a risk management process.
- IEC 62304 Edition 1.1 2015-06, medical device software - software life cycle processes.
- IEC 62366-1 Edition 1.1 2020-06 CONSOLIDATED VERSION, Medical devices - Part 1: Application of usability engineering to medical devices.

These non-clinical tests relied on in this premarket notification submission can support the determination of substantial equivalence of the subject device.

8. Clinical Studies

Not applicable. The subject of this submission, Resona I8W, Resona I8, Resona I8 Exp,

Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System, does not require clinical studies to support substantial equivalence.

9. Summary

Based on the performance data as documented in the study, Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System was found to have a safety and effectiveness profile that is similar to the predicate device.

10. Conclusion:

Intended uses and other key features are consistent with traditional clinical practices, FDA guidelines and established methods of patient examination. The design, development and quality process of the manufacturer confirms with 21 CFR 820, ISO 9001 and ISO 13485 quality systems. The device conforms to applicable medical device safety standards. Therefore, the Resona I8W, Resona I8, Resona I8 Exp, Resona I8S, Resona I8T, Resona IY, Resona I8 Easi, Eagus I8 Easi, Nueva I8W, Nueva I8, Nueva I8 Exp, Nueva I8S, Nueva I8T, Imagyn I8, Imagyn I8S, Imagyn I8T, Imagyn I8 Exp, Nueva IY Diagnostic Ultrasound System is substantially equivalent with respect to safety and effectiveness to the predicate device Resona I9.